

# LYMED<sup>®</sup>

**LYMED CATALOGUE**

MEDICAL DEVICES

**1/2026**



**SKIN  
COMPRESSION  
SENSE  
INTERIM & LIGHT  
POST-OPERATION**

**ISO 13485**

CERTIFIED QUALITY MANAGEMENT SYSTEM



**VALPAS**  
CERTIFICATIONS



**All up-to-date information:**

**LYMED.FI**

**FOR PROFESSIONALS:**

**Catalogue**

**Order forms**

**Measuring instructions**

**FOR USERS:**

**Instructional videos for donning**

**Wear & care instructions**

**Customer stories**

**Frequently asked questions**

# LYMED CATALOGUE

## TABLE OF CONTENTS

### PRODUCT INFORMATION

Company	4
What are pressure garments?	5
Lymed medical devices	6
Product groups and context of use	7
Guidance for selecting the appropriate product	12
Ordering Lymed products	14
Pressure and compression	16
Lymed materials	18
Choosing fabric and pressure levels for custom-made products	20
Choosing fabric and pressure levels for Sense products	21
Colour and emblem options	22
Silicone and elastane bands	23
Additions and special solutions	24
Ordering and product codes	27
Additions for custom-made products	28
Customisation options for standard-sized products	34
Sense functional supports	38

### SKIN™ Pressure garments for scar management

Vests and abdominal support	41
Sternal straps	42
Bodys	44
Pants	45
Legs and stockings	46
Sleeves and gloves	48
Web spacer and digital cap	52
Head and neck products	54

### COMPRESSION™ Medical compression garments

Vests and bodys	58
Bras and bra tops	60
Compression bras	62
Pants	64
Suspensor and penis tube	72
Maternity pants	73
Legs and stockings	74
Sleeves and gloves	76
Head products	80

### SENSE™ Neurological pressure garments

Scapula support vest and body	83
Vests	84
Overalls	85
Bodys	86
Pants	88
Legs and stockings	89
Sleeves and gloves	90

### INTERIM & LIGHT™ Interim care and light pressure garments

Vests	97
Pants	100
Gloves and sleeves	102
Stockings	105
Head products	106

### POST-OPERATION™ Post-operational support garments

Pants for women	108
Vest and sleeves	113
Bras and breast wrap	116
Abdominal bands	123
Head products	125



reddot award 2014  
winner



Lymed Oy is a Finnish family-owned company founded in 1993 in Tampere that designs, manufactures, and sells medical pressure garments and related wellness products.

From the outset, Lymed has operated internationally as a developer and pioneer of high-quality pressure garments. Lymed's product range is the most extensive in the world, and our product selection covers all applications of pressure garment therapy.

Lymed operates as a medical aid manufacturer specialising in highly customised, client-specific solutions, based on our extensive expertise in pressure garment manufacturing.

Lymed's multidisciplinary team combines technology with strong textile industry expertise to deliver the best possible pressure garments for our clients.

Lymed's wide-ranging expertise is reflected in the fact that the company also manufactures products for animals such as dogs, horses, and camels.

### MANUFACTURING OF LYMED PRODUCTS

The aim of every Lymed product is to provide the best possible fit, excellent comfort, and a refined, presentable appearance.

Lymed products meet the highest quality standards without compromising on comfort. Unique custom-made products and special solutions are an integral part of our offering, as individual needs—such as challenges related to donning and doffing, work and hobbies, and physical limitations—can be carefully considered in the design process.

All Lymed custom-made and standard-sized products are designed in Tampere, Finland. Our custom-made products are manufactured from start to finish in Tampere. Cutting and sewing of standard-sized products takes place in Tampere as well as Estonia.

### SHARING OUR EXPERTISE AND TRAINING NEW MEASURERS

Lymed and our country-specific distributors continuously provide training for measurers, purchasers, and all healthcare professionals interested in compression garments.

Our measurers are typically occupational therapists, physiotherapists, or experts in neurological occupational therapy and physiotherapy, working in both the private and public sectors.



# WHAT ARE PRESSURE GARMENTS?

A pressure garment is a soft and elastic medical device to support bodily function by utilising externally applied pressure. Pressure garments can be used by users of different ages in a wide range of use contexts as part of a care and rehabilitation process related to diseases, injuries, and functional capacity challenges.

In practice, a pressure garment is a closely fitting product that can be designed according to its primary intended use, the intended support effect, and the body area to which pressure is applied. The general term pressure garment may be used to describe both pressure and support garments that provide even pressure, as well as compression garments in which the pressure is graduated and decreases towards the central parts of the body.

Pressure garments are used as part of a treatment plan guided by a healthcare professional, based on an individual assessment and the recommendation of the treating professional. Pressure garments do not replace other medical treatment or the instructions provided by healthcare professionals.

## IN WHICH CONTEXTS ARE PRESSURE GARMENTS COMMONLY USED?

Pressure garments are part of comprehensive treatment and recovery-supporting practices in which external pressure supports bodily function across different applications.

### SCAR MANAGEMENT

Pressure garments are used as part of scar treatment to support scar management and the scar maturation process. They are suitable, for example, for:

- **burn scars**
- **hypertrophic scars**
- **keloids**

Pressure garments support the scar maturation process and help manage scar-related thickening, stiffness, and discomfort.

### OEDEMA MANAGEMENT

Compression garments are used to support the management of oedema in situations where the body's fluid and blood circulation is compromised. They may be utilised, for example, in individuals whose oedema is associated with:

- **venous insufficiency**
- **lymphatic oedema**
- **postoperative or post-procedural conditions**
- **prolonged immobility or physical strain**

Compression garments support the body's natural blood and fluid circulation through external compression.

### SENSORIMOTOR APPLICATIONS

Pressure garments can be used as support in sensorimotor applications, such as:

- **situations related to movement disorders and challenges in motor control**
- **situations related to pain conditions and bodily tension**
- **situations involving postural misalignments and load-related challenges**

Pressure garments support body awareness, proprioceptive feedback, and movement control, and may help alleviate discomfort and support relaxation and bodily calming during daily activities and rehabilitation.

### POSTOPERATIVE AFTERCARE

Pressure garments are used after surgery as part of the recovery process, for example:

- **following various invasive procedures**

Pressure garments support the management of swelling, bruising and scars, provide external support to the operated area, and may help alleviate recovery-related discomfort.

## MOST PERSONALISATION OPTIONS


Challenging product solutions and individual requests can be achieved in custom-made products.



Read more on page 24

## EASY TO CARE FOR

The products are machine washable, dryer-safe, and quick-drying.

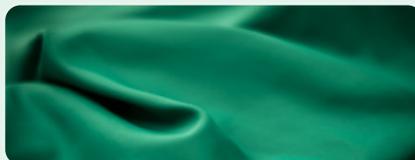

## LONG LIFESPAN

The products are designed to withstand everyday use. Repairs and modifications are possible, even after production.



Read more on page 15

## COMFORT THROUGH DESIGN

Anatomically shaped and contoured joint areas to achieve a functional fit.



## THIN, NON-ALLERGENIC FABRICS

Better wearing comfort with the same pressure levels: Thin products allow movement and do not hinder everyday life.

Lymed materials do not contain natural rubber or other allergens.

Read more on page 18 forward

## FLAT SEAM STRUCTURE

Our seams are tailored to be as comfortable as possible against the skin.



Read more on page 18

# LOOKS LIKE CLOTHING, FEELS LIKE CLOTHING, BUT IS A MEDICAL DEVICE

A medical pressure garment is intended to be used as part of the treatment and rehabilitation of a disease or injury and is classified as a medical device in accordance with the EU Medical Device Regulation (MDR).

The use of a medical pressure garment differs from ordinary clothing: its use is planned and guided according to instructions.

### LYMED-CUSTOM-MADE PRODUCTS BY PRODUCT CATEGORY:

- Skin™
- Compression™
- Sense™

Device class: Class I medical device (MDR)  
Device type: Custom-made medical device

### LYMED-STANDARD-SIZED PRODUCTS BY PRODUCT CATEGORY:

- Skin™
- Compression™
- Sense™
- Interim & Light™
- Post-operation™

Device class: Class I medical device (MDR)

# LYMED PRODUCT INFORMATION

## Product groups and context of use



# SKIN™

## PRESSURE GARMENTS FOR SCAR MANAGEMENT

CUSTOM-MADE

STANDARD-SIZE



Pressure garments are used as part of scar treatment to support scar management and the scar maturation process, particularly following burn injuries, extensive skin grafts, and surgical procedures. They are intended for the management of hypertrophic scars and keloids and may also be used preventively when wound or skin graft healing is prolonged and the risk of hypertrophic scar formation is increased.

Pressure garments support scar management through external pressure. Even pressure may help support the scar maturation process, manage scar thickening and stiffness, and alleviate scar-related discomfort such as itching, pain-related sensations, and redness. The benefits of pressure garments are greatest when they are used during the scar maturation phase as part of a structured scar management plan.

Pressure garments may be used as part of an individual treatment plan in certain skin-related conditions, as assessed by a healthcare professional, and may also assist in keeping dressings or wound coverings in place as part of an overall treatment, support, or recovery plan.

In pressure garments intended for scar management, the pressure is even throughout the product. Pressure garments are recommended to be ordered as custom-made to ensure optimal fit and to reduce the risk of irritation to healing skin. Pressure garment therapy may be initiated with lower-pressure products to allow gradual adaptation to the actual pressure garment. Interim pressure garments with lower pressure levels can be used earlier, even before the start of active scar management; see the Interim & Light product group.

Lymed Skin products are primarily custom-made for all body parts. The products are manufactured with pressure levels intended for scar management. In addition, a lightened pressure level is available, which is particularly suitable for children.

### TYPICAL CONTEXTS OF USE

- Hypertrophic scars
- Burn scars
- Keloids
- Other post-surgical or post-injury scars

### MOST COMMON GOALS OF USE

- Supporting scar shaping and preventing scar thickening
- Supporting the scar maturation process
- Management of scar-related redness, itching, and pain-related sensations

# LYMED PRODUCT INFORMATION

## Product groups and context of use



## COMPRESSION™

### MEDICAL COMPRESSION GARMENTS

CUSTOM-MADE

STANDARD-SIZE



Pressure garments used for the management and prevention of oedema are referred to as compression garments.

Medical compression garments are used as part of oedema management and prevention in various conditions associated with swelling and circulatory load. Compression garments support the body's blood and fluid circulation through external compression.

Compression garments may also be used preventively in situations where the load on fluid circulation in the lower limbs or other parts of the body is increased. Such situations include prolonged sitting or standing work, travel, pregnancy, and the recovery phase following surgical procedures.

Compression garments may be used in selected inflammatory or connective tissue-related conditions when the goal is to support the management of associated oedema as part of an overall treatment plan. Compression garments may also be used in venous conditions where wound dressings are used on the lower limbs, to support the management of oedema and swelling and to assist in keeping dressings or wound coverings in place, as part of an overall treatment, support, or recovery plan.

The pressure in compression garments is built so that it gradually decreases from distal to proximal areas, toward the centre of the body, with the strongest pressure at the extremities. This may help reduce fluid accumulation in the tissues and support oedema management. Lymed Compression products are primarily custom-made, and the pressure they provide is divided into four compression classes based on strength. Lymed Compression products can generally be used both during the day and at night.

### TYPICAL CONTEXTS OF USE

- Primary and secondary lymphoedema
- Lipoedema
- Venous circulatory load conditions such as venous insufficiency and varicose veins
- Venous lower limb conditions with wound dressings
- Postoperative or post-injury swelling

### MOST COMMON GOALS OF USE

- Supporting oedema management and preventing recurrence of swelling
- Supporting blood and fluid circulation
- Supporting the management of swelling and swelling-related discomfort
- Supporting the placement of dressings and wound coverings

# LYMED PRODUCT INFORMATION

## Product groups and context of use



## SENSE™ NEUROLOGICAL PRESSURE GARMENTS

Pressure garments are used as rehabilitation aids and tools in neurological physiotherapy and occupational therapy as part of a professionally guided rehabilitation and treatment plan.

The effect of neurological pressure garments is based on external pressure that supports body control, movement regulation, and body awareness. The pressure applied by the pressure garment may support proprioceptive feedback, which can help support muscle activation and movement control during rehabilitation. Pressure garments may also provide external support to joints, which can help manage joint hypermobility and support predictability and control of movement in everyday situations.

In clinical practice, pressure garments are also used as part of pain management alongside other treatment modalities, particularly in situations where pain is associated with impaired body control, joint hypermobility, sensorimotor deviations, or tissue load.

According to professional experience, the even external pressure provided by pressure garments and their body-awareness-enhancing effect may support the management of pain-related discomfort as part of an

CUSTOM-MADE

STANDARD-SIZE



overall treatment approach. In clinical practice, pressure garments are used, for example, to support pain management in patients with CRPS and EDS, as part of a treatment plan assessed by a healthcare professional, without being considered a replacement for other medical or therapeutic treatment.


Lymed Sense pressure garments can be extensively customised according to individual goals to support daily functioning and rehabilitation objectives.

### TYPICAL CONTEXTS OF USE

- Disorders of movement regulation and motor control
- Neurodevelopmental disorders and conditions with sensorimotor symptoms
- Central or peripheral nervous system injuries and dysfunctions
- Musculoskeletal load-related conditions
- As part of pain management
- Musculoskeletal diseases and conditions

### MOST COMMON GOALS OF USE

- Supporting sensorimotor function and movement regulation
- Strengthening body schema and body awareness
- Supporting proprioceptive feedback
- Supporting movement quality and control
- Providing external joint support for managing hypermobility
- Supporting pain management
- Supporting functional ability and daily performance

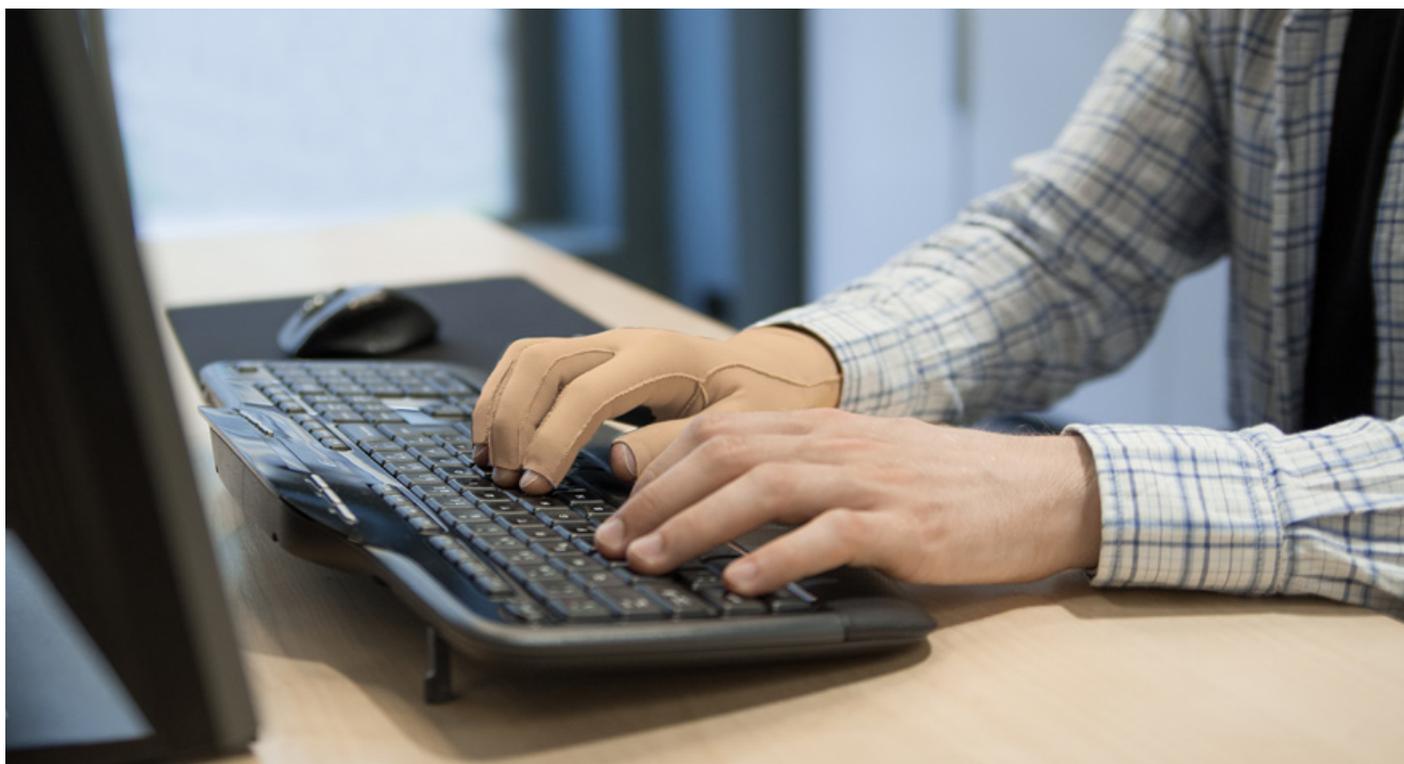

## INTERIM & LIGHT™

### INTERIM CARE AND LIGHT PRESSURE GARMENTS

STANDARD-SIZE



Lymed Interim & Light products are standard-sized pressure garments that provide light and even external pressure, primarily used as part of scar management. The products are designed for short-term use during the transition phase before moving to custom-made products or as part of the early phase of treatment when pressure garment therapy cannot yet be initiated or when the user is being accustomed to pressure garment use. The products may also be used to help keep dressings in place.

Lymed Interim & Light products may also be used as part of pain management alongside other treatment modalities, particularly when light external support and a body-awareness-enhancing effect are needed in the early phase of treatment prior to the introduction of a custom-made product.

In addition, Lymed Interim & Light products may be used as part of an individual treatment plan in certain skin-related conditions when a healthcare professional assesses that light external pressure supports the treatment goals.

#### TYPICAL CONTEXTS OF USE

- Management of various types of scars
- Short-term use as part of pain management
- Supporting the retention of dressings

#### MOST COMMON GOALS OF USE

- Supporting scar management and the maturation process in the early phase
- Acclimatising the user to pressure garment use
- Providing light pressure prior to the use of optimal pressure garment

# LYMED PRODUCT INFORMATION

## Product groups and context of use



## POST-OPERATION™

### POST-OPERATIONAL SUPPORT GARMENTS

STANDARD-SIZE



Postoperative support garments are pressure and support garments used as part of the aftercare following various invasive surgical procedures.

Support garments provide external pressure and/or external support, which supports postoperative recovery by offering external support to the operated body area.

The pressure and external support may help support the management of oedema, swelling, bruising, and scars, alleviate recovery-related discomfort, and support movement and functional activity during recovery.

Postoperative support garments may also be used to assist in keeping dressings or wound coverings in place as part of an overall treatment or recovery plan.

Lymed Post-operation products are standard-sized products with even pressure.

#### TYPICAL CONTEXTS OF USE

- Abdominal and lower body procedures
- Breast reconstruction and reshaping procedures
- Facial and upper arm procedures
- Liposuction procedures
- Recovery after childbirth and caesarean section
- Aftercare following various invasive surgical procedures


#### MOST COMMON GOALS OF USE

- Providing external support to the operated area
- Supporting the management of oedema, swelling and bruising
- Supporting surgical scar management
- Supporting the management of pain and discomfort related to the surgical area
- Supporting the placement of dressings and wound coverings

# LYMED PRODUCT INFORMATION

Guidance for selecting the appropriate product


## Scar management

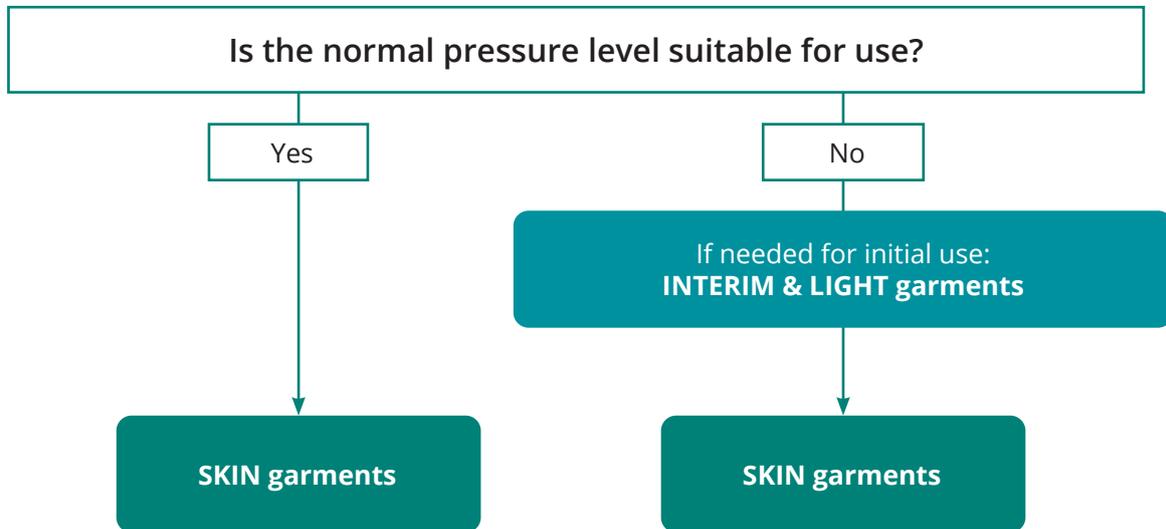


## Support of fluid and blood circulation Venous insufficiency



**Sensorimotor and neurological applications**  
**Pain management**

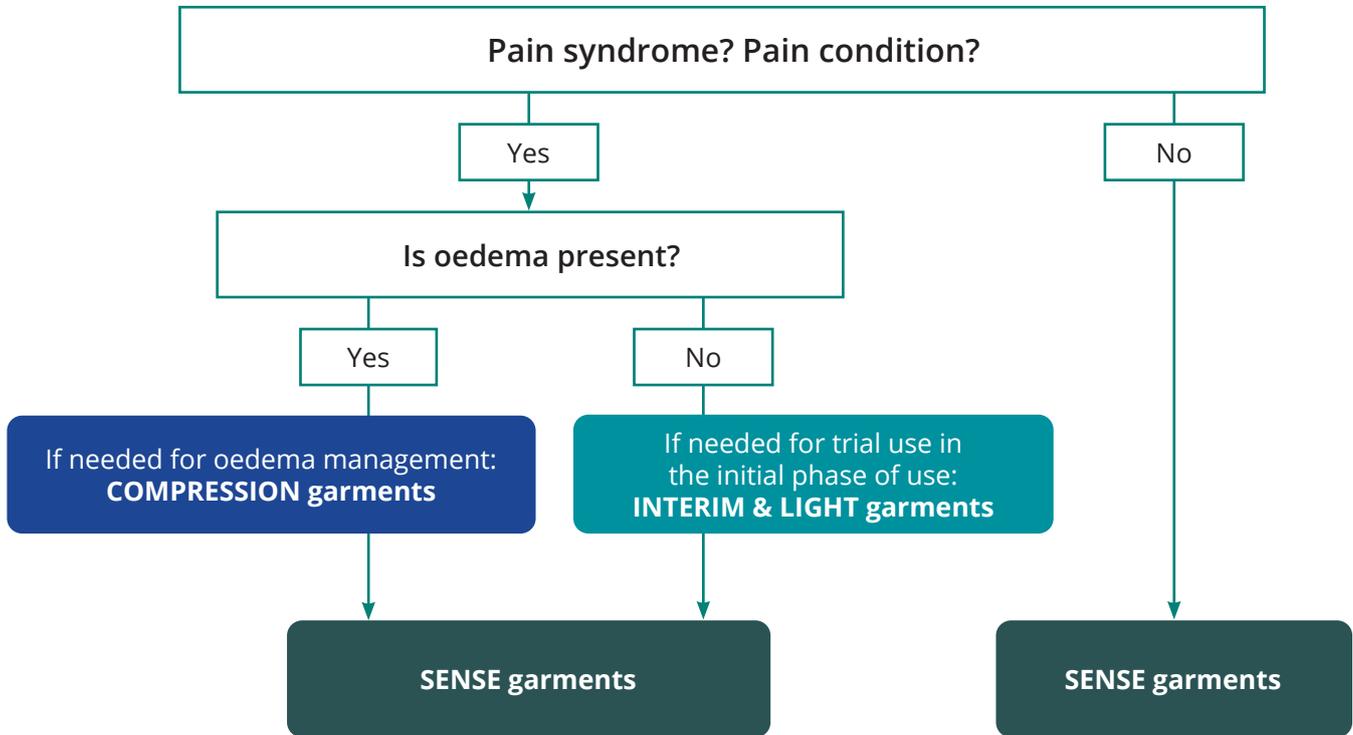

**Post-operative aftercare**

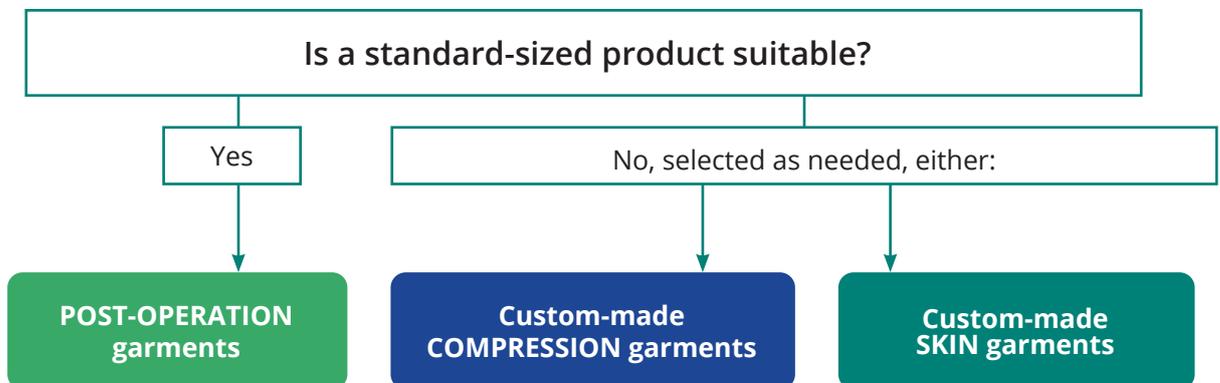

# ORDERING LYMED PRODUCTS

Lymed products are available for both the public and private healthcare sectors. Trained Lymed measurers can be found across all of our distributing countries.

Our measurers are primarily involved in compression treatment, occupational therapy, physiotherapy, and neurological physiotherapy. They manage the entire ordering process for custom-made Lymed products.

Standard-sized products are available through our distributors. Customers in Finland can also purchase directly through Lymed.

For more details, please visit our website at [www.lymed.fi](http://www.lymed.fi).

## ORDER PROCESS:

### CUSTOM-MADE PRODUCT

#### 1. Choosing a product

- The party responsible for the treatment assesses the need for the use of a pressure garment and selects an appropriate product based on the product information presented in the catalogue.

#### 2. Measuring and ordering

- Measuring the patient according to Lymed's instructions.
- Providing additional information, special requirements and photos.

#### 3. Sending orders to a Lymed distributor

- The order will be reviewed, and the customer will be contacted if necessary.

#### 4. Delivery

- The products are delivered as agreed.

### STANDARD-SIZED PRODUCT

#### 1. Choosing a product

- The party responsible for the treatment assesses the need for the use of a pressure garment and selects an appropriate product based on the product information presented in the catalogue.

#### 2. Choosing a size

- Choosing the correct size based on the instructions in the catalogue.

#### 3. Sending orders to a Lymed distributor

- The order will be reviewed, and the customer will be contacted if necessary.

#### 4. Delivery

- The products are delivered as agreed.

## WHEN SHOULD I ORDER A NEW PRODUCT?

Lymed products are durable and designed for long-term use. Due to their functional properties, the products are subject to significant mechanical stress, which causes them to wear faster than ordinary clothing. For this reason, their service life cannot be compared to that of regular garments or socks.

With regular use, the service life of Lymed products typically ranges from three to six months. The estimated service life is based on reasonably foreseeable external influences and environmental conditions. In high wear and tear products such as a stockings and gloves, the service life may vary considerably between users.

Pressure garments are recommended to be replaced at least every six months, and products in active use every three months.

For active users, such as children and working adults, replacement may be required more frequently. The condition and fit of the product should be actively monitored during the period of use.

### **The rate of wear is affected by e.g.:**

- Purpose and frequency of use
- Strength of pressure or compression
- Individual solutions and anatomy of the user
- The rubbing effect of external factors and the use of other aids
- Acidity of skin
- Care and washing of the product

### **A PRESSURE GARMENT NEEDS TO BE REPLACED WHEN**

- Patient's weight or measurements change
- Washing no longer cleans the material properly and the feel of the product becomes greasy
- Elasticity of the product decreases and the original pressure levels change
- The product or its part breaks

### **TWO LAST LONGER THAN ONE**

- If you wear a pressure garment continuously, it is recommended to have multiple products to use in rotation.
- Alternating between multiple products makes washing easier and allows the fabric to rest between uses.
- Alternating helps extend the service life of both products.

## REPAIRS AND MODIFICATIONS

As medical devices, medical pressure garments should not be repaired by the user, even if they have sewing skills.

Lymed offers repair and modification services for its products. The availability of the service may vary depending on your country of residence.

Repairs and modifications are carried out in accordance with the terms and conditions of Lymed, and each repair or modification is assessed on a case-by-case basis.

If the product is heavily worn, Lymed recommends considering ordering a new product instead of having it repaired.

More information about modifications and repairs from your local Lymed distributor.

**Read Lymed's terms and conditions in full at:**  
**[www.lymed.fi](http://www.lymed.fi)**

**The terms and conditions may vary depending on the product's granting party or your country of residence.**

### HOW PRESSURE IS CREATED WITH PRODUCTS

The effects of compression and pressure garments are based on the controlled implementation of pressure designed into the product.

The fabric itself does not generate any pressure; instead, pressure is created through the negative relationship between the circumference measurements of the treated area and the product's circumference, based on the fabric's elasticity and recovery properties.

The pressure provided by the product is generally divided into two different types according to the treatment goals. Compression garments are designed with progressive compression, providing the strongest support in the extremities, which gradually decreases towards the midsection. In even pressure garments, the pressure is evenly distributed throughout the product.

#### CUSTOM-MADE VS. STANDARD-SIZED PRODUCT


When acquiring a pressure garment, it should be assessed whether a standard-sized product can be used or whether a custom-made product is required.

As a general principle, Lymed always recommends the use of a custom-made product. Custom-made products generally provide a better fit than standard-sized products. Significant swelling, anatomical characteristics, and individual special requirements generally guide the choice towards a custom-made product.


The fit of a standard-sized product must be carefully verified: the more closely the user's measurements correspond to the size chart, the more likely it is that the fit is correct and that the pressure is applied appropriately.

#### COMPRESSION

#### EVEN PRESSURE

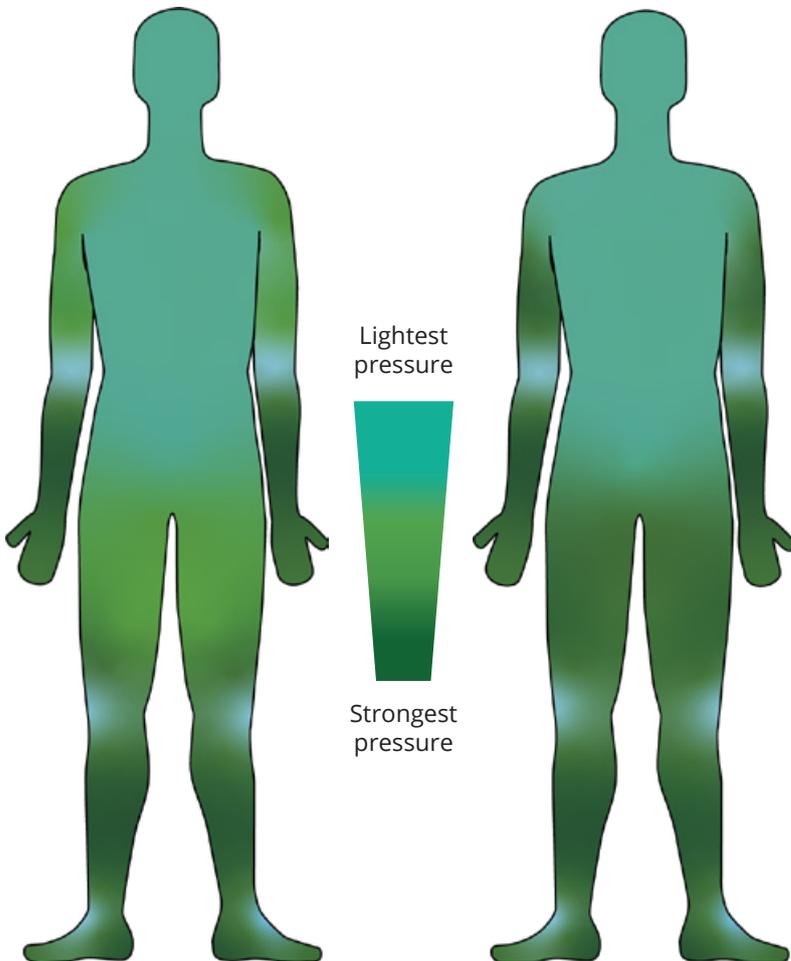

#### COMPRESSION VS. EVEN PRESSURE

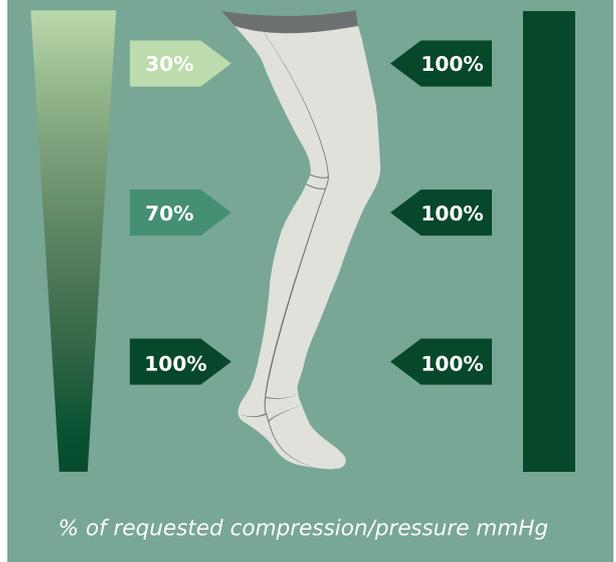

## COMPRESSION CLASSES AND PRESSURE LEVELS OF LYMED PRODUCTS

The pressure level or compression class of a product is selected based on the treatment need and goal, as well as the intended area of use. Lymed products are available with either even pressure or compression, depending on the product group and its specified purpose of use.

The unit used to measure the pressure produced by pressure garments is millimeters of mercury (mmHg).

A product's pressure class indicates the minimum and maximum pressure within which the actual average pressure typically falls. The realization of pressure on the treated area is influenced by the structure of the tissue and the user's level of activity. Firmer tissue or activating muscles provide counterpressure to the product, in which case the pressure applied to the tissue may in practice be higher.

When the product is worn, the actual pressure can be measured using pressure meters designed for pressure garment use. Pressure meters can also be used to monitor pressure during use and to assess the need for product replacement.

### COMPRESSION CLASSES

The compression classification of Lymed products (classes 1–4) is based on the European pre-standard ENV 12718:2001.

### PRESSURE LEVELS

The product-group-specific pressure levels offered by Lymed, as well as the possible general recommendations presented based on them, are based on reference values defined in the literature, use-related recommendations, and experience accumulated from customer use and needs.

COMPRESSION CLASSES CCL	
Progressive pressure	
<b>Compression products</b>	
<b>Ccl 1</b>	<b>15 - 21 mmHg</b>
<b>Ccl 2</b>	<b>23 - 32 mmHg</b>
<b>Ccl 3</b>	<b>34 - 46 mmHg</b>
<b>Ccl 4</b>	<b>49 mmHg and higher</b>

PRESSURE LEVELS	
Even pressure	
<b>Skin products</b>	
<i>Lightened pressure</i>	<b>16 - 22 mmHg</b>
<i>Normal pressure</i>	<b>23 - 30 mmHg</b>
<b>Sense products</b>	
<i>Light pressure</i>	<b>9 - 13 mmHg</b>
<i>Medium pressure</i>	<b>14 - 20 mmHg</b>
<i>Strong pressure</i>	<b>21 - 30 mmHg</b>
<b>Interim &amp; Light products</b>	<b>10 - 18 mmHg</b>
<b>Post-operation products</b>	<b>23 - 30 mmHg</b>



**Guidelines for the use of compression classes can be found in your country's healthcare guidelines or in international publications.**

# LYMED MATERIALS AND FABRICS


The experience of a pressure garment's comfort is influenced by many factors. The most important are the pressure level selected for the user and the feel of the fabric against the skin.

Lymed's fabrics are designed to withstand long-term use in pressure garment applications while remaining comfortable, breathable, and easy to care for. The fabrics are designed to be smooth, which makes donning easier.

We use three different fabrics with varying levels of rigidity, from which different pressure levels can be achieved to support specific treatment goals.

Lymed products are made from flat-knitted warp knit fabrics composed of a polyamide-elastane blend. Characteristics of warp knits include excellent resistance to pressure and the ability to manufacture pressure garments with complex shapes. In addition, warp knits cannot unravel.

By describing the treatment goals, the required pressure level, and the fabric choice that best supports the user's daily activities, we are able to create an individual medical aid that is also pleasant to wear.

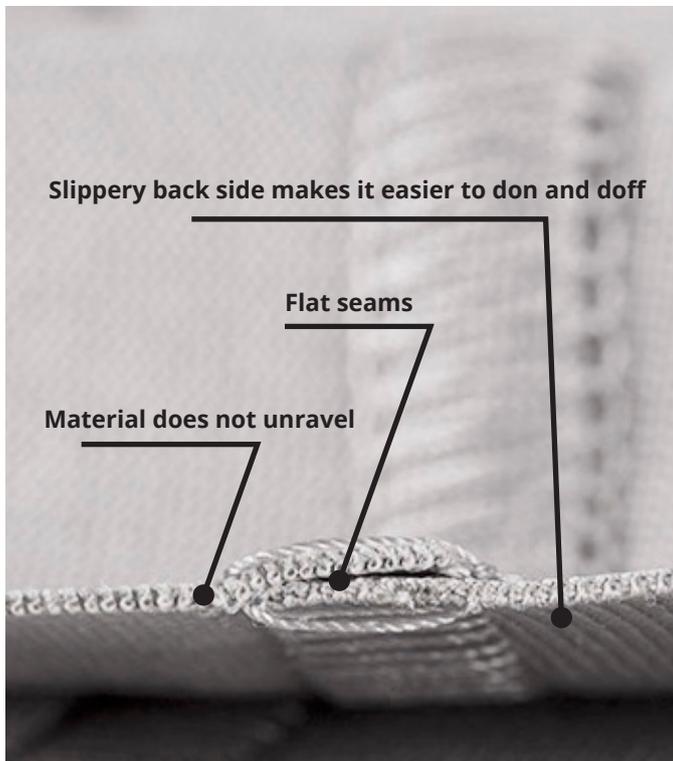


## Our textile materials comply with Oeko-Tex 100 certified.

Our materials are Oeko-Tex certified and do not contain rubber or other known allergens. The materials are responsibly manufactured in Europe.

The fabrics are tested for durability, colour fastness, and other properties. They pass even the most stringent tests—their abrasion resistance exceeds 150,000 Martindales (SFS-EN ISO 12947-2).

In addition, Lymed has developed non-standard, practice-simulating testing methods for continuous quality control in collaboration with Finnish research institutes.


The fabrics also provide sun protection with UPF 98 %.

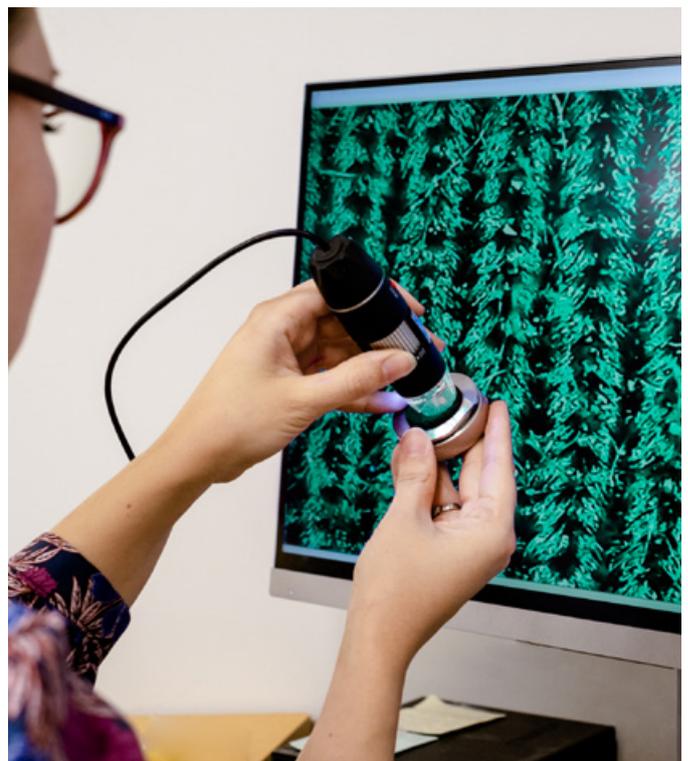

# THREE HIGH-QUALITY, LIGHTWEIGHT FABRICS



	<b>STRONG FABRIC</b>	<b>MEDIUM FABRIC</b>	<b>LIGHT FABRIC</b>
<b>Fabric description for ordering a custom-made product</b>	<p>Our most popular and widely used fabric among our customers.</p> <ul style="list-style-type: none"> <li>• Firm feel</li> <li>• A good primary choice if lighter option isn't needed.</li> <li>• The sole option for ccl 4 products.</li> </ul>	<p>An option for those who prefer a lighter feel, including children, the elderly, or people with sensitive skin.</p> <ul style="list-style-type: none"> <li>• Lighter and smoother than heavier fabrics.</li> <li>• May be easier to put on.</li> </ul>	<p>Only for light pressure Sense products for highly sensitive users.</p> <ul style="list-style-type: none"> <li>• Ultra lightweight feel</li> <li>• Especially suitable for babies.</li> </ul>
<b>Composition</b>	61 % polyamide 39 % elastane	54 % polyamide 46 % elastane	83 % polyamide 17 % elastane
<b>Structure</b>	Flat knitted warp knit	Flat knitted warp knit	Flat knitted warp knit
<b>Abrasion resistance (Martindale)</b>	>150 000	>150 000	90 000
<b>Breathability</b>	447 l/m <sup>2</sup> /s	466 l/m <sup>2</sup> /s	246 l/m <sup>2</sup> /s
<b>Thickness</b>	0,64 mm	0,57 mm	0,48 mm
<b>UV protection</b>	98 % UV protection	98 % UV protection	98 % UV protection

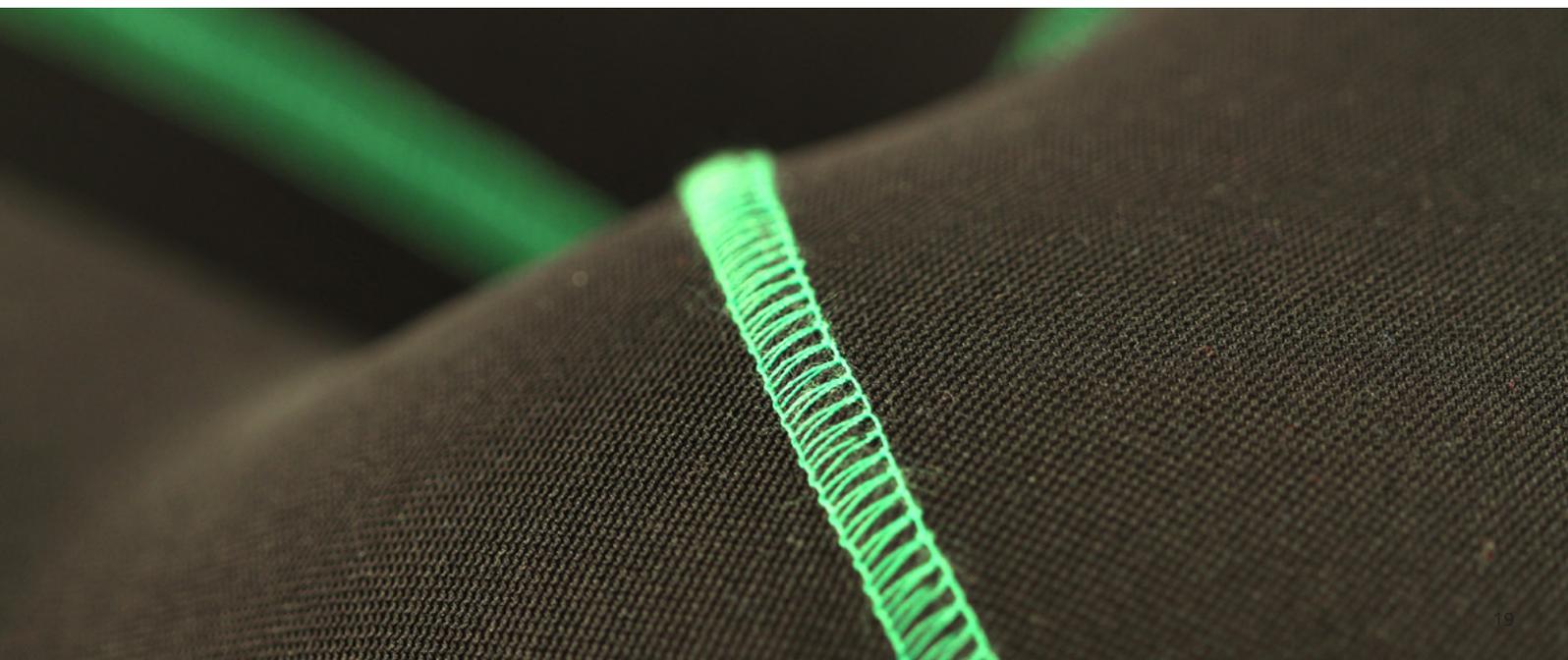

# LYMED PRODUCT INFORMATION

## Choosing fabric and pressure levels for custom-made products

The pressure level or compression class is selected according to the intended use and the goals of the overall treatment or rehabilitation plan, as well as the area of use of the product.

The user's response to the product may also vary over time, in which case product selections may need to be reviewed. If you are uncertain about fabric and pressure selections, always start with recommended options. The choice of material can affect the feel of the product and

## RECOMMENDED MATERIALS AND PRESSURE LEVELS

Product-group-specific, compatible pressure and fabric options ensure the product's performance.

Deviating solutions in fabric selection may in some cases be possible. Requests are evaluated on a case-by-case basis from a manufacturing perspective. The fabric options of standard-sized products cannot be changed.

### COMPRESSION

**CCL 1**

-  Strong fabric
-  Medium fabric

**CCL 2**

-  Strong fabric
-  Medium fabric

**CCL 3**

-  Strong fabric
-  Medium fabric

**CCL 4**

-  Strong fabric

thereby the perceived effect during use.

If the user requires a more supportive feel but a higher pressure level or compression class is not suitable, we recommend selecting a firmer fabric for the next product. If needed, the supportive feel can be enhanced with various reinforcements and double-layer fabrics; see additional options.

### SKIN

**NORMAL PRESSURE LEVEL**

-  Strong fabric
-  Medium fabric

**LIGHTENED PRESSURE LEVEL**

-  Strong fabric
-  Medium fabric

### SENSE [READ MORE ABOUT SENSE PRODUCTS](#)

**LIGHT PRESSURE**

-  Medium fabric\*
-  Light fabric
-  Strong fabric

**MEDIUM PRESSURE**

-  Medium fabric\*\*
-  Strong fabric

**STRONG PRESSURE**

-  Strong fabric
-  Medium fabric

## PRESSURE AND FABRIC CHOICES FOR SENSE PRODUCTS

The recommendations presented below are based on usage experience and customer feedback collected by Lymed, as well as on professionals' clinical practice.

- \*The most common Sense product for adults and children to support functionality is a light pressure product with medium fabric. There is rarely a need to order pressure garments for functionality support with strong pressure and a strong fabric.
- \*\*The first Sense product for an adult with pain syndrome is typically one with medium pressure and a medium fabric.

### PAIN CONDITIONS AND PAIN SYNDROMES

- The first Sense product for an adult with pain is typically one with medium pressure and a medium fabric.
- This group of users often use several products with different pressure levels and/or fabric types, which can be alternated even on a daily basis according to the desired pressure and feel.
- This user group may also more frequently choose a strong-pressure product with a strong fabric compared to other Sense product users.

### USERS WITH PAIN AND OEDEMA

- If the user presents with oedema, oedema management can, if necessary, first be supported with a medical compression product (Compression product group) before transitioning to Sense products.

### EHLERS-DANLOS SYNDROME (EDS)


- Pressure garment selection among users with EDS varies considerably. Some users require very light pressure, while others prefer the strongest possible pressure level.
- A typical challenge for the EDS user group in long-term use of pressure garments is variation in body circumferences, which affects the fit and feel of the product.
- If it is difficult to determine whether light or firmer feeling is needed, a product with light pressure and a medium fabric can be a good starting point.

### PRODUCTS FOR SCAPULA WINGING

- For this user group, the most typical pressure level is light, with the product made from either a strong or a medium fabric.

### SENSE PRODUCTS FOR CHILDREN

- For children under 10 years of age, the first Sense product ordered is often a product with light pressure and a medium fabric.
- For babies, we recommend a light fabric and light pressure.
- For children with a sensitive gag reflex, it is appropriate to consider a product with light pressure and a light fabric.

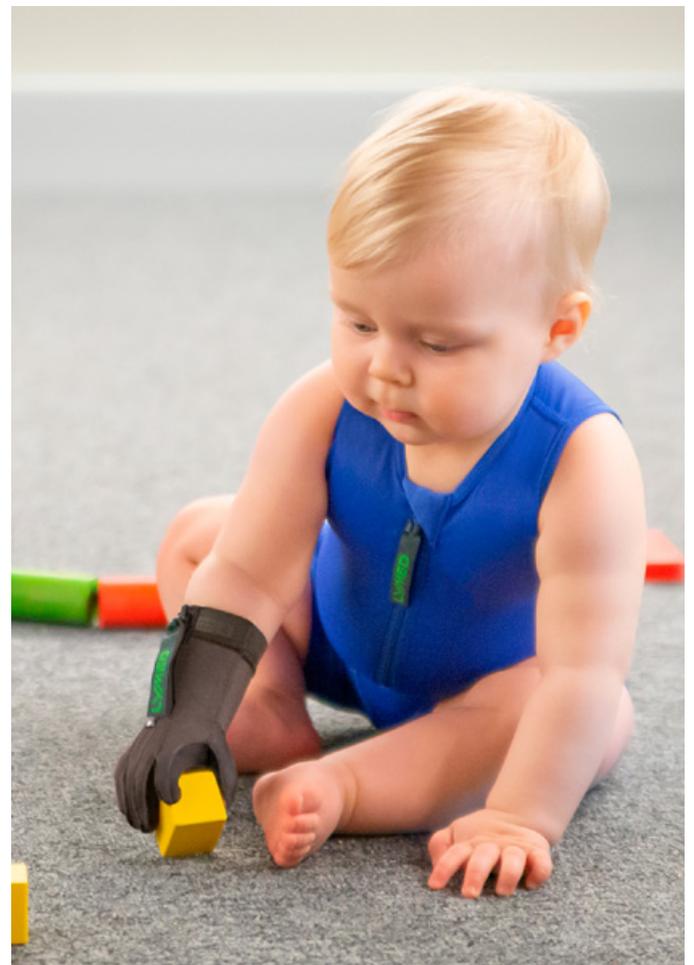

# LYMED PRODUCT INFORMATION

## Colour and emblem options

*Lymed materials do not contain rubber, coated rubber or other known allergens.*

### FABRIC COLOURS



**F1** Beige



**F2** Black



**F3** White



**F4** Grey



**F5** Blue



**F6** Red



**F7** Green

### SEAM COLOURS



**S1** Beige



**S2** Black



**S3** White



**S4** Grey



**S5** Blue



**S6** Red



**S7** Green



**S8** Pink



**S9** Bright green



**S10** Yellow



**S11** Orange



**S12** Bright blue



**S13** Purple

### DECORATED EMBLEMS

SIZE (DIAMETER): 2,2 CM



**P1** Cat



**P2** Dog



**P3** Fox



**P4** Bunny



**P5** Duck



**P6** Strawberry



**P7** Snowflake



**P8** Dino



**P9** Bear



**P10** Beetle



**P11** Ladybug



**P12** Turtle



**P13** Whale



**P14** Flower

*Our designers determine the placement of the emblem on each product.*

## IRRITATION CAUSED BY SILICONE BANDS

Based on literature, most reactions caused by silicones are non-allergic and non-specific. They are typically a result of irritation, which may be caused by factors such as the skin's properties, skin care products, hygiene, or the combination of all these elements. Individual factors, like skin pH, microbes, sweat, skin lotions, and personal hygiene, can contribute to irritation.



Irritation may worsen when sweating increases, such as in higher temperatures. The irritation can be reduced by cleaning the silicone band and the skin beneath it more frequently.

Each product can be manufactured without the silicone band or with the band repositioned. Several edging band options are available, and they can be swapped into the product as needed. In some cases, switching to a model that does not require silicone bands at all could be beneficial (e.g., using different strap options or models with more coverage). The best silicone band options for sensitive skin are SIL 1 and SIL 2.

# LYMED PRODUCT INFORMATION

## Silicone and elastane bands

Lymed elastane and silicone bands do not contain rubber, coated rubber, or other known allergens. The properties of each Lymed product are outlined on the product pages, including any elastane and silicone bands used in the individual product.

### ELASTANE BANDS




**ELA 1**  
Elastane band  
Width: 3 cm

**ELA 2**  
Elastane band  
Width: 2 cm

**ELA 3**  
Edging band / elastane band  
Width: 0,75 cm / 1,5 cm

**ELA 4**  
Elastane band  
Width: 5 cm

### SILICONE BANDS

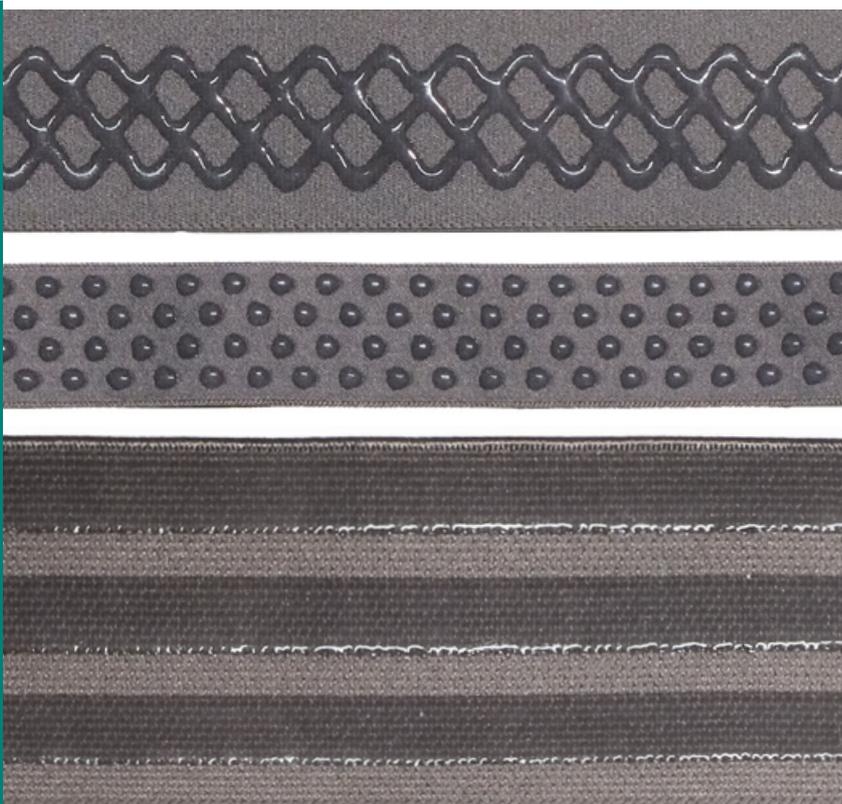

**SIL 1**  
Honeycomb silicone  
Width: 3 cm

**SIL 2**  
Droplet silicone  
Width: 2 cm

**SIL 4**  
3-stripe silicone  
Width: 5 cm

## CUSTOMISING LYMED PRODUCTS TO MEET USER NEEDS

HOW CAN WE SUPPORT INDEPENDENT DONNING AND DOFFING?

HOW CAN I HELP ENSURE THAT THE LYMED PRODUCT STAYS IN PLACE?

With our range of additional options, Lymed products can be extensively customised to meet the user's needs. Our catalogue presents the basic Lymed product models as well as the most common additions, enabling us to provide the user with a pressure garment that fits optimally and supports the goals of the overall treatment or care plan.

We are available to answer all product-related questions and to assist in designing a product that meets the needs of both the user and care personnel.

In complex cases, photographs, preliminary measurements, and detailed background information can help clarify the situation.

If there is no known need for additional solutions prior to the first order, Lymed recommends selecting a basic Lymed product model. Additional features may be considered when it is known that the basic model does not meet the needs, independent use of the basic model is difficult, or there is a desire to reduce the need for assisted donning.

### OUR SPECIALISED EXPERTISE

- Small-sized products for infants, as well as exceptionally large-sized products.
- Products adapted to individual body shapes, including solutions for dysmelia, extensive tissue defects, and stumps.
- Localised additional support and reinforcement, using double fabrics, splint pockets, and neoprene padding.
- Adjustable straps and hook-and-loop fasteners, to support functional body guidance.
- Special solutions for the genital area, including additional pressure flaps, pockets, and openings.

## EXAMPLES OF CUSTOM-MADE PRODUCTS BASED ON USER NEEDS



A customised solution for the amputation area

### THE LARGEST SELECTION OF STOCKING CHOICES AVAILABLE!

#### ADDITIONS FOR EASY USE

There are different solutions available for donning and doffing of Lymed stockings. The most popular is the zipper, which can be manufactured almost anywhere on the product, in all lengths.

Additions to products can be considered when there are difficulties in using the basic models, or there is a need for independent donning and doffing.



Anatomical ankle construction

#### TIP CHOICES FOR STOCKINGS

Stockings can be manufactured with various tip models depending on the client's personal needs.



Toes closed   Toes open   Toe sock, toe tips closed   Toe sock, toe tips open   Tabi sock   Big toe sock



Foot glove

#### ZIPPER CHOICES FOR LYMED STOCKINGS

A zipper is a helpful and fast solution in assistive donning and doffing. In independent donning a zipper may remove the need for other donning and doffing aids.

Zipper solutions are handy in wound care and changing of wound dressings, e.g. in leg ulcers.

A zipper can be placed almost everywhere on the product, based on the preference of the client and how the stocking will be donned. Dressing aid hooks can be added beneath the zipper to ease with the process of closing the zipper.



Full length, open edge   Full length, intact edge   Partial length

# LYMED PRODUCT INFORMATION

## Additions and special solutions



Zipper pull tab attached to the zipper slider to facilitate the closing of the zipper.



Pull-up loop attached to the product to aid in dressing.



Hook fastening.



Zipper guard to protect the skin and prevent skin or hair from getting caught in the zipper.

# ORDERING ADDITIONALS

Additional features are always ordered together with the product by entering the product code followed by the code(s) of the desired additional feature(s). We are happy to provide a cost estimate if the product requires complex features that differ from the basic model.

## EXAMPLE OF ORDERING A STANDARD-SIZED PRODUCT WITH ADDITIONAL FEATURE:

Ordering an Interim & Light vest for men with zipper fastening and dressing aid hook

↓  
How to order:



**STIN224336F1Z AHZS**

**STIN224336F1Z**

### PRODUCT CODE

- Interim & Light Vest
- Men
- Long sleeves
- Size 6
- Beige
- Zipper

**AHZS**

### ADDITION

- Additional aid hook for zipper

## EXAMPLE OF ORDERING A CUSTOM-MADE PRODUCT WITH ADDITIONAL FEATURES:

Ordering a compression knee-high stocking with zipper and zipper guard

↓  
How to order:



**CO20629F2 AZ AZF**

**CO20629F2**

### PRODUCT CODE

- Compression knee-high stocking
- Toes closed
- Black

**AZ, AZF**

### ADDITIONS TO PRODUCT

- Additional zipper
- Zipper guard under the zipper





*Specify zipper placement in your order*



# ADDITIONS FOR CUSTOM-MADE PRODUCTS

- Not all additions are available for all product options.
- Lymed will contact the ordering party if the feature cannot be implemented.
- Finishing options are mainly the most common options for adult models. The suitability of these options is assessed on the basis of the order.
- When planning additional options, the product orderer and the healthcare professional responsible for pressure garment therapy should jointly assess the suitability of the solutions for the user's individual situation, context of use, and daily activities.

## Vests and bodys

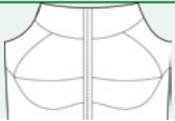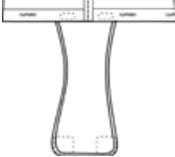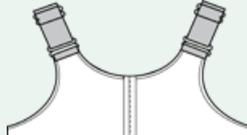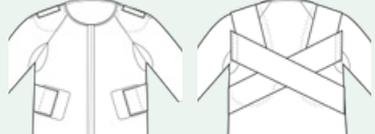	Addition code	
<p><b>MAMMARY CUPS</b></p> <p>Includes one or two breast cups made according to the customer's measurements.</p>	<b>MC</b>	
<p><b>TURTLE NECK</b></p> <p>Raised collar in double fabric.</p>	<b>AC</b>	
<p><b>TURTLE NECK WITH NEOPRENE</b></p> <p>Raised collar in double fabric with neoprene filling.</p>	<b>ACN</b>	
<p><b>CROTCH PANEL WITH FASTENING</b></p> <p>A fabric crotch panel attached to a shirt or abdominal support, running between the legs to keep the lower edge in place and prevent the garment from rolling up. The panel is fixed at one end and finished with a snap-fastening at the opposite end. A velcro closure is available on request. Different from a body-style design.</p>	<b>ADF</b>	
<p><b>ADJUSTABLE STRAPS FOR SLEEVELESS GARMENTS</b></p> <p>Fixed and adjustable 3 cm wide straps for sleeveless vests and bodysuits. Available on request with detachable straps. Available only in beige, black, white. Products in other colours are made with black straps.</p>	<b>ABV</b>	
<p><b>SHORT LEGS FOR BODY</b></p> <p>Short legs extending up to just above the knee.</p> <p>For adult Skin-body</p> <p>For adult Compression-body</p> <p>For adult Sense-body</p> <p>For children Skin-body</p> <p>For children Compression-body</p> <p>For children Sense-body</p>	<p><b>PRA21L</b></p> <p><b>COA21L</b></p> <p><b>SEA21L</b></p> <p><b>PRA10L</b></p> <p><b>COA10L</b></p> <p><b>SEA10L</b></p>	
<p><b>SENSE- DOUBLE SCAPULA SUPPORT</b></p> <p>For scapula support vest or scapula support body when double-sided scapula support is needed.</p>	<b>SEE2</b>	

## Gloves

	Addition code	
<p><b>ADDITIONAL DIGIT TO PALM-GLOVE</b></p> <p>Addition of one finger. For adding multiple fingers, use the code for glove. The thumb is part of the basic gauntlet model.</p>	<b>ADP</b>	

# LYMED PRODUCT INFORMATION

## Additions for custom-made products

### Sleeves

#### Addition code

#### SLEEVE FASTENING MODEL 1


Elastic strap running under the breast to hold the sleeve up. Suitable especially for users with large breasts.

For adult sleeves

**A20S1**

For children's sleeves

**A10S1**

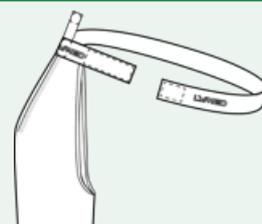

#### SLEEVE FASTENING MODEL 2


Elastic strap running under the axilla to hold the sleeve up.

For adult sleeves

**A20S2**

For children's sleeves

**A10S2**

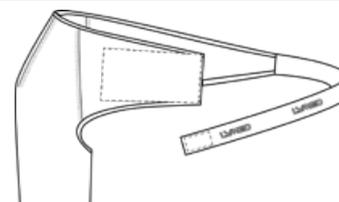

#### SLEEVE FASTENING MODEL 3

Fastening model used to join two axilla length sleeves. Two sleeves need to be ordered with the fastening.

For adult sleeves

**A20S3**

For children's sleeves

**A10S3**




### Pants and stockings

#### Addition code

#### SHOULDER STRAPS FOR PANTS

Fixed and adjustable 3 cm wide elastic straps for pants. Available only in beige, black, and white. Products in other colours are made with black straps.

**AB**

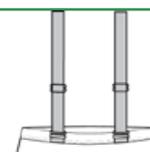

#### ADDITIONAL FLAP FOR GENITAL AREA

Used in pants to create local additional pressure. Fixed to the crotch, with a velcro fastener on the lower abdomen.

**AF**



#### ADDITIONAL FLAP WITH NEOPRENE FOR GENITAL AREA

Used in pants to create local additional pressure. Fixed to the crotch, neoprene filling and with a velcro fastener on the lower abdomen.

**AFN**



#### HIGH WAIST, CUSTOM

Increasing the waist height of pants, requires additional circumference measurement.

**HW**



#### MATERNITY BELLY FOR PANTS

A maternity-style belly section for custom-made pants. Image on page 73.


**AM**



#### DRAWSTRING CHANNEL AT THE WAIST

A fabric drawstring channel for the pants waist. The channel includes an ELA1 band.

**ALP**

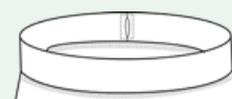

#### ADDITIONAL TOE FOR A FOOT WITH OPEN TOES OR CLOSED TOES

Addition of a toe for a foot, with open or closed toes. To order the Tabi-sock model. For adding multiple toes, use the code for a foot glove.

**ATP**



#### WAIST STRAP

Non-removable, adjustable waist band to help keep the thigh-high stocking or thigh-high leg tube in place.

Adults

**A20B**

Children

**A10B**



# LYMED PRODUCT INFORMATION

## Additions for custom-made products


### Use, dressing and fastenings (1/2)

Addition code

#### ADDITIONAL ZIPPER FASTENING

Length up to 45 cm. Sufficient for example a full-length zipper on a knee-high stocking. Includes a soft zipper pull tab. The zipper is always supplied in the same colour as the fabric.

AZ

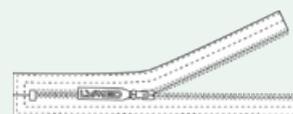

#### ADDITIONAL LONG ZIPPER FASTENING


Length over 45 cm, for example a full-length zipper for a thigh-high stocking. Includes a soft zipper pull tab. The zipper is always supplied in the same colour as the fabric.

AZ2

#### ZIPPER GUARD

A soft protective strip made of ELA1 band. Placed underneath the zipper to protect the skin and prevent skin or hair from getting caught in the zipper. For very small products, ELA2 band is used.


AZF

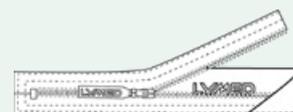

#### FABRIC ZIPPER GUARD

A softer fabric guard placed under the zipper instead of ELA band. Placed underneath the zipper to protect the skin and prevent skin or hair from getting caught in the zipper. Standard width is 4 cm, or another width, according to the customer's preference.


AZFF

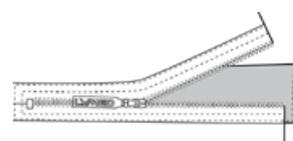

#### ZIPPER PULL TAB

The zipper pull tab attached to the zipper slider to facilitate the closing of the zipper. There are two options for the length of the pull tab: short (9 cm) and long (15 cm). The default length is short.

AZL

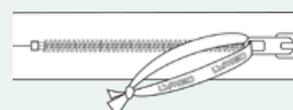

#### DRESSING AID HOOKS

Individual hooks placed under the zipper to facilitate zipping up. Unless otherwise requested, the default is 1 piece for a knee-high stocking, 2 pieces for a thigh-high stocking, and 2 - 3 pieces for a vest. Available upon request in a colour different from the fabric. Priced in sets of 3.

AHZ



#### PULL-UP LOOP

A loop attached to the product to aid in dressing. The length and material are determined during pattern making.


APH



#### ADDITIONAL HOOK FASTENING

An additional hook fastening or an extra row of hooks not included in the basic product model, in addition to any basic fastening.


AH

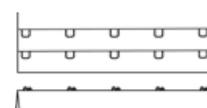

#### ADDITIONAL SNAP FASTENERS

An additional snap fastening or an extra row of snaps not included in the basic product model, in addition to any basic fastening.

ASB

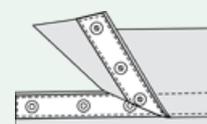

#### VELCRO FASTENING

Replacement of the fastening included in the product model with Velcro, or addition of extra Velcro fastening. Priced per piece for each started 25 cm of length. The manufacturer determines the width and properties of the Velcro, taking the customer's preferences into account. Velcro fastening is available only in beige, black, and white. Products in other colours are made with black Velcro.

AVF



#### ADDITIONAL VELCRO TABS

Small individual Velcro tabs. Used, for example, on top of a zipper to make closing it easier. Cannot be applied for functional opening, closing, or adjustment purposes of the product. Priced in sets of 4.


AV



#### VELCRO FASTENING TO ATTACH PRODUCTS TOGETHER

Eg. attaching vest and pants. Price includes max 4 velcro fasteners and protective velcro pads.

AVSE

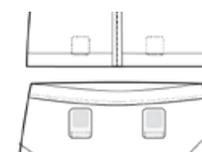

# LYMED PRODUCT INFORMATION

## Additions for custom-made products

### Use, dressing and fastenings (2/2)

#### Addition code

#### GRIP SILICONE - INSIDE

Additional silicone band to prevent for example sleeves or stockings from slipping. Can be used to double the edge band or to increase friction on the inside of the fabric. Free choice of silicone band style.

**ASI2**



#### GRIP SILICONE - OUTSIDE

A silicone strip placed on the outside of the product to improve grip, for example, on fingertips or the sole of a foot.

**ASI**



#### FUNCTIONAL LOOP OR PULL

A functional loop or pull band with Velcro fastening, made according to the customer's specifications. Produced using the standard range of bands.

**ALS1**



### Linings and reinforcements


#### Addition code

#### SEAM REINFORCEMENT/LINING WITH FABRIC STRIP

Reinforcing/lining a seam with an additional strip of fabric. Priced per piece and for every started 30 cm of length.

Sewing the strip adds more stitching compared to the original product design. Not suitable for small children's or adult stockings with a narrow forefoot.

**ASRF**

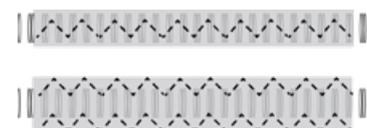

#### SEAM REINFORCEMENT WITH ADDITIONAL STITCHING

Reinforcement stitching for seams subjected to exceptional stress. Priced per piece and for every started 30 cm of length. Full-product reinforcement is not recommended. Reinforcement slightly changes the feel of the seam. Not available for small children's stockings or for adult stockings with a narrow forefoot.

**ASRZ**



#### STOCKING: REINFORCEMENT STITCHING

Reinforcement stitching in four seams: at the heel, ankle, and foot seams. Reinforcement slightly changes the feel of the seam. Not available for small children's stockings or for adult stockings with a narrow forefoot.

**ASRZ1**



#### SMALL PARTIAL LINING WITH FABRIC

Partial lining of the product according to the desired design. Used to add pressure, durability or local protection. Requires additional stitching compared to the original product design.

Small partial lining, approx 10 cm x 15 cm without crossing any seams, e.g. ankle, elbow bend, heel.

**AL1**

Medium-sized partial lining less than 20 cm x 15 cm or lining that extends to two pieces of the product.

**AL2**

Large partial lining over about 20 cm x 15 cm or lining that extends to three pieces of the product, e.g. shoulder lining.

**AL3**

#### SMALL SIZE DOUBLE FABRIC

An entire piece of the product is made with double material to add support or durability. The code is used when doubling is identical to the original piece. Doubling a piece does not require additional stitching. The double fabric often makes the product more rigid, which can affect the dressing.

Small size double fabric

**ALD1**

Medium-sized double fabric

**ALD2**

Big size double fabric

**ALD3**

# LYMED PRODUCT INFORMATION

## Additions for custom-made products

### Pockets

#### Addition code

#### POCKET

A feature in a product for inserting soft materials, fillings or small devices.

Small, approx. 10 cm x 15 cm, that does not exceed the product's seam. **AP1**

Medium, approx. 20 cm x 15 cm, or extending to two pieces of the product. **AP2**

Large, approx. 20 cm x 15 cm, or extending to three pieces of the product. **AP3**

#### POCKET WITH NEOPRENE

A pocket containing neoprene to create additional pressure or to fill hollow areas in the body

Small, approx. 10 cm x 15 cm, that does not exceed the product's seam. **APN1**

Medium, approx. 20 cm x 15 cm, or extending to two pieces of the product. **APN2**

Large, approx. 20 cm x 15 cm, or extending to three pieces of the product. **APN3**

#### POCKET FOR PROSTHESIS

Pocket for breast prosthesis. Side opening for inserting the prosthesis.

**APP**



### Sense functional supports


#### Addition code

Read more on page 38.

#### LIGHT PELVIC FUNCTIONAL SUPPORT

A support made of strong fabric, placed on the outside of the product. Sewn with a stronger seam type than the product's standard seams.

**ASPL**

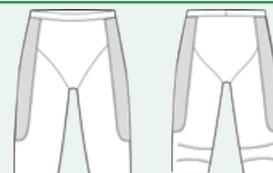

#### STRONG PELVIC FUNCTIONAL SUPPORT

A support made of strong fabric, placed on the outside of the product. Sewn with a stronger seam type than the product's standard seams.

**ASPS**



#### FUNCTIONAL SUPPORT, BACK

A support made of strong fabric, placed on the outside of the product. Sewn with a stronger seam type than the product's standard seams.


**ASVS**



#### FUNCTIONAL SUPPORT, SHOULDERS

A support made of strong fabric, placed on the outside of the product. Sewn with a stronger seam type than the product's standard seams.


**ASVC**

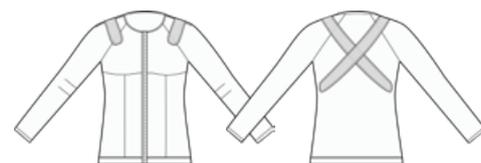

#### STRONG FUNCTIONAL SUPPORT, SHOULDERS

A support made of strong fabric, placed on the outside of the product. Sewn with a stronger seam type than the product's standard seams.

**ASVU**

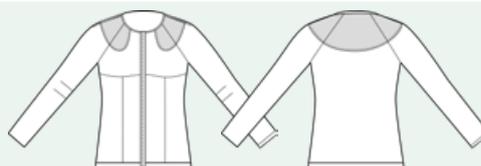

#### OTHER FUNCTIONAL SUPPORT, PRICE FORM

A support made of strong fabric, placed on the outside of the product and customized according to the customer's requirements. Sewn with a stronger seam type than the product's standard seams. Priced on a case-by-case basis.

**ASFM**



# LYMED PRODUCT INFORMATION

## Additions for custom-made products

### Miscellaneous

### Addition code

#### ADDITIONAL OBLIQUE SHAPING OF AN EDGE

Available for the hem of vests, upper edge of sleeves, leg tubes, and thigh-high or knee-high stockings. A customised edge design that differs from the standard shape, including asymmetrical or otherwise customer-specified shaping. For example, a raised outer edge on a sleeve or thigh-high stocking, or an extended vest hem.


AME



#### POUCH FOR LIMB STUMP

Shaping the end of a sleeve or pant leg into a pouch. Used for protecting and shaping the stump. Can be used in conjunction with a prosthesis. The shape of the stump determines the pattern: a round base or shaped seams.

AAC

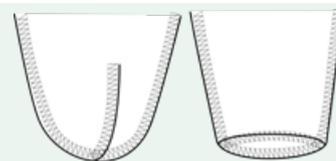

#### OVERLAPPING SLIT

An openable horizontal slit with overlapping fabric edges, forming an open-style closure. Can be used, for example, for a PEG button. The manufacturer determines the size and construction in proportion to the ordered product, taking the customer's preferences into account.


AOS



#### OPENING

An opening in a customer-specified size, for example for a stoma or PEG button, as a ponytail opening in a hood, or as a heel opening in a stocking. The size and placement of the opening in relation to the ordered product may affect the pressure the product provides around the opening area. Opening edges: ELA3 binding tape, or edge stitching for very small openings. Can also be added as a product modification after manufacturing, in which case shipping costs are paid by the customer.

ASTO

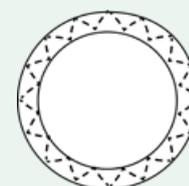

#### FABRIC FLAP FOR OPENING

A fabric flap to cover the opening. Velcro fastening. Can be used in addition to ASTO code.


AFC



#### ADDITIONAL NEOPRENE

An additional neoprene piece that is typically identical to the one made for the pocket. Price is for a piece about the size of a palm.

AN

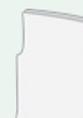

#### ADDITIONAL EMBLEM

The product includes one free emblem. More emblems are priced per piece. Read more on page 24.

DE

#### MULTICOLOUR

Manufacturing the product using two fabric colours. The product may consist of only two different fabric colours. The manufacturer determines the placement of the colours, taking the customer's preferences into account.

AMC

# CUSTOMISATION OPTIONS FOR STANDARD-SIZED PRODUCTS

- Not all additions are available for all product options.
- Lymed will contact the ordering party if the feature cannot be implemented.
- When planning additional options, the product orderer and the healthcare professional responsible for pressure garment therapy should jointly assess the suitability of the solutions for the user's individual situation, context of use, and daily activities.


## Standard-sized vests and sleeves

### Addition code

#### ENLARGED NECKLINE

Enlarged neckline, max 3 cm lower than the standard model. Indicate desired enlargement in centimetres.

**ANR**

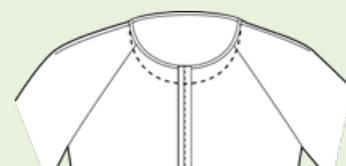

#### HEMLINE EXTENSION

Indicate desired extension in centimetres. The circumference of the extension is the same as in the original hem.

**AVLW**

#### HEMLINE SHORTENING

Indicate desired shortening in centimetres.

**AVSW**

#### SLEEVE LENGTHENING

For vest sleeves and separate sleeves. Indicate desired extension in centimetres. The circumference of the extension is the same as in the original sleeve opening.

**ASLR**

#### SLEEVE SHORTENING

For vest sleeves and separate sleeves. Indicate desired shortening in centimetres.

**ASSR**

## Standard-sized pants (1/2)

### Addition code

#### OPEN GENITAL AREA

Open crotch area for women's pants.


**OS**



#### SHOULDER STRAPS

Fixed and adjustable 3 cm wide elastic straps for pants.

**ABS**

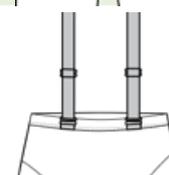

#### WAIST HEIGHT INCREASE

Indicate desired increase in centimetres. The circumference of the extension is the same as in the original waist.

**AHW**

# LYMED PRODUCT INFORMATION

## Customisation options for standard-sized products

### Standard-sized pants (2/2)

Addition code

#### PANT LEGS LENGTHENING

Extension of pant legs. Indicate desired increase in centimetres. The circumference of the extension is the same as in the original leg opening.

**ALLL**

#### PANT LEGS LENGTH SHORTENING

Shortening of pant legs. Indicate the amount of shortening in centimeters.

**ALLS**

#### MEN'S SEAMLESS FRONT PIECE FOR COMPRESSION PANTS

Front piece without anatomical design, no pouch on genital area.

**AMFP**




### Standard-sized gloves

Addition code

#### FINGER LENGTH SHORTENING

Shortening only for gloves with open fingertips. The shortening is the same length for all fingers. Indicate desired shortening in centimetres.


**ALF**

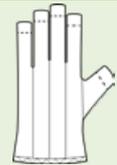

#### GLOVE WRIST LENGTHENING

Maximum lengthening of the wrist is 5 cm. The circumference of the cuff grows moderately towards the end. Indicate lengthening in centimetres.

**AGR**

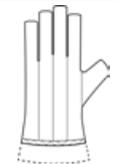

#### POCKET FOR GLOVE

Pocket on the backhand side of the glove. Desired filling can be added.


**AP**



#### POCKET WITH NEOPRENE FOR GLOVE

Pocket with neoprene on the backhand side of the glove. Neoprene is removable.

**APN**

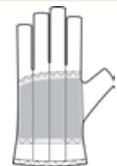


# LYMED PRODUCT INFORMATION

## Customisation options for standard-sized products

Fastenings and pulls for standard-sized products		Addition code	
<b>ADDITIONAL ZIPPER</b> Indicate the desired location of the zipper in the order form.	Left	<b>AZSL</b>	
	Right	<b>AZSR</b>	
	Middle	<b>AZSM</b>	
	Back	<b>AZSB</b>	
<b>DRESSING AID HOOKS FOR ZIPPER</b> Individual hooks underneath the zipper to facilitate zipping up. The amount is standard based on a product from 1 to 3 hooks.	1 pcs	<b>AHZS</b>	
	2 pcs	<b>AHZS2</b>	
	3 pcs	<b>AHZS3</b>	
<b>SILICONE STRIP</b> Silicone strip to hold the product in place. Pattern designer decides the location depending on the product.	1 pcs	<b>ASI2S</b>	
	2 pcs	<b>ASI2S2</b>	
	3 pcs	<b>ASI2S3</b>	
<b>FASTENING REMOVAL</b> Manufacturing the product without a fastening. Not applicable to bras, support bands, or headwear.		<b>AHL</b>	
<b>FASTENING CHANGE TO ZIPPER</b> Changing the fastening to a zipper fastening. Not applicable to bras, support bands, or headwear.		<b>AHLZ</b>	
<b>FASTENING CHANGE TO HOOKS</b> Changing the fastening to a hook fastening. Not applicable to bras, support bands, or headwear.		<b>AHLH</b>	

## Standard-sized bras

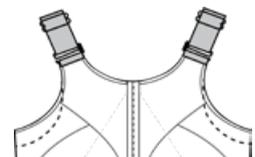Standard-sized bras		Addition code	
<b>FIXED BRA STRAPS</b> Fixed, adjustable straps for models that normally have detachable straps.		<b>ABB</b>	
<b>COMPRESSION BRA: LOW ARMHOLES</b> 2 cm lower armholes on a standard compression bra.		<b>AAR</b>	
<b>STRONG SUPPORT BRA: POCKET FOR PROSTHESIS</b> Prosthesis pocket with a side opening for inserting the prosthesis.	Left	<b>APPBL</b>	
	Right	<b>APPBR</b>	
	Both	<b>APPBB</b>	

# LYMED PRODUCT INFORMATION

## Customisation options for standard-sized products

### Miscellaneous for standard-sized products

Addition code

#### DECORATIVE EMBLEM

Cat	<b>DESP1</b>	Dino	<b>DESP8</b>
Dog	<b>DESP2</b>	Bear	<b>DESP9</b>
Fox	<b>DESP3</b>	Beetle	<b>DESP10</b>
Bunny	<b>DESP4</b>	Ladybug	<b>DESP11</b>
Duck	<b>DESP5</b>	Turtle	<b>DESP12</b>
Strawberry	<b>DESP6</b>	Whale	<b>DESP13</b>
Snowflake	<b>DESP7</b>	Flower	<b>DESP14</b>

#### EDGING OPTION CHANGE

Create the code by selecting the location of the edge, and then the code for the desired edging option.

#### Examples:

Silicon 2 to upper edge of sleeve: **ACBU SIL2**

Silicone 4 to pants waist: **ACBW SIL4**

Vest sleeve with finishing: **ACBS finishing**

Silicone 1 to vest hemline: **ACBH SIL1**

	LOCATION	EDGING	
	Hemline	<b>ACBH</b>	<b>ELA1</b>
	Waist	<b>ACBW</b>	<b>ELA2</b>
	Upper edge	<b>ACBU</b>	<b>ELA3</b>
	Lower edge	<b>ACBD</b>	<b>ELA4</b>
	Leg	<b>ACBL</b>	<b>SIL1</b>
	Sleeve	<b>ACBS</b>	<b>SIL2</b>
	Arm of glove	<b>ACBG</b>	<b>SIL4</b>
	Gauntlet	<b>ACBP</b>	<b>no finishing</b>
	Armhole	<b>ACBB</b>	<b>finishing</b>

#### MODIFIED LENGTH FOR SUPPORT BELT OR SUSPENSOR

Maximum modification to standard size - 40 cm / +1 5 cm.

The shortest suitable product length is chosen as the basic product for the customer.

**AABLR**



# SENSE FUNCTIONAL SUPPORTS

## WHEN TO CHOOSE FUNCTIONAL SUPPORTS?

Supports can be used when the support provided by regular product features is not sufficient or when there is a desire to have a more significant impact on the user's body posture experience.

Functional supports work best when you want to enhance body awareness in a specific area of the body or remind the user to maintain a certain posture. Please note that the experience is always individual.

## FUNCTIONAL SUPPORT OR DOUBLE FABRIC?

If the aim of the Sense product is to provide a more supportive effect, you can also order parts of the product with double fabric or lining.

It should be noted that extensive reinforcements will increase the overall pressure level of the product, increase the product's stiffness, and may pose additional challenges when donning.

## MORE IS NOT ALWAYS BETTER

When aiming for a localised feeling of additional support, the functional support must be targeted and proportionately small compared to the rest of the product.

If linings, reinforcements and double fabrics are added to the product excessively, it increases the overall pressure and stiffness, causing the sense of localised, targeted support to disappear. In such cases, donning may become more difficult and the sensation may even become uncomfortable.




- We recommend ordering a basic model as a first Lymed product, before trying functional supports.
- We recommend ordering functional supports primarily for long-legged and long-sleeved products.
- The intensity of the sensation provided by the supports is significantly influenced by the customer's body shape, as well as the material and pressure level of the product ordered as a base.
- Supports are made from strong fabric and sewn on the outside of the product.
- The design and size of the supports are standardised and adjusted to the custom-made product.
- Functional supports use a sturdier seam type, different from the other seams of the product.




# LYMED PRODUCT INFORMATION

## Sense functional supports

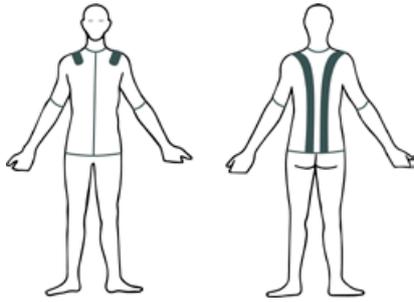


### FUNCTIONAL SUPPORT, BACK

Provides a sensation of support to the upper body and shoulders, aiding in perceiving upper body posture. The most significant experiential impact is on maintaining a straight back and relaxing the shoulders.

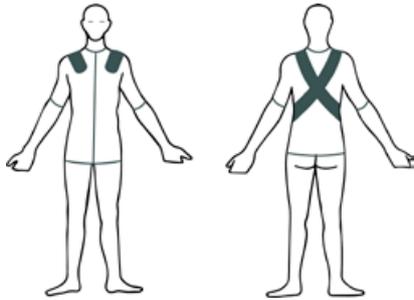


### FUNCTIONAL SUPPORT, SHOULDERS

Provides a sense of support to the upper body and shoulders, while aiding in perceiving upper body posture. The most significant experiential impact is on relaxing the shoulders and guiding them backward.

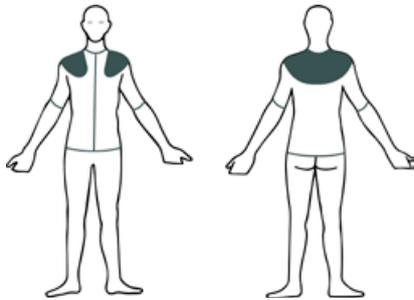


### STRONG FUNCTIONAL SUPPORT, SHOULDERS

Provides an extremely strong sensation of support to the shoulders, upper arms, and upper back.

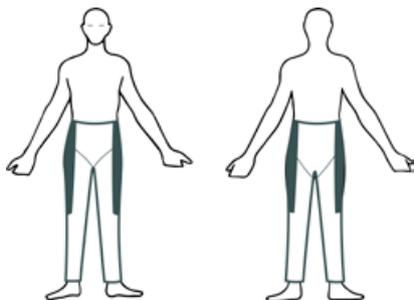

### LIGHT PELVIC FUNCTIONAL SUPPORT

Provides a sense of support and posture to the midriff. Helps to perceive the position of the pelvis if the support provided by regular pressure pants is insufficient.

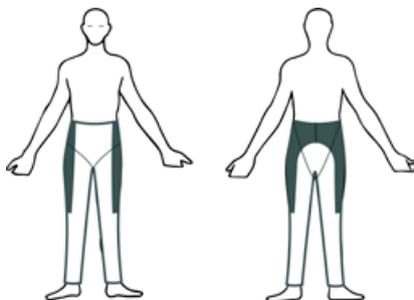

### STRONG PELVIC FUNCTIONAL SUPPORT

Provides a strong sense of support and posture to the midriff. Helps to perceive the position of the pelvis if the support provided by regular pressure pants is insufficient.

**Addition codes for Sense functional supports are listed on page 32**

### OTHER FUNCTIONAL SUPPORT

Choose the shape and location freely.




**SKIN**<sup>™</sup>

**PRESSURE GARMENTS FOR  
SCAR MANAGEMENT**

**CUSTOM-MADE**

**STANDARD-SIZE**

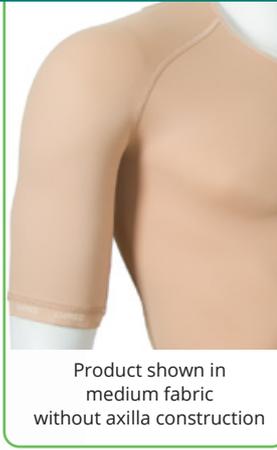

Product shown in medium fabric without axilla construction



Product picture, code: PR22423F5

VEST		CUSTOM-MADE			
Model	Product / Art.No				
	Sleeveless	Short sleeves	Long and short sleeve	Long sleeves	
Women	PR21411	PR21422	PR21423	PR21433	
Men	PR22411	PR22422	PR22423	PR22433	
Children	PR10411	PR10422	PR10423	PR10433	

Children's models are made for children under 10.

**Product information:**

The long-sleeved vest features anatomically shaped elbow bends. In products made from strong fabric, the axilla area uses medium fabric construction. Products made from medium fabric do not include axilla construction. The women's vest features shaping seams at the breasts. The women's model can be manufactured without bust shaping seams free of charge upon request.

Material:	Strong or medium
Colours:	All
Pressure level:	Selectable
Fastening:	2-row hooks / zipper with guard / no fastening
Details:	Hemline SIL 1
	Neckline ELA 3
	Edge of sleeve ELA 2
	Sleeveless model, edging ELA 3



ABDOMINAL SUPPORT		CUSTOM-MADE
Model	Art.No	
Women	PR21400	
Men	PR22400	
Children	PR10400	

Children's models are made for children under 10.

**Product information:**

Used whenever extra pressure is needed on the midsection.

Material:	Strong or medium
Colours:	All
Pressure level:	Selectable
Fastening:	2-row hooks / zipper with guard / no fastening
Details:	Upper edge SIL 1
	Lower edge SIL 1

Product picture, code: PR22400F1



Product picture, code: STPR714F2

STERNAL STRAP   NARROW FRONT			STANDARD-SIZE
Size	Width* cm	Circumf.** cm	Art.No
XS	4,0	50 - 62	STPR712F2
S	4,4	63 - 75	STPR713F2
M	4,8	76 - 88	STPR714F2
L	5,3	89 - 101	STPR715F2
XL	5,7	102 - 114	STPR716F2

**Product information:**

A two-piece product with hook-and-loop fastening at the sides and shoulders.

The product is designed to apply pressure to the sternal area.

A splint pocket is integrated in the sternal area.

The design is particularly suited for women.

The width of the front section at its narrowest point is 4–5.7 cm, depending on size.

\*Width of the front piece at the narrowest part

\*\*Circumference of the chest below breasts.

Material: Polyamide elastane blend

Colour: Black

Pressure level: Adjustable

Fastening: Velcro fastening



Product picture, code: STPR724F2

STERNAL STRAP   WIDE FRONT			STANDARD-SIZE
Size	Width* cm	Circumf.** cm	Art.No
S	10	63 - 75	STPR723F2
M	10	76 - 88	STPR724F2
L	10	89 - 101	STPR725F2
XL	10	102 - 114	STPR726F2

**Product information:**


A two-piece product with hook-and-loop fastening at the sides and shoulders.  
The product is designed to apply pressure to the sternal area.  
A splint pocket with a removable pad for additional pressure is integrated in the sternal area.

\*Width of the front piece is 10 cm at the narrowest part.  
\*\*Circumference of the chest below breasts.

Material:	Polyamide elastane blend, polyester polyurethane pad
Colour:	Black
Pressure level:	Adjustable
Fastening:	Velcro fastening



Product picture, code:  
PR104033F7

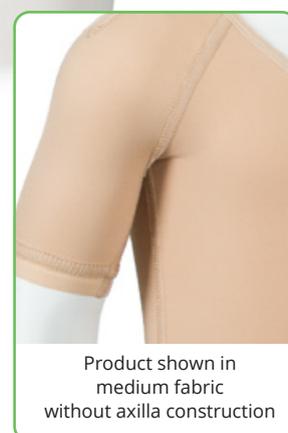

BODY				CUSTOM-MADE
Model	Product / Art.No			
	Sleeveless	Short sleeves	Long and short sleeve	Long sleeves
Women	PR214011	PR214022	PR214023	PR214033
Children	PR104011	PR104022	PR104023	PR104033

Children's models are made for children under 10.

**Product information:**

The long-sleeved body features anatomically shaped elbow bends. In products made from strong fabric, the axilla area uses medium fabric construction. Products made from medium fabric do not include axilla construction.

The women's body features shaping seams at the breasts. The women's model can be manufactured without bust shaping seams free of charge upon request. Short legs are available as an addition.

Material:	Strong or medium
Colours:	All
Pressure level:	Selectable
Fastening:	<b>Women:</b> 2-row hooks/ zipper with guard 2-row snap fasteners at the crotch Short legs: no crotch closure <b>Children:</b> 2-row hooks / zipper with guard 2-row snap fasteners at the crotch Short legs: 1-row snap fasteners
Details:	Neckline and leg opening ELA 3 Edge of sleeve ELA 2 Sleeveless model, edging ELA 3 Short legs: edge of leg ELA 2



**Product information:**

Long-leg pants with anatomical construction at the knee.  
Stocking legs with anatomical construction at the ankle and knee.

When a stocking leg is ordered in strong material, the constructions are made from medium material.  
Men's models with contouring at the front.

Material:	Strong or medium
Colours:	All
Pressure level:	Selectable
Fastening:	Available separately
Details:	Waist ELA 1 Leg opening ELA 2

**PANTS** CUSTOM-MADE

Model	Product / Art. No			
	Short legs	Short and long legs	Short leg and leg with stocking OPEN	Short leg and leg with stocking CLOSED
Women	PR21522	PR21523	PR215248	PR215249
Men	PR22522	PR22523	PR225248	PR225249
Girls	PR11522	PR11523	PR115248	PR115249
Boys	PR12522	PR12523	PR125248	PR125249

Product picture, code: PR225249F4

Children's models are made for children under 10.

Model	Product / Art.No						
	Long legs	Long leg and leg with stocking OPEN	Long leg and leg with stocking CLOSED	Short leg and leg with toe sock, toe tips OPEN	Short leg and leg with toe sock, toe tips CLOSED	Long leg and leg with toe sock, toe tips OPEN	Long leg and leg with toe sock, toe tips CLOSED
Women	PR21533	PR215348	PR215349	PR21524FG8	PR21524FG9	PR21534FG8	PR21534FG9
Men	PR22533	PR225348	PR225349	PR22524FG8	PR22524FG9	PR22534FG8	PR22534FG9
Girls	PR11533	PR115348	PR115349	PR11524FG8	PR11524FG9	PR11534FG8	PR11534FG9
Boys	PR12533	PR125348	PR125349	PR12524FG8	PR12524FG9	PR12534FG8	PR12534FG9

Children's models are made for children under 10.

**STOCKINGS** CUSTOM-MADE

Model	Product / Art.No					
	Toes OPEN	Toes CLOSED	One leg with toe sock, toe tips OPEN	One leg with toe sock, toe tips CLOSED	With toe socks, toe tips OPEN	With toe socks, toe tips CLOSED
Women	PR215448	PR215449	PR21544FG8	PR21544FG9	PR21544FG28	PR21544FG29
Men	PR225448	PR225449	PR22544FG8	PR22544FG9	PR22544FG28	PR22544FG29
Girls	PR115448	PR115449	PR11544FG8	PR11544FG9	PR11544FG28	PR11544FG29
Boys	PR125448	PR125449	PR12544FG8	PR12544FG9	PR12544FG28	PR12544FG29

Children's models are made for children under 10.



Product picture, code: PR20603F6



Product picture, code: PR20603F1

LEG		CUSTOM-MADE	
Model	Product / Art.No		
	Ankle-knee	Knee-thigh	Ankle-thigh
Adults	PR20601	PR20602	PR20603
Children	PR10601	PR10602	PR10603

Children's models are made for children under 10.

**Product information:**

The leg sleeve features an anatomically shaped construction in the knee bend area.

When the product is ordered in strong fabric, the knee bend construction is made from medium fabric.

Material: Strong, medium

Colours: All

Pressure level: Selectable

Fastening: Available separately

Details: Upper edge SIL 4, upper edge in knee-high SIL 1, lower edge ELA 2.



Product picture, code:  
PR20639F4 A20B



Product picture, code:  
PR20629F4 AZ



Product picture, code:  
PR20619F4



Product picture, code:  
PRA20T9F2

**STOCKING**

CUSTOM-MADE

Model	Toes	Product / Art.No		
		Ankle-high	Knee-high	Thigh-high
Adults	Closed	PR20619	PR20629	PR20639
Adults	Open	PR20618	PR20628	PR20638
Adults	Toe sock, closed	PR20619FG	PR20629FG	PR20639FG
Adults	Toe sock, open	PR20618FG	PR20628FG	PR20638FG
Children	Closed	PR10619	PR10629	PR10639
Children	Open	PR10618	PR10628	PR10638
Children	Toe sock, closed	PR10619FG	PR10629FG	PR10639FG
Children	Toe sock, open	PR10618FG	PR10628FG	PR10638FG

Children's models are made for children under 10.

**Product information:**

In ankle-length, knee-length, and thigh-length stockings, the ankle features an anatomically shaped construction.

In thigh-length stockings, there is additionally an anatomically shaped knee bend.

When the stocking is ordered in strong material, the constructions are made from medium material. In open-toe stockings, the toe is lightly beveled.

The ankle sock's shaft length is measured upwards from the L/R61 circumference. Please note that the shaft length can be at most the same as the L/R67 measurement.

Material:	Strong or medium
Colours:	All
Pressure level:	Selectable
Fastening:	Available separately
Details:	Upper edge SIL 1, upper edge in thigh-high SIL 4, lower edge, tip open ELA 2.

**FOOT GLOVE**

CUSTOM-MADE

Model	Toe tips	Art.No
Adults	Closed	PRA20T9
Adults	Open	PRA20T8
Children	Closed	PRA10T9
Children	Open	PRA10T8

Children's models are made for children under 10.

**Product information:**

The product can be used together with an open-toe stocking or separately.

Material:	Strong or medium
Colours:	All
Pressure level:	Selectable
Details:	No finishing



Product picture, code: PR20303F1



Product picture, code: PR2031038F1



Product picture, code: PR2031138F1

SLEEVE		CUSTOM-MADE	
Model	Product / Art.No		
	Wrist-elbow	Elbow-axilla	Wrist-axilla
Adults	PR20301	PR20302	PR20303
Children	PR10301	PR10302	PR10303

Children's models are made for children under 10.

SLEEVE GAUNTLET		CUSTOM-MADE	
Model	Thumb	Product / Art.No	
		To elbow	To axilla
Adults	Closed	PR2031019	PR2031039
Adults	Open	PR2031018	PR2031038
Children	Closed	PR1031019	PR1031039
Children	Open	PR1031018	PR1031038

Children's models are made for children under 10.

SLEEVE GLOVE		CUSTOM-MADE	
Model	Fingertips	Product / Art.No	
		To elbow	To axilla
Adults	Closed	PR2031119	PR2031139
Adults	Open	PR2031118	PR2031138
Children	Closed	PR1031119	PR1031139
Children	Open	PR1031118	PR1031138

Children's models are made for children under 10.

**Product information:**

The wrist-to-axilla model features an anatomical elbow bend.

Material:	Strong or medium
Colours:	All
Pressure level:	Selectable
Fastening:	Available separately
Details:	Upper edge SIL 1 Lower edge ELA 2

**Product information:**

Axilla-length model with anatomical elbow construction. Anatomical construction in the thumb crotch.

Material:	Strong or medium
Colours:	All
Pressure level:	Selectable
Fastening:	Available separately
Details:	Upper edge SIL 1 Lower edge ELA 2

**Product information:**

Axilla-length model with anatomical elbow construction. Anatomical construction between the fingers and in the thumb crotch.

Material:	Strong or medium
Colours:	All
Pressure level:	Selectable
Fastening:	Available separately
Details:	Upper edge SIL 1

### SLEEVE FASTENING - MODEL 1 CUSTOM-MADE


Model	Product / Art.No
<b>Model 1 - Shoulder flap</b>	
Adults	A20S1
Children	A10S1

Children's models are made for children under 10.

#### Product information:

Elastic strap running under the breast to hold the sleeve up. Fastening with velcro. Suitable especially for users with large breasts. Available for all sleeves reaching up to axilla. Women's sleeve with a bra strap holder.

Material:	ELA 1
Colour:	Same as sleeve

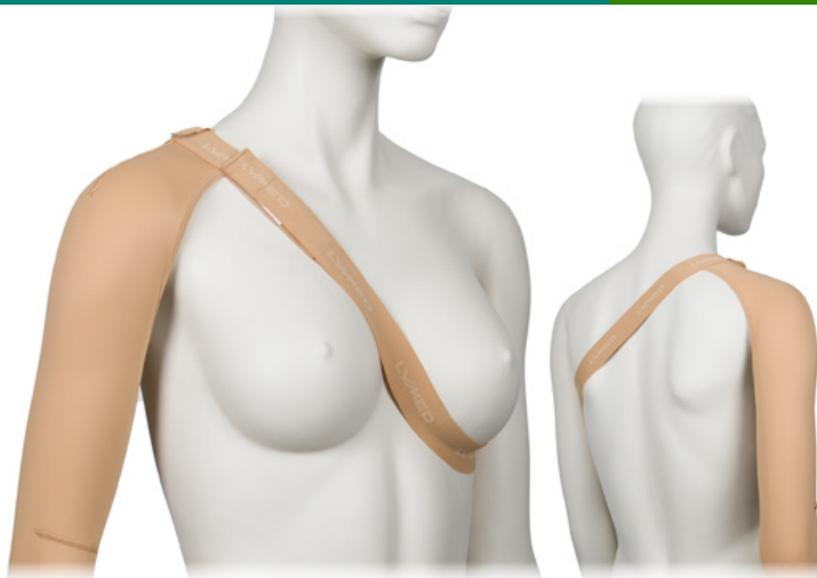

### SLEEVE FASTENING - MODEL 2 CUSTOM-MADE


Model	Product / Art.No
<b>Model 2 - Shoulder flap</b>	
Adults	A20S2
Children	A10S2

Children's models are made for children under 10.

#### Product information:

Elastic strap running under the axilla to hold the sleeve up. Fastening with velcro. Available for all sleeves reaching up to axilla.

Material:	ELA 1
Colour:	Same as sleeve

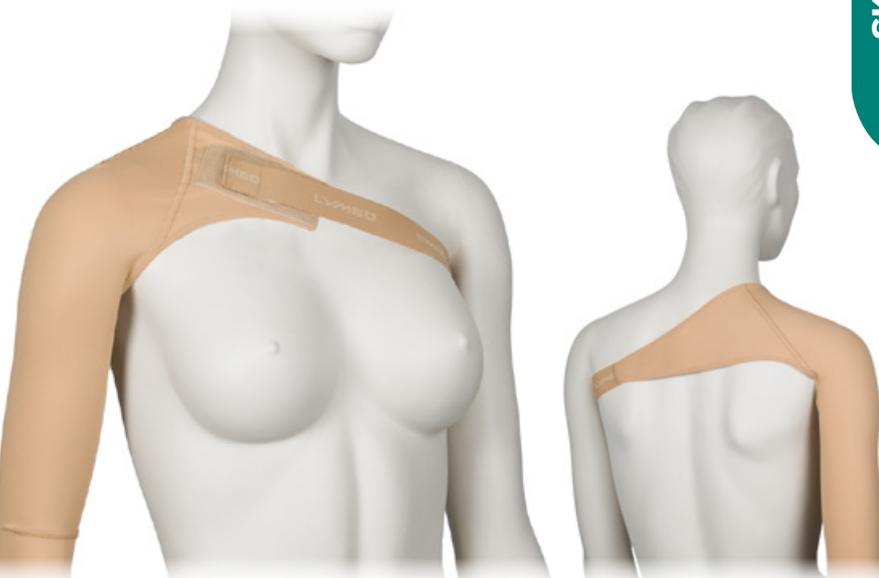

### SLEEVE FASTENING - MODEL 3 CUSTOM-MADE

Model	Product / Art.No
<b>Model 3 - Opposite sleeve</b>	
Adults	A20S3
Children	A10S3

Children's models are made for children under 10.


#### Product information:

In the sleeve attachment model, a counter-sleeve connects two axilla-length sleeves. The model is suitable for situations where maintaining the position of one or both sleeves is challenging. The counter-sleeve configuration can be used in different setups: to connect two sleeves or to support the positioning of a single sleeve.

Two sleeves must be ordered in conjunction with the attachment. If the ordered sleeves are made from different materials, the attachment component is manufactured by default using the lighter material.

Material:	Strong or medium
Colour:	Same as sleeve
Fastening:	2-row hooks / zipper with guard
Details:	Neckline ELA 3
	Hemline SIL 2
	Edge of sleeves ELA 2

Product pictured, made from medium material

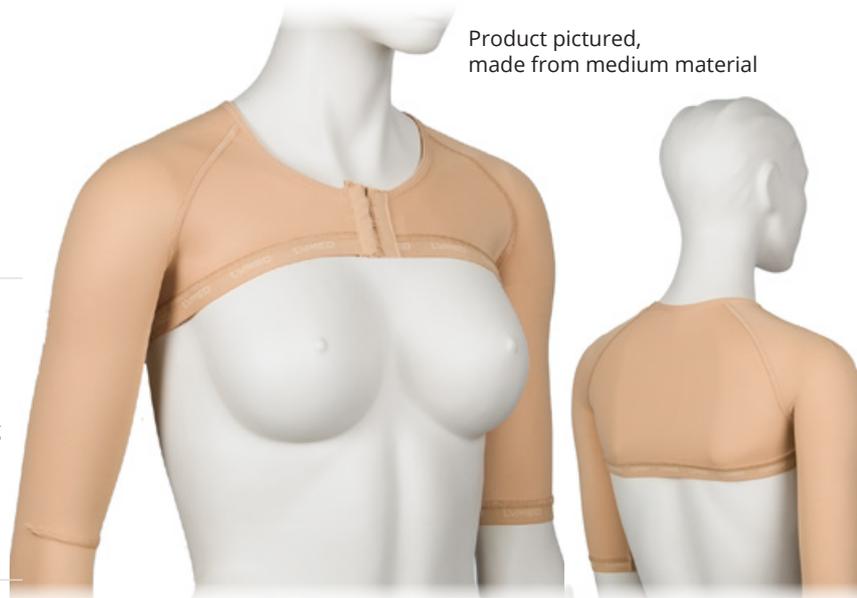

See also vest model on p. 58

GLOVE		CUSTOM-MADE
Model	Product / Art. No	
	Fingertips closed	Fingertips open
Adults	PR203119	PR203118
Children	PR103119	PR103118

Children's models are made for children under 10.

**Product information:**

Anatomical construction between the fingers and in the thumb fourchette. Number of backhand seams/darts in a glove may vary due to individual measurements and pressure/compression levels. Order a sleeve-glove when the length of the shaft is greater than the length of the palm.

Material:	Strong or medium
Colours:	All
Pressure level:	Selectable
Fastening:	Available separately
Details:	Wrist edging ELA 3



Product picture, code:  
PR203119F1 (Custom-made)  
STPR31194RF1 (Standard-size)

GLOVE		STANDARD-SIZE				
Size	Circumf.*	Colour	Product / Art.No			
cm			Fingertips closed		Fingertips open	
			Right	Left	Right	Left
XXS	14 - 16	Beige	STPR31191RF1	STPR31191LF1	STPR31181RF1	STPR31181LF1
XXS	14 - 16	Black	STPR31191RF2	STPR31191LF2	STPR31181RF2	STPR31181LF2
XS	17 - 18	Beige	STPR31192RF1	STPR31192LF1	STPR31182RF1	STPR31182LF1
XS	17 - 18	Black	STPR31192RF2	STPR31192LF2	STPR31182RF2	STPR31182LF2
S	19 - 20	Beige	STPR31193RF1	STPR31193LF1	STPR31183RF1	STPR31183LF1
S	19 - 20	Black	STPR31193RF2	STPR31193LF2	STPR31183RF2	STPR31183LF2
M	21 - 22	Beige	STPR31194RF1	STPR31194LF1	STPR31184RF1	STPR31184LF1
M	21 - 22	Black	STPR31194RF2	STPR31194LF2	STPR31184RF2	STPR31184LF2
L	23 - 24	Beige	STPR31195RF1	STPR31195LF1	STPR31185RF1	STPR31185LF1
L	23 - 24	Black	STPR31195RF2	STPR31195LF2	STPR31185RF2	STPR31185LF2
XL	25 - 26	Beige	STPR31196RF1	STPR31196LF1	STPR31186RF1	STPR31186LF1
XL	25 - 26	Black	STPR31196RF2	STPR31196LF2	STPR31186RF2	STPR31186LF2
XXL	27 - 28	Beige	STPR31197RF1	STPR31197LF1	STPR31187RF1	STPR31187LF1
XXL	27 - 28	Black	STPR31197RF2	STPR31197LF2	STPR31187RF2	STPR31187LF2

Product picture, code:  
PR203118F1 (Custom-made)  
STPR31184RF1 (Standard-size)

**Product information:**

Anatomical construction between the fingers.

\*Circumference at MCP

Material:	Strong
Colours:	Beige, black
Pressure level:	23 - 30 mmHg
Details:	Wrist edging ELA 3



Product picture, code:  
PR203108F1 (Custom-made)  
STPR3104RF1 (Standard-size)

GAUNTLET		CUSTOM-MADE
Model	Product / Art.No	
	Thumb closed	Thumb open
Adults	PR203109	PR203108
Children	PR103109	PR103108

Children's models are made for children under 10.

**Product information:**

Anatomical construction in the thumb fourchette. Order a sleeve-gauntlet when the length of the shaft is greater than the length of the palm.

Material:	Strong or medium
Colours:	All
Pressure level:	Selectable
Fastening:	Available separately
Details:	Wrist edging ELA 3 Palm edging ELA 2

GAUNTLET			STANDARD-SIZE	
Size	Circumf.*	Colour	Product / Art.No	
	cm		Thumb open	
			Right	Left
XXS	14 - 16	Beige	STPR3101RF1	STPR3101LF1
XXS	14 - 16	Black	STPR3101RF2	STPR3101LF2
XS	17 - 18	Beige	STPR3102RF1	STPR3102LF1
XS	17 - 18	Black	STPR3102RF2	STPR3102LF2
S	19 - 20	Beige	STPR3103RF1	STPR3103LF1
S	19 - 20	Black	STPR3103RF2	STPR3103LF2
M	21 - 22	Beige	STPR3104RF1	STPR3104LF1
M	21 - 22	Black	STPR3104RF2	STPR3104LF2
L	23 - 24	Beige	STPR3105RF1	STPR3105LF1
L	23 - 24	Black	STPR3105RF2	STPR3105LF2
XL	25 - 26	Beige	STPR3106RF1	STPR3106LF1
XL	25 - 26	Black	STPR3106RF2	STPR3106LF2
XXL	27 - 28	Beige	STPR3107RF1	STPR3107LF1
XXL	27 - 28	Black	STPR3107RF2	STPR3107LF2

**Product information:**

\*Circumference at MCP

Material:	Strong
Colours:	Beige, black
Pressure level:	23 - 30 mmHg
Details:	Wrist edging ELA 3 Palm edging ELA 2



Product picture, code:  
STPR2037F2

WEB SPACER		STANDARD-SIZE
<i>Model</i>	<i>Colour</i>	<i>Product / Art.No</i>
<b>One size</b>		
Adults	Beige	STPR2037F1
Adults	Black	STPR2037F2
Children	Beige	STPR1037F1
Children	Black	STPR1037F2

**Product information:**

Web spacer gives extra pressure between fingers. Product is adjustable and suitable for both hands. Intended to be worn over a pressure glove.

Material:	Elastic velcro, elastane strap
Colours:	Beige, black
Pressure level:	Adjustable
Fastening:	Velcro



Product picture, code:  
PR203128F1 (Custom-made)  
STPR334F1 (Standard-size)

DIGITAL CAP		CUSTOM-MADE
Model	Product / Art.No	
	Tip closed	Tip open
Adults	PR203129	PR203128
Children	PR103129	PR103128

Children's models are made for children under 10.

**Product information:**

The digital cap is available in tip open and tip closed versions.

Material:	Strong or medium
Colours:	All
Pressure level:	Selectable
Fastening:	No fastening
Details:	No finishing

DIGITAL CAP				STANDARD-SIZE
Model	Product length	Circumf.*	Colour	Art.No
	cm	cm		Tip open
XS	5,0	4 - 5	Beige	STPR332F1
S	5,5	5 - 6	Beige	STPR333F1
M	6,5	6 - 7	Beige	STPR334F1
L	7,5	7 - 8	Beige	STPR335F1
XL	8,5	8 - 9	Beige	STPR336F1

**Product information:**

Can be used on the fingers of both hands.

\*Circumference around the PIP joint.

Material:	Medium
Colour:	Beige
Pressure level:	23 - 30 mmHg
Fastening:	No fastening
Details:	No finishing



Product picture, code:  
PR2011F1 (Custom-made)  
STPR114F1 (Standard-size)

Product picture, code:  
PR2012F1 (Custom-made)  
STPR124F1 (Standard-size)

CHIN STRAP		CUSTOM-MADE
Model	Product / Art.No	
	Ear area open	Ear area closed
Adults	PR2011	PR2012
Children	PR1011	PR1012

Children's models are made for children under 10.

**Product information:**

Velcro fastenings on the crown of the head and back of the neck.

Material:	Strong or medium
Colours:	All
Pressure level:	Selectable
Fastening:	ELA 1 with velcro
Details:	Edging ELA 3

CHIN STRAP		STANDARD-SIZE	
Size	Circumf.*	Product / Art.No	
	cm	Ear area open	Ear area closed
XXS	50 - 54	STPR111F1	STPR121F1
XS	55 - 58	STPR112F1	STPR122F1
S	59 - 62	STPR113F1	STPR123F1
M	63 - 66	STPR114F1	STPR124F1
L	67 - 70	STPR115F1	STPR125F1
XL	71 - 74	STPR116F1	STPR126F1

**Product information:**

Velcro fastenings on the crown of the head and back of the neck.

\*Circumference from crown to chin.

Material:	Strong
Colour:	Beige
Pressure level:	Adjustable
Fastening:	ELA 1 with velcro
Details:	Edging ELA 3



Product picture, code:  
PR2010F1

HEADBAND		CUSTOM-MADE
Model	Product / Art.No	
Adults	PR2010	
Children	PR1010	

Children's models are made for children under 10.

**Product information:**

Velcro fastening at the back of the head.

Material:	Strong or medium
Colours:	All
Pressure level:	Selectable
Fastening:	Velcro
Details:	Edging ELA 3



Product picture, code:  
PR20138F1



Product picture, code:  
PR20139F1

HELMET		CUSTOM-MADE
Model	Product / Art.No	
	Face open	Face closed
Adults	PR20138	PR20139
Children	PR10138	PR10139

Children's models are made for children under 10.

**Product information:**

Two different models are available, one of which features a closed face area. The closed-face model is designed to keep scar and wound care products, dressings, or a facial mask securely in place. If necessary, openings can be cut into the face area for the mouth, nose, and eyes. The material does not fray after cutting.

Material:	Strong or medium
Colours:	All
Pressure level:	Selectable
Fastening:	1-row hooks / zipper with guard From back of head to neck
Details:	Edging ELA 3



Product picture, code:  
PR202F1 (Custom-made)  
STPR24F1 (Standard-size)

PRESSURE COLLAR		CUSTOM-MADE
Model	Art.No	
Adults	PR202	
Children	PR102	

Children's models are made for children under 10.

**Product information:**

Collar with 3 mm neoprene filling.  
Velcro fastening in the back of the neck.

Material:	Strong Neoprene
Colours:	All
Pressure level:	Selectable
Pressure level:	Adjustable
Fastening:	Velcro

PRESSURE COLLAR					STANDARD-SIZE
Size	Model	Circumf.* cm	Height		Art.No
			Front	Back	
XXS	Children	20 - 24	4,5	3,0	STPR21F1
XS	Children	25 - 30	4,5	3,0	STPR22F1
S	Adults	31 - 35	7,0	5,5	STPR23F1
M	Adults	36 - 40	7,5	5,5	STPR24F1
L	Adults	41 - 45	8,0	5,5	STPR25F1
XL	Adults	46 - 50	9,0	6,0	STPR26F1

**Product information:**

Collar with 3 mm neoprene filling. Velcro fastening in the back of the neck.  
\*Circumference around the neck.

Material:	Strong with neoprene
Colour:	Beige
Pressure level:	Adjustable
Fastening:	Velcro



# COMPRESSION<sup>TM</sup>

## MEDICAL COMPRESSION GARMENTS

CUSTOM-MADE

STANDARD-SIZE



Product shown in medium fabric without axilla construction



Product picture, code: CO21433F7



Product picture, code: CO21433F1, measured to stop under the breasts  
Smaller picture: Vest with mammary cups addition (MC)

**VEST** **CUSTOM-MADE**

Model	Product / Art.No			
	Sleeveless	Short sleeves	Long and short sleeve	Long sleeves
Women	CO21411	CO21422	CO21423	CO21433
Men	CO22411	CO22422	CO22423	CO22433
Children	CO10411	CO10422	CO10423	CO10433

Children's models are made for children under 10.

Separate abdominal support available for midsection: please see product details from Lymed Skin product group page 41

**Product information:**

The long-sleeved shirt features anatomically shaped elbow construction. In products made from strong fabric, the axilla area uses medium fabric construction. Products made from medium fabric do not include axilla construction. The women's vest features shaping seams at the breasts. The women's model can be manufactured without bust shaping seams free of charge upon request.

Material:	Strong (Ccl 1-4) or medium (Ccl 1-3)
Colours:	All
Compression level:	All available
Fastening:	2-row hooks / zipper with guard / none
Details:	Hemline: SIL 1
	Neckline: ELA 3
	Edge of sleeve ELA 2
	Sleeveless model: edging ELA 3



Product picture, code: CO214022F4



Product shown in medium fabric without axilla construction



**Product information:**

The long-sleeved body features anatomically shaped elbow bends. In products made from strong fabric, the underarm area uses medium fabric construction. Products made from medium fabric do not include axilla construction. The women's body features shaping seams at the breasts. The women's model can be manufactured without bust shaping seams free of charge upon request. Short legs are available as an addition.

Material: Strong (Ccl 1-4) or medium (Ccl 1-3)  
 Colours: All  
 Compression level: All available

**Women:** 2-row hooks / zipper with guard  
 2-row snap fasteners at the crotch  
 Short legs: no crotch closure

**Children:** 2-row hooks / zipper with guard  
 2-row snap fasteners at the crotch  
 Short legs: 1-row snap fasteners

Neckline and leg opening ELA 3  
 Edge of sleeve ELA 2  
 Sleeveless model: edging ELA 3  
 Short legs: edge of leg ELA 2

**BODY** **CUSTOM-MADE**

Model	Product / Art.No			
	Sleeveless	Short sleeves	Long and short sleeve	Long sleeves
Women	CO214011	CO214022	CO214023	CO214033
Children	CO104011	CO104022	CO104023	CO104033

Children's models are made for children under 10.



Product picture, code: CO7464F3



Product picture, code: CO7466F3

BRA		CUSTOM-MADE
Model	Product / Art.No	
Two cups	CO7464	
One cup	CO7465	
Without cups	CO7466	

**Product information:**

Sturdy compression bra designed for oedema management, particularly in the side and underarm areas. Shirt-style design with front fastening positioned between the breasts.


Available with two cups, one cup, or without cups. Prosthesis pockets are available separately as an addition.

Material:	Strong (Ccl 1-4) or medium (Ccl 1-3)
Colours:	All*
Compression level:	All available
Fastenings:	1-row hooks / 2-row hooks / zipper with guard and two dressing aid hooks
Details:	Neckline ELA 3
	Edging ELA 3
	Lower edge selectable

\* 3 cm width adjustable straps available only in beige, black or white

## BRA AND BRA TOP STANDARD SELECTION

### STRAPS

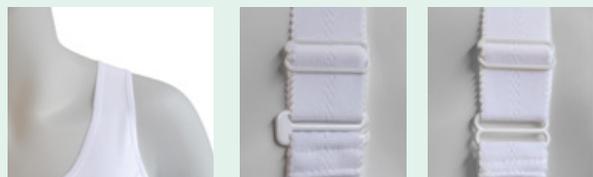

Fabric,  
non-adjustable

Open hook,  
adjustable

Closed hook,  
adjustable

### BACK NECKLINE



High

Low

### FRONT NECKLINE



Medium height

Low

### ARMHOLE



High armhole

Low armhole



Product picture, code:  
CO7474F3

Product picture, code:  
CO7476F3

BRA TOP		CUSTOM-MADE
Model	Product / Art.no	
Two cups	CO7474	
One cup	CO7475	
Without cups	CO7476	

\* 3 cm width adjustable straps available only in beige, black or white

#### Product information:

A sturdy compression bra designed for oedema management, particularly in the side and underarm areas. Vest style design with front fastening positioned between the breasts. Available with two cups, one cup, or without cups. Prosthesis pockets are available separately as an addition.

Material:	Strong (Ccl 1-4) or medium (Ccl 1-3)
Colours:	All*
Compression level:	All available
Fastenings:	1-row hooks / 2-row hooks / zipper with guard and three dressing aid hooks
Details:	Neckline ELA 3
	Edging ELA 3
	Lower edge selectable



Product picture, code: STCO754F3H

**COMPRESSION BRA**

STANDARD-SIZE

**Product information:**

A sturdy compression bra designed for oedema management, particularly in the side and underarm areas. Vest style design with front fastening positioned between the breasts. Available with two cups, one cup, or without cups, with or without prosthesis pockets.

*\*Circumference at the chest below the breasts.*

Material:	Strong Pocket for prosthesis: Medium
Colours:	Black, white
Compression level:	Ccl 2
Fastening:	1-row hooks / zipper with guard
Details:	Hemline ELA 1 Edging ELA 3



**COMPRESSION BRA | HOOKS**

STANDARD-SIZE

Size	Circumf.* cm	Colour	Product / Art.No						
			Two cups	Without cups	Cup on the right	Cup on the left	Prosthesis pocket, both	Prosthesis pocket, right	Prosthesis pocket, left
XS	68 - 76	Black	STCO752F2H	STCO762F2H	STCO762RF2H	STCO762LF2H	STCO752PPF2H	STCO752PRF2H	STCO752PLF2H
XS	68 - 76	White	STCO752F3H	STCO762F3H	STCO762RF3H	STCO762LF3H	STCO752PPF3H	STCO752PRF3H	STCO752PLF3H
S	76 - 84	Black	STCO753F2H	STCO763F2H	STCO763RF2H	STCO763LF2H	STCO753PPF2H	STCO753PRF2H	STCO753PLF2H
S	76 - 84	White	STCO753F3H	STCO763F3H	STCO763RF3H	STCO763LF3H	STCO753PPF3H	STCO753PRF3H	STCO753PLF3H
M	84 - 89	Black	STCO754F2H	STCO764F2H	STCO764RF2H	STCO764LF2H	STCO754PPF2H	STCO754PRF2H	STCO754PLF2H
M	84 - 89	White	STCO754F3H	STCO764F3H	STCO764RF3H	STCO764LF3H	STCO754PPF3H	STCO754PRF3H	STCO754PLF3H
L	89 - 94	Black	STCO755F2H	STCO765F2H	STCO765RF2H	STCO765LF2H	STCO755PPF2H	STCO755PRF2H	STCO755PLF2H
L	89 - 94	White	STCO755F3H	STCO765F3H	STCO765RF3H	STCO765LF3H	STCO755PPF3H	STCO755PRF3H	STCO755PLF3H
XL	94 - 104	Black	STCO756F2H	STCO766F2H	STCO766RF2H	STCO766LF2H	STCO756PPF2H	STCO756PRF2H	STCO756PLF2H
XL	94 - 104	White	STCO756F3H	STCO766F3H	STCO766RF3H	STCO766LF3H	STCO756PPF3H	STCO756PRF3H	STCO756PLF3H
XXL	104 - 114	Black	STCO757F2H	STCO767F2H	STCO767RF2H	STCO767LF2H	STCO757PPF2H	STCO757PRF2H	STCO757PLF2H
XXL	104 - 114	White	STCO757F3H	STCO767F3H	STCO767RF3H	STCO767LF3H	STCO757PPF3H	STCO757PRF3H	STCO757PLF3H
XXXL	114 - 127	Black	STCO758F2H	STCO768F2H	STCO768RF2H	STCO768LF2H	STCO758PPF2H	STCO758PRF2H	STCO758PLF2H
XXXL	114 - 127	White	STCO758F3H	STCO768F3H	STCO768RF3H	STCO768LF3H	STCO758PPF3H	STCO758PRF3H	STCO758PLF3H

**COMPRESSION BRA | ZIPPER**

STANDARD-SIZE

Size	Circumf.* cm	Colour	Product / Art.No						
			Two cups	Without cups	Cup on the right	Cup on the left	Prosthesis pocket, both	Prosthesis pocket, right	Prosthesis pocket, left
XS	68 - 76	Black	STCO752F2Z	STCO762F2Z	STCO762RF2Z	STCO762LF2Z	STCO752PPF2Z	STCO752PRF2Z	STCO752PLF2Z
XS	68 - 76	White	STCO752F3Z	STCO762F3Z	STCO762RF3Z	STCO762LF3Z	STCO752PPF3Z	STCO752PRF3Z	STCO752PLF3Z
S	76 - 84	Black	STCO753F2Z	STCO763F2Z	STCO763RF2Z	STCO763LF2Z	STCO753PPF2Z	STCO753PRF2Z	STCO753PLF2Z
S	76 - 84	White	STCO753F3Z	STCO763F3Z	STCO763RF3Z	STCO763LF3Z	STCO753PPF3Z	STCO753PRF3Z	STCO753PLF3Z
M	84 - 89	Black	STCO754F2Z	STCO764F2Z	STCO764RF2Z	STCO764LF2Z	STCO754PPF2Z	STCO754PRF2Z	STCO754PLF2Z
M	84 - 89	White	STCO754F3Z	STCO764F3Z	STCO764RF3Z	STCO764LF3Z	STCO754PPF3Z	STCO754PRF3Z	STCO754PLF3Z
L	89 - 94	Black	STCO755F2Z	STCO765F2Z	STCO765RF2Z	STCO765LF2Z	STCO755PPF2Z	STCO755PRF2Z	STCO755PLF2Z
L	89 - 94	White	STCO755F3Z	STCO765F3Z	STCO765RF3Z	STCO765LF3Z	STCO755PPF3Z	STCO755PRF3Z	STCO755PLF3Z
XL	94 - 104	Black	STCO756F2Z	STCO766F2Z	STCO766RF2Z	STCO766LF2Z	STCO756PPF2Z	STCO756PRF2Z	STCO756PLF2Z
XL	94 - 104	White	STCO756F3Z	STCO766F3Z	STCO766RF3Z	STCO766LF3Z	STCO756PPF3Z	STCO756PRF3Z	STCO756PLF3Z
XXL	104 - 114	Black	STCO757F2Z	STCO767F2Z	STCO767RF2Z	STCO767LF2Z	STCO757PPF2Z	STCO757PRF2Z	STCO757PLF2Z
XXL	104 - 114	White	STCO757F3Z	STCO767F3Z	STCO767RF3Z	STCO767LF3Z	STCO757PPF3Z	STCO757PRF3Z	STCO757PLF3Z
XXXL	114 - 127	Black	STCO758F2Z	STCO768F2Z	STCO768RF2Z	STCO768LF2Z	STCO758PPF2Z	STCO758PRF2Z	STCO758PLF2Z
XXXL	114 - 127	White	STCO758F3Z	STCO768F3Z	STCO768RF3Z	STCO768LF3Z	STCO758PPF3Z	STCO758PRF3Z	STCO758PLF3Z



Product picture, code:  
CO22533F1

**PANTS & STOCKINGS**

CUSTOM-MADE

**Product information:**

Long-leg pants with anatomical construction at the knee. Stocking legs with anatomical construction at the ankle and knee.

When a stocking leg is ordered in strong material with compression classes 1-2, the constructions are made from medium material.

Men's models with contouring at the front.

Material:	Strong (Ccl 1-4) or medium (Ccl 1-3)
Colours:	All
Compression level:	All available
Fastening:	Available separately
Details:	Waist ELA 1
	Leg opening ELA 2

PANTS						CUSTOM-MADE
Model	Product / Art.No					
	Short legs	Short and long leg	Short leg and leg with stocking OPEN	Short leg and leg with stocking CLOSED	Long legs	
Women	CO21522	CO21523	CO215248	CO215249	CO21533	
Men	CO22522	CO22523	CO225248	CO225249	CO22533	
Girls	CO11522	CO11523	CO115248	CO115249	CO11533	
Boys	CO12522	CO12523	CO125248	CO125249	CO12533	

Children's models are made for children under 10.

Model	Product / Art.No					
	Long leg and leg with stocking OPEN	Long leg and leg with stocking CLOSED	Short leg and leg with toe sock, toe tips OPEN	Short leg and leg with toe sock, toe tips CLOSED	Long leg and leg with toe sock, toe tips OPEN	Long leg and leg with toe sock, toe tips CLOSED
Women	CO215348	CO215349	CO21524FG8	CO21524FG9	CO21534FG8	CO21534FG9
Men	CO225348	CO225349	CO22524FG8	CO22524FG9	CO22534FG8	CO22534FG9
Girls	CO115348	CO115349	CO11524FG8	CO11524FG9	CO11534FG8	CO11534FG9
Boys	CO125348	CO125349	CO12524FG8	CO12524FG9	CO12534FG8	CO12534FG9

Children's models are made for children under 10.

STOCKINGS							CUSTOM-MADE
Model	Product / Art.No						
	Toes OPEN	Toes CLOSED	One leg with toe sock, toe tips OPEN	One leg with toe sock, toe tips CLOSED	With toe socks, toe tips OPEN	With toe socks, toe tips CLOSED	
Women	CO215448	CO215449	CO21544FG8	CO21544FG9	CO21544FG28	CO21544FG29	
Men	CO225448	CO225449	CO22544FG8	CO22544FG9	CO22544FG28	CO22544FG29	
Girls	CO115448	CO115449	CO11544FG8	CO11544FG9	CO11544FG28	CO11544FG29	
Boys	CO125448	CO125449	CO12544FG8	CO12544FG9	CO12544FG28	CO12544FG29	

Children's models are made for children under 10.



Product picture, code:  
STCO21514F1



Product picture, code:  
STCO21514HWF1

COMPRESSION PANTS FOR WOMEN

STANDARD-SIZE

**Product information:**

Normal-waist and high-waist models are available in different leg lengths.  
Optional additional pressure configurations (FS, FNS, NI) are also available for the genital area.

Recommended addition: shoulder straps.

\*Circumference at widest point of pelvis and upper thigh.

Material:	Strong
Colours:	Beige, black
Compression level:	Ccl 2
Details:	Waist SIL 1, Leg opening ELA 2

Product picture, code:  
STCO21514FNSF1 ABS

Normal waist  
with additional flap (FNS),  
shoulder straps



COMPRESSION PANTS FOR WOMEN   NORMAL WAIST						STANDARD-SIZE
Model	Circumf.*		Colour	Product / Art.No		
	Pelvis (cm)	Thigh (cm)		Knee length	Calf length	Ankle length
XS	86 - 90	50,0 - 52,5	Beige	STCO21512F1	STCO21522F1	STCO21532F1
XS	86 - 90	50,0 - 52,5	Black	STCO21512F2	STCO21522F2	STCO21532F2
S	91 - 95	53,0 - 56,0	Beige	STCO21513F1	STCO21523F1	STCO21533F1
S	91 - 95	53,0 - 56,0	Black	STCO21513F2	STCO21523F2	STCO21533F2
M	96 - 101	56,5 - 59,0	Beige	STCO21514F1	STCO21524F1	STCO21534F1
M	96 - 101	56,5 - 59,0	Black	STCO21514F2	STCO21524F2	STCO21534F2
L	102 - 110	59,5 - 62,0	Beige	STCO21515F1	STCO21525F1	STCO21535F1
L	102 - 110	59,5 - 62,0	Black	STCO21515F2	STCO21525F2	STCO21535F2
XL	111 - 121	62,5 - 65,0	Beige	STCO21516F1	STCO21526F1	STCO21536F1
XL	111 - 121	62,5 - 65,0	Black	STCO21516F2	STCO21526F2	STCO21536F2
XXL	122 - 132	65,5 - 68,0	Beige	STCO21517F1	STCO21527F1	STCO21537F1
XXL	122 - 132	65,5 - 68,0	Black	STCO21517F2	STCO21527F2	STCO21537F2
XXXL	133 - 138	68,5 - 71,5	Beige	STCO21518F1	STCO21528F1	STCO21538F1
XXXL	133 - 138	68,5 - 71,5	Black	STCO21518F2	STCO21528F2	STCO21538F2
XXXXL	139 - 144	72,0 - 74,5	Beige	STCO21519F1	STCO21529F1	STCO21539F1
XXXXL	139 - 144	72,0 - 74,5	Black	STCO21519F2	STCO21529F2	STCO21539F2

COMPRESSION PANTS FOR WOMEN   NORMAL WAIST						STANDARD-SIZE
Fabric flap, velcro adjustment						
Model	Circumf.*		Colour	Product / Art.No		
	Pelvis (cm)	Thigh (cm)		Knee length	Calf length	Ankle length
XS	86 - 90	50,0 - 52,5	Beige	STCO21512FSF1	STCO21522FSF1	STCO21532FSF1
XS	86 - 90	50,0 - 52,5	Black	STCO21512FSF2	STCO21522FSF2	STCO21532FSF2
S	91 - 95	53,0 - 56,0	Beige	STCO21513FSF1	STCO21523FSF1	STCO21533FSF1
S	91 - 95	53,0 - 56,0	Black	STCO21513FSF2	STCO21523FSF2	STCO21533FSF2
M	96 - 101	56,5 - 59,0	Beige	STCO21514FSF1	STCO21524FSF1	STCO21534FSF1
M	96 - 101	56,5 - 59,0	Black	STCO21514FSF2	STCO21524FSF2	STCO21534FSF2
L	102 - 110	59,5 - 62,0	Beige	STCO21515FSF1	STCO21525FSF1	STCO21535FSF1
L	102 - 110	59,5 - 62,0	Black	STCO21515FSF2	STCO21525FSF2	STCO21535FSF2
XL	111 - 121	62,5 - 65,0	Beige	STCO21516FSF1	STCO21526FSF1	STCO21536FSF1
XL	111 - 121	62,5 - 65,0	Black	STCO21516FSF2	STCO21526FSF2	STCO21536FSF2
XXL	122 - 132	65,5 - 68,0	Beige	STCO21517FSF1	STCO21527FSF1	STCO21537FSF1
XXL	122 - 132	65,5 - 68,0	Black	STCO21517FSF2	STCO21527FSF2	STCO21537FSF2
XXXL	133 - 138	68,5 - 71,5	Beige	STCO21518FSF1	STCO21528FSF1	STCO21538FSF1
XXXL	133 - 138	68,5 - 71,5	Black	STCO21518FSF2	STCO21528FSF2	STCO21538FSF2
XXXXL	139 - 144	72,0 - 74,5	Beige	STCO21519FSF1	STCO21529FSF1	STCO21539FSF1
XXXXL	139 - 144	72,0 - 74,5	Black	STCO21519FSF2	STCO21529FSF2	STCO21539FSF2

**COMPRESSION PANTS FOR WOMEN | NORMAL WAIST**

Flap with neoprene reinforcement, velcro adjustment

**STANDARD-SIZE**

Model	Circumf.*		Colour	Product / Art.No		
	Pelvis (cm)	Thigh (cm)		Knee length	Calf length	Ankle length
XS	86 - 90	50,0 - 52,5	Beige	STCO21512FNSF1	STCO21522FNSF1	STCO21532FNSF1
XS	86 - 90	50,0 - 52,5	Black	STCO21512FNSF2	STCO21522FNSF2	STCO21532FNSF2
S	91 - 95	53,0 - 56,0	Beige	STCO21513FNSF1	STCO21523FNSF1	STCO21533FNSF1
S	91 - 95	53,0 - 56,0	Black	STCO21513FNSF2	STCO21523FNSF2	STCO21533FNSF2
M	96 - 101	56,5 - 59,0	Beige	STCO21514FNSF1	STCO21524FNSF1	STCO21534FNSF1
M	96 - 101	56,5 - 59,0	Black	STCO21514FNSF2	STCO21524FNSF2	STCO21534FNSF2
L	102 - 110	59,5 - 62,0	Beige	STCO21515FNSF1	STCO21525FNSF1	STCO21535FNSF1
L	102 - 110	59,5 - 62,0	Black	STCO21515FNSF2	STCO21525FNSF2	STCO21535FNSF2
XL	111 - 121	62,5 - 65,0	Beige	STCO21516FNSF1	STCO21526FNSF1	STCO21536FNSF1
XL	111 - 121	62,5 - 65,0	Black	STCO21516FNSF2	STCO21526FNSF2	STCO21536FNSF2
XXL	122 - 132	65,5 - 68,0	Beige	STCO21517FNSF1	STCO21527FNSF1	STCO21537FNSF1
XXL	122 - 132	65,5 - 68,0	Black	STCO21517FNSF2	STCO21527FNSF2	STCO21537FNSF2
XXXL	133 - 138	68,5 - 71,5	Beige	STCO21518FNSF1	STCO21528FNSF1	STCO21538FNSF1
XXXL	133 - 138	68,5 - 71,5	Black	STCO21518FNSF2	STCO21528FNSF2	STCO21538FNSF2
XXXXL	139 - 144	72,0 - 74,5	Beige	STCO21519FNSF1	STCO21529FNSF1	STCO21539FNSF1
XXXXL	139 - 144	72,0 - 74,5	Black	STCO21519FNSF2	STCO21529FNSF2	STCO21539FNSF2

**COMPRESSION PANTS FOR WOMEN | NORMAL WAIST**

Integrated centerpiece with neoprene reinforcement

**STANDARD-SIZE**

Model	Circumf.*		Colour	Product / Art.No		
	Pelvis (cm)	Thigh (cm)		Knee length	Calf length	Ankle length
XS	86 - 90	50,0 - 52,5	Beige	STCO21512NIF1	STCO21522NIF1	STCO21532NIF1
XS	86 - 90	50,0 - 52,5	Black	STCO21512NIF2	STCO21522NIF2	STCO21532NIF2
S	91 - 95	53,0 - 56,0	Beige	STCO21513NIF1	STCO21523NIF1	STCO21533NIF1
S	91 - 95	53,0 - 56,0	Black	STCO21513NIF2	STCO21523NIF2	STCO21533NIF2
M	96 - 101	56,5 - 59,0	Beige	STCO21514NIF1	STCO21524NIF1	STCO21534NIF1
M	96 - 101	56,5 - 59,0	Black	STCO21514NIF2	STCO21524NIF2	STCO21534NIF2
L	102 - 110	59,5 - 62,0	Beige	STCO21515NIF1	STCO21525NIF1	STCO21535NIF1
L	102 - 110	59,5 - 62,0	Black	STCO21515NIF2	STCO21525NIF2	STCO21535NIF2
XL	111 - 121	62,5 - 65,0	Beige	STCO21516NIF1	STCO21526NIF1	STCO21536NIF1
XL	111 - 121	62,5 - 65,0	Black	STCO21516NIF2	STCO21526NIF2	STCO21536NIF2
XXL	122 - 132	65,5 - 68,0	Beige	STCO21517NIF1	STCO21527NIF1	STCO21537NIF1
XXL	122 - 132	65,5 - 68,0	Black	STCO21517NIF2	STCO21527NIF2	STCO21537NIF2
XXXL	133 - 138	68,5 - 71,5	Beige	STCO21518NIF1	STCO21528NIF1	STCO21538NIF1
XXXL	133 - 138	68,5 - 71,5	Black	STCO21518NIF2	STCO21528NIF2	STCO21538NIF2
XXXXL	139 - 144	72,0 - 74,5	Beige	STCO21519NIF1	STCO21529NIF1	STCO21539NIF1
XXXXL	139 - 144	72,0 - 74,5	Black	STCO21519NIF2	STCO21529NIF2	STCO21539NIF2

**COMPRESSION PANTS FOR WOMEN | HIGH WAIST**

STANDARD-SIZE

Model	Circumf.*		Colour	Product / Art.No		
	Pelvis (cm)	Thigh (cm)		Knee length	Calf length	Ankle length
XS	86 - 90	50,0 - 52,5	Beige	STCO21512HWF1	STCO21522HWF1	STCO21532HWF1
XS	86 - 90	50,0 - 52,5	Black	STCO21512HWF2	STCO21522HWF2	STCO21532HWF2
S	91 - 95	53,0 - 56,0	Beige	STCO21513HWF1	STCO21523HWF1	STCO21533HWF1
S	91 - 95	53,0 - 56,0	Black	STCO21513HWF2	STCO21523HWF2	STCO21533HWF2
M	96 - 101	56,5 - 59,0	Beige	STCO21514HWF1	STCO21524HWF1	STCO21534HWF1
M	96 - 101	56,5 - 59,0	Black	STCO21514HWF2	STCO21524HWF2	STCO21534HWF2
L	102 - 110	59,5 - 62,0	Beige	STCO21515HWF1	STCO21525HWF1	STCO21535HWF1
L	102 - 110	59,5 - 62,0	Black	STCO21515HWF2	STCO21525HWF2	STCO21535HWF2
XL	111 - 121	62,5 - 65,0	Beige	STCO21516HWF1	STCO21526HWF1	STCO21536HWF1
XL	111 - 121	62,5 - 65,0	Black	STCO21516HWF2	STCO21526HWF2	STCO21536HWF2
XXL	122 - 132	65,5 - 68,0	Beige	STCO21517HWF1	STCO21527HWF1	STCO21537HWF1
XXL	122 - 132	65,5 - 68,0	Black	STCO21517HWF2	STCO21527HWF2	STCO21537HWF2
XXXL	133 - 138	68,5 - 71,5	Beige	STCO21518HWF1	STCO21528HWF1	STCO21538HWF1
XXXL	133 - 138	68,5 - 71,5	Black	STCO21518HWF2	STCO21528HWF2	STCO21538HWF2
XXXXL	139 - 144	72,0 - 74,5	Beige	STCO21519HWF1	STCO21529HWF1	STCO21539HWF1
XXXXL	139 - 144	72,0 - 74,5	Black	STCO21519HWF2	STCO21529HWF2	STCO21539HWF2

**COMPRESSION PANTS FOR WOMEN | HIGH WAIST**

Integrated centerpiece with neoprene reinforcement

STANDARD-SIZE

Model	Circumf.*		Colour	Product / Art.No		
	Pelvis (cm)	Thigh (cm)		Knee length	Calf length	Ankle length
XS	86 - 90	50,0 - 52,5	Beige	STCO21512HWNIF1	STCO21522HWNIF1	STCO21532HWNIF1
XS	86 - 90	50,0 - 52,5	Black	STCO21512HWNIF2	STCO21522HWNIF2	STCO21532HWNIF2
S	91 - 95	53,0 - 56,0	Beige	STCO21513HWNIF1	STCO21523HWNIF1	STCO21533HWNIF1
S	91 - 95	53,0 - 56,0	Black	STCO21513HWNIF2	STCO21523HWNIF2	STCO21533HWNIF2
M	96 - 101	56,5 - 59,0	Beige	STCO21514HWNIF1	STCO21524HWNIF1	STCO21534HWNIF1
M	96 - 101	56,5 - 59,0	Black	STCO21514HWNIF2	STCO21524HWNIF2	STCO21534HWNIF2
L	102 - 110	59,5 - 62,0	Beige	STCO21515HWNIF1	STCO21525HWNIF1	STCO21535HWNIF1
L	102 - 110	59,5 - 62,0	Black	STCO21515HWNIF2	STCO21525HWNIF2	STCO21535HWNIF2
XL	111 - 121	62,5 - 65,0	Beige	STCO21516HWNIF1	STCO21526HWNIF1	STCO21536HWNIF1
XL	111 - 121	62,5 - 65,0	Black	STCO21516HWNIF2	STCO21526HWNIF2	STCO21536HWNIF2
XXL	122 - 132	65,5 - 68,0	Beige	STCO21517HWNIF1	STCO21527HWNIF1	STCO21537HWNIF1
XXL	122 - 132	65,5 - 68,0	Black	STCO21517HWNIF2	STCO21527HWNIF2	STCO21537HWNIF2
XXXL	133 - 138	68,5 - 71,5	Beige	STCO21518HWNIF1	STCO21528HWNIF1	STCO21538HWNIF1
XXXL	133 - 138	68,5 - 71,5	Black	STCO21518HWNIF2	STCO21528HWNIF2	STCO21538HWNIF2
XXXXL	139 - 144	72,0 - 74,5	Beige	STCO21519HWNIF1	STCO21529HWNIF1	STCO21539HWNIF1
XXXXL	139 - 144	72,0 - 74,5	Black	STCO21519HWNIF2	STCO21529HWNIF2	STCO21539HWNIF2



Product picture, code: STCO22534F1

**Product information:**

Men's compression pants in various leg lengths.  
Men's models with contouring at the front.

Recommended addition: shoulder straps.  
\*Circumference at widest point of pelvis and upper thigh.

Material:	Strong
Colours:	Beige, black
Compression level:	Ccl 2
Details:	Waist SIL 1, Leg opening ELA 2

**COMPRESSION PANTS FOR MEN | NORMAL WAIST**

STANDARD-SIZE

Model	Circumf.		Colour	Product / Art.No		
	Pelvis (cm)	Thigh (cm)		Knee length	Calf length	Ankle length
XS	84 - 89	55,0 - 57,5	Beige	STCO22512F1	STCO22522F1	STCO22532F1
XS	84 - 89	55,0 - 57,5	Black	STCO22512F2	STCO22522F2	STCO22532F2
S	90 - 95	58,0 - 60,5	Beige	STCO22513F1	STCO22523F1	STCO22533F1
S	90 - 95	58,0 - 60,5	Black	STCO22513F2	STCO22523F2	STCO22533F2
M	96 - 101	61,0 - 63,5	Beige	STCO22514F1	STCO22524F1	STCO22534F1
M	96 - 101	61,0 - 63,5	Black	STCO22514F2	STCO22524F2	STCO22534F2
L	102 - 107	64,0 - 66,5	Beige	STCO22515F1	STCO22525F1	STCO22535F1
L	102 - 107	64,0 - 66,5	Black	STCO22515F2	STCO22525F2	STCO22535F2
XL	108 - 114	67,0 - 69,5	Beige	STCO22516F1	STCO22526F1	STCO22536F1
XL	108 - 114	67,0 - 69,5	Black	STCO22516F2	STCO22526F2	STCO22536F2
XXL	115 - 121	70,0 - 72,5	Beige	STCO22517F1	STCO22527F1	STCO22537F1
XXL	115 - 121	70,0 - 72,5	Black	STCO22517F2	STCO22527F2	STCO22537F2

Product picture, code: STCO22534FSF1 ABS



**Product information:**

The front panel of the trousers features an opening instead of contouring, along with an additional adjustable support flap secured with velcro to provide genital support.

Recommended addition: shoulder straps.  
\*Circumference at widest point of pelvis and upper thigh.

Material:	Strong
Colours:	Beige, black
Compression level:	Ccl 2
Details:	Waist SIL 1, Leg opening ELA 2

**COMPRESSION PANTS FOR MEN | NORMAL WAIST**

Fabric flap, velcro adjustment

STANDARD-SIZE

Model	Circumf.		Colour	Product / Art.No		
	Pelvis (cm)	Thigh (cm)		Knee length	Calf length	Ankle length
XS	84 - 89	55,0 - 57,5	Beige	STCO22512FSF1	STCO22522FSF1	STCO22532FSF1
XS	84 - 89	55,0 - 57,5	Black	STCO22512FSF2	STCO22522FSF2	STCO22532FSF2
S	90 - 95	58,0 - 60,5	Beige	STCO22513FSF1	STCO22523FSF1	STCO22533FSF1
S	90 - 95	58,0 - 60,5	Black	STCO22513FSF2	STCO22523FSF2	STCO22533FSF2
M	96 - 101	61,0 - 63,5	Beige	STCO22514FSF1	STCO22524FSF1	STCO22534FSF1
M	96 - 101	61,0 - 63,5	Black	STCO22514FSF2	STCO22524FSF2	STCO22534FSF2
L	102 - 107	64,0 - 66,5	Beige	STCO22515FSF1	STCO22525FSF1	STCO22535FSF1
L	102 - 107	64,0 - 66,5	Black	STCO22515FSF2	STCO22525FSF2	STCO22535FSF2
XL	108 - 114	67,0 - 69,5	Beige	STCO22516FSF1	STCO22526FSF1	STCO22536FSF1
XL	108 - 114	67,0 - 69,5	Black	STCO22516FSF2	STCO22526FSF2	STCO22536FSF2
XXL	115 - 121	70,0 - 72,5	Beige	STCO22517FSF1	STCO22527FSF1	STCO22537FSF1
XXL	115 - 121	70,0 - 72,5	Black	STCO22517FSF2	STCO22527FSF2	STCO22537FSF2



Product picture, code: STCO22504F2

**SUSPENSOR**

STANDARD-SIZE

Model	Estim. volume dl	Height cm	Length cm	Art.No
S	6	18	120	STCO22503F2
M	8	18	120	STCO22504F2
L	12	18	120	STCO22505F2
XL	14	18	120	STCO22506F2

**Product information:**

Suspensor for men's genital area oedema.  
Adjustable velcro fastening and straps.

Material:	Polyamide elastane knit, support band with velcro
Colour:	Black
Pressure level:	Adjustable
Fastening:	Velcro fastening
Details:	Front edging: ELA 3 Straps: velcro straps

**PENIS TUBE**

CUSTOM-MADE

Model	Tip	Art.No
Adults	Closed	CO22809
Adults	Open	CO22808
Children	Closed	CO12809
Children	Open	CO12808

Children's models are made for children under 10.

**Product information:**

Anatomical construction in the base of the tube. We recommend velcro fastening for easier donning.

If the circumference of penis is over 20 cm, the product will be priced on a case-by-case basis.

Material:	Strong (Ccl 1-4) or medium (Ccl 1-3)
Colours:	All
Compression level:	All available
Fastening:	Velcro fastening available separately*
Details:	No finishing

\* Fastening colour: white, black or beige, which is chosen by our production.



Product picture, code: CO22808F1

Product picture, code: STCOM21534F2



COMPRESSION™

MATERNITY PANTS						STANDARD-SIZE
Model	Circumf.*		Colour	Product / Art. No		
	Pelvis (cm)	Thigh (cm)		Knee length	Calf length	Ankle length
XS	86 - 90	50,0 - 52,5	Beige	STCOM21512F1	STCOM21522F1	STCOM21532F1
XS	86 - 90	50,0 - 52,5	Black	STCOM21512F2	STCOM21522F2	STCOM21532F2
S	91 - 95	53,0 - 56,0	Beige	STCOM21513F1	STCOM21523F1	STCOM21533F1
S	91 - 95	53,0 - 56,0	Black	STCOM21513F2	STCOM21523F2	STCOM21533F2
M	96 - 101	56,5 - 59,0	Beige	STCOM21514F1	STCOM21524F1	STCOM21534F1
M	96 - 101	56,5 - 59,0	Black	STCOM21514F2	STCOM21524F2	STCOM21534F2
L	102 - 110	59,5 - 62,0	Beige	STCOM21515F1	STCOM21525F1	STCOM21535F1
L	102 - 110	59,5 - 62,0	Black	STCOM21515F2	STCOM21525F2	STCOM21535F2
XL	111 - 121	62,5 - 65,0	Beige	STCOM21516F1	STCOM21526F1	STCOM21536F1
XL	111 - 121	62,5 - 65,0	Black	STCOM21516F2	STCOM21526F2	STCOM21536F2
XXL	122 - 132	65,5 - 68,0	Beige	STCOM21517F1	STCOM21527F1	STCOM21537F1
XXL	122 - 132	65,5 - 68,0	Black	STCOM21517F2	STCOM21527F2	STCOM21537F2
XXXL	133 - 138	68,5 - 71,5	Beige	STCOM21518F1	STCOM21528F1	STCOM21538F1
XXXL	133 - 138	68,5 - 71,5	Black	STCOM21518F2	STCOM21528F2	STCOM21538F2

**Product information:**

The maternity model features a more flexible abdominal section for a growing abdomen. The abdominal section adapts to the measurements of most expectant mothers from mid-pregnancy through to the later stages of pregnancy.

Material:	Strong
	Abdominal area: Light
Colours:	Beige, black
Compression level:	Ccl 2
	Waist SIL 1
Details:	Leg opening ELA 2

**Tip! Can be used together with maternity support belt and bras.**





Product picture, code: CO20603F6



Product picture, code: CO20603F1

LEG		CUSTOM-MADE	
Model	Product / Art.No		
	Ankle-knee	Knee-thigh	Ankle-thigh
Adults	CO20601	CO20602	CO20603
Children	CO10601	CO10602	CO10603

Children's models are made for children under 10.

**Product information:**

The leg sleeve features an anatomically shaped construction in the knee bend area. When the product is ordered in strong fabric with compression class CCL 1-2, the knee bend construction is made from medium fabric.

Material: Strong (Ccl 1-4) or medium (Ccl 1-3)

Colours: All

Compression level: All available

Fastening: Available separately

Details: Upper edge SIL 4,  
Upper edge in knee-high SIL 1,  
Lower edge ELA 2



Product picture, code:  
CO20639F4 A20B



Product picture, code:  
CO20629F4 AZ



Product picture, code:  
CO20619F4



Product picture, code:  
COA20T9F2

**STOCKING** **CUSTOM-MADE**

Model	Toes	Product / Art.No		
		Ankle-high	Knee-high	Thigh-high
Adults	Closed	CO20619	CO20629	CO20639
Adults	Open	CO20618	CO20628	CO20638
Adults	Toe sock, closed	CO20619FG	CO20629FG	CO20639FG
Adults	Toe sock, open	CO20618FG	CO20628FG	CO20638FG
Children	Closed	CO10619	CO10629	CO10639
Children	Open	CO10618	CO10628	CO10638
Children	Toe sock, closed	CO10619FG	CO10629FG	CO10639FG
Children	Toe sock, open	CO10618FG	CO10628FG	CO10638FG

Children's models are made for children under 10.

**Product information:**

The stockings feature anatomically shaped construction at the ankle and knee bend. When the product is ordered in strong fabric with compression class CCL 1-2, the ankle and knee bend constructions are made from medium fabric. In open-toe stockings, the toe is lightly beveled.

The ankle sock's shaft length is measured upwards from the L/R61 circumference. Please note that the shaft length can be at most the same as the L/R67 measurement.

Material:	Strong (Ccl 1-4) or medium (Ccl 1-3)
Colours:	All
Compression level:	All available
Fastening:	Available separately
Details:	Upper edge SIL 1, upper edge in thigh-high SIL 4, lower edge, tip open ELA 2.

**FOOT GLOVE** **CUSTOM-MADE**

Model	Toe tips	Art.No
Adults	Closed	COA20T9
Adults	Open	COA20T8
Children	Closed	COA10T9
Children	Open	COA10T8

Children's models are made for children under 10.

**Product information:**

For the treatment of foot or toes. Can be used by itself or e.g. with an open-toe stocking.

Material:	Strong (Ccl 1-4) or medium (Ccl 1-3)
Colours:	All
Compression level:	All available
Details:	No finishing



Product picture, code: CO20303F1



Product picture, code: CO2031038F1



Product picture, code: CO2031138F1

**SLEEVE** CUSTOM-MADE

Model	Product / Art.No		
	Wrist-elbow	Elbow-axilla	Wrist-axilla
Adults	CO20301	CO20302	CO20303
Children	CO10301	CO10302	CO10303

Children's models are made for children under 10.

**Product information:**

The wrist-to-axilla model features an anatomical elbow bend.

Material:	Strong (Ccl 1-4) or medium (Ccl 1-3)
Colours:	All
Compression level:	All available
Fastening:	Available separately
Details:	Upper edge SIL 1 Lower edge ELA 2

**SLEEVE GAUNTLET** CUSTOM-MADE

Model	Thumb	Product / Art.No	
		To elbow	To axilla
Adults	Closed	CO2031019	CO2031039
Adults	Open	CO2031018	CO2031038
Children	Closed	CO1031019	CO1031039
Children	Open	CO1031018	CO1031038

Children's models are made for children under 10.

**Product information:**

Axilla-length model with anatomical elbow construction. Anatomical construction in the thumb fourchette.

Material:	Strong (Ccl 1-4) or medium (Ccl 1-3)
Colours:	All
Compression level:	All available
Fastening:	Available separately
Details:	Upper edge SIL 1 Lower edge ELA 2

**SLEEVE GLOVE** CUSTOM-MADE

Model	Fingertips	Product / Art.No	
		To elbow	To axilla
Adults	Closed	CO2031119	CO2031139
Adults	Open	CO2031118	CO2031138
Children	Closed	CO1031119	CO1031139
Children	Open	CO1031118	CO1031138

Children's models are made for children under 10.

**Product information:**

Axilla-length model with anatomical elbow construction. Anatomical construction between the fingers and in the thumb fourchette.

Material:	Strong (Ccl 1-4) or medium (Ccl 1-3)
Colours:	All
Compression level:	All available
Fastening:	Available separately
Details:	Upper edge SIL 1

**SLEEVE FASTENING - MODEL 1** CUSTOM-MADE


Model	Product / Art.No
<b>Model 1 Shoulder flap</b>	
Adults	A20S1
Children	A10S1

Children's models are made for children under 10.

**Product information:**

Elastic strap running under the breast to hold the sleeve up. Fastening with velcro. Suitable especially for users with large breasts. Available for all sleeves reaching up to axilla. Women's sleeve with a bra strap holder.

Material:	ELA 1
Colour:	Same as sleeve

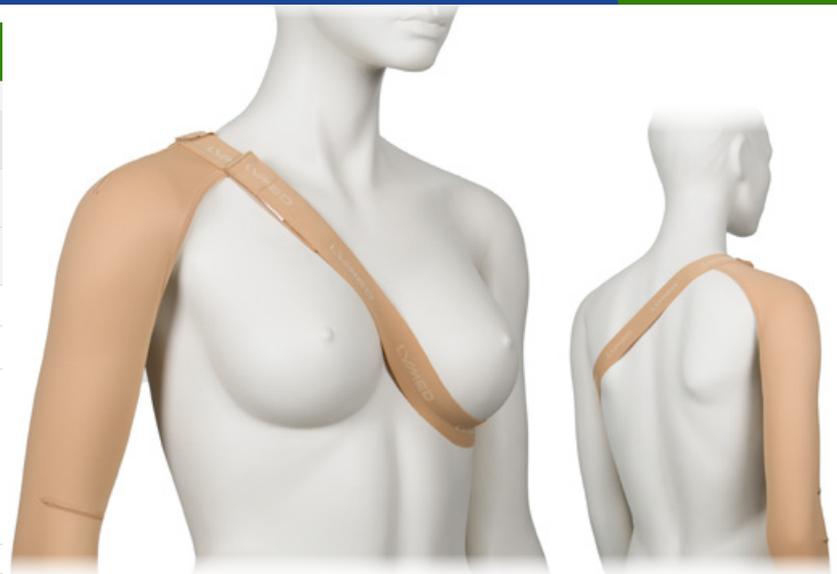

**SLEEVE FASTENING - MODEL 2** CUSTOM-MADE


Model	Product / Art.No
<b>Model 2 Shoulder flap</b>	
Adults	A20S2
Children	A10S2

Children's models are made for children under 10.

**Product information:**

Elastic strap running under the axilla to hold the sleeve up. Fastening with velcro. Available for all sleeves reaching up to axilla.

Material:	ELA 1
Colour:	Same as sleeve

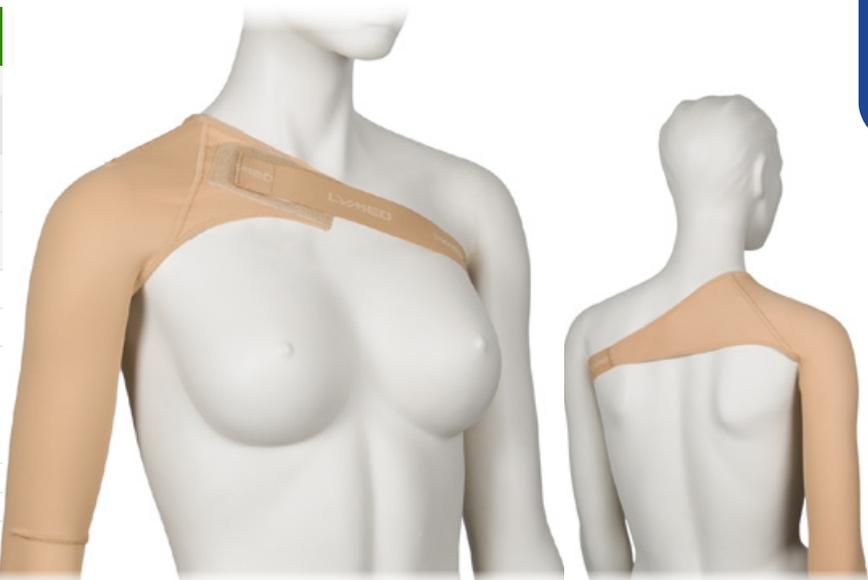

**SLEEVE FASTENING - MODEL 3** CUSTOM-MADE

Model	Product / Art.No
<b>Model 3 Opposite sleeve</b>	
Adults	A20S3
Children	A10S3


Children's models are made for children under 10.

**Product information:**

In the sleeve attachment model, a counter-sleeve connects two axilla-length sleeves. The model is suitable for situations where maintaining the position of one or both sleeves is challenging. The counter-sleeve configuration can be used in different setups: to connect two sleeves or to support the positioning of a single sleeve. Two sleeves must be ordered in conjunction with the attachment. If the ordered sleeves are made from different materials, the attachment component is manufactured by default using the lighter material.

Material:	Strong or medium
Colour:	Same as sleeve
Fastening:	2-row hooks / zipper with guard
Details:	Neckline ELA 3
	Hemline SIL 2
	Edge of sleeves ELA 2

Product pictured, made from medium material

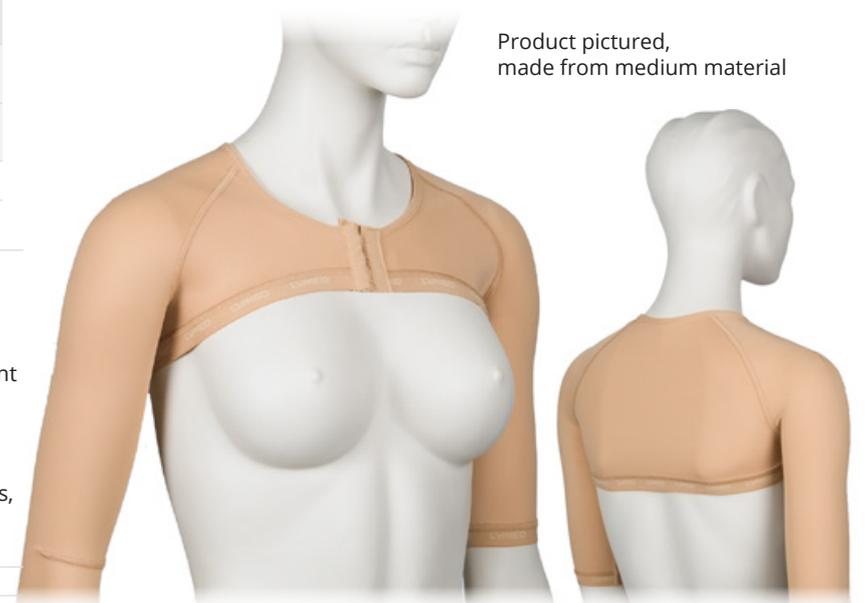

See also vest model on p. 58



Product picture, code:  
CO203119F1 (Custom-made)  
STCO31194RF1 (Standard-size)

Product picture, code:  
CO203118F1 (Custom-made)  
STCO31184RF1 (Standard-size)

GLOVE		CUSTOM-MADE
Model	Product / Art.No	
	Fingertips closed	Fingertips open
Adults	CO203119	CO203118
Children	CO103119	CO103118

Children's models are made for children under 10.

**Product information:**

Anatomical construction between the fingers and in the thumb fourchette. Number of backhand seams/darts in a glove may vary due to individual measurements and pressure/compression levels.  
Order a sleeve-glove when the length of the shaft is greater than the length of the palm.

Material:	Strong (Ccl 1-4) or medium (Ccl 1-3)
Colours:	All
Compression level:	All available
Fastening:	Available separately
Details:	Wrist edging ELA 3

GLOVE		STANDARD-SIZE				
Size	Circumf.*	Colour	Product / Art.No			
	cm		Fingertips closed		Fingertips open	
			Right	Left	Right	Left
XXS	14 - 16	Beige	STCO31191RF1	STCO31191LF1	STCO31181RF1	STCO31181LF1
XXS	14 - 16	Black	STCO31191RF2	STCO31191LF2	STCO31181RF2	STCO31181LF2
XS	17 - 18	Beige	STCO31192RF1	STCO31192LF1	STCO31182RF1	STCO31182LF1
XS	17 - 18	Black	STCO31192RF2	STCO31192LF2	STCO31182RF2	STCO31182LF2
S	19 - 20	Beige	STCO31193RF1	STCO31193LF1	STCO31183RF1	STCO31183LF1
S	19 - 20	Black	STCO31193RF2	STCO31193LF2	STCO31183RF2	STCO31183LF2
M	21 - 22	Beige	STCO31194RF1	STCO31194LF1	STCO31184RF1	STCO31184LF1
M	21 - 22	Black	STCO31194RF2	STCO31194LF2	STCO31184RF2	STCO31184LF2
L	23 - 24	Beige	STCO31195RF1	STCO31195LF1	STCO31185RF1	STCO31185LF1
L	23 - 24	Black	STCO31195RF2	STCO31195LF2	STCO31185RF2	STCO31185LF2
XL	25 - 26	Beige	STCO31196RF1	STCO31196LF1	STCO31186RF1	STCO31186LF1
XL	25 - 26	Black	STCO31196RF2	STCO31196LF2	STCO31186RF2	STCO31186LF2
XXL	27 - 28	Beige	STCO31197RF1	STCO31197LF1	STCO31187RF1	STCO31187LF1
XXL	27 - 28	Black	STCO31197RF2	STCO31197LF2	STCO31187RF2	STCO31187LF2

**Product information:**

Anatomical construction between the fingers.

\*Circumference at MCP.

Material:	Strong
Colours:	Beige, black
Compression level:	Ccl 2
Details:	Wrist edging ELA 3



Product picture, code:  
CO203108F1 (Custom-made)  
STCO3104RF1 (Standard-size)

GAUNTLET		CUSTOM-MADE
Model	Product / Art.No	
	Thumb closed	Thumb open
Adults	CO203109	CO203108
Children	CO103109	CO103108

Children's models are made for children under 10.

**Product information:**

Anatomical construction in the thumb fourchette. Order a sleeve-gauntlet when the length of the shaft is greater than the length of the palm.

Material:	Strong (Ccl 1-4) or medium (Ccl 1-3)
Colours:	All
Compression level:	All available
Fastening:	Available separately
Details:	Wrist edging ELA 3 Palm edging ELA 2

GAUNTLET			STANDARD-SIZE	
Size	Circumf.*	Colour	Product / Art No.	
	cm		Thumb open	
			Right	Left
XXS	14 - 16	Beige	STCO3101RF1	STCO3101LF1
XXS	14 - 16	Black	STCO3101RF2	STCO3101LF2
XS	17 - 18	Beige	STCO3102RF1	STCO3102LF1
XS	17 - 18	Black	STCO3102RF2	STCO3102LF2
S	19 - 20	Beige	STCO3103RF1	STCO3103LF1
S	19 - 20	Black	STCO3103RF2	STCO3103LF2
M	21 - 22	Beige	STCO3104RF1	STCO3104LF1
M	21 - 22	Black	STCO3104RF2	STCO3104LF2
L	23 - 24	Beige	STCO3105RF1	STCO3105LF1
L	23 - 24	Black	STCO3105RF2	STCO3105LF2
XL	25 - 26	Beige	STCO3106RF1	STCO3106LF1
XL	25 - 26	Black	STCO3106RF2	STCO3106LF2
XXL	27 - 28	Beige	STCO3107RF1	STCO3107LF1
XXL	27 - 28	Black	STCO3107RF2	STCO3107LF2

**Product information:**

Thumb tip open.

\*Circumference at MCP.

Material:	Strong
Colours:	Beige, black
Compression level:	Ccl 2
Details:	Wrist edging ELA 3 Palm edging ELA 2



Product picture, code:  
CO2011F1



Product picture, code:  
CO2012F1

CHIN STRAP		CUSTOM-MADE
Model	Product / Art.No	
	Ear area open	Ear area closed
Adults	CO2011	CO2012
Children	CO1011	CO1012

Children's models are made for children under 10.

**Product information:**

Velcro fastenings on the crown of the head and back of the neck.

Material: Strong (Ccl 1-4) or medium (Ccl 1-3)

Colours: All

Compression level: All available

Fastening: ELA 1 with velcro

Details: Edging: ELA 3



Product picture, code:  
CO20138F1



Product picture, code:  
CO20139F1

HELMET		CUSTOM-MADE
Model	Product / Art.No	
	Face open	Face closed
Adults	CO20138	CO20139
Children	CO10138	CO10139

Children's models are made for children under 10.

**Product information:**

Two different models are available, one of which features a closed face area. The closed-face model is designed to keep scar and wound care products, dressings, or a facial mask securely in place. If necessary, openings can be cut into the face area for the mouth, nose, and eyes. The material does not fray after cutting.

Material:	Strong (Ccl 1-4) or medium (Ccl 1-3)
Colours:	All
Compression level:	All available
Fastening:	1-row hooks / zipper with guard From back of head to neck
Details:	Edging ELA 3

# SENSE™

NEUROLOGICAL PRESSURE GARMENTS

CUSTOM-MADE

STANDARD-SIZE





Product picture, code:  
SEE10422F5

SCAPULA SUPPORT VEST			CUSTOM-MADE
Model	Product / Art.No		
	Long sleeves	Long and short sleeve	Short sleeves
Women	SEE21433	SEE21423	SEE21422
Men	SEE22433	SEE22423	SEE22422
Children	SEE10433	SEE10423	SEE10422

Children's models are made for children under 10.

**Product information:**

The product is intended for use in connection with Erb's palsy and serratus palsy. Adjustable, elastic hook-and-loop straps and a neoprene reinforcement in the scapular and shoulder area. The straps are made of elastane tape. Available as an additional option with bilateral scapular support. The long-sleeved shirt features anatomically shaped elbow bends. The product features axilla constructions made from light fabric. The women's shirt features shaping seams at the breasts. The women's model can be manufactured without bust shaping seams free of charge upon request.

Material:	Strong or medium
Colours:	All
Pressure level:	Selectable
Fastening:	2-row hooks / zipper with guard
Details:	Hemline SIL 1 Neckline ELA 3 Edge of sleeve ELA 2


SCAPULA SUPPORT BODY			CUSTOM-MADE
Model	Product / Art.No		
	Long sleeves	Long and short sleeve	Short sleeves
Children	SEE104033	SEE104023	SEE104022

Children's models are made for children under 10.

**Product information:**

The product is intended for use in connection with Erb's palsy and serratus palsy. Adjustable, elastic hook-and-loop straps and a neoprene reinforcement in the scapular and shoulder area. The straps are made of elastane tape. Available with bilateral scapular support and short legs. Adjustable snap fastening in the crotch. The long-sleeved shirt features anatomically shaped elbow bends. The product features axilla constructions made from light fabric.

Material:	Strong or medium
Colours:	All
Pressure level:	Selectable
Fastening:	2-row hooks / zipper with guard 2-row snap fasteners at the crotch Short legs: 1-row snap fasteners Neckline and leg opening ELA 3
Details:	Edge of sleeve ELA 2 Short legs: ELA 2

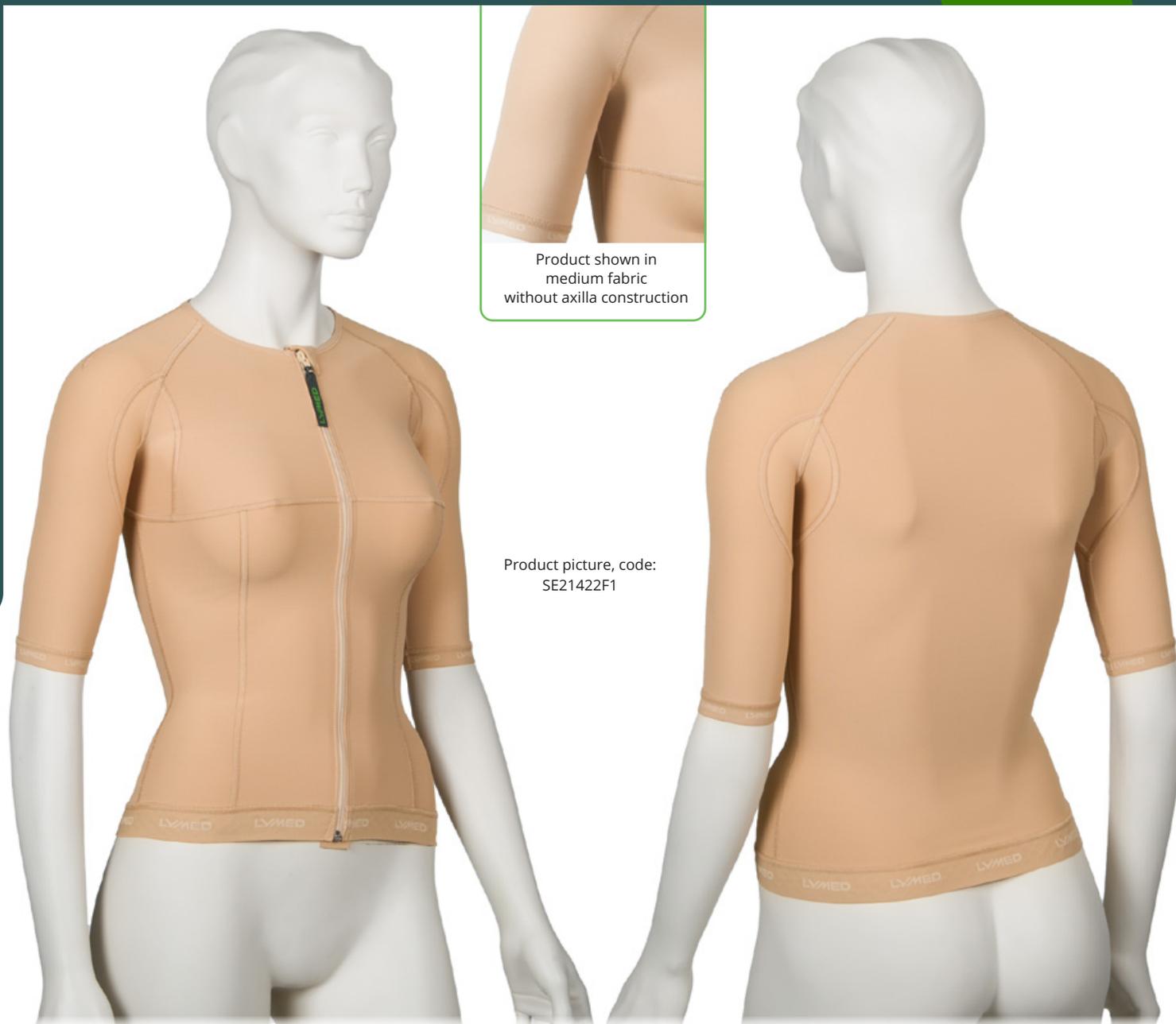

Product shown in medium fabric without axilla construction

Product picture, code: SE21422F1

VEST		CUSTOM-MADE		
Model	Product / Art.No			
	Sleeveless	Short sleeves	Long and short sleeve	Long sleeves
Women	SE21411	SE21422	SE21423	SE21433
Men	SE22411	SE22422	SE22423	SE22433
Children	SE10411	SE10422	SE10423	SE10433

Children's models are made for children under 10.

Separate abdominal support available for midsection: please see product details from Lymed Skin -product group page 41

**Product information:**

The long-sleeved vest features anatomically shaped elbow bends. In products made from strong fabric, the axilla area uses light fabric construction. Products made from medium or light fabric do not include axilla construction. The women's vest features shaping seams at the breasts. The women's model can be manufactured without bust shaping seams free of charge upon request.

Material:	Strong, medium, light (only light pressure)
Colours:	All
Pressure level:	Selectable
Fastening:	2-row hooks / zipper with guard / no fastening
Details:	Helpline SIL 1
	Neckline ELA 3
	Edge of sleeve ELA 2
	Sleeveless model: edging ELA 3



Product shown in medium fabric without axilla construction

Product picture, code:  
SE1045333F6

<b>OVERALL</b>		<b>CUSTOM-MADE</b>
<i>Model</i>	<i>Product / Art.No</i>	
<b>Long sleeves and legs</b>		
Children	SE10453333	

Children's models are made for children under 10.

**Product information:**

Overall with anatomical elbow and knee constructions. In products made from strong fabric, the underarm are uses medium fabric construction. Products made from medium or light fabric do not include underarm construction.

Material:	Strong, medium, light (only light pressure)
Colours available:	All
Pressure level:	Selectable
Fastening:	2-row hooks / zipper with guard
Details:	Neckline ELA 3
	Leg opening ELA 2
	Edge of sleeve ELA 2



Product picture, code:  
SE104011F5 SEA10L

**BODY**

**CUSTOM-MADE**

Model	Product / Art.No			
	Sleeveless	Short sleeves	Long and short sleeve	Long sleeves
Women	SE214011	SE214022	SE214023	SE214033
Children	SE104011	SE104022	SE104023	SE104033

Children's models are made for children under 10.

**Product information:**

The long-sleeved body features anatomically shaped elbow bends and in products made from strong fabric, the axilla area uses medium fabric construction. Products made from medium or light fabric do not include underarm construction.

The women's body features shaping seams at the breasts. The women's model can be manufactured without bust shaping seams free of charge upon request. Short legs are available as an addition.

Material:	Strong, medium, light (only light pressure)
Colours:	All
Pressure level:	Selectable
Fastening:	<b>Women:</b> 2-row hooks / zipper with guard 2-row snap fasteners at the crotch Short legs: no crotch closure
Details:	<b>Children:</b> 2-row hooks / zipper with guard 2-row snap fasteners at the crotch Short legs: 1-row snap fasteners
	Neckline and leg opening ELA 3
	Edge of sleeve ELA 2
	Sleeveless model: edging ELA 3
	Short legs: edge of leg ELA 2



Product picture, code:  
STSE404F7

**ADJUSTABLE  
BODY**

STANDARD-SIZE


Size	Art. No	Colourcode
<b>Short sleeves</b>		
1	STSE401	F1 / F2 / F3 / F4 / F5 / F6 / F7
2	STSE402	F1 / F2 / F3 / F4 / F5 / F6 / F7
3	STSE403	F1 / F2 / F3 / F4 / F5 / F6 / F7
4	STSE404	F1 / F2 / F3 / F4 / F5 / F6 / F7
5	STSE405	F1 / F2 / F3 / F4 / F5 / F6 / F7
6	STSE406	F1 / F2 / F3 / F4 / F5 / F6 / F7
7	STSE407	F1 / F2 / F3 / F4 / F5 / F6 / F7

Sizes	Measurements (cm)					
	B30	B11	B17	B2	B9	B12
1	80 - 86	24	30	43	40	42
2	84 - 90	25	31	47	43	45
3	88 - 96	26	33	51	46	48
4	94 - 102	27	35	55	49	52
5	100 - 108	28,5	37	59	52	56
6	107 - 115	30	39	63	55	60
7	114 - 122	31,5	41	67	58	64

**Product information:**

The product can be used in particular as an assistive garment in physiotherapy and occupational therapy. Adjustable hook-and-loop fastening at the back. The lighter fabric in the abdominal area adds adaptability to the fit. The product features axilla constructions made from medium fabric. Adjustable snap fastening in the crotch.

Material:	Strong Abdominal area: medium
Colours:	All
Pressure level:	Adjustable
Fastening:	Velcro at the back, 3- row snap fasteners at the crotch
Details:	Neckline and leg opening ELA 3 Edge of sleeve ELA 2

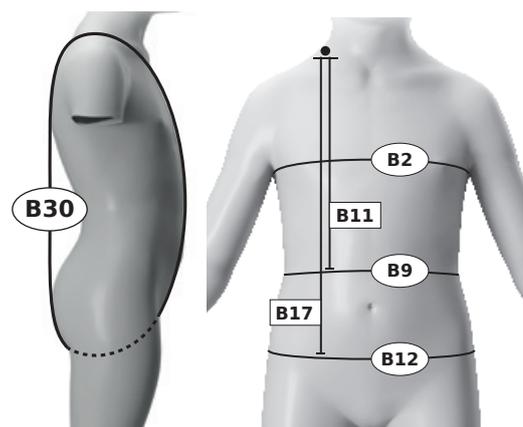



Product picture, code:  
SE22522F4

**Product information:**

Long-leg pants with anatomical construction at the knee.  
Stocking legs with anatomical construction at the ankle and knee.  
When a stocking leg is ordered in strong material, the constructions are made from medium material.  
Men's models with contouring at the front.

Material:	Strong, medium, light (only light pressure)
Colours:	All
Pressure level:	Selectable
Fastening:	Available separately
Details:	Waist ELA 1 Leg opening ELA 2

**PANTS**

**CUSTOM-MADE**

Model	Product / Art.No			
	Short legs	Short and long leg	Short leg and leg with stocking OPEN	Short leg and leg with stocking CLOSED
Women	SE21522	SE21523	SE215248	SE215249
Men	SE22522	SE22523	SE225248	SE225249
Girls	SE11522	SE11523	SE115248	SE115249
Boys	SE12522	SE12523	SE125248	SE125249

Children's models are made for children under 10.

Model	Product / Art.No						
	Long legs	Long leg and leg with stocking OPEN	Long leg and leg with stocking CLOSED	Short leg and leg with toe sock, toe tips OPEN	Short leg and leg with toe sock, toe tips CLOSED	Long leg and leg with toe sock, toe tips OPEN	Long leg and leg with toe sock, toe tips CLOSED
Women	SE21533	SE215348	SE215349	SE21524FG8	SE21524FG9	SE21534FG8	SE21534FG9
Men	SE22533	SE225348	SE225349	SE22524FG8	SE22524FG9	SE22534FG8	SE22534FG9
Girls	SE11533	SE115348	SE115349	SE11524FG8	SE11524FG9	SE11534FG8	SE11534FG9
Boys	SE12533	SE125348	SE125349	SE12524FG8	SE12524FG9	SE12534FG8	SE12534FG9

Children's models are made for children under 10.

**STOCKINGS**

**CUSTOM-MADE**

Model	Product / Art.No					
	Toes OPEN	Toes CLOSED	One leg with toe sock, toe tips OPEN	One leg with toe sock, toe tips CLOSED	With toe socks, toe tips OPEN	With toe socks, toe tips CLOSED
Women	SE215448	SE215449	SE21544FG8	SE21544FG9	SE21544FG28	SE21544FG29
Men	SE225448	SE225449	SE22544FG8	SE22544FG9	SE22544FG28	SE22544FG29
Girls	SE115448	SE115449	SE11544FG8	SE11544FG9	SE11544FG28	SE11544FG29
Boys	SE125448	SE125449	SE12544FG8	SE12544FG9	SE12544FG28	SE12544FG29

Children's models are made for children under 10.



Product picture, code:  
SE20603F1



Product picture, code:  
SE10639F5



Product picture, code:  
SE10629F6

LEG		CUSTOM-MADE		
Model	Product / Art.No			
	Ankle-knee	Knee-thigh	Ankle-thigh	
Adults	SE20601	SE20602	SE20603	
Children	SE10601	SE10602	SE10603	

Children's models are made for children under 10.

**Product information:**

The leg sleeve features an anatomically shaped construction in the knee bend area.  
When the product is ordered in strong fabric, the knee bend construction is made from medium fabric.

Material:	Strong, medium, light (only light pressure)
Colours:	All
Pressure level:	Selectable
Fastening:	Available separately
Details:	Knee-high: upper edge SIL 1
	Upper edge SIL 4, upper edge in knee-high SIL 1, lower edge ELA 2.

STOCKING		CUSTOM-MADE		
Model	Toes	Product / Art.No		
		Ankle-high	Knee-high	Thigh-high
Adults	Closed	SE20619	SE20629	SE20639
Adults	Open	SE20618	SE20628	SE20638
Adults	Toe sock, closed	SE20619FG	SE20629FG	SE20639FG
Adults	Toe sock, open	SE20618FG	SE20628FG	SE20638FG
Children	Closed	SE10619	SE10629	SE10639
Children	Open	SE10618	SE10628	SE10638
Children	Toe sock, closed	SE10619FG	SE10629FG	SE10639FG
Children	Toe sock, open	SE10618FG	SE10628FG	SE10638FG

Children's models are made for children under 10.

**Product information:**

Stockings feature anatomical ankle and knee bend shaping.  
When the stocking is ordered in strong material, the anatomical shapings are made from medium material. In open-toe stockings, the toe is lightly beveled. The ankle sock's shaft length is measured upwards from the L/R61 circumference. Please note that the shaft length can be at most the same as the L/R67 measurement.

Material:	Strong, medium, light (only light pressure)
Colours:	All
Pressure level:	Selectable
Fastening:	Available separately
Details:	Upper edge SIL 1, upper-edge in thigh-high SIL 4, lower edge, tip open ELA 2.



Product picture, code: SE20303F1



Product picture, code: SE2031038F1



Product picture, code: SE2031138F1

**SLEEVE** **CUSTOM-MADE**

Model	Product / Art.No		
	Wrist-elbow	Elbow-axilla	Wrist-axilla
Adults	SE20301	SE20302	SE20303
Children	SE10301	SE10302	SE10303

Children's models are made for children under 10.

**Product information:**

The wrist-to-axilla model features an anatomical elbow bend.

Material:	Strong, medium, light (only light pressure)
Colours:	All
Pressure level:	Selectable
Fastening:	Available separately
Details:	Upper edge SIL 1 Lower edge ELA 2

**SLEEVE GAUNTLET** **CUSTOM-MADE**

Model	Thumb	Product / Art.No	
		To elbow	To axilla
Adults	Closed	SE2031019	SE2031039
Adults	Open	SE2031018	SE2031038
Children	Closed	SE1031019	SE1031039
Children	Open	SE1031018	SE1031038

Children's models are made for children under 10.

**Product information:**

Axilla-length model with anatomical elbow construction. Anatomical construction in the thumb fourchette.

Material:	Strong, medium, light (only light pressure)
Colours:	All
Pressure level:	Selectable
Fastening:	Available separately
Details:	Upper edge SIL 1 Lower edge ELA 2

**SLEEVE GLOVE** **CUSTOM-MADE**

Model	Fingertips	Product / Art.No	
		To elbow	To axilla
Adults	Closed	SE2031119	SE2031139
Adults	Open	SE2031118	SE2031138
Children	Closed	SE1031119	SE1031139
Children	Open	SE1031118	SE1031138

Children's models are made for children under 10.

**Product information:**

Axilla-length model with anatomical elbow construction. Anatomical construction between the fingers and in the thumb fourchette.

Material:	Strong, medium, light (only light pressure)
Colours:	All
Pressure level:	Selectable
Fastening:	Available separately
Details:	Upper edge SIL 1

## SLEEVE FASTENING - MODEL 1 CUSTOM-MADE


Model	Product / Art.No
<b>Model 1 - Shoulder flap</b>	
Adults	A20S1
Children	A10S1

Children's models are made for children under 10.

### Product information:

Elastic strap running under the breast to hold the sleeve up. Fastening with velcro. Suitable especially for users with large breasts. Available for all sleeves reaching up to axilla. Women's sleeve with a bra strap holder.

Material:	ELA 1
Colour:	Same as sleeve

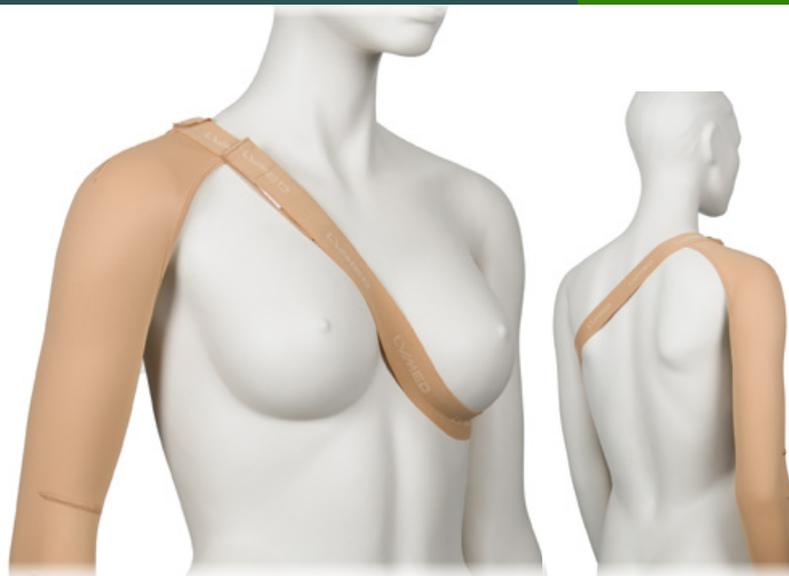

## SLEEVE FASTENING - MODEL 2 CUSTOM-MADE


Model	Product / Art.No
<b>Model 2 - Shoulder flap</b>	
Adults	A20S2
Children	A10S2

Children's models are made for children under 10.

### Product information:

Elastic strap running under the axilla to hold the sleeve up. Fastening with velcro. Available for all sleeves reaching up to axilla.

Material:	ELA 1
Colour:	Same as sleeve

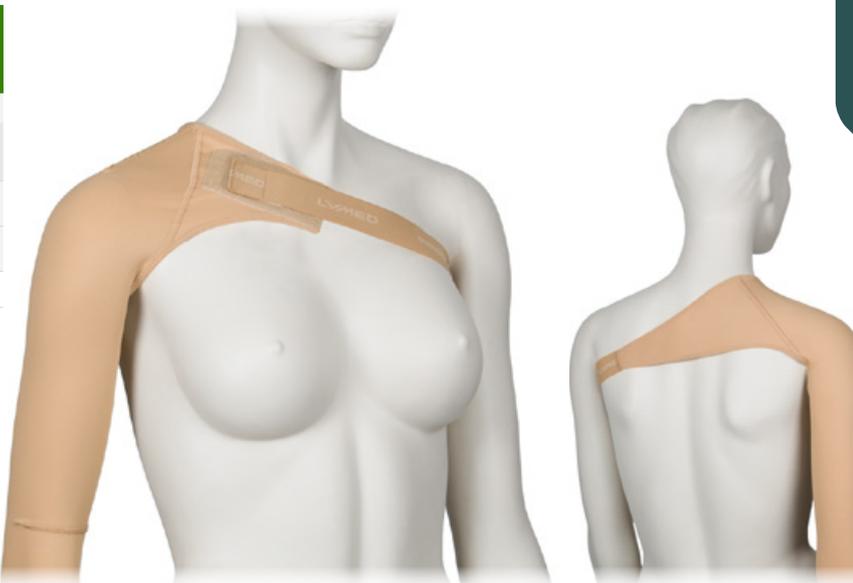

## SLEEVE FASTENING - MODEL 3 CUSTOM-MADE

Model	Product / Art.No
<b>Model 3 - Opposite sleeve</b>	
Adults	A20S3
Children	A10S3


Children's models are made for children under 10.

### Product information:

In the sleeve attachment model, a counter-sleeve connects two axilla-length sleeves. The model is suitable for situations where maintaining the position of one or both sleeves is challenging. The counter-sleeve configuration can be used in different setups: to connect two sleeves or to support the positioning of a single sleeve. Two sleeves must be ordered in conjunction with the attachment. If the ordered sleeves are made from different materials, the attachment component is manufactured by default using the lighter material.

Material:	Strong, medium, light (only light pressure)
Colour:	Same as sleeve
Fastening:	2-row hooks / zipper with guard
Details:	Neckline ELA 3
	Hemline SIL 2
	Edge of sleeves ELA 2

Product pictured, made from medium material

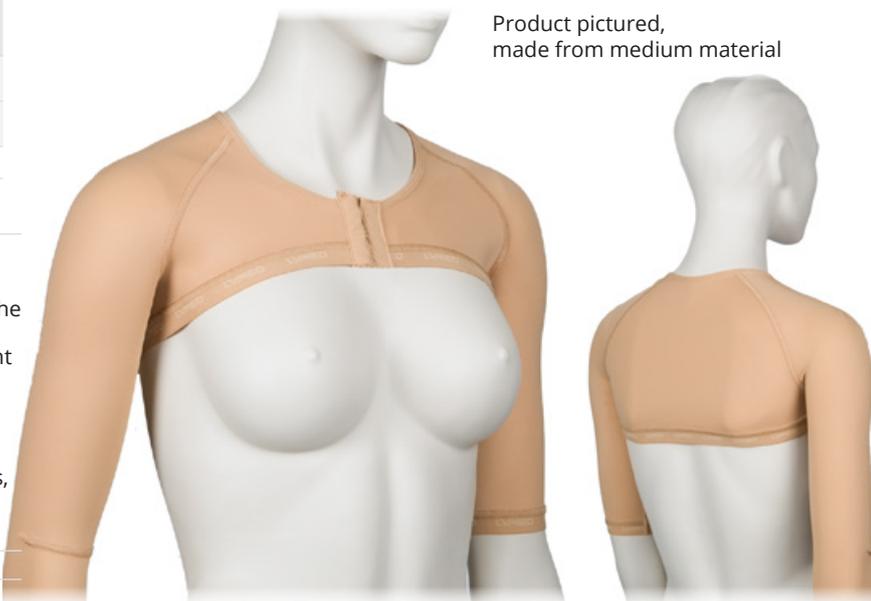

See also vest model on p. 58



Product picture, code:  
SE203119F1



Product picture, code:  
SE203118F1

GLOVE		CUSTOM-MADE
Model	Product / Art.No	
	Fingertips closed	Fingertips open
Adults	SE203119	SE203118
Children	SE103119	SE103118

Children's models are made for children under 10.

**Product information:**

Anatomical construction between the fingers and in the thumb fourchette. Number of backhand seams/darts in glove may vary due to individual measurements and pressure/compression levels.

Order a sleeve-glove when the length of the shaft is greater than the length of the palm.

Material:	Strong, medium, light (only light pressure)
Colours:	All
Pressure level:	Selectable
Fastening:	Available separately
Details:	Wrist edging ELA 3



Product picture, code:  
SE203108F1

GAUNTLET		CUSTOM-MADE
Model	Product / Art.No	
	Thumb closed	Thumb open
Adults	SE203109	SE203108
Children	SE103109	SE103108

Children's models are made for children under 10.

**Product information:**

Anatomical construction in the thumb fourchette. Order a sleeve-glove when the length of the shaft is greater than the length of the palm.

Material:	Strong, medium, light (only light pressure)
Colours:	All
Pressure level:	Selectable
Fastening:	Available separately
Details:	Wrist edging ELA 3 Palm edging ELA 2



Product picture, code: STSE3115LF7

**GLOVE WITH  
SPLINT POCKET**

STANDARD-SIZE

Size	Product / Art.No		Colour code
	Right	Left	
1	STSE3111R	STSE3111L	F1 / F2 / F3 / F4 / F5 / F6 / F7
2	STSE3112R	STSE3112L	F1 / F2 / F3 / F4 / F5 / F6 / F7
3	STSE3113R	STSE3113L	F1 / F2 / F3 / F4 / F5 / F6 / F7
4	STSE3114R	STSE3114L	F1 / F2 / F3 / F4 / F5 / F6 / F7
5	STSE3115R	STSE3115L	F1 / F2 / F3 / F4 / F5 / F6 / F7
6	STSE3116R	STSE3116L	F1 / F2 / F3 / F4 / F5 / F6 / F7
7	STSE3117R	STSE3117L	F1 / F2 / F3 / F4 / F5 / F6 / F7
8	STSE3118R	STSE3118L	F1 / F2 / F3 / F4 / F5 / F6 / F7
9	STSE3119R	STSE3119L	F1 / F2 / F3 / F4 / F5 / F6 / F7
10	STSE3110R	STSE3110L	F1 / F2 / F3 / F4 / F5 / F6 / F7

**GAUNTLET WITH  
SPLINT POCKET**

STANDARD-SIZE

Size	Product / Art.No		Colour code
	Right	Left	
1	STSE3101R	STSE3101L	F1 / F2 / F3 / F4 / F5 / F6 / F7
2	STSE3102R	STSE3102L	F1 / F2 / F3 / F4 / F5 / F6 / F7
3	STSE3103R	STSE3103L	F1 / F2 / F3 / F4 / F5 / F6 / F7
4	STSE3104R	STSE3104L	F1 / F2 / F3 / F4 / F5 / F6 / F7
5	STSE3105R	STSE3105L	F1 / F2 / F3 / F4 / F5 / F6 / F7
6	STSE3106R	STSE3106L	F1 / F2 / F3 / F4 / F5 / F6 / F7
7	STSE3107R	STSE3107L	F1 / F2 / F3 / F4 / F5 / F6 / F7
8	STSE3108R	STSE3108L	F1 / F2 / F3 / F4 / F5 / F6 / F7
9	STSE3109R	STSE3109L	F1 / F2 / F3 / F4 / F5 / F6 / F7
10	STSE31010R	STSE31010L	F1 / F2 / F3 / F4 / F5 / F6 / F7

**Product information:**

Sense glove and gauntlet include a zipper and a pocket for optional splint or other reinforcements. The product is used in connection with different tonus deficiencies or hemiplegia.

Glove with open fingertips. Gauntlet with open thumb tip.

Material:	Strong
Colours:	All
Pressure level:	14 - 20 mmHg
Fastening:	Zipper
Details:	Wrist edging ELA 3

Sizes	Measurements (cm)			
	MCP	L11	L16	L15
1	11	10,5	14	6
2	12	11	14,5	6
3	13	11,5	15,5	8,5
4	14	12	16	8,5
5	15	12,5	16,5	9
6	16	13,5	17	9
7	17	14,5	17,5	10
8	18	16	18,5	11
9	19	16,5	20	11
10	20	17,5	23	12

Helmets and chin straps available, please see details from Lymed Skin product group pages 54-55.



Product picture, code: STSE3133RF1

CI GLOVE		STANDARD-SIZE	
Size	Circumf.*	Product / Art.No	
	cm	Right	Left
1	12,0 - 13,5	STSE3131RF1	STSE3131LF1
2	13,5 - 15,0	STSE3132RF1	STSE3132LF1
3	15,0 - 16,5	STSE3133RF1	STSE3133LF1
4	16,5 - 18,0	STSE3134RF1	STSE3134LF1
5	18,0 - 19,5	STSE3135RF1	STSE3135LF1

**Product information:**

Used in constraint-included movement therapy, for the healthy hand.

\*Circumference at MCP

Material:	Strong
Colour:	Beige
Pressure level:	No pressure
Details:	Wrist edging ELA 3

# INTERIM & LIGHT™

INTERIM CARE AND LIGHT PRESSURE GARMENTS

STANDARD-SIZE





**Product information:**

Short or long sleeved model with long hemline reaching down to the pelvis.

*\*Circumference of chest under axillae, above breasts.*

Material:	Light
Colours:	Beige, black
Pressure level:	10 - 18 mmHg
Fastening:	2-row hooks / zipper with guard
Details:	Neckline ELA 3, hemline SIL 1, edge of sleeve ELA 2

Product picture, code: STIN224226F1H

**MEN'S VEST - WITH HOOKS** STANDARD-SIZE

Size	Circumf.* cm	Colour	Product / Art.No	
			Short sleeves	Long sleeves
1	81 - 84	Beige	STIN224221F1H	STIN224331F1H
1	81 - 84	Black	STIN224221F2H	STIN224331F2H
2	85 - 88	Beige	STIN224222F1H	STIN224332F1H
2	85 - 88	Black	STIN224222F2H	STIN224332F2H
3	89 - 92	Beige	STIN224223F1H	STIN224333F1H
3	89 - 92	Black	STIN224223F2H	STIN224333F2H
4	93 - 96	Beige	STIN224224F1H	STIN224334F1H
4	93 - 96	Black	STIN224224F2H	STIN224334F2H
5	97 - 100	Beige	STIN224225F1H	STIN224335F1H
5	97 - 100	Black	STIN224225F2H	STIN224335F2H
6	101 - 104	Beige	STIN224226F1H	STIN224336F1H
6	101 - 104	Black	STIN224226F2H	STIN224336F2H
7	105 - 108	Beige	STIN224227F1H	STIN224337F1H
7	105 - 108	Black	STIN224227F2H	STIN224337F2H
8	109 - 112	Beige	STIN224228F1H	STIN224338F1H
8	109 - 112	Black	STIN224228F2H	STIN224338F2H
9	113 - 116	Beige	STIN224229F1H	STIN224339F1H
9	113 - 116	Black	STIN224229F2H	STIN224339F2H
10	117 - 120	Beige	STIN2242210F1H	STIN2243310F1H
10	117 - 120	Black	STIN2242210F2H	STIN2243310F2H

**MEN'S VEST - WITH ZIPPER** STANDARD-SIZE

Size	Circumf.* cm	Colour	Product / Art.No	
			Short sleeves	Long sleeves
1	81 - 84	Beige	STIN224221F1Z	STIN224331F1Z
1	81 - 84	Black	STIN224221F2Z	STIN224331F2Z
2	85 - 88	Beige	STIN224222F1Z	STIN224332F1Z
2	85 - 88	Black	STIN224222F2Z	STIN224332F2Z
3	89 - 92	Beige	STIN224223F1Z	STIN224333F1Z
3	89 - 92	Black	STIN224223F2Z	STIN224333F2Z
4	93 - 96	Beige	STIN224224F1Z	STIN224334F1Z
4	93 - 96	Black	STIN224224F2Z	STIN224334F2Z
5	97 - 100	Beige	STIN224225F1Z	STIN224335F1Z
5	97 - 100	Black	STIN224225F2Z	STIN224335F2Z
6	101 - 104	Beige	STIN224226F1Z	STIN224336F1Z
6	101 - 104	Black	STIN224226F2Z	STIN224336F2Z
7	105 - 108	Beige	STIN224227F1Z	STIN224337F1Z
7	105 - 108	Black	STIN224227F2Z	STIN224337F2Z
8	109 - 112	Beige	STIN224228F1Z	STIN224338F1Z
8	109 - 112	Black	STIN224228F2Z	STIN224338F2Z
9	113 - 116	Beige	STIN224229F1Z	STIN224339F1Z
9	113 - 116	Black	STIN224229F2Z	STIN224339F2Z
10	117 - 120	Beige	STIN2242210F1Z	STIN2243310F1Z
10	117 - 120	Black	STIN2242210F2Z	STIN2243310F2Z



### Product information:

Short or long sleeved model with long hemline reaching down to the pelvis. Women's vest with anatomical breast construction.

\*Circumference of chest under axillae, above breasts.

Material:	Light
Colours:	Beige, black
Pressure level:	10 - 18 mmHg
Fastening:	2-row hooks / zipper with guard
Details:	Neckline ELA 3, hemline SIL 1, edge of sleeve ELA 2

Product picture, code: STIN214226F1H

## WOMEN'S VEST - WITH HOOKS STANDARD-SIZE

Size	Circumf.* cm	Colour	Product / Art.No	
			Short sleeves	Long sleeves
1	76 - 80	Beige	STIN214221F1H	STIN214331F1H
1	76 - 80	Black	STIN214221F2H	STIN214331F2H
2	81 - 84	Beige	STIN214222F1H	STIN214332F1H
2	81 - 84	Black	STIN214222F2H	STIN214332F2H
3	85 - 88	Beige	STIN214223F1H	STIN214333F1H
3	85 - 88	Black	STIN214223F2H	STIN214333F2H
4	89 - 92	Beige	STIN214224F1H	STIN214334F1H
4	89 - 92	Black	STIN214224F2H	STIN214334F2H
5	93 - 96	Beige	STIN214225F1H	STIN214335F1H
5	93 - 96	Black	STIN214225F2H	STIN214335F2H
6	97 - 100	Beige	STIN214226F1H	STIN214336F1H
6	97 - 100	Black	STIN214226F2H	STIN214336F2H
7	101 - 104	Beige	STIN214227F1H	STIN214337F1H
7	101 - 104	Black	STIN214227F2H	STIN214337F2H
8	105 - 108	Beige	STIN214228F1H	STIN214338F1H
8	105 - 108	Black	STIN214228F2H	STIN214338F2H
9	109 - 112	Beige	STIN214229F1H	STIN214339F1H
9	109 - 112	Black	STIN214229F2H	STIN214339F2H
10	113 - 116	Beige	STIN2142210F1H	STIN2143310F1H
10	113 - 116	Black	STIN2142210F2H	STIN2143310F2H
11	117 - 119	Beige	STIN2142211F1H	STIN2143311F1H
11	117 - 119	Black	STIN2142211F2H	STIN2143311F2H

## WOMEN'S VEST - WITH ZIPPER STANDARD-SIZE

Size	Circumf.* cm	Colour	Product / Art.No	
			Short sleeves	Long sleeves
1	76 - 80	Beige	STIN214221F1Z	STIN214331F1Z
1	76 - 80	Black	STIN214221F2Z	STIN214331F2Z
2	81 - 84	Beige	STIN214222F1Z	STIN214332F1Z
2	81 - 84	Black	STIN214222F2Z	STIN214332F2Z
3	85 - 88	Beige	STIN214223F1Z	STIN214333F1Z
3	85 - 88	Black	STIN214223F2Z	STIN214333F2Z
4	89 - 92	Beige	STIN214224F1Z	STIN214334F1Z
4	89 - 92	Black	STIN214224F2Z	STIN214334F2Z
5	93 - 96	Beige	STIN214225F1Z	STIN214335F1Z
5	93 - 96	Black	STIN214225F2Z	STIN214335F2Z
6	97 - 100	Beige	STIN214226F1Z	STIN214336F1Z
6	97 - 100	Black	STIN214226F2Z	STIN214336F2Z
7	101 - 104	Beige	STIN214227F1Z	STIN214337F1Z
7	101 - 104	Black	STIN214227F2Z	STIN214337F2Z
8	105 - 108	Beige	STIN214228F1Z	STIN214338F1Z
8	105 - 108	Black	STIN214228F2Z	STIN214338F2Z
9	109 - 112	Beige	STIN214229F1Z	STIN214339F1Z
9	109 - 112	Black	STIN214229F2Z	STIN214339F2Z
10	113 - 116	Beige	STIN2142210F1Z	STIN2143310F1Z
10	113 - 116	Black	STIN2142210F2Z	STIN2143310F2Z
11	117 - 119	Beige	STIN2142211F1Z	STIN2143311F1Z
11	117 - 119	Black	STIN2142211F2Z	STIN2143311F2Z



**Product information:**

Short or long sleeved model with long hemline reaching down to the pelvis.

\*Circumference of chest under axillae, above breasts.

Material:	Light
Colours:	Beige, black
Pressure level:	10 - 18 mmHg
Fastening:	2-row hooks / zipper with guard
Details:	Neckline ELA 3, hemline SIL 1, edge of sleeve ELA 3

Product picture, code: STIN104226F1H

**CHILDREN'S VEST - WITH HOOKS** STANDARD-SIZE

Size	Circumf.* cm	Colour	Product / Art.No	
			Short sleeves	Long sleeves
1	43 - 45	Beige	STIN104221FH	STIN104331F1H
1	43 - 45	Black	STIN104221F2H	STIN104331F2H
2	46 - 49	Beige	STIN104222F1H	STIN104332F1H
2	46 - 49	Black	STIN104222F2H	STIN104332F2H
3	50 - 54	Beige	STIN104223F1H	STIN104333F1H
3	50 - 54	Black	STIN104223F2H	STIN104333F2H
4	55 - 59	Beige	STIN104224F1H	STIN104334F1H
4	55 - 59	Black	STIN104224F2H	STIN104334F2H
5	60 - 63	Beige	STIN104225F1H	STIN104335F1H
5	60 - 63	Black	STIN104225F2H	STIN104335F2H
6	64 - 67	Beige	STIN104226F1H	STIN104336F1H
6	64 - 67	Black	STIN104226F2H	STIN104336F2H
7	68 - 71	Beige	STIN104227F1H	STIN104337F1H
7	68 - 71	Black	STIN104227F2H	STIN104337F2H
8	72 - 75	Beige	STIN104228F1H	STIN104338F1H
8	72 - 75	Black	STIN104228F2H	STIN104338F2H
9	76 - 79	Beige	STIN104229F1H	STIN104339F1H
9	76 - 79	Black	STIN104229F2H	STIN104339F2H
10	80 - 83	Beige	STIN1042210F1H	STIN1043310F1H
10	80 - 83	Black	STIN1042210F2H	STIN1043310F2H

**CHILDREN'S VEST - WITH ZIPPER** STANDARD-SIZE

Size	Circumf.* cm	Colour	Product / Art.No	
			Short sleeves	Long sleeves
1	43 - 45	Beige	STIN104221F1Z	STIN104331F1Z
1	43 - 45	Black	STIN104221F2Z	STIN104331F2Z
2	46 - 49	Beige	STIN104222F1Z	STIN104332F1Z
2	46 - 49	Black	STIN104222F2Z	STIN104332F2Z
3	50 - 54	Beige	STIN104223F1Z	STIN104333F1Z
3	50 - 54	Black	STIN104223F2Z	STIN104333F2Z
4	55 - 59	Beige	STIN104224F1Z	STIN104334F1Z
4	55 - 59	Black	STIN104224F2Z	STIN104334F2Z
5	60 - 63	Beige	STIN104225F1Z	STIN104335F1Z
5	60 - 63	Black	STIN104225F2Z	STIN104335F2Z
6	64 - 67	Beige	STIN104226F1Z	STIN104336F1Z
6	64 - 67	Black	STIN104226F2Z	STIN104336F2Z
7	68 - 71	Beige	STIN104227F1Z	STIN104337F1Z
7	68 - 71	Black	STIN104227F2Z	STIN104337F2Z
8	72 - 75	Beige	STIN104228F1Z	STIN104338F1Z
8	72 - 75	Black	STIN104228F2Z	STIN104338F2Z
9	76 - 79	Beige	STIN104229F1Z	STIN104339F1Z
9	76 - 79	Black	STIN104229F2Z	STIN104339F2Z
10	80 - 83	Beige	STIN1042210F1Z	STIN1043310F1Z
10	80 - 83	Black	STIN1042210F2Z	STIN1043310F2Z



Product picture, code: STIN225336F1

### Product information:

Normal waist. Men's models with contouring in the front.

\*Circumference at widest point of pelvis.

Material:	Light
Colours:	Beige, black
Pressure level:	10 - 18 mmHg
Details:	Waist: ELA 1, Leg opening ELA 2, Children's pants leg opening ELA3

## MEN'S PANTS STANDARD-SIZE

Size	Circumf.* cm	Colour	Product / Art.No	
			Short legs	Long legs
1	87 - 92	Beige	STIN225221F1	STIN225331F1
1	87 - 92	Black	STIN225221F2	STIN225331F2
2	93 - 96	Beige	STIN225222F1	STIN225332F1
2	93 - 96	Black	STIN225222F2	STIN225332F2
3	97 - 100	Beige	STIN225223F1	STIN225333F1
3	97 - 100	Black	STIN225223F2	STIN225333F2
4	101 - 104	Beige	STIN225224F1	STIN225334F1
4	101 - 104	Black	STIN225224F2	STIN225334F2
5	105 - 108	Beige	STIN225225F1	STIN225335F1
5	105 - 108	Black	STIN225225F2	STIN225335F2
6	109 - 112	Beige	STIN225226F1	STIN225336F1
6	109 - 112	Black	STIN225226F2	STIN225336F2
7	113 - 116	Beige	STIN225227F1	STIN225337F1
7	113 - 116	Black	STIN225227F2	STIN225337F2
8	117 - 120	Beige	STIN225228F1	STIN225338F1
8	117 - 120	Black	STIN225228F2	STIN225338F2
9	121 - 124	Beige	STIN225229F1	STIN225339F1
9	121 - 124	Black	STIN225229F2	STIN225339F2
10	125 - 128	Beige	STIN2252210F1	STIN2253310F1
10	125 - 128	Black	STIN2252210F2	STIN2253310F2

## WOMEN'S PANTS STANDARD-SIZE

Size	Circumf.* cm	Colour	Product / Art.No	
			Short legs	Long legs
1	87 - 91	Beige	STIN215221F1	STIN215331F1
1	87 - 91	Black	STIN215221F2	STIN215331F2
2	92 - 95	Beige	STIN215222F1	STIN215332F1
2	92 - 95	Black	STIN215222F2	STIN215332F2
3	96 - 99	Beige	STIN215223F1	STIN215333F1
3	96 - 99	Black	STIN215223F2	STIN215333F2
4	100 - 103	Beige	STIN215224F1	STIN215334F1
4	100 - 103	Black	STIN215224F2	STIN215334F2
5	104 - 107	Beige	STIN215225F1	STIN215335F1
5	104 - 107	Black	STIN215225F2	STIN215335F2
6	108 - 111	Beige	STIN215226F1	STIN215336F1
6	108 - 111	Black	STIN215226F2	STIN215336F2
7	112 - 115	Beige	STIN215227F1	STIN215337F1
7	112 - 115	Black	STIN215227F2	STIN215337F2
8	116 - 119	Beige	STIN215228F1	STIN215338F1
8	116 - 119	Black	STIN215228F2	STIN215338F2
9	120 - 123	Beige	STIN215229F1	STIN215339F1
9	120 - 123	Black	STIN215229F2	STIN215339F2
10	124 - 127	Beige	STIN2152210F1	STIN2153310F1
10	124 - 127	Black	STIN2152210F2	STIN2153310F2



Product picture, code: STIN215336F1



Product picture, code: STIN105336F1

**CHILDREN'S PANTS**

STANDARD-SIZE

Size	Circumf.* cm	Colour	Product / Art.No	
			Short legs	Long legs
1	47 - 50	Beige	STIN105221F1	STIN105331F1
1	47 - 50	Black	STIN105221F2	STIN105331F2
2	51 - 54	Beige	STIN105222F1	STIN105332F1
2	51 - 54	Black	STIN105222F2	STIN105332F2
3	55 - 58	Beige	STIN105223F1	STIN105333F1
3	55 - 58	Black	STIN105223F2	STIN105333F2
4	59 - 61	Beige	STIN105224F1	STIN105334F1
4	59 - 61	Black	STIN105224F2	STIN105334F2
5	62 - 66	Beige	STIN105225F1	STIN105335F1
5	62 - 66	Black	STIN105225F2	STIN105335F2
6	67 - 70	Beige	STIN105226F1	STIN105336F1
6	67 - 70	Black	STIN105226F2	STIN105336F2
7	71 - 74	Beige	STIN105227F1	STIN105337F1
7	71 - 74	Black	STIN105227F2	STIN105337F2
8	75 - 78	Beige	STIN105228F1	STIN105338F1
8	75 - 78	Black	STIN105228F2	STIN105338F2
9	79 - 82	Beige	STIN105229F1	STIN105339F1
9	79 - 82	Black	STIN105229F2	STIN105339F2
10	83 - 86	Beige	STIN1052210F1	STIN1053310F1
10	83 - 86	Black	STIN1052210F2	STIN1053310F2



Product picture, code:  
STIN2031194RF1



Product picture, code:  
STIN2031184RF1

**GLOVE**

STANDARD-SIZE

**Product information:**

Anatomical construction between the fingers.

\*Circumference at MCP.

Material:	Light
Colours:	Beige, black
Pressure level:	10 - 18 mmHg
Details:	Wrist edging ELA 3

ADULT'S GLOVE							STANDARD-SIZE
Size	Circumf.*	Colour	Product / Art. No				
	cm		Fingertips closed right	Fingertips closed left	Fingertips open right	Fingertips open left	
1	13 - 14	Beige	STIN2031191RF1	STIN2031191LF1	STIN2031181RF1	STIN2031181LF1	
1	13 - 14	Black	STIN2031191RF2	STIN2031191LF2	STIN2031181RF2	STIN2031181LF2	
2	15 - 16	Beige	STIN2031192RF1	STIN2031192LF1	STIN2031182RF1	STIN2031182LF1	
2	15 - 16	Black	STIN2031192RF2	STIN2031192LF2	STIN2031182RF2	STIN2031182LF2	
3	17 - 18	Beige	STIN2031193RF1	STIN2031193LF1	STIN2031183RF1	STIN2031183LF1	
3	17 - 18	Black	STIN2031193RF2	STIN2031193LF2	STIN2031183RF2	STIN2031183LF2	
4	19 - 20	Beige	STIN2031194RF1	STIN2031194LF1	STIN2031184RF1	STIN2031184LF1	
4	19 - 20	Black	STIN2031194RF2	STIN2031194LF2	STIN2031184RF2	STIN2031184LF2	
5	21 - 22	Beige	STIN2031195RF1	STIN2031195LF1	STIN2031185RF1	STIN2031185LF1	
5	21 - 22	Black	STIN2031195RF2	STIN2031195LF2	STIN2031185RF2	STIN2031185LF2	
6	23 - 24	Beige	STIN2031196RF1	STIN2031196LF1	STIN2031186RF1	STIN2031186LF1	
6	23 - 24	Black	STIN2031196RF2	STIN2031196LF2	STIN2031186RF2	STIN2031186LF2	
7	25 - 26	Beige	STIN2031197RF1	STIN2031197LF1	STIN2031187RF1	STIN2031187LF1	
7	25 - 26	Black	STIN2031197RF2	STIN2031197LF2	STIN2031187RF2	STIN2031187LF2	
8	27 - 28	Beige	STIN2031198RF1	STIN2031198LF1	STIN2031188RF1	STIN2031188LF1	
8	27 - 28	Black	STIN2031198RF2	STIN2031198LF2	STIN2031188RF2	STIN2031188LF2	

CHILDREN'S GLOVE							STANDARD-SIZE
Size	Circumf.*	Colour	Product / Art. No				
	cm		Fingertips closed right	Fingertips closed left	Fingertips open right	Fingertips open left	
1	10 - 11	Beige	STIN1031191RF1	STIN1031191LF1	STIN1031181RF1	STIN1031181LF1	
1	10 - 11	Black	STIN1031191RF2	STIN1031191LF2	STIN1031181RF2	STIN1031181LF2	
2	12 - 13	Beige	STIN1031192RF1	STIN1031192LF1	STIN1031182RF1	STIN1031182LF1	
2	12 - 13	Black	STIN1031192RF2	STIN1031192LF2	STIN1031182RF2	STIN1031182LF2	



ADULT'S SLEEVE			STANDARD-SIZE
Size	Circumf.* cm	Colour	Art. No
1	18 - 20	Beige	STIN20301F1
1	18 - 20	Black	STIN20301F2
2	20 - 22	Beige	STIN20302F1
2	20 - 22	Black	STIN20302F2
3	22 - 24	Beige	STIN20303F1
3	22 - 24	Black	STIN20303F2
4	24 - 26	Beige	STIN20304F1
4	24 - 26	Black	STIN20304F2
5	26 - 28	Beige	STIN20305F1
5	26 - 28	Black	STIN20305F2
6	28 - 30	Beige	STIN20306F1
6	28 - 30	Black	STIN20306F2
7	30 - 32	Beige	STIN20307F1
7	30 - 32	Black	STIN20307F2
8	32 - 34	Beige	STIN20308F1
8	32 - 34	Black	STIN20308F2
9	34 - 36	Beige	STIN20309F1
9	34 - 36	Black	STIN20309F2
10	36 - 38	Beige	STIN203010F1
10	36 - 38	Black	STIN203010F2
11	38 - 40	Beige	STIN203011F1
11	38 - 40	Black	STIN203011F2

**Product information:**

Sleeve length in size 6: 43,5 cm.  
Wrist-axilla.  
*\*Circumference at widest point of upper arm.*

Material:	Light
Colours:	Beige, black
Pressure level:	10 - 18 mmHg
Details:	Upper edge SIL 2 Lower edge ELA 2

Product picture, code: STIN20305F1

CHILDREN'S SLEEVE			STANDARD-SIZE
Size	Circumf.* cm	Colour	Art. No
1	10 - 12	Beige	STIN10301F1
1	10 - 12	Black	STIN10301F2
2	12 - 14	Beige	STIN10302F1
2	12 - 14	Black	STIN10302F2
3	14 - 16	Beige	STIN10303F1
3	14 - 16	Black	STIN10303F2
4	16 - 18	Beige	STIN10304F1
4	16 - 18	Black	STIN10304F2
5	18 - 20	Beige	STIN10305F1
5	18 - 20	Black	STIN10305F2

**Product information:**

Sleeve length in size 3: 26,5 cm.  
Wrist-axilla.  
*\*Circumference at widest point of upper arm.*

Material:	Light
Colours:	Beige, black
Pressure level:	10 - 18 mmHg
Details:	Upper edge SIL 2 Lower edge ELA 3



Product picture, code:  
STIN20692F1

Product picture, code:  
STIN20682F1

## ADULT'S STOCKING

STANDARD-SIZE

Size	Circumf.* cm	Shoe size EU	Colour	Product / Art.No Toes closed
1	28 - 32	35 - 38	Beige	STIN20691F1
1	28 - 32	35 - 38	Black	STIN20691F2
2	33 - 37	38 - 40	Beige	STIN20692F1
2	33 - 37	38 - 40	Black	STIN20692F2
3	38 - 42	40 - 43	Beige	STIN20693F1
3	38 - 42	40 - 43	Black	STIN20693F2
4	43 - 46	43 - 46	Beige	STIN20694F1
4	43 - 46	43 - 46	Black	STIN20694F2
cm				Toes open
1	28 - 32		Beige	STIN20681F1
1	28 - 32		Black	STIN20681F2
2	33 - 37		Beige	STIN20682F1
2	33 - 37		Black	STIN20682F2
3	38 - 42		Beige	STIN20683F1
3	38 - 42		Black	STIN20683F2
4	43 - 46		Beige	STIN20684F1
4	43 - 46		Black	STIN20684F2

### Product information:

Stocking that fits both left and right foot.  
\*Circumference of widest point of calf.

Material:	Light
Colours:	Beige, black
Pressure level:	10 - 18 mmHg
Details:	Upper edge SIL 2 Edging, tip open ELA 2

## CHILDREN'S STOCKING

STANDARD-SIZE

Size	Circumf.* cm	Shoe size EU	Colour	Product / Art.No Toes closed
1	16 - 19	23 - 26	Beige	STIN10691F1
1	16 - 19	23 - 26	Black	STIN10691F2
2	20 - 23	26 - 31	Beige	STIN10692F1
2	20 - 23	26 - 31	Black	STIN10692F2
3	24 - 27	31 - 36	Beige	STIN10693F1
3	24 - 27	31 - 36	Black	STIN10693F2
cm				Toes open
1	16 - 19		Beige	STIN10681F1
1	16 - 19		Black	STIN10681F2
2	20 - 23		Beige	STIN10682F1
2	20 - 23		Black	STIN10682F2
3	24 - 27		Beige	STIN10683F1
3	24 - 27		Black	STIN10683F2

### Product information:

Stocking that fits both left and right foot.  
\*Circumference of widest point of calf.

Material:	Light
Colours:	Beige, black
Pressure level:	10 - 18 mmHg
Details:	Upper edge SIL 2 Edging, tip open ELA 3

**Product information:**

\*Circumference of the head.

Material:	Light
Colours:	Beige, black
Pressure level:	10 - 18 mmHg
Fastening:	Velcro in the back
Details:	Edges: overlocking stitch

**CHIN STRAP**

STANDARD-SIZE

Size	Circumf.* cm	Colour	Product / Art. No
1	47 - 50	Beige	STIN1381F1
1	47 - 50	Black	STIN1381F2
2	51 - 53	Beige	STIN1382F1
2	51 - 53	Black	STIN1382F2
3	54 - 56	Beige	STIN1383F1
3	54 - 56	Black	STIN1383F2
4	57 - 59	Beige	STIN1384F1
4	57 - 59	Black	STIN1384F2
5	60 - 62	Beige	STIN1385F1
5	60 - 62	Black	STIN1385F2

**HELMET**

STANDARD-SIZE

Size	Circumf.* cm	Colour	Product / Art. No
1	47 - 50	Beige	STIN1391F1
1	47 - 50	Black	STIN1391F2
2	51 - 53	Beige	STIN1392F1
2	51 - 53	Black	STIN1392F2
3	54 - 56	Beige	STIN1393F1
3	54 - 56	Black	STIN1393F2
4	57 - 59	Beige	STIN1394F1
4	57 - 59	Black	STIN1394F2
5	60 - 62	Beige	STIN1395F1
5	60 - 62	Black	STIN1395F2

**Product information:**

Features a closed face area. The closed-face model is designed to keep scar and wound care products, dressings, or a facial mask securely in place. If necessary, openings can be cut into the face area for the mouth, nose, and eyes. The material does not fray after cutting.

\*Circumference of the head.

Material:	Light
Colours:	Beige, black
Pressure level:	10 - 18 mmHg
Fastening:	Occipital hooks
Details:	Edges: overlocking stitch



Product picture, code: STIN1383F1



Product picture, code: STIN1393F1



# POST-OPERATION™

POST-OPERATIONAL SUPPORT GARMENTS

STANDARD-SIZE



Product picture, code:  
STPOP21535HWF2C




Product picture, code:  
STPOP21525HWF2C



Product picture, code:  
STPOP21515HWF2C



Product picture, code:  
STPOP21505HWF2C

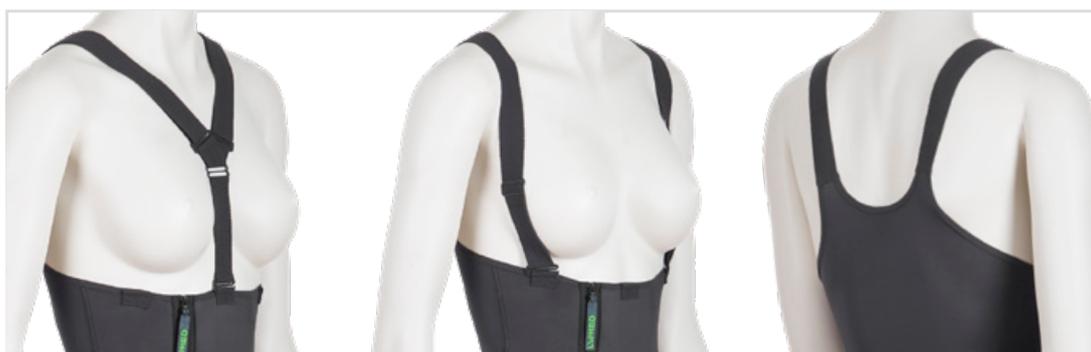

Ways to wear shoulder straps

**PANTS FOR WOMEN | HIGH WAIST**

STANDARD-SIZE

**Product information:**

Women's support pants intended for use in connection with mid- and lower-body surgical procedures. Also suitable for use after childbirth. The pants feature a high waist and double-layer fabric in the abdominal area. Elastic and adjustable suspenders with front opening. An additional component is included, allowing the suspenders to be configured in a Y-shape. \*Circumference at widest point of pelvis and upper thigh.

Material:	Medium
Colour:	Beige, black
Pressure level:	23 - 30 mmHg
Fastening:	Zipper with guard and hooks on the side of pelvis.
Details:	Waist ELA 3
	Leg opening ELA 2

**PANTS FOR WOMEN | HIGH WAIST | GENITAL AREA CLOSED**
**STANDARD-SIZE**

Size	Circumf.*		Colour	Product / Art. No			
	Pelvis cm	Thigh cm		Ankle length	Calf length	Knee length	Short length
XS	86 - 90	50,0 - 53,0	Beige	STPOP21532HWF1C	STPOP21522HWF1C	STPOP21512HWF1C	STPOP21502HWF1C
XS	86 - 90	50,0 - 53,0	Black	STPOP21532HWF2C	STPOP21522HWF2C	STPOP21512HWF2C	STPOP21502HWF2C
S	91 - 95	53,5 - 56,0	Beige	STPOP21533HWF1C	STPOP21523HWF1C	STPOP21513HWF1C	STPOP21503HWF1C
S	91 - 95	53,5 - 56,0	Black	STPOP21533HWF2C	STPOP21523HWF2C	STPOP21513HWF2C	STPOP21503HWF2C
M	96 - 101	56,5 - 59,5	Beige	STPOP21534HWF1C	STPOP21524HWF1C	STPOP21514HWF1C	STPOP21504HWF1C
M	96 - 101	56,5 - 59,5	Black	STPOP21534HWF2C	STPOP21524HWF2C	STPOP21514HWF2C	STPOP21504HWF2C
L	102 - 110	60,0 - 62,5	Beige	STPOP21535HWF1C	STPOP21525HWF1C	STPOP21515HWF1C	STPOP21505HWF1C
L	102 - 110	60,0 - 62,5	Black	STPOP21535HWF2C	STPOP21525HWF2C	STPOP21515HWF2C	STPOP21505HWF2C
XL	111 - 121	63,0 - 66,0	Beige	STPOP21536HWF1C	STPOP21526HWF1C	STPOP21516HWF1C	STPOP21506HWF1C
XL	111 - 121	63,0 - 66,0	Black	STPOP21536HWF2C	STPOP21526HWF2C	STPOP21516HWF2C	STPOP21506HWF2C
XXL	122 - 132	66,5 - 69,0	Beige	STPOP21537HWF1C	STPOP21527HWF1C	STPOP21517HWF1C	STPOP21507HWF1C
XXL	122 - 132	66,5 - 69,0	Black	STPOP21537HWF2C	STPOP21527HWF2C	STPOP21517HWF2C	STPOP21507HWF2C
XXXL	133 - 138	69,5 - 71,5	Beige	STPOP21538HWF1C	STPOP21528HWF1C	STPOP21518HWF1C	STPOP21508HWF1C
XXXL	133 - 138	69,5 - 71,5	Black	STPOP21538HWF2C	STPOP21528HWF2C	STPOP21518HWF2C	STPOP21508HWF2C
XXXXL	139 - 144	72,0 - 74,5	Beige	STPOP21539HWF1C	STPOP21529HWF1C	STPOP21519HWF1C	STPOP21509HWF1C
XXXXL	139 - 144	72,0 - 74,5	Black	STPOP21539HWF2C	STPOP21529HWF2C	STPOP21519HWF2C	STPOP21509HWF2C

**PANTS FOR WOMEN | HIGH WAIST | GENITAL AREA OPEN**
**STANDARD-SIZE**

Size	Circumf.*		Colour	Product / Art. No			
	Pelvis cm	Thigh cm		Ankle length	Calf length	Knee length	Short length
XS	86 - 90	50,0 - 53,0	Beige	STPOP21532HWF1O	STPOP21522HWF1O	STPOP21512HWF1O	STPOP21502HWF1C
XS	86 - 90	50,0 - 53,0	Black	STPOP21532HWF2O	STPOP21522HWF2O	STPOP21512HWF2O	STPOP21502HWF2O
S	91 - 95	53,5 - 56,0	Beige	STPOP21533HWF1O	STPOP21523HWF1O	STPOP21513HWF1O	STPOP21503HWF1O
S	91 - 95	53,5 - 56,0	Black	STPOP21533HWF2O	STPOP21523HWF2O	STPOP21513HWF2O	STPOP21503HWF2O
M	96 - 101	56,5 - 59,5	Beige	STPOP21534HWF1O	STPOP21524HWF1O	STPOP21514HWF1O	STPOP21504HWF1O
M	96 - 101	56,5 - 59,5	Black	STPOP21534HWF2O	STPOP21524HWF2O	STPOP21514HWF2O	STPOP21504HWF2O
L	102 - 110	60,0 - 62,5	Beige	STPOP21535HWF1O	STPOP21525HWF1O	STPOP21515HWF1O	STPOP21505HWF1O
L	102 - 110	60,0 - 62,5	Black	STPOP21535HWF2O	STPOP21525HWF2O	STPOP21515HWF2O	STPOP21505HWF2O
XL	111 - 121	63,0 - 66,0	Beige	STPOP21536HWF1O	STPOP21526HWF1O	STPOP21516HWF1O	STPOP21506HWF1O
XL	111 - 121	63,0 - 66,0	Black	STPOP21536HWF2O	STPOP21526HWF2O	STPOP21516HWF2O	STPOP21506HWF2O
XXL	122 - 132	66,5 - 69,0	Beige	STPOP21537HWF1O	STPOP21527HWF1O	STPOP21517HWF1O	STPOP21507HWF1O
XXL	122 - 132	66,5 - 69,0	Black	STPOP21537HWF2O	STPOP21527HWF2O	STPOP21517HWF2O	STPOP21507HWF2O
XXXL	133 - 138	69,5 - 71,5	Beige	STPOP21538HWF1O	STPOP21528HWF1O	STPOP21518HWF1O	STPOP21508HWF1O
XXXL	133 - 138	69,5 - 71,5	Black	STPOP21538HWF2O	STPOP21528HWF2O	STPOP21518HWF2O	STPOP21508HWF2O
XXXXL	139 - 144	72,0 - 74,5	Beige	STPOP21539HWF1O	STPOP21529HWF1O	STPOP21519HWF1O	STPOP21509HWF1O
XXXXL	139 - 144	72,0 - 74,5	Black	STPOP21539HWF2O	STPOP21529HWF2O	STPOP21519HWF2O	STPOP21509HWF2O

### PANTS FOR WOMEN | NORMAL WAIST

STANDARD-SIZE

#### Product information:

Women's support pants intended for use in connection with mid- and lower-body surgical procedures. The pants feature a normal waist. Available with the genital area open or closed. Zipper on the right side.

\*Circumference at widest point of pelvis and upper thigh.

Material:	Medium
Colour:	Beige, black
Pressure level:	23 - 30 mmHg
Fastening:	Zipper with guard and hooks on the side of pelvis.
Details:	Waist SIL 1 Leg opening ELA 2



Product picture, code:  
STPOP21535F2ZC



Product picture, code:  
STPOP21525F2ZC



Product picture, code:  
STPOP21515F2ZC

**PANTS FOR WOMEN | NORMAL WAIST | WITHOUT ZIPPER | GENITAL AREA CLOSED**

STANDARD-SIZE

Size	Circumf.*		Colour	Product / Art. No		
	Pelvis cm	Thigh cm		Ankle length	Calf length	Short length
XS	86 - 90	50,0 - 53,0	Beige	STPOP21532F1C	STPOP21522F1C	STPOP21512F1C
XS	86 - 90	50,0 - 53,0	Black	STPOP21532F2C	STPOP21522F2C	STPOP21512F2C
S	91 - 95	53,5 - 56,0	Beige	STPOP21533F1C	STPOP21523F1C	STPOP21513F1C
S	91 - 95	53,5 - 56,0	Black	STPOP21533F2C	STPOP21523F2C	STPOP21513F2C
M	96 - 101	56,5 - 59,5	Beige	STPOP21534F1C	STPOP21524F1C	STPOP21514F1C
M	96 - 101	56,5 - 59,5	Black	STPOP21534F2C	STPOP21524F2C	STPOP21514F2C
L	102 - 110	60,0 - 62,5	Beige	STPOP21535F1C	STPOP21525F1C	STPOP21515F1C
L	102 - 110	60,0 - 62,5	Black	STPOP21535F2C	STPOP21525F2C	STPOP21515F2C
XL	111 - 121	63,0 - 66,0	Beige	STPOP21536F1C	STPOP21526F1C	STPOP21516F1C
XL	111 - 121	63,0 - 66,0	Black	STPOP21536F2C	STPOP21526F2C	STPOP21516F2C
XXL	122 - 132	66,5 - 69,0	Beige	STPOP21537F1C	STPOP21527F1C	STPOP21517F1C
XXL	122 - 132	66,5 - 69,0	Black	STPOP21537F2C	STPOP21527F2C	STPOP21517F2C
XXXL	133 - 138	69,5 - 71,5	Beige	STPOP21538F1C	STPOP21528F1C	STPOP21518F1C
XXXL	133 - 138	69,5 - 71,5	Black	STPOP21538F2C	STPOP21528F2C	STPOP21518F2C
XXXXL	139 - 144	72,0 - 74,5	Beige	STPOP21539F1C	STPOP21529F1C	STPOP21519F1C
XXXXL	139 - 144	72,0 - 74,5	Black	STPOP21539F2C	STPOP21529F2C	STPOP21519F2C

**PANTS FOR WOMEN | NORMAL WAIST | WITHOUT ZIPPER | GENITAL AREA OPEN**

STANDARD-SIZE

Size	Circumf.*		Colour	Product / Art. No		
	Pelvis cm	Thigh cm		Ankle length	Calf length	Short length
XS	86 - 90	50,0 - 53,0	Beige	STPOP21532F1O	STPOP21522F1O	STPOP21512F1O
XS	86 - 90	50,0 - 53,0	Black	STPOP21532F2O	STPOP21522F2O	STPOP21512F2O
S	91 - 95	53,5 - 56,0	Beige	STPOP21533F1O	STPOP21523F1O	STPOP21513F1O
S	91 - 95	53,5 - 56,0	Black	STPOP21533F2O	STPOP21523F2O	STPOP21513F2O
M	96 - 101	56,5 - 59,5	Beige	STPOP21534F1O	STPOP21524F1O	STPOP21514F1O
M	96 - 101	56,5 - 59,5	Black	STPOP21534F2O	STPOP21524F2O	STPOP21514F2O
L	102 - 110	60,0 - 62,5	Beige	STPOP21535F1O	STPOP21525F1O	STPOP21515F1O
L	102 - 110	60,0 - 62,5	Black	STPOP21535F2O	STPOP21525F2O	STPOP21515F2O
XL	111 - 121	63,0 - 66,0	Beige	STPOP21536F1O	STPOP21526F1O	STPOP21516F1O
XL	111 - 121	63,0 - 66,0	Black	STPOP21536F2O	STPOP21526F2O	STPOP21516F2O
XXL	122 - 132	66,5 - 69,0	Beige	STPOP21537F1O	STPOP21527F1O	STPOP21517F1O
XXL	122 - 132	66,5 - 69,0	Black	STPOP21537F2O	STPOP21527F2O	STPOP21517F2O
XXXL	133 - 138	69,5 - 71,5	Beige	STPOP21538F1O	STPOP21528F1O	STPOP21518F1O
XXXL	133 - 138	69,5 - 71,5	Black	STPOP21538F2O	STPOP21528F2O	STPOP21518F2O
XXXXL	139 - 144	72,0 - 74,5	Beige	STPOP21539F1O	STPOP21529F1O	STPOP21519F1O
XXXXL	139 - 144	72,0 - 74,5	Black	STPOP21539F2O	STPOP21529F2O	STPOP21519F2O

## PANTS FOR WOMEN | NORMAL WAIST | WITH ZIPPER | GENITAL AREA CLOSED

STANDARD-SIZE

Size	Circumf.*		Colour	Product / Art. No		
	Pelvis cm	Thigh cm		Ankle length	Calf length	Short length
XS	86 - 90	50,0 - 53,0	Beige	STPOP21532F1ZC	STPOP21522F1ZC	STPOP21512F1ZC
XS	86 - 90	50,0 - 53,0	Black	STPOP21532F2ZC	STPOP21522F2ZC	STPOP21512F2ZC
S	91 - 95	53,5 - 56,0	Beige	STPOP21533F1ZC	STPOP21523F1ZC	STPOP21513F1ZC
S	91 - 95	53,5 - 56,0	Black	STPOP21533F2ZC	STPOP21523F2ZC	STPOP21513F2ZC
M	96 - 101	56,5 - 59,5	Beige	STPOP21534F1ZC	STPOP21524F1ZC	STPOP21514F1ZC
M	96 - 101	56,5 - 59,5	Black	STPOP21534F2ZC	STPOP21524F2ZC	STPOP21514F2ZC
L	102 - 110	60,0 - 62,5	Beige	STPOP21535F1ZC	STPOP21525F1ZC	STPOP21515F1ZC
L	102 - 110	60,0 - 62,5	Black	STPOP21535F2ZC	STPOP21525F2ZC	STPOP21515F2ZC
XL	111 - 121	63,0 - 66,0	Beige	STPOP21536F1ZC	STPOP21526F1ZC	STPOP21516F1ZC
XL	111 - 121	63,0 - 66,0	Black	STPOP21536F2ZC	STPOP21526F2ZC	STPOP21516F2ZC
XXL	122 - 132	66,5 - 69,0	Beige	STPOP21537F1ZC	STPOP21527F1ZC	STPOP21517F1ZC
XXL	122 - 132	66,5 - 69,0	Black	STPOP21537F2ZC	STPOP21527F2ZC	STPOP21517F2ZC
XXXL	133 - 138	69,5 - 71,5	Beige	STPOP21538F1ZC	STPOP21528F1ZC	STPOP21518F1ZC
XXXL	133 - 138	69,5 - 71,5	Black	STPOP21538F2ZC	STPOP21528F2ZC	STPOP21518F2ZC
XXXXL	139 - 144	72,0 - 74,5	Beige	STPOP21539F1ZC	STPOP21529F1ZC	STPOP21519F1ZC
XXXXL	139 - 144	72,0 - 74,5	Black	STPOP21539F2ZC	STPOP21529F2ZC	STPOP21519F2ZC

## PANTS FOR WOMEN | NORMAL WAIST | WITH ZIPPER | GENITAL AREA OPEN

STANDARD-SIZE

Size	Circumf.*		Colour	Product / Art. No		
	Pelvis cm	Thigh cm		Ankle length	Calf length	Short length
XS	86 - 90	50,0 - 53,0	Beige	STPOP21532F1ZO	STPOP21522F1ZO	STPOP21512F1ZO
XS	86 - 90	50,0 - 53,0	Black	STPOP21532F2ZO	STPOP21522F2ZO	STPOP21512F2ZO
S	91 - 95	53,5 - 56,0	Beige	STPOP21533F1ZO	STPOP21523F1ZO	STPOP21513F1ZO
S	91 - 95	53,5 - 56,0	Black	STPOP21533F2ZO	STPOP21523F2ZO	STPOP21513F2ZO
M	96 - 101	56,5 - 59,5	Beige	STPOP21534F1ZO	STPOP21524F1ZO	STPOP21514F1ZO
M	96 - 101	56,5 - 59,5	Black	STPOP21534F2ZO	STPOP21524F2ZO	STPOP21514F2ZO
L	102 - 110	60,0 - 62,5	Beige	STPOP21535F1ZO	STPOP21525F1ZO	STPOP21515F1ZO
L	102 - 110	60,0 - 62,5	Black	STPOP21535F2ZO	STPOP21525F2ZO	STPOP21515F2ZO
XL	111 - 121	63,0 - 66,0	Beige	STPOP21536F1ZO	STPOP21526F1ZO	STPOP21516F1ZO
XL	111 - 121	63,0 - 66,0	Black	STPOP21536F2ZO	STPOP21526F2ZO	STPOP21516F2ZO
XXL	122 - 132	66,5 - 69,0	Beige	STPOP21537F1ZO	STPOP21527F1ZO	STPOP21517F1ZO
XXL	122 - 132	66,5 - 69,0	Black	STPOP21537F2ZO	STPOP21527F2ZO	STPOP21517F2ZO
XXXL	133 - 138	69,5 - 71,5	Beige	STPOP21538F1ZO	STPOP21528F1ZO	STPOP21518F1ZO
XXXL	133 - 138	69,5 - 71,5	Black	STPOP21538F2ZO	STPOP21528F2ZO	STPOP21518F2ZO
XXXXL	139 - 144	72,0 - 74,5	Beige	STPOP21539F1ZO	STPOP21529F1ZO	STPOP21519F1ZO
XXXXL	139 - 144	72,0 - 74,5	Black	STPOP21539F2ZO	STPOP21529F2ZO	STPOP21519F2ZO



Product picture, code: STPOP2134F1

**SLEEVES FOR WOMEN**


STANDARD-SIZE

Size	Circumf. 1* cm	Circumf. 2** cm	Colour	Art. No
XS	76 - 84	24,0 - 26,0	Beige	STPOP2132F1
XS	76 - 84	24,0 - 26,0	Black	STPOP2132F2
S	85 - 92	26,5 - 28,5	Beige	STPOP2133F1
S	85 - 92	26,5 - 28,5	Black	STPOP2133F2
M	93 - 100	29,0 - 31,0	Beige	STPOP2134F1
M	93 - 100	29,0 - 31,0	Black	STPOP2134F2
L	101 - 107	31,5 - 33,5	Beige	STPOP2135F1
L	101 - 107	31,5 - 33,5	Black	STPOP2135F2
XL	108 - 114	34,0 - 36,0	Beige	STPOP2136F1
XL	108 - 114	34,0 - 36,0	Black	STPOP2136F2
XXL	115 - 124	36,5 - 38,5	Beige	STPOP2137F1
XXL	115 - 124	36,5 - 38,5	Black	STPOP2137F2

**Product information:**

Women's support sleeves intended for use in connection with upper limb surgical procedures. Anatomically shaped elbow bends.

\*Circumference 1 under axillae, above breasts.  
\*Circumference 2 at widest point of upper arm.

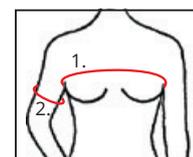

Material:	Sleeves: medium Bodice: light
Colours:	Beige, black
Pressure level:	23 - 30 mmHg in sleeves No pressure in torso
Fastening:	2-row hooks Hemline ELA 1
Details:	Neckline ELA 3 Edge of sleeve ELA 2



Product picture, code:  
STPOP2244F2

## VEST FOR MEN

STANDARD-SIZE

Size	Chest circumference under axillae	Waist circumference	Vest length from shoulder to hem	Colour	Product / Art. No
	cm	cm	cm		
XXXS	75 - 79	69 - 72	53,5	Beige	STPOP2240F1
XXXS	75 - 79	69 - 72	53,5	Black	STPOP2240F2
XXS	80 - 84	72 - 75	54,5	Beige	STPOP2241F1
XXS	80 - 84	72 - 75	54,5	Black	STPOP2241F2
XS	85 - 89	75 - 78	55,5	Beige	STPOP2242F1
XS	85 - 89	75 - 78	55,5	Black	STPOP2242F2
S	90 - 95	79 - 84	56,5	Beige	STPOP2243F1
S	90 - 95	79 - 84	56,5	Black	STPOP2243F2
M	96 - 101	84 - 90	59,5	Beige	STPOP2244F1
M	96 - 101	84 - 90	59,5	Black	STPOP2244F2
L	102 - 107	90 - 96	62,5	Beige	STPOP2245F1
L	102 - 107	90 - 96	62,5	Black	STPOP2245F2
XL	108 - 113	96 - 102	65,5	Beige	STPOP2246F1
XL	108 - 113	96 - 102	65,5	Black	STPOP2246F2
XXL	114 - 119	102 - 108	68,5	Beige	STPOP2247F1
XXL	114 - 119	102 - 108	68,5	Black	STPOP2247F2
XXXL	120 - 125	108 - 114	71,5	Beige	STPOP2248F1
XXXL	120 - 125	108 - 114	71,5	Black	STPOP2248F2

### Product information:

Men's support shirt intended for use in connection with post-operative care following upper body surgical procedures.

Material: Medium

Colour: Beige, black

Pressure level: 23 - 30 mmHg

Fastening: 2-row hooks

Details: Hemline SIL 1

Neckline and armholes ELA 3



Product picture, code:  
STPOP2271VF2

**SHORT VEST FOR MEN**

STANDARD-SIZE

Size	Chest circumference under axillae	Circumference at the hem of the vest	Vest length from shoulder to hem*	Colour	Product / Art. No
	cm	cm	cm		
XXXS	75 - 79	69 - 72	35,5	Beige	STPOP2270VF1
XXXS	75 - 79	69 - 72	35,5	Black	STPOP2270VF2
XXS	80 - 84	72 - 75	36	Beige	STPOP2271VF1
XXS	80 - 84	72 - 75	36	Black	STPOP2271VF2
XS	85 - 89	75 - 78	36,5	Beige	STPOP2272VF1
XS	85 - 89	75 - 78	36,5	Black	STPOP2272VF2
S	90 - 95	79 - 84	37	Beige	STPOP2273VF1
S	90 - 95	79 - 84	37	Black	STPOP2273VF2
M	96 - 101	84 - 90	39,5	Beige	STPOP2274VF1
M	96 - 101	84 - 90	39,5	Black	STPOP2274VF2
L	102 - 107	90 - 96	42	Beige	STPOP2275VF1
L	102 - 107	90 - 96	42	Black	STPOP2275VF2
XL	108 - 113	96 - 102	44,5	Beige	STPOP2276VF1
XL	108 - 113	96 - 102	44,5	Black	STPOP2276VF2
XXL	114 - 119	102 - 108	47	Beige	STPOP2277VF1
XXL	114 - 119	102 - 108	47	Black	STPOP2277VF2
XXXL	120 - 125	108 - 114	49,5	Beige	STPOP2278VF1
XXXL	120 - 125	108 - 114	49,5	Black	STPOP2278VF2

**Product information:**

Men's support shirt intended for use in connection with post-operative care following upper body surgical procedures. Adjustable velcro straps.

\*The shoulder adjustment affects the length of the vest.

Material:	Medium
Colour:	Beige, black
Pressure level:	23 - 30 mmHg
Fastening:	2-row hooks, adjustable velcro straps
Details:	Hemline SIL 1
	Neckline and armholes ELA 3

Product picture, code: STPOP714F3



**BRA | LIGHT SUPPORT**

STANDARD-SIZE

**Product information:**

The light support bra is intended for use after surgery to keep dressings in place. Adjustable 3 cm wide front-opening straps allow inspection and changing of dressings without removing the bra.

An adjustable hook-and-loop fastening at the back facilitates donning. Available with two cups and with one or two prosthesis pockets.

*\*Circumference at the chest below the breasts.*

Material:	Light
Colours:	Black, white
Pressure level:	No pressure
Fastening:	Velcro at the back, width 7 cm
Details:	Lower end ELA 1 Edging ELA 3



Product picture, code: STPOP714F3

**BRA | LIGHT SUPPORT**

STANDARD-SIZE

Size	Circumf.* cm	Colour	Product / Art. No	Two cups	Prosthesis pocket, right	Prosthesis pocket, left	Prosthesis pocket, both
XS	70 - 75	Black	STPOP712F2	STPOP712PRF2	STPOP712PLF2	STPOP712PPF2	
XS	70 - 75	White	STPOP712F3	STPOP712PRF3	STPOP712PLF3	STPOP712PPF3	
S	73 - 78	Black	STPOP713F2	STPOP713PRF2	STPOP713PLF2	STPOP713PPF2	
S	73 - 78	White	STPOP713F3	STPOP713PRF3	STPOP713PLF3	STPOP713PPF3	
M	76 - 83	Black	STPOP714F2	STPOP714PRF2	STPOP714PLF2	STPOP714PPF2	
M	76 - 83	White	STPOP714F3	STPOP714PRF3	STPOP714PLF3	STPOP714PPF3	
L	81 - 88	Black	STPOP715F2	STPOP715PRF2	STPOP715PLF2	STPOP715PPF2	
L	81 - 88	White	STPOP715F3	STPOP715PRF3	STPOP715PLF3	STPOP715PPF3	
XL	86 - 93	Black	STPOP716F2	STPOP716PRF2	STPOP716PLF2	STPOP716PPF2	
XL	86 - 93	White	STPOP716F3	STPOP716PRF3	STPOP716PLF3	STPOP716PPF3	
XXL	91 - 98	Black	STPOP717F2	STPOP717PRF2	STPOP717PLF2	STPOP717PPF2	
XXL	91 - 98	White	STPOP717F3	STPOP717PRF3	STPOP717PLF3	STPOP717PPF3	

Product picture, code: STPOP724F3



**BRA | MEDIUM SUPPORT**

STANDARD-SIZE

**Product information:**

The medium support bra is intended for use after surgery to keep dressings in place.

The material is stronger than in the light support bra.

Adjustable 3 cm wide front-opening straps allow inspection and changing of dressings without removing the bra.

An adjustable hook-and-loop fastening at the back facilitates donning.

Also available with one or two prosthesis pockets.

*\*Circumference at the chest below the breasts.*

Material:	Medium
Colours:	Black, white
Pressure level:	No pressure
Fastening:	Velcro at the back, width 7 cm
Details:	Lower end ELA 1 Edging ELA 3



Product picture, code: STPOP724F3

**BRA | MEDIUM SUPPORT**

STANDARD-SIZE

Size	Circumf.* cm	Colour	Product / Art. No	Two cups	Prosthesis pocket, right	Prosthesis pocket, left	Prosthesis pocket, both
XS	70 - 75	Black	STPOP722F2	STPOP722PRF2	STPOP722PLF2	STPOP722PPF2	
XS	70 - 75	White	STPOP722F3	STPOP722PRF3	STPOP722PLF3	STPOP722PPF3	
S	73 - 78	Black	STPOP723F2	STPOP723PRF2	STPOP723PLF2	STPOP723PPF2	
S	73 - 78	White	STPOP723F3	STPOP723PRF3	STPOP723PLF3	STPOP723PPF3	
M	76 - 83	Black	STPOP724F2	STPOP724PRF2	STPOP724PLF2	STPOP724PPF2	
M	76 - 83	White	STPOP724F3	STPOP724PRF3	STPOP724PLF3	STPOP724PPF3	
L	81 - 88	Black	STPOP725F2	STPOP725PRF2	STPOP725PLF2	STPOP725PPF2	
L	81 - 88	White	STPOP725F3	STPOP725PRF3	STPOP725PLF3	STPOP725PPF3	
XL	86 - 93	Black	STPOP726F2	STPOP726PRF2	STPOP726PLF2	STPOP726PPF2	
XL	86 - 93	White	STPOP726F3	STPOP726PRF3	STPOP726PLF3	STPOP726PPF3	
XXL	91 - 98	Black	STPOP727F2	STPOP727PRF2	STPOP727PLF2	STPOP727PPF2	
XXL	91 - 98	White	STPOP727F3	STPOP727PRF3	STPOP727PLF3	STPOP727PPF3	

Product picture, code: STPOP7380DF3H



**Product information:**

Strong support bras for breast support after surgery.  
Can also be used as a dressing bra.  
Front fastening between the breasts and a supportive back panel.  
3 cm wide, adjustable and openable straps.

*\*Circumference at the chest below the breasts.*

Material:	Strong
Colours:	Black, white
Pressure level:	No pressure
Fastening:	1-row hooks / zipper with guard and assistive hooks
Details:	Lower end ELA 1 Edging ELA 3

**BRA | STRONG SUPPORT | SHORT**

STANDARD-SIZE

<i>Circumf.*</i>	<i>Art. No (Code + Cup size + Colour + Fastening)</i>			
cm	Code	Cup Size	Colour black / white	Fastening hooks / zipper
65 - 69	STPOP7365	A / B / C / D / E / F / G	F2 / F3	H / Z
70 - 74	STPOP7370	A / B / C / D / E / F / G	F2 / F3	H / Z
75 - 79	STPOP7375	A / B / C / D / E / F / G	F2 / F3	H / Z
80 - 84	STPOP7380	A / B / C / D / E / F / G	F2 / F3	H / Z
85 - 89	STPOP7385	A / B / C / D / E / F / G	F2 / F3	H / Z
90 - 94	STPOP7390	A / B / C / D / E / F / G	F2 / F3	H / Z
95 - 99	STPOP7395	A / B / C / D / E / F / G	F2 / F3	H / Z
100 - 104	STPOP73100	A / B / C / D / E / F / G	F2 / F3	H / Z

Product picture, code: STPOP7480DF3Z



POST-OPERATION™

**Product information:**

Strong support bras for breast support after surgery.  
The long model is longer than the short model and is particularly suitable for larger bust sizes. Can also be used as a dressing bra. Front fastening with auxiliary hooks between the breasts and a supportive back panel. 3 cm wide, adjustable and openable straps.

\*Circumference at the chest below the breasts.

Material:	Strong
Colours:	Black, white
Pressure level:	No pressure
Fastening:	Zipper with guard and assistive hooks / 1-row hooks
Details:	Lower end ELA 1 Edging ELA 3

**BRA | STRONG SUPPORT | LONG**

STANDARD-SIZE

<i>Circumf.*</i>	<i>Art. No (Code + Cup size + Colour + Fastening)</i>			
cm	Code	Cup Size	Colour black / white	Fastening hooks / zipper
65 - 69	STPOP7465	A / B / C / D / E / F / G	F2 / F3	H / Z
70 - 74	STPOP7470	A / B / C / D / E / F / G	F2 / F3	H / Z
75 - 79	STPOP7475	A / B / C / D / E / F / G	F2 / F3	H / Z
80 - 84	STPOP7480	A / B / C / D / E / F / G	F2 / F3	H / Z
85 - 89	STPOP7485	A / B / C / D / E / F / G	F2 / F3	H / Z
90 - 94	STPOP7490	A / B / C / D / E / F / G	F2 / F3	H / Z
95 - 99	STPOP7495	A / B / C / D / E / F / G	F2 / F3	H / Z
100 - 104	STPOP74100	A / B / C / D / E / F / G	F2 / F3	H / Z



**S-SHAPED BREAST WRAP**

STANDARD-SIZE

Size	Circumf.* cm	Art.No
XS	60 - 75	STPOP752F3
S	70 - 90	STPOP753F3
M	85 - 105	STPOP754F3

**Product information:**

S-shape band to support the breasts in the aftercare of breast reconstructions. Intended to be worn on top of the bra to provide extra support after surgery. Can be used with strong support bra (see previous page).

Separate band enables the use of the products separately.

Fabric on the sides, adjustable velcro fastening at the front and back. Height of the product is 8 cm from the front.

*\*Circumference above the breasts.*

Material:	Strong, elastic 8 cm wide support band
Colour:	White
Pressure level:	Adjustable
Fastening:	Velcro
Details:	Upper and lower edge ELA 3

Product picture, code: STPOP753F3



**LIGHT ABDOMINAL BAND**

STANDARD-SIZE

<i>Prod.length</i> cm	<i>Height</i> cm	<i>Art.No</i>
100	18	STPOP4042100F2
120	18	STPOP4042120F2
140	18	STPOP4042140F2

**Product information:**

An adjustable light support belt with a two-part hook-and-loop fastening.  
The fastening attaches along the entire length of the product.  
Transparent and breathable material.  
Select the belt length closest to the required circumference.

Material:	Velcro support band
Colour:	Black
Pressure level:	Adjustable
Fastening:	Velcro

Product picture, code: STPOP4042100F2



Product picture, code: STPOP403180F1



Product picture, code: STPOP403380F1

## STRONG ABDOMINAL BAND

STANDARD-SIZE

Prod.length cm	Height cm	Art.No
80	16	STPOP403180F1
100	16	STPOP4031100F1
120	16	STPOP4031120F1
140	16	STPOP4031140F1
80	26	STPOP403380F1
100	26	STPOP4033100F1
120	26	STPOP4033120F1
140	26	STPOP4033140F1

### Product information:

A strong and flexible support belt.

Wide, adjustable hook-and-loop fastening across the full width of the belt. Can also be used, for example, as a back support and as a light hernia belt. Select the belt length closest to the required circumference.

For the lowest height option, select a belt shorter than the circumference.

Material:	Strong support band polyamide 86 %, elasthane 9 %, polypropylene 5 %
Colour:	Beige
Pressure level:	Adjustable
Fastening:	Velcro

Product picture, code: STM4041110F1

## MATERNITY SUPPORT BELT

STANDARD-SIZE

Prod.length cm	Height cm	Art.No
110	10	STM4041110F1

### Product information:

Strong elastic band to support the growing belly.

Material:	Strong support band polyamide 86 %, elasthane 9 %, polypropylene 5 %
Colour:	Beige
Pressure level:	Adjustable
Fastening:	Velcro





Product picture, code:  
STPOP1114F1

Product picture, code:  
STPOP1124F1

**FACIAL SUPPORT**

STANDARD-SIZE

Size	Circumf.*	Product / Art.No	
	cm	Ear area open	Ear area closed
S	58 - 62	STPOP1113F1	STPOP1123F1
M	62 - 66	STPOP1114F1	STPOP1124F1
L	66 - 72	STPOP1115F1	STPOP1125F1
XL	72 - 76	STPOP1116F1	STPOP1126F1

**Product information:**

Velcro fastenings on the crown of the head and back of the neck.

\*Circumference from crown to chin.

Material:	Medium
Colour:	Beige
Pressure level:	Adjustable
Fastening:	Velcro
Details:	Neck strap: ELA 1 Edging: ELA 3



Product picture, code:  
STPOP115F1

**STRONG CHIN STRAP**

STANDARD-SIZE

Size	Colour	Art. No
One size	Beige	STPOP115F1
One size	White	STPOP115F3

**Product information:**

Consists of several bands with adjustable velcro fastenings.

Material:	Strong, elastic 8 cm wide support band
Colour:	Beige, white
Pressure level:	Adjustable
Fastening:	Velcro

**HELMET FOR HAIR  
TRANSPLANT OPERATIONS**

STANDARD-SIZE

Size	Circumf.* cm	Art.No
S	50 - 54	STPOP1393F2
M	54 - 58	STPOP1394F2

**Product information:**

Helmet for keeping dressings in place.

*\*Circumference of the head.*

Material:	Light
Colour:	Black
Pressure level:	No pressure
Fastening:	Velcro below the chin



Product picture, code: STPOP1394F2











SKIN™ 40

COMPRESSION™ 57

SENSE™ 82

INTERIM & LIGHT™ 96

POST-OPERATION™ 107

**LYMED**®

Manufacturer:

**Lymed Oy**

**Pyhäjärvenkatu 5 A , 33200 Tampere Finland**

**Tel.: +358 20 779 2233**

**Email: [orders@lymed.fi](mailto:orders@lymed.fi)**

**Distributors worldwide: [www.lymed.fi](http://www.lymed.fi)**

Lymed Oy | 1/2026 | Please see [www.lymed.fi](http://www.lymed.fi) for the latest catalogue.