

LYMED[®]

LYMED CATALOGUE

MEDICAL DEVICES

1/2026



**SKIN
COMPRESSION
SENSE
INTERIM & LIGHT
POST-OPERATION**

ISO 13485

CERTIFIED QUALITY MANAGEMENT SYSTEM



VALPAS
CERTIFICATIONS



All up-to-date information:

LYMED.FI

FOR PROFESSIONALS:

Catalogue

Order forms

Measuring instructions

FOR USERS:

Instructional videos for donning

Wear & care instructions

Customer stories

Frequently asked questions

LYMED CATALOGUE

TABLE OF CONTENTS

PRODUCT INFORMATION

| | |
|--|----|
| Company | 4 |
| What are pressure garments? | 5 |
| Lymed medical devices | 6 |
| Product groups and context of use | 7 |
| Guidance for selecting the appropriate product | 12 |
| Ordering Lymed products | 14 |
| Pressure and compression | 16 |
| Lymed materials | 18 |
| Choosing fabric and pressure levels for custom-made products | 20 |
| Choosing fabric and pressure levels for Sense products | 21 |
| Colour and emblem options | 22 |
| Silicone and elastane bands | 23 |
| Additions and special solutions | 24 |
| Ordering and product codes | 27 |
| Additions for custom-made products | 28 |
| Customisation options for standard-sized products | 34 |
| Sense functional supports | 38 |

SKIN™ Pressure garments for scar management

| | |
|-----------------------------|----|
| Vests and abdominal support | 41 |
| Sternal straps | 42 |
| Bodys | 44 |
| Pants | 45 |
| Legs and stockings | 46 |
| Sleeves and gloves | 48 |
| Web spacer and digital cap | 52 |
| Head and neck products | 54 |

COMPRESSION™ Medical compression garments

| | |
|--------------------------|----|
| Vests and bodys | 58 |
| Bras and bra tops | 60 |
| Compression bras | 62 |
| Pants | 64 |
| Suspensor and penis tube | 72 |
| Maternity pants | 73 |
| Legs and stockings | 74 |
| Sleeves and gloves | 76 |
| Head products | 80 |

SENSE™ Neurological pressure garments

| | |
|-------------------------------|----|
| Scapula support vest and body | 83 |
| Vests | 84 |
| Overalls | 85 |
| Bodys | 86 |
| Pants | 88 |
| Legs and stockings | 89 |
| Sleeves and gloves | 90 |

INTERIM & LIGHT™ Interim care and light pressure garments

| | |
|--------------------|-----|
| Vests | 97 |
| Pants | 100 |
| Gloves and sleeves | 102 |
| Stockings | 105 |
| Head products | 106 |

POST-OPERATION™ Post-operational support garments

| | |
|----------------------|-----|
| Pants for women | 108 |
| Vest and sleeves | 113 |
| Bras and breast wrap | 116 |
| Abdominal bands | 123 |
| Head products | 125 |



reddot award 2014
winner



Joint Award to Industry for
Innovation in the Treatment of Lymphoedema



QUALITY
INNOVATION
THE YEAR
2013



FINNISH
TEXTILE &
FASHION
HONORABLE
MENTION 2018

Lymed Oy is a Finnish family-owned company founded in 1993 in Tampere that designs, manufactures, and sells medical pressure garments and related wellness products.

From the outset, Lymed has operated internationally as a developer and pioneer of high-quality pressure garments. Lymed's product range is the most extensive in the world, and our product selection covers all applications of pressure garment therapy.

Lymed operates as a medical aid manufacturer specialising in highly customised, client-specific solutions, based on our extensive expertise in pressure garment manufacturing.

Lymed's multidisciplinary team combines technology with strong textile industry expertise to deliver the best possible pressure garments for our clients.

Lymed's wide-ranging expertise is reflected in the fact that the company also manufactures products for animals such as dogs, horses, and camels.

MANUFACTURING OF LYMED PRODUCTS

The aim of every Lymed product is to provide the best possible fit, excellent comfort, and a refined, presentable appearance.

Lymed products meet the highest quality standards without compromising on comfort. Unique custom-made products and special solutions are an integral part of our offering, as individual needs—such as challenges related to donning and doffing, work and hobbies, and physical limitations—can be carefully considered in the design process.

All Lymed custom-made and standard-sized products are designed in Tampere, Finland. Our custom-made products are manufactured from start to finish in Tampere. Cutting and sewing of standard-sized products takes place in Tampere as well as Estonia.

SHARING OUR EXPERTISE AND TRAINING NEW MEASURERS

Lymed and our country-specific distributors continuously provide training for measurers, purchasers, and all healthcare professionals interested in compression garments.

Our measurers are typically occupational therapists, physiotherapists, or experts in neurological occupational therapy and physiotherapy, working in both the private and public sectors.



WHAT ARE PRESSURE GARMENTS?

A pressure garment is a soft and elastic medical device to support bodily function by utilising externally applied pressure. Pressure garments can be used by users of different ages in a wide range of use contexts as part of a care and rehabilitation process related to diseases, injuries, and functional capacity challenges.

In practice, a pressure garment is a closely fitting product that can be designed according to its primary intended use, the intended support effect, and the body area to which pressure is applied. The general term pressure garment may be used to describe both pressure and support garments that provide even pressure, as well as compression garments in which the pressure is graduated and decreases towards the central parts of the body.

Pressure garments are used as part of a treatment plan guided by a healthcare professional, based on an individual assessment and the recommendation of the treating professional. Pressure garments do not replace other medical treatment or the instructions provided by healthcare professionals.

IN WHICH CONTEXTS ARE PRESSURE GARMENTS COMMONLY USED?

Pressure garments are part of comprehensive treatment and recovery-supporting practices in which external pressure supports bodily function across different applications.

SCAR MANAGEMENT

Pressure garments are used as part of scar treatment to support scar management and the scar maturation process. They are suitable, for example, for:

- **burn scars**
- **hypertrophic scars**
- **keloids**

Pressure garments support the scar maturation process and help manage scar-related thickening, stiffness, and discomfort.

OEDEMA MANAGEMENT

Compression garments are used to support the management of oedema in situations where the body's fluid and blood circulation is compromised. They may be utilised, for example, in individuals whose oedema is associated with:

- **venous insufficiency**
- **lymphatic oedema**
- **postoperative or post-procedural conditions**
- **prolonged immobility or physical strain**

Compression garments support the body's natural blood and fluid circulation through external compression.

SENSORIMOTOR APPLICATIONS

Pressure garments can be used as support in sensorimotor applications, such as:

- **situations related to movement disorders and challenges in motor control**
- **situations related to pain conditions and bodily tension**
- **situations involving postural misalignments and load-related challenges**

Pressure garments support body awareness, proprioceptive feedback, and movement control, and may help alleviate discomfort and support relaxation and bodily calming during daily activities and rehabilitation.

POSTOPERATIVE AFTERCARE

Pressure garments are used after surgery as part of the recovery process, for example:

- **following various invasive procedures**

Pressure garments support the management of swelling, bruising and scars, provide external support to the operated area, and may help alleviate recovery-related discomfort.

MOST PERSONALISATION OPTIONS


Challenging product solutions and individual requests can be achieved in custom-made products.



Read more on page 24


EASY TO CARE FOR

The products are machine washable, dryer-safe, and quick-drying.

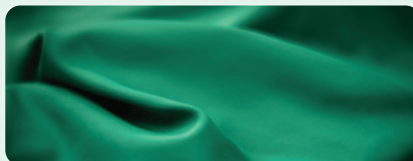

LONG LIFESPAN

The products are designed to withstand everyday use. Repairs and modifications are possible, even after production.

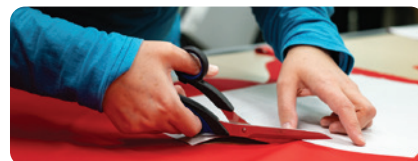

Read more on page 15

COMFORT THROUGH DESIGN

Anatomically shaped and contoured joint areas to achieve a functional fit.



THIN, NON-ALLERGENIC FABRICS

Better wearing comfort with the same pressure levels: Thin products allow movement and do not hinder everyday life.

Lymed materials do not contain natural rubber or other allergens.

Read more on page 18 forward

FLAT SEAM STRUCTURE

Our seams are tailored to be as comfortable as possible against the skin.



Read more on page 18

LOOKS LIKE CLOTHING, FEELS LIKE CLOTHING, BUT IS A MEDICAL DEVICE

A medical pressure garment is intended to be used as part of the treatment and rehabilitation of a disease or injury and is classified as a medical device in accordance with the EU Medical Device Regulation (MDR).

The use of a medical pressure garment differs from ordinary clothing: its use is planned and guided according to instructions.

LYMED-CUSTOM-MADE PRODUCTS BY PRODUCT CATEGORY:

- Skin™
- Compression™
- Sense™

Device class: Class I medical device (MDR)
Device type: Custom-made medical device

LYMED-STANDARD-SIZED PRODUCTS BY PRODUCT CATEGORY:

- Skin™
- Compression™
- Sense™
- Interim & Light™
- Post-operation™

Device class: Class I medical device (MDR)

LYMED PRODUCT INFORMATION

Product groups and context of use



SKIN™

PRESSURE GARMENTS FOR SCAR MANAGEMENT

CUSTOM-MADE

STANDARD-SIZE



Pressure garments are used as part of scar treatment to support scar management and the scar maturation process, particularly following burn injuries, extensive skin grafts, and surgical procedures. They are intended for the management of hypertrophic scars and keloids and may also be used preventively when wound or skin graft healing is prolonged and the risk of hypertrophic scar formation is increased.

Pressure garments support scar management through external pressure. Even pressure may help support the scar maturation process, manage scar thickening and stiffness, and alleviate scar-related discomfort such as itching, pain-related sensations, and redness. The benefits of pressure garments are greatest when they are used during the scar maturation phase as part of a structured scar management plan.

Pressure garments may be used as part of an individual treatment plan in certain skin-related conditions, as assessed by a healthcare professional, and may also assist in keeping dressings or wound coverings in place as part of an overall treatment, support, or recovery plan.

In pressure garments intended for scar management, the pressure is even throughout the product. Pressure garments are recommended to be ordered as custom-made to ensure optimal fit and to reduce the risk of irritation to healing skin. Pressure garment therapy may be initiated with lower-pressure products to allow gradual adaptation to the actual pressure garment. Interim pressure garments with lower pressure levels can be used earlier, even before the start of active scar management; see the Interim & Light product group.

Lymed Skin products are primarily custom-made for all body parts. The products are manufactured with pressure levels intended for scar management. In addition, a lightened pressure level is available, which is particularly suitable for children.

TYPICAL CONTEXTS OF USE

- Hypertrophic scars
- Burn scars
- Keloids
- Other post-surgical or post-injury scars

MOST COMMON GOALS OF USE

- Supporting scar shaping and preventing scar thickening
- Supporting the scar maturation process
- Management of scar-related redness, itching, and pain-related sensations

LYMED PRODUCT INFORMATION

Product groups and context of use



COMPRESSION™

MEDICAL COMPRESSION GARMENTS

CUSTOM-MADE

STANDARD-SIZE



Pressure garments used for the management and prevention of oedema are referred to as compression garments.

Medical compression garments are used as part of oedema management and prevention in various conditions associated with swelling and circulatory load. Compression garments support the body's blood and fluid circulation through external compression.

Compression garments may also be used preventively in situations where the load on fluid circulation in the lower limbs or other parts of the body is increased. Such situations include prolonged sitting or standing work, travel, pregnancy, and the recovery phase following surgical procedures.

Compression garments may be used in selected inflammatory or connective tissue-related conditions when the goal is to support the management of associated oedema as part of an overall treatment plan. Compression garments may also be used in venous conditions where wound dressings are used on the lower limbs, to support the management of oedema and swelling and to assist in keeping dressings or wound coverings in place, as part of an overall treatment, support, or recovery plan.

The pressure in compression garments is built so that it gradually decreases from distal to proximal areas, toward the centre of the body, with the strongest pressure at the extremities. This may help reduce fluid accumulation in the tissues and support oedema management. Lymed Compression products are primarily custom-made, and the pressure they provide is divided into four compression classes based on strength. Lymed Compression products can generally be used both during the day and at night.

TYPICAL CONTEXTS OF USE

- Primary and secondary lymphoedema
- Lipoedema
- Venous circulatory load conditions such as venous insufficiency and varicose veins
- Venous lower limb conditions with wound dressings
- Postoperative or post-injury swelling

MOST COMMON GOALS OF USE

- Supporting oedema management and preventing recurrence of swelling
- Supporting blood and fluid circulation
- Supporting the management of swelling and swelling-related discomfort
- Supporting the placement of dressings and wound coverings

LYMED PRODUCT INFORMATION

Product groups and context of use



SENSE™ NEUROLOGICAL PRESSURE GARMENTS

Pressure garments are used as rehabilitation aids and tools in neurological physiotherapy and occupational therapy as part of a professionally guided rehabilitation and treatment plan.

The effect of neurological pressure garments is based on external pressure that supports body control, movement regulation, and body awareness. The pressure applied by the pressure garment may support proprioceptive feedback, which can help support muscle activation and movement control during rehabilitation. Pressure garments may also provide external support to joints, which can help manage joint hypermobility and support predictability and control of movement in everyday situations.

In clinical practice, pressure garments are also used as part of pain management alongside other treatment modalities, particularly in situations where pain is associated with impaired body control, joint hypermobility, sensorimotor deviations, or tissue load.

According to professional experience, the even external pressure provided by pressure garments and their body-awareness-enhancing effect may support the management of pain-related discomfort as part of an

CUSTOM-MADE

STANDARD-SIZE



overall treatment approach. In clinical practice, pressure garments are used, for example, to support pain management in patients with CRPS and EDS, as part of a treatment plan assessed by a healthcare professional, without being considered a replacement for other medical or therapeutic treatment.

Lymed Sense pressure garments can be extensively customised according to individual goals to support daily functioning and rehabilitation objectives.

TYPICAL CONTEXTS OF USE

- Disorders of movement regulation and motor control
- Neurodevelopmental disorders and conditions with sensorimotor symptoms
- Central or peripheral nervous system injuries and dysfunctions
- Musculoskeletal load-related conditions
- As part of pain management
- Musculoskeletal diseases and conditions

MOST COMMON GOALS OF USE

- Supporting sensorimotor function and movement regulation
- Strengthening body schema and body awareness
- Supporting proprioceptive feedback
- Supporting movement quality and control
- Providing external joint support for managing hypermobility
- Supporting pain management
- Supporting functional ability and daily performance



INTERIM & LIGHT™

INTERIM CARE AND LIGHT PRESSURE GARMENTS

STANDARD-SIZE



Lymed Interim & Light products are standard-sized pressure garments that provide light and even external pressure, primarily used as part of scar management. The products are designed for short-term use during the transition phase before moving to custom-made products or as part of the early phase of treatment when pressure garment therapy cannot yet be initiated or when the user is being accustomed to pressure garment use. The products may also be used to help keep dressings in place.

Lymed Interim & Light products may also be used as part of pain management alongside other treatment modalities, particularly when light external support and a body-awareness-enhancing effect are needed in the early phase of treatment prior to the introduction of a custom-made product.

In addition, Lymed Interim & Light products may be used as part of an individual treatment plan in certain skin-related conditions when a healthcare professional assesses that light external pressure supports the treatment goals.

TYPICAL CONTEXTS OF USE

- Management of various types of scars
- Short-term use as part of pain management
- Supporting the retention of dressings

MOST COMMON GOALS OF USE

- Supporting scar management and the maturation process in the early phase
- Acclimatising the user to pressure garment use
- Providing light pressure prior to the use of optimal pressure garment

LYMED PRODUCT INFORMATION

Product groups and context of use



POST-OPERATION™

POST-OPERATIONAL SUPPORT GARMENTS

STANDARD-SIZE



Postoperative support garments are pressure and support garments used as part of the aftercare following various invasive surgical procedures.

Support garments provide external pressure and/or external support, which supports postoperative recovery by offering external support to the operated body area.

The pressure and external support may help support the management of oedema, swelling, bruising, and scars, alleviate recovery-related discomfort, and support movement and functional activity during recovery.

Postoperative support garments may also be used to assist in keeping dressings or wound coverings in place as part of an overall treatment or recovery plan.

Lymed Post-operation products are standard-sized products with even pressure.


TYPICAL CONTEXTS OF USE

- Abdominal and lower body procedures
- Breast reconstruction and reshaping procedures
- Facial and upper arm procedures
- Liposuction procedures
- Recovery after childbirth and caesarean section
- Aftercare following various invasive surgical procedures


MOST COMMON GOALS OF USE

- Providing external support to the operated area
- Supporting the management of oedema, swelling and bruising
- Supporting surgical scar management
- Supporting the management of pain and discomfort related to the surgical area
- Supporting the placement of dressings and wound coverings


Scar management

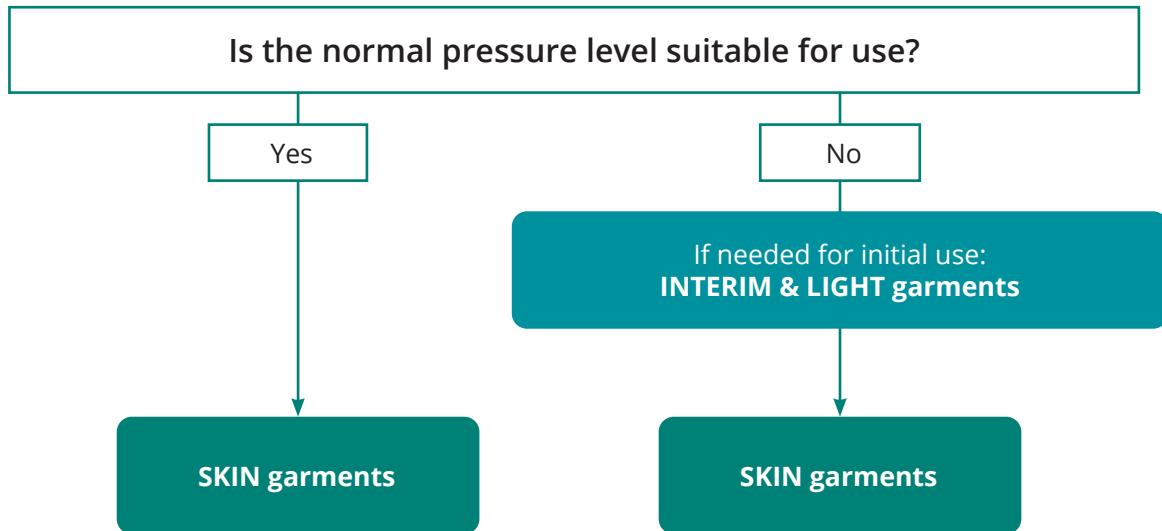


Support of fluid and blood circulation Venous insufficiency

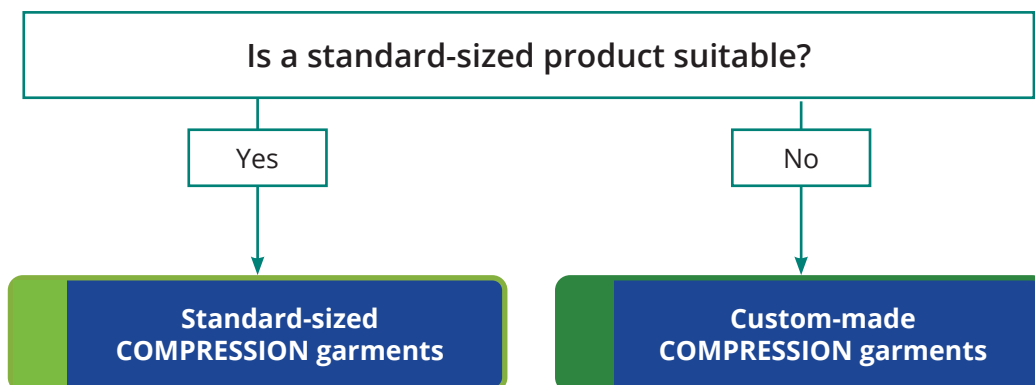

Sensorimotor and neurological applications
Pain management

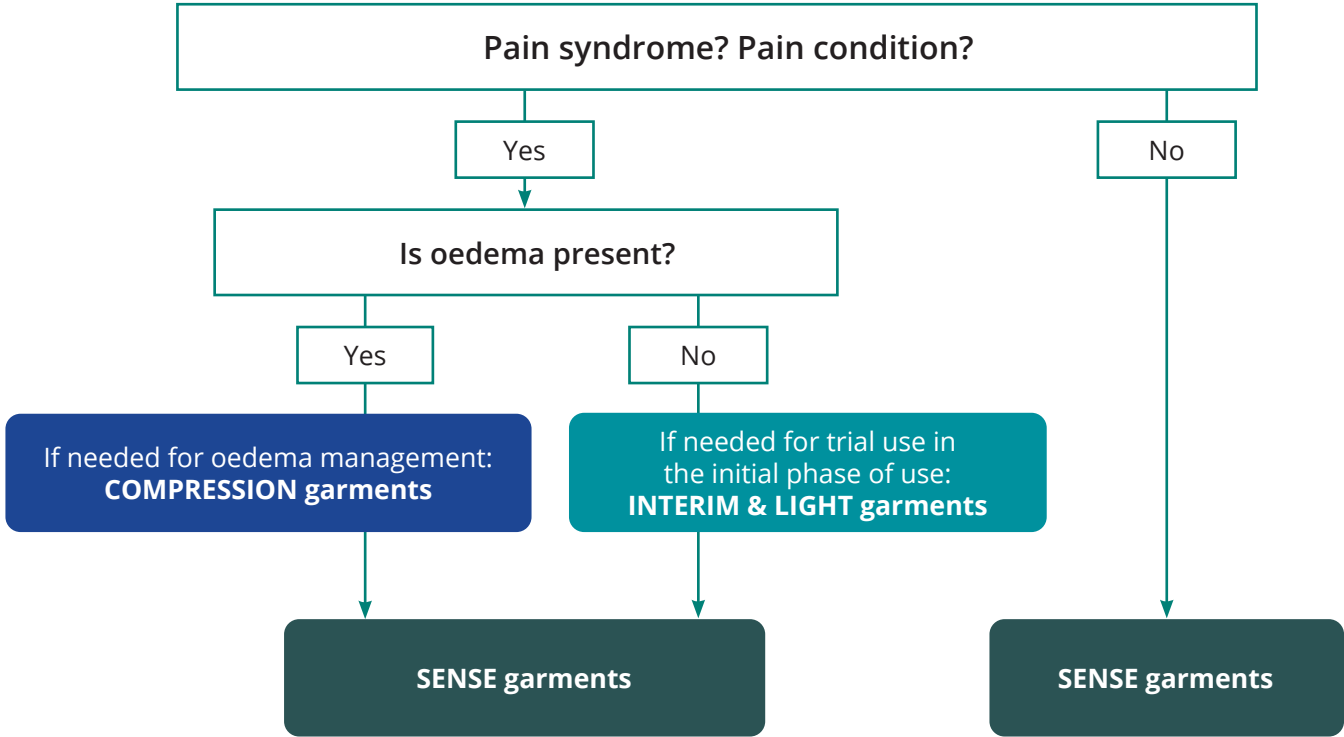

Post-operative aftercare

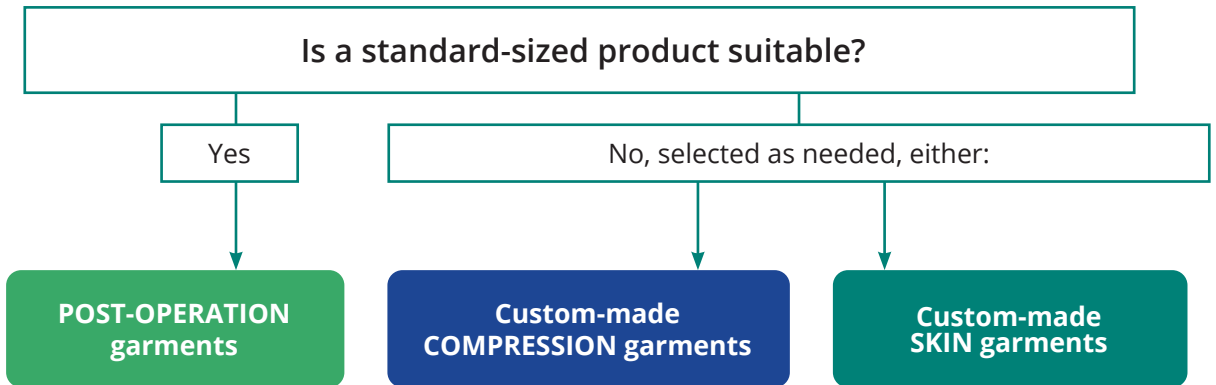

ORDERING LYMED PRODUCTS

Lymed products are available for both the public and private healthcare sectors. Trained Lymed measurers can be found across all of our distributing countries.

Our measurers are primarily involved in compression treatment, occupational therapy, physiotherapy, and neurological physiotherapy. They manage the entire ordering process for custom-made Lymed products.

Standard-sized products are available through our distributors. Customers in Finland can also purchase directly through Lymed.

For more details, please visit our website at www.lymed.fi.

ORDER PROCESS:

CUSTOM-MADE PRODUCT

1. Choosing a product

- The party responsible for the treatment assesses the need for the use of a pressure garment and selects an appropriate product based on the product information presented in the catalogue.

2. Measuring and ordering

- Measuring the patient according to Lymed's instructions.
- Providing additional information, special requirements and photos.

3. Sending orders to a Lymed distributor

- The order will be reviewed, and the customer will be contacted if necessary.

4. Delivery

- The products are delivered as agreed.

STANDARD-SIZED PRODUCT

1. Choosing a product

- The party responsible for the treatment assesses the need for the use of a pressure garment and selects an appropriate product based on the product information presented in the catalogue.

2. Choosing a size

- Choosing the correct size based on the instructions in the catalogue.

3. Sending orders to a Lymed distributor

- The order will be reviewed, and the customer will be contacted if necessary.

4. Delivery

- The products are delivered as agreed.

WHEN SHOULD I ORDER A NEW PRODUCT?

Lymed products are durable and designed for long-term use. Due to their functional properties, the products are subject to significant mechanical stress, which causes them to wear faster than ordinary clothing. For this reason, their service life cannot be compared to that of regular garments or socks.

With regular use, the service life of Lymed products typically ranges from three to six months. The estimated service life is based on reasonably foreseeable external influences and environmental conditions. In high wear and tear products such as a stockings and gloves, the service life may vary considerably between users.

Pressure garments are recommended to be replaced at least every six months, and products in active use every three months.

For active users, such as children and working adults, replacement may be required more frequently. The condition and fit of the product should be actively monitored during the period of use.

The rate of wear is affected by e.g.:

- Purpose and frequency of use
- Strength of pressure or compression
- Individual solutions and anatomy of the user
- The rubbing effect of external factors and the use of other aids
- Acidity of skin
- Care and washing of the product

A PRESSURE GARMENT NEEDS TO BE REPLACED WHEN

- Patient's weight or measurements change
- Washing no longer cleans the material properly and the feel of the product becomes greasy
- Elasticity of the product decreases and the original pressure levels change
- The product or its part breaks

TWO LAST LONGER THAN ONE

- If you wear a pressure garment continuously, it is recommended to have multiple products to use in rotation.
- Alternating between multiple products makes washing easier and allows the fabric to rest between uses.
- Alternating helps extend the service life of both products.

REPAIRS AND MODIFICATIONS

As medical devices, medical pressure garments should not be repaired by the user, even if they have sewing skills.

Lymed offers repair and modification services for its products. The availability of the service may vary depending on your country of residence.

Repairs and modifications are carried out in accordance with the terms and conditions of Lymed, and each repair or modification is assessed on a case-by-case basis.

If the product is heavily worn, Lymed recommends considering ordering a new product instead of having it repaired.

More information about modifications and repairs from your local Lymed distributor.

Read Lymed's terms and conditions in full at:
www.lymed.fi

The terms and conditions may vary depending on the product's granting party or your country of residence.

HOW PRESSURE IS CREATED WITH PRODUCTS

The effects of compression and pressure garments are based on the controlled implementation of pressure designed into the product.

The fabric itself does not generate any pressure; instead, pressure is created through the negative relationship between the circumference measurements of the treated area and the product's circumference, based on the fabric's elasticity and recovery properties.

The pressure provided by the product is generally divided into two different types according to the treatment goals. Compression garments are designed with progressive compression, providing the strongest support in the extremities, which gradually decreases towards the midsection. In even pressure garments, the pressure is evenly distributed throughout the product.


CUSTOM-MADE VS. STANDARD-SIZED PRODUCT

When acquiring a pressure garment, it should be assessed whether a standard-sized product can be used or whether a custom-made product is required.


As a general principle, Lymed always recommends the use of a custom-made product. Custom-made products generally provide a better fit than standard-sized products. Significant swelling, anatomical characteristics, and individual special requirements generally guide the choice towards a custom-made product.

The fit of a standard-sized product must be carefully verified: the more closely the user's measurements correspond to the size chart, the more likely it is that the fit is correct and that the pressure is applied appropriately.

COMPRESSION

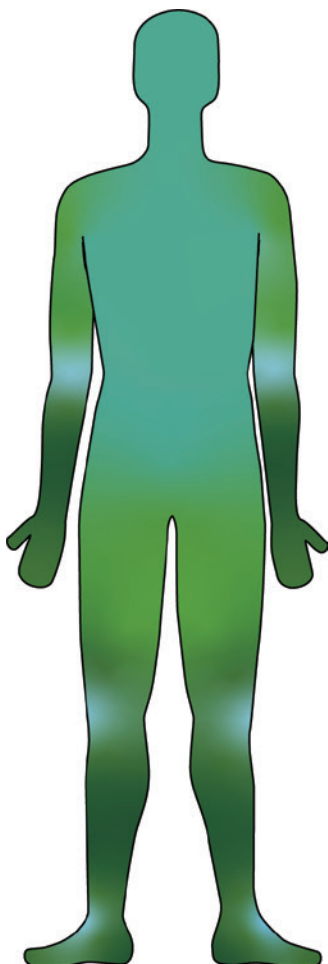

EVEN PRESSURE

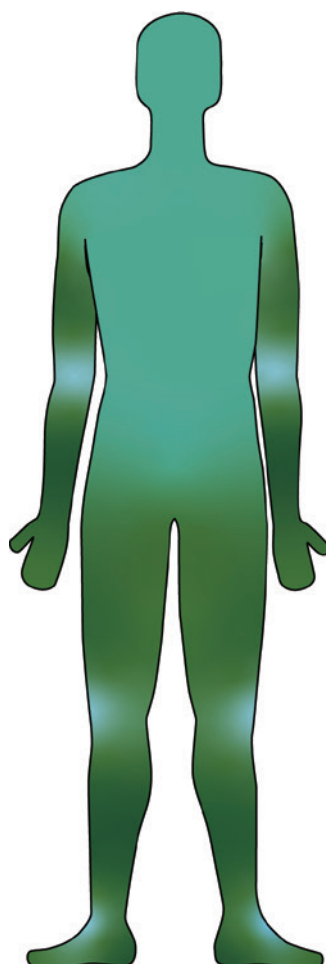


Lightest pressure



Strongest pressure

COMPRESSION VS. EVEN PRESSURE

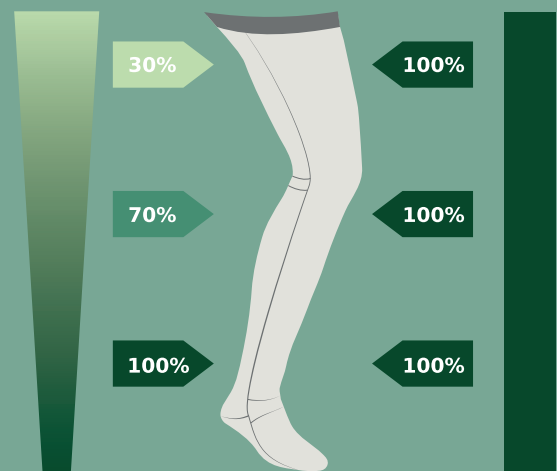

% of requested compression/pressure mmHg

COMPRESSION CLASSES AND PRESSURE LEVELS OF LYMED PRODUCTS

The pressure level or compression class of a product is selected based on the treatment need and goal, as well as the intended area of use. Lymed products are available with either even pressure or compression, depending on the product group and its specified purpose of use.

The unit used to measure the pressure produced by pressure garments is millimeters of mercury (mmHg).

A product's pressure class indicates the minimum and maximum pressure within which the actual average pressure typically falls. The realization of pressure on the treated area is influenced by the structure of the tissue and the user's level of activity. Firmer tissue or activating muscles provide counterpressure to the product, in which case the pressure applied to the tissue may in practice be higher.

When the product is worn, the actual pressure can be measured using pressure meters designed for pressure garment use. Pressure meters can also be used to monitor pressure during use and to assess the need for product replacement.

COMPRESSION CLASSES

The compression classification of Lymed products (classes 1–4) is based on the European pre-standard ENV 12718:2001.

PRESSURE LEVELS

The product-group-specific pressure levels offered by Lymed, as well as the possible general recommendations presented based on them, are based on reference values defined in the literature, use-related recommendations, and experience accumulated from customer use and needs.

COMPRESSION CLASSES CCL

Progressive pressure

Compression products

Ccl 1 **15 - 21 mmHg**

Ccl 2 **23 - 32 mmHg**

Ccl 3 **34 - 46 mmHg**

Ccl 4 **49 mmHg and higher**

PRESSURE LEVELS

Even pressure

Skin products

Lightened pressure **16 - 22 mmHg**

Normal pressure **23 - 30 mmHg**

Sense products

Light pressure **9 - 13 mmHg**

Medium pressure **14 - 20 mmHg**

Strong pressure **21 - 30 mmHg**

Interim & Light products **10 - 18 mmHg**

Post-operation products **23 - 30 mmHg**



Guidelines for the use of compression classes can be found in your country's healthcare guidelines or in international publications.

LYMED MATERIALS AND FABRICS


The experience of a pressure garment's comfort is influenced by many factors. The most important are the pressure level selected for the user and the feel of the fabric against the skin.

Lymed's fabrics are designed to withstand long-term use in pressure garment applications while remaining comfortable, breathable, and easy to care for. The fabrics are designed to be smooth, which makes donning easier.

We use three different fabrics with varying levels of rigidity, from which different pressure levels can be achieved to support specific treatment goals.

Lymed products are made from flat-knitted warp knit fabrics composed of a polyamide-elastane blend. Characteristics of warp knits include excellent resistance to pressure and the ability to manufacture pressure garments with complex shapes. In addition, warp knits cannot unravel.

By describing the treatment goals, the required pressure level, and the fabric choice that best supports the user's daily activities, we are able to create an individual medical aid that is also pleasant to wear.

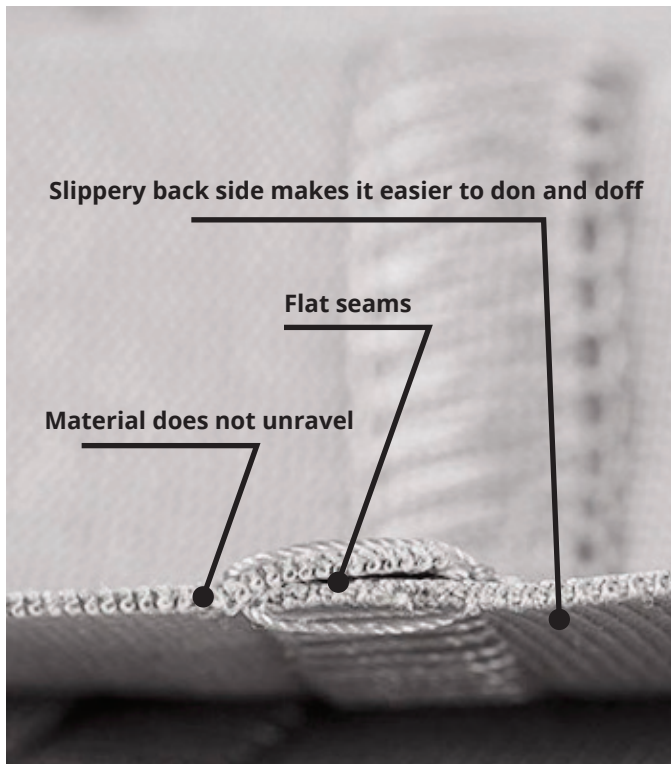

Our textile materials comply with Oeko-Tex 100 certified.

Our materials are Oeko-Tex certified and do not contain rubber or other known allergens. The materials are responsibly manufactured in Europe.

The fabrics are tested for durability, colour fastness, and other properties. They pass even the most stringent tests—their abrasion resistance exceeds 150,000 Martindales (SFS-EN ISO 12947-2).

In addition, Lymed has developed non-standard, practice-simulating testing methods for continuous quality control in collaboration with Finnish research institutes.

The fabrics also provide sun protection with UPF 98 %.



THREE HIGH-QUALITY, LIGHTWEIGHT FABRICS



| | STRONG FABRIC | MEDIUM FABRIC | LIGHT FABRIC |
|--|---|--|---|
| Fabric description for ordering a custom-made product | <p>Our most popular and widely used fabric among our customers.</p> <ul style="list-style-type: none"> • Firm feel • A good primary choice if lighter option isn't needed. • The sole option for ccl 4 products. | <p>An option for those who prefer a lighter feel, including children, the elderly, or people with sensitive skin.</p> <ul style="list-style-type: none"> • Lighter and smoother than heavier fabrics. • May be easier to put on. | <p>Only for light pressure Sense products for highly sensitive users.</p> <ul style="list-style-type: none"> • Ultra lightweight feel • Especially suitable for babies. |
| Composition | 61 % polyamide 39 % elastane | 54 % polyamide 46 % elastane | 83 % polyamide 17 % elastane |
| Structure | Flat knitted warp knit | Flat knitted warp knit | Flat knitted warp knit |
| Abrasion resistance (Martindale) | >150 000 | >150 000 | 90 000 |
| Breathability | 447 l/m ² /s | 466 l/m ² /s | 246 l/m ² /s |
| Thickness | 0,64 mm | 0,57 mm | 0,48 mm |
| UV protection | 98 % UV protection | 98 % UV protection | 98 % UV protection |



LYMED PRODUCT INFORMATION

Choosing fabric and pressure levels for custom-made products

The pressure level or compression class is selected according to the intended use and the goals of the overall treatment or rehabilitation plan, as well as the area of use of the product.

The user's response to the product may also vary over time, in which case product selections may need to be reviewed. If you are uncertain about fabric and pressure selections, always start with recommended options. The choice of material can affect the feel of the product and

RECOMMENDED MATERIALS AND PRESSURE LEVELS

Product-group-specific, compatible pressure and fabric options ensure the product's performance.

Deviating solutions in fabric selection may in some cases be possible. Requests are evaluated on a case-by-case basis from a manufacturing perspective. The fabric options of standard-sized products cannot be changed.

COMPRESSION

CCL 1

-  Strong fabric
-  Medium fabric

CCL 2

-  Strong fabric
-  Medium fabric

CCL 3

-  Strong fabric
-  Medium fabric

CCL 4

-  Strong fabric

thereby the perceived effect during use.

If the user requires a more supportive feel but a higher pressure level or compression class is not suitable, we recommend selecting a firmer fabric for the next product. If needed, the supportive feel can be enhanced with various reinforcements and double-layer fabrics; see additional options.

SKIN

NORMAL PRESSURE LEVEL



-  Strong fabric
-  Medium fabric

LIGHTENED PRESSURE LEVEL



-  Strong fabric
-  Medium fabric

SENSE [READ MORE ABOUT SENSE PRODUCTS](#)



LIGHT PRESSURE

-  Medium fabric*
-  Light fabric
-  Strong fabric

MEDIUM PRESSURE

-  Medium fabric**
-  Strong fabric

STRONG PRESSURE

-  Strong fabric
-  Medium fabric

PRESSURE AND FABRIC CHOICES FOR SENSE PRODUCTS

The recommendations presented below are based on usage experience and customer feedback collected by Lymed, as well as on professionals' clinical practice.

- *The most common Sense product for adults and children to support functionality is a light pressure product with medium fabric. There is rarely a need to order pressure garments for functionality support with strong pressure and a strong fabric.
- **The first Sense product for an adult with pain syndrome is typically one with medium pressure and a medium fabric.

PAIN CONDITIONS AND PAIN SYNDROMES

- The first Sense product for an adult with pain is typically one with medium pressure and a medium fabric.
- This group of users often use several products with different pressure levels and/or fabric types, which can be alternated even on a daily basis according to the desired pressure and feel.
- This user group may also more frequently choose a strong-pressure product with a strong fabric compared to other Sense product users.

USERS WITH PAIN AND OEDEMA

- If the user presents with oedema, oedema management can, if necessary, first be supported with a medical compression product (Compression product group) before transitioning to Sense products.

EHLERS-DANLOS SYNDROME (EDS)

- Pressure garment selection among users with EDS varies considerably. Some users require very light pressure, while others prefer the strongest possible pressure level.
- A typical challenge for the EDS user group in long-term use of pressure garments is variation in body circumferences, which affects the fit and feel of the product.
- If it is difficult to determine whether light or firmer feeling is needed, a product with light pressure and a medium fabric can be a good starting point.

PRODUCTS FOR SCAPULA WINGING

- For this user group, the most typical pressure level is light, with the product made from either a strong or a medium fabric.

SENSE PRODUCTS FOR CHILDREN

- For children under 10 years of age, the first Sense product ordered is often a product with light pressure and a medium fabric.
- For babies, we recommend a light fabric and light pressure.
- For children with a sensitive gag reflex, it is appropriate to consider a product with light pressure and a light fabric.



LYMED PRODUCT INFORMATION


Colour and emblem options

Lymed materials do not contain rubber, coated rubber or other known allergens.

FABRIC COLOURS



F1 Beige

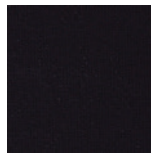

F2 Black



F3 White



F4 Grey



F5 Blue



F6 Red




F7 Green

SEAM COLOURS




S1 Beige

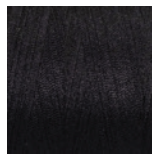

S2 Black



S3 White

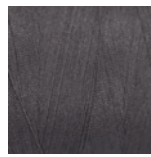

S4 Grey



S5 Blue



S6 Red



S7 Green



S8 Pink



S9 Bright green



S10 Yellow



S11 Orange



S12 Bright blue



S13 Purple

DECORATED EMBLEMS

SIZE (DIAMETER): 2,2 CM



P1 Cat



P2 Dog



P3 Fox



P4 Bunny



P5 Duck



P6 Strawberry



P7 Snowflake



P8 Dino



P9 Bear



P10 Beetle



P11 Ladybug



P12 Turtle



P13 Whale



P14 Flower

Our designers determine the placement of the emblem on each product.

IRRITATION CAUSED BY SILICONE BANDS

Based on literature, most reactions caused by silicones are non-allergic and non-specific. They are typically a result of irritation, which may be caused by factors such as the skin's properties, skin care products, hygiene, or the combination of all these elements. Individual factors, like skin pH, microbes, sweat, skin lotions, and personal hygiene, can contribute to irritation.



Irritation may worsen when sweating increases, such as in higher temperatures. The irritation can be reduced by cleaning the silicone band and the skin beneath it more frequently.

Each product can be manufactured without the silicone band or with the band repositioned. Several edging band options are available, and they can be swapped into the product as needed. In some cases, switching to a model that does not require silicone bands at all could be beneficial (e.g., using different strap options or models with more coverage). The best silicone band options for sensitive skin are SIL 1 and SIL 2.

LYMED PRODUCT INFORMATION

Silicone and elastane bands

Lymed elastane and silicone bands do not contain rubber, coated rubber, or other known allergens. The properties of each Lymed product are outlined on the product pages, including any elastane and silicone bands used in the individual product.

ELASTANE BANDS




ELA 1
Elastane band
Width: 3 cm

ELA 2
Elastane band
Width: 2 cm

ELA 3
Edging band / elastane band
Width: 0,75 cm / 1,5 cm

ELA 4
Elastane band
Width: 5 cm

SILICONE BANDS

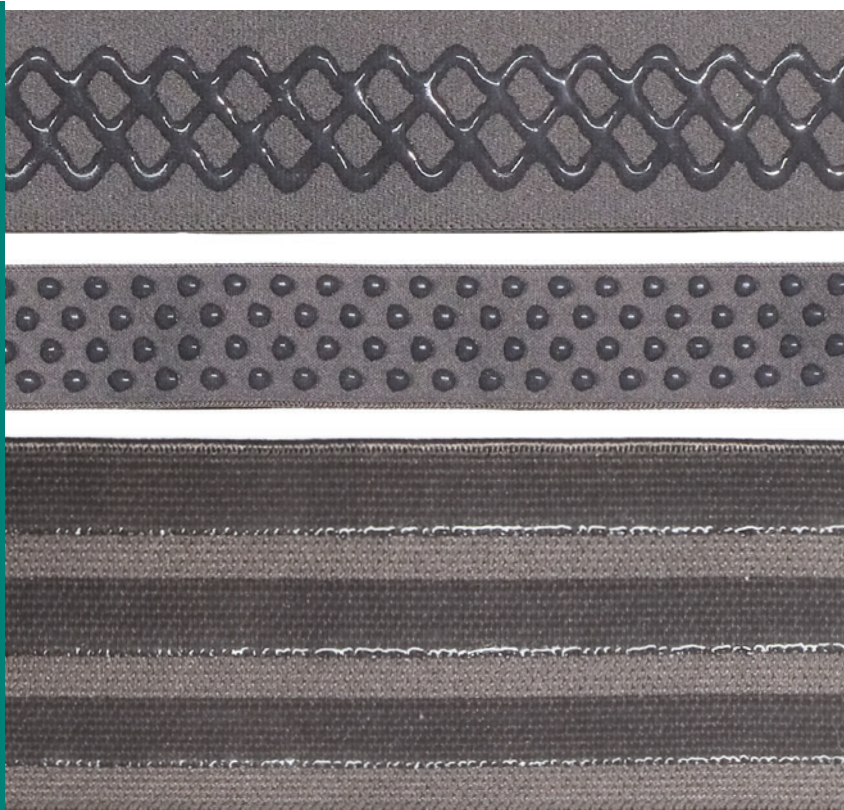

SIL 1
Honeycomb silicone
Width: 3 cm

SIL 2
Droplet silicone
Width: 2 cm

SIL 4
3-stripe silicone
Width: 5 cm

CUSTOMISING LYMED PRODUCTS TO MEET USER NEEDS

HOW CAN WE SUPPORT INDEPENDENT DONNING AND DOFFING?

HOW CAN I HELP ENSURE THAT THE LYMED PRODUCT STAYS IN PLACE?

With our range of additional options, Lymed products can be extensively customised to meet the user's needs. Our catalogue presents the basic Lymed product models as well as the most common additions, enabling us to provide the user with a pressure garment that fits optimally and supports the goals of the overall treatment or care plan.

We are available to answer all product-related questions and to assist in designing a product that meets the needs of both the user and care personnel.

In complex cases, photographs, preliminary measurements, and detailed background information can help clarify the situation.

If there is no known need for additional solutions prior to the first order, Lymed recommends selecting a basic Lymed product model. Additional features may be considered when it is known that the basic model does not meet the needs, independent use of the basic model is difficult, or there is a desire to reduce the need for assisted donning.

OUR SPECIALISED EXPERTISE

- Small-sized products for infants, as well as exceptionally large-sized products.
- Products adapted to individual body shapes, including solutions for dysmelia, extensive tissue defects, and stumps.
- Localised additional support and reinforcement, using double fabrics, splint pockets, and neoprene padding.
- Adjustable straps and hook-and-loop fasteners, to support functional body guidance.
- Special solutions for the genital area, including additional pressure flaps, pockets, and openings.

EXAMPLES OF CUSTOM-MADE PRODUCTS BASED ON USER NEEDS



A customised solution for the amputation area

THE LARGEST SELECTION OF STOCKING CHOICES AVAILABLE!

ADDITIONS FOR EASY USE

There are different solutions available for donning and doffing of Lymed stockings. The most popular is the zipper, which can be manufactured almost anywhere on the product, in all lengths.

Additions to products can be considered when there are difficulties in using the basic models, or there is a need for independent donning and doffing.



Anatomical ankle construction

TIP CHOICES FOR STOCKINGS

Stockings can be manufactured with various tip models depending on the client's personal needs.



Toes closed Toes open Toe sock, toe tips closed Toe sock, toe tips open Tabi sock Big toe sock



Foot glove

ZIPPER CHOICES FOR LYMED STOCKINGS

A zipper is a helpful and fast solution in assistive donning and doffing. In independent donning a zipper may remove the need for other donning and doffing aids.

Zipper solutions are handy in wound care and changing of wound dressings, e.g. in leg ulcers.

A zipper can be placed almost everywhere on the product, based on the preference of the client and how the stocking will be donned. Dressing aid hooks can be added beneath the zipper to ease with the process of closing the zipper.



Full length, open edge Full length, intact edge Partial length

LYMED PRODUCT INFORMATION

Additions and special solutions



Zipper pull tab attached to the zipper slider to facilitate the closing of the zipper.



Pull-up loop attached to the product to aid in dressing.



Hook fastening.



Zipper guard to protect the skin and prevent skin or hair from getting caught in the zipper.

ORDERING ADDITIONALS

Additional features are always ordered together with the product by entering the product code followed by the code(s) of the desired additional feature(s). We are happy to provide a cost estimate if the product requires complex features that differ from the basic model.

EXAMPLE OF ORDERING A STANDARD-SIZED PRODUCT WITH ADDITIONAL FEATURE:

Ordering an Interim & Light vest for men with zipper fastening and dressing aid hook

How to order:



STIN224336F1Z AHZS

STIN224336F1Z

PRODUCT CODE

- Interim & Light Vest
- Men
- Long sleeves
- Size 6
- Beige
- Zipper

AHZS

ADDITION

- Additional aid hook for zipper

EXAMPLE OF ORDERING A CUSTOM-MADE PRODUCT WITH ADDITIONAL FEATURES:

Ordering a compression knee-high stocking with zipper and zipper guard

How to order:



CO20629F2 AZ AZF

CO20629F2

PRODUCT CODE

- Compression knee-high stocking
- Toes closed
- Black

AZ, AZF

ADDITIONS TO PRODUCT

- Additional zipper
- Zipper guard under the zipper







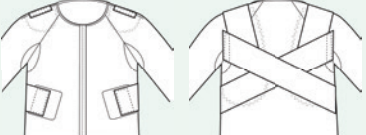
Specify zipper placement in your order




ADDITIONS FOR CUSTOM-MADE PRODUCTS

- Not all additions are available for all product options.
- Lymed will contact the ordering party if the feature cannot be implemented.
- Finishing options are mainly the most common options for adult models. The suitability of these options is assessed on the basis of the order.
- When planning additional options, the product orderer and the healthcare professional responsible for pressure garment therapy should jointly assess the suitability of the solutions for the user's individual situation, context of use, and daily activities.

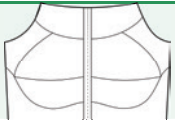Vests and bodys
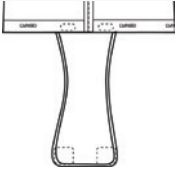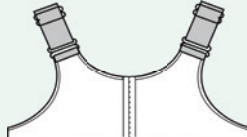
| | Addition code | |
|--|---|---|
| <p>MAMMARY CUPS</p> <p>Includes one or two breast cups made according to the customer's measurements.</p> | MC |  |
| <p>TURTLE NECK</p> <p>Raised collar in double fabric.</p> | AC |  |
| <p>TURTLE NECK WITH NEOPRENE</p> <p>Raised collar in double fabric with neoprene filling.</p> | ACN |  |
| <p>CROTCH PANEL WITH FASTENING</p> <p>A fabric crotch panel attached to a shirt or abdominal support, running between the legs to keep the lower edge in place and prevent the garment from rolling up. The panel is fixed at one end and finished with a snap-fastening at the opposite end. A velcro closure is available on request. Different from a body-style design.</p> | ADF |  |
| <p>ADJUSTABLE STRAPS FOR SLEEVELESS GARMENTS</p> <p>Fixed and adjustable 3 cm wide straps for sleeveless vests and bodysuits. Available on request with detachable straps. Available only in beige, black, white. Products in other colours are made with black straps.</p> | ABV |  |
| <p>SHORT LEGS FOR BODY</p> <p>Short legs extending up to just above the knee.</p> <p>For adult Skin-body</p> <p>For adult Compression-body</p> <p>For adult Sense-body</p> <p>For children Skin-body</p> <p>For children Compression-body</p> <p>For children Sense-body</p> | <p>PRA21L</p> <p>COA21L</p> <p>SEA21L</p> <p>PRA10L</p> <p>COA10L</p> <p>SEA10L</p> |  |
| <p>SENSE- DOUBLE SCAPULA SUPPORT</p> <p>For scapula support vest or scapula support body when double-sided scapula support is needed.</p> | SEE2 |  |

Gloves

| | Addition code | |
|---|---------------|---|
| <p>ADDITIONAL DIGIT TO PALM-GLOVE</p> <p>Addition of one finger. For adding multiple fingers, use the code for glove. The thumb is part of the basic gauntlet model.</p> | ADP |  |

LYMED PRODUCT INFORMATION

Additions for custom-made products

Sleeves

Addition code

SLEEVE FASTENING MODEL 1


Elastic strap running under the breast to hold the sleeve up. Suitable especially for users with large breasts.

For adult sleeves

A20S1

For children's sleeves

A10S1

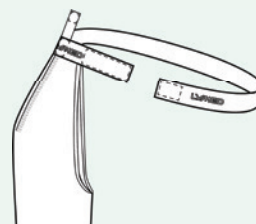

SLEEVE FASTENING MODEL 2


Elastic strap running under the axilla to hold the sleeve up.

For adult sleeves

A20S2

For children's sleeves

A10S2

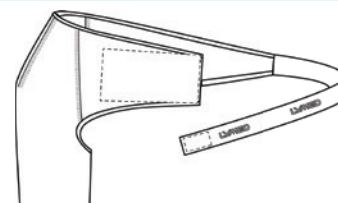

SLEEVE FASTENING MODEL 3


Fastening model used to join two axilla length sleeves. Two sleeves need to be ordered with the fastening.

For adult sleeves

A20S3

For children's sleeves

A10S3

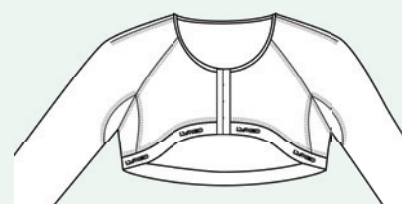


Pants and stockings

Addition code

SHOULDER STRAPS FOR PANTS

Fixed and adjustable 3 cm wide elastic straps for pants. Available only in beige, black, and white. Products in other colours are made with black straps.

AB

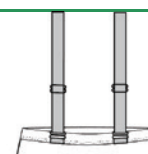

ADDITIONAL FLAP FOR GENITAL AREA

Used in pants to create local additional pressure. Fixed to the crotch, with a velcro fastener on the lower abdomen.

AF



ADDITIONAL FLAP WITH NEOPRENE FOR GENITAL AREA

Used in pants to create local additional pressure. Fixed to the crotch, neoprene filling and with a velcro fastener on the lower abdomen.

AFN



HIGH WAIST, CUSTOM

Increasing the waist height of pants, requires additional circumference measurement.

HW



MATERNITY BELLY FOR PANTS

A maternity-style belly section for custom-made pants. Image on page 73.


AM



DRAWSTRING CHANNEL AT THE WAIST

A fabric drawstring channel for the pants waist. The channel includes an ELA1 band.

ALP

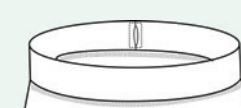

ADDITIONAL TOE FOR A FOOT WITH OPEN TOES OR CLOSED TOES

Addition of a toe for a foot, with open or closed toes. To order the Tabi-sock model. For adding multiple toes, use the code for a foot glove.

ATP



WAIST STRAP

Non-removable, adjustable waist band to help keep the thigh-high stocking or thigh-high leg tube in place.

Adults

A20B

Children

A10B



LYMED PRODUCT INFORMATION

Additions for custom-made products


Use, dressing and fastenings (1/2)

Addition code

ADDITIONAL ZIPPER FASTENING

Length up to 45 cm. Sufficient for example a full-length zipper on a knee-high stocking. Includes a soft zipper pull tab. The zipper is always supplied in the same colour as the fabric.

AZ

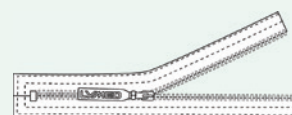

ADDITIONAL LONG ZIPPER FASTENING

Length over 45 cm, for example a full-length zipper for a thigh-high stocking. Includes a soft zipper pull tab. The zipper is always supplied in the same colour as the fabric.

AZ2

ZIPPER GUARD

A soft protective strip made of ELA1 band. Placed underneath the zipper to protect the skin and prevent skin or hair from getting caught in the zipper. For very small products, ELA2 band is used.


AZF



FABRIC ZIPPER GUARD

A softer fabric guard placed under the zipper instead of ELA band. Placed underneath the zipper to protect the skin and prevent skin or hair from getting caught in the zipper. Standard width is 4 cm, or another width, according to the customer's preference.

AZFF

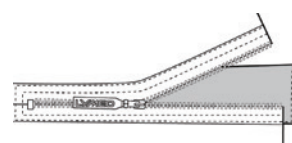

ZIPPER PULL TAB

The zipper pull tab attached to the zipper slider to facilitate the closing of the zipper. There are two options for the length of the pull tab: short (9 cm) and long (15 cm). The default length is short.

AZL



DRESSING AID HOOKS

Individual hooks placed under the zipper to facilitate zipping up. Unless otherwise requested, the default is 1 piece for a knee-high stocking, 2 pieces for a thigh-high stocking, and 2 - 3 pieces for a vest. Available upon request in a colour different from the fabric. Priced in sets of 3.

AHZ



PULL-UP LOOP

A loop attached to the product to aid in dressing. The length and material are determined during pattern making.

APH



ADDITIONAL HOOK FASTENING

An additional hook fastening or an extra row of hooks not included in the basic product model, in addition to any basic fastening.


AH



ADDITIONAL SNAP FASTENERS

An additional snap fastening or an extra row of snaps not included in the basic product model, in addition to any basic fastening.

ASB

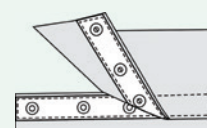

VELCRO FASTENING

Replacement of the fastening included in the product model with Velcro, or addition of extra Velcro fastening. Priced per piece for each started 25 cm of length. The manufacturer determines the width and properties of the Velcro, taking the customer's preferences into account. Velcro fastening is available only in beige, black, and white. Products in other colours are made with black Velcro.

AVF



ADDITIONAL VELCRO TABS

Small individual Velcro tabs. Used, for example, on top of a zipper to make closing it easier. Cannot be applied for functional opening, closing, or adjustment purposes of the product. Priced in sets of 4.


AV



VELCRO FASTENING TO ATTACH PRODUCTS TOGETHER

Eg. attaching vest and pants. Price includes max 4 velcro fasteners and protective velcro pads.

AVSE

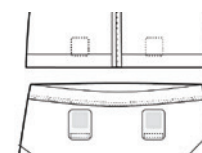

LYMED PRODUCT INFORMATION

Additions for custom-made products

Use, dressing and fastenings (2/2)

Addition code

GRIP SILICONE - INSIDE

Additional silicone band to prevent for example sleeves or stockings from slipping. Can be used to double the edge band or to increase friction on the inside of the fabric. Free choice of silicone band style.


ASI2



GRIP SILICONE - OUTSIDE

A silicone strip placed on the outside of the product to improve grip, for example, on fingertips or the sole of a foot.

ASI

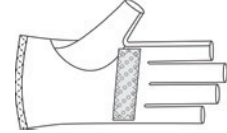

FUNCTIONAL LOOP OR PULL

A functional loop or pull band with Velcro fastening, made according to the customer's specifications. Produced using the standard range of bands.

ALS1



Linings and reinforcements


Addition code

SEAM REINFORCEMENT/LINING WITH FABRIC STRIP

Reinforcing/lining a seam with an additional strip of fabric. Priced per piece and for every started 30 cm of length.

Sewing the strip adds more stitching compared to the original product design. Not suitable for small children's or adult stockings with a narrow forefoot.

ASRF

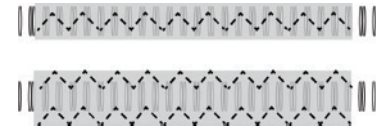

SEAM REINFORCEMENT WITH ADDITIONAL STITCHING

Reinforcement stitching for seams subjected to exceptional stress. Priced per piece and for every started 30 cm of length. Full-product reinforcement is not recommended. Reinforcement slightly changes the feel of the seam. Not available for small children's stockings or for adult stockings with a narrow forefoot.

ASRZ



STOCKING: REINFORCEMENT STITCHING

Reinforcement stitching in four seams: at the heel, ankle, and foot seams. Reinforcement slightly changes the feel of the seam. Not available for small children's stockings or for adult stockings with a narrow forefoot.

ASRZ1



SMALL PARTIAL LINING WITH FABRIC

Partial lining of the product according to the desired design. Used to add pressure, durability or local protection. Requires additional stitching compared to the original product design.

Small partial lining, approx 10 cm x 15 cm without crossing any seams, e.g. ankle, elbow bend, heel.

AL1

Medium-sized partial lining less than 20 cm x 15 cm or lining that extends to two pieces of the product.

AL2

Large partial lining over about 20 cm x 15 cm or lining that extends to three pieces of the product, e.g. shoulder lining.

AL3

SMALL SIZE DOUBLE FABRIC

An entire piece of the product is made with double material to add support or durability. The code is used when doubling is identical to the original piece. Doubling a piece does not require additional stitching. The double fabric often makes the product more rigid, which can affect the dressing.

Small size double fabric

ALD1

Medium-sized double fabric

ALD2

Big size double fabric

ALD3

LYMED PRODUCT INFORMATION

Additions for custom-made products

Pockets

Addition code

POCKET

A feature in a product for inserting soft materials, fillings or small devices.

Small, approx. 10 cm x 15 cm, that does not exceed the product's seam. **AP1**

Medium, approx. 20 cm x 15 cm, or extending to two pieces of the product. **AP2**

Large, approx. 20 cm x 15 cm, or extending to three pieces of the product. **AP3**

POCKET WITH NEOPRENE

A pocket containing neoprene to create additional pressure or to fill hollow areas in the body

Small, approx. 10 cm x 15 cm, that does not exceed the product's seam. **APN1**

Medium, approx. 20 cm x 15 cm, or extending to two pieces of the product. **APN2**

Large, approx. 20 cm x 15 cm, or extending to three pieces of the product. **APN3**

POCKET FOR PROSTHESIS

Pocket for breast prosthesis. Side opening for inserting the prosthesis.

APP



Sense functional supports

Addition code

Read more on page 38.

LIGHT PELVIC FUNCTIONAL SUPPORT

A support made of strong fabric, placed on the outside of the product. Sewn with a stronger seam type than the product's standard seams.

ASPL



STRONG PELVIC FUNCTIONAL SUPPORT

A support made of strong fabric, placed on the outside of the product. Sewn with a stronger seam type than the product's standard seams.


ASPS



FUNCTIONAL SUPPORT, BACK

A support made of strong fabric, placed on the outside of the product. Sewn with a stronger seam type than the product's standard seams.


ASVS

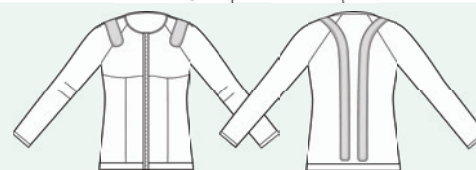

FUNCTIONAL SUPPORT, SHOULDERS

A support made of strong fabric, placed on the outside of the product. Sewn with a stronger seam type than the product's standard seams.

ASVC

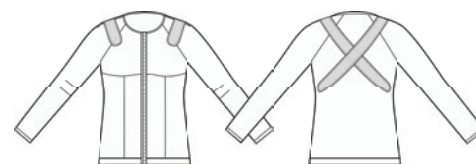

STRONG FUNCTIONAL SUPPORT, SHOULDERS

A support made of strong fabric, placed on the outside of the product. Sewn with a stronger seam type than the product's standard seams.

ASVU



OTHER FUNCTIONAL SUPPORT, PRICE FORM

A support made of strong fabric, placed on the outside of the product and customized according to the customer's requirements. Sewn with a stronger seam type than the product's standard seams. Priced on a case-by-case basis.

ASFM



LYMED PRODUCT INFORMATION

Additions for custom-made products


Miscellaneous

Addition code

ADDITIONAL OBLIQUE SHAPING OF AN EDGE

Available for the hem of vests, upper edge of sleeves, leg tubes, and thigh-high or knee-high stockings. A customised edge design that differs from the standard shape, including asymmetrical or otherwise customer-specified shaping. For example, a raised outer edge on a sleeve or thigh-high stocking, or an extended vest hem.


AME

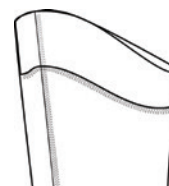

POUCH FOR LIMB STUMP

Shaping the end of a sleeve or pant leg into a pouch. Used for protecting and shaping the stump. Can be used in conjunction with a prosthesis. The shape of the stump determines the pattern: a round base or shaped seams.


AAC

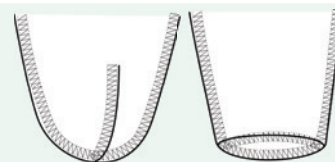

OVERLAPPING SLIT

An openable horizontal slit with overlapping fabric edges, forming an open-style closure. Can be used, for example, for a PEG button. The manufacturer determines the size and construction in proportion to the ordered product, taking the customer's preferences into account.


AOS

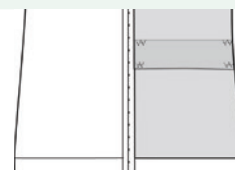

OPENING

An opening in a customer-specified size, for example for a stoma or PEG button, as a ponytail opening in a hood, or as a heel opening in a stocking. The size and placement of the opening in relation to the ordered product may affect the pressure the product provides around the opening area. Opening edges: ELA3 binding tape, or edge stitching for very small openings. Can also be added as a product modification after manufacturing, in which case shipping costs are paid by the customer.


ASTO

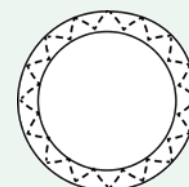

FABRIC FLAP FOR OPENING

A fabric flap to cover the opening. Velcro fastening. Can be used in addition to ASTO code.

AFC

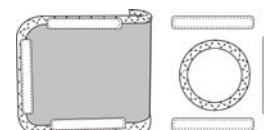

ADDITIONAL NEOPRENE

An additional neoprene piece that is typically identical to the one made for the pocket. Price is for a piece about the size of a palm.

AN



ADDITIONAL EMBLEM

The product includes one free emblem. More emblems are priced per piece. Read more on page 24.

DE

MULTICOLOUR

Manufacturing the product using two fabric colours. The product may consist of only two different fabric colours. The manufacturer determines the placement of the colours, taking the customer's preferences into account.

AMC

CUSTOMISATION OPTIONS FOR STANDARD-SIZED PRODUCTS

- Not all additions are available for all product options.
- Lymed will contact the ordering party if the feature cannot be implemented.
- When planning additional options, the product orderer and the healthcare professional responsible for pressure garment therapy should jointly assess the suitability of the solutions for the user's individual situation, context of use, and daily activities.


Standard-sized vests and sleeves

Addition code

ENLARGED NECKLINE

Enlarged neckline, max 3 cm lower than the standard model. Indicate desired enlargement in centimetres.

ANR

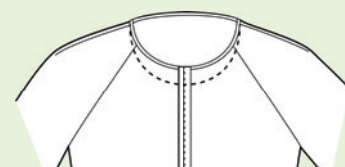

HEMLINE EXTENSION

Indicate desired extension in centimetres. The circumference of the extension is the same as in the original hem.

AVLW

HEMLINE SHORTENING

Indicate desired shortening in centimetres.

AVSW

SLEEVE LENGTHENING

For vest sleeves and separate sleeves. Indicate desired extension in centimetres. The circumference of the extension is the same as in the original sleeve opening.

ASLR

SLEEVE SHORTENING

For vest sleeves and separate sleeves. Indicate desired shortening in centimetres.

ASSR

Standard-sized pants (1/2)

Addition code

OPEN GENITAL AREA

Open crotch area for women's pants.


OS



Shoulder straps

Fixed and adjustable 3 cm wide elastic straps for pants.

ABS

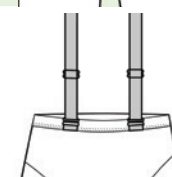

WAIST HEIGHT INCREASE

Indicate desired increase in centimetres. The circumference of the extension is the same as in the original waist.

AHW

LYMED PRODUCT INFORMATION

Customisation options for standard-sized products

Standard-sized pants (2/2)

Addition code

PANT LEGS LENGTHENING

Extension of pant legs. Indicate desired increase in centimetres. The circumference of the extension is the same as in the original leg opening.

ALLL

PANT LEGS LENGTH SHORTENING

Shortening of pant legs. Indicate the amount of shortening in centimeters.

ALLS

MEN'S SEAMLESS FRONT PIECE FOR COMPRESSION PANTS

Front piece without anatomical design, no pouch on genital area.

AMFP




Standard-sized gloves

Addition code

FINGER LENGTH SHORTENING

Shortening only for gloves with open fingertips. The shortening is the same length for all fingers. Indicate desired shortening in centimetres.


ALF

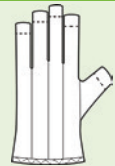

GLOVE WRIST LENGTHENING

Maximum lengthening of the wrist is 5 cm. The circumference of the cuff grows moderately towards the end. Indicate lengthening in centimetres.


AGR

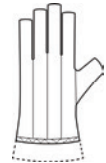

POCKET FOR GLOVE

Pocket on the backhand side of the glove. Desired filling can be added.


AP

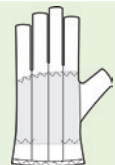

POCKET WITH NEOPRENE FOR GLOVE


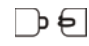
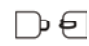
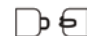
Pocket with neoprene on the backhand side of the glove. Neoprene is removable.

APN


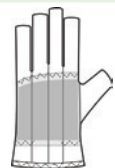



LYMED PRODUCT INFORMATION

Customisation options for standard-sized products

| Fastenings and pulls for standard-sized products | | Addition code | |
|---|--------|---------------|---|
| ADDITIONAL ZIPPER Indicate the desired location of the zipper in the order form. | Left | AZSL |  |
| | Right | AZSR | |
| | Middle | AZSM | |
| | Back | AZSB | |
| DRESSING AID HOOKS FOR ZIPPER Individual hooks underneath the zipper to facilitate zipping up. The amount is standard based on a product from 1 to 3 hooks. | 1 pcs | AHZS |  |
| | 2 pcs | AHZS2 |  |
| | 3 pcs | AHZS3 |  |
| SILICONE STRIP Silicone strip to hold the product in place. Pattern designer decides the location depending on the product. | 1 pcs | ASI2S | |
| | 2 pcs | ASI2S2 | |
| | 3 pcs | ASI2S3 | |
| FASTENING REMOVAL Manufacturing the product without a fastening. Not applicable to bras, support bands, or headwear. | | AHL | |
| FASTENING CHANGE TO ZIPPER Changing the fastening to a zipper fastening. Not applicable to bras, support bands, or headwear. | | AHLZ | |
| FASTENING CHANGE TO HOOKS Changing the fastening to a hook fastening. Not applicable to bras, support bands, or headwear. | | AHLH | |

Standard-sized bras
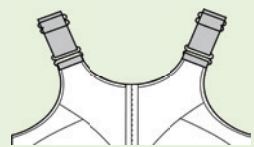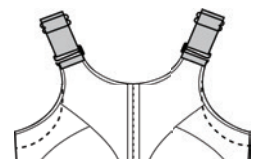
| Standard-sized bras | | Addition code | |
|---|-------|---------------|---|
| FIXED BRA STRAPS Fixed, adjustable straps for models that normally have detachable straps. | | ABB |  |
| COMPRESSION BRA: LOW ARMHOLES 2 cm lower armholes on a standard compression bra. | | AAR |  |
| STRONG SUPPORT BRA: POCKET FOR PROSTHESIS Prosthesis pocket with a side opening for inserting the prosthesis. | Left | APPBL |  |
| | Right | APPBR | |
| | Both | APPBB | |

LYMED PRODUCT INFORMATION

Customisation options for standard-sized products

Miscellaneous for standard-sized products

Addition code

DECORATIVE EMBLEM

| | | | |
|------------|--------------|---------|---------------|
| Cat | DESP1 | Dino | DESP8 |
| Dog | DESP2 | Bear | DESP9 |
| Fox | DESP3 | Beetle | DESP10 |
| Bunny | DESP4 | Ladybug | DESP11 |
| Duck | DESP5 | Turtle | DESP12 |
| Strawberry | DESP6 | Whale | DESP13 |
| Snowflake | DESP7 | Flower | DESP14 |

EDGING OPTION CHANGE

Create the code by selecting the location of the edge, and then the code for the desired edging option.

Examples:

Silicon 2 to upper edge of sleeve: **ACBU SIL2**

Silicone 4 to pants waist: **ACBW SIL4**

Vest sleeve with finishing: **ACBS finishing**

Silicone 1 to vest hemline: **ACBH SIL1**

| | LOCATION | EDGING | |
|--|--------------|-------------|---------------------|
| | Hemline | ACBH | ELA1 |
| | Waist | ACBW | ELA2 |
| | Upper edge | ACBU | ELA3 |
| | Lower edge | ACBD | ELA4 |
| | Leg | ACBL | SIL1 |
| | Sleeve | ACBS | SIL2 |
| | Arm of glove | ACBG | SIL4 |
| | Gauntlet | ACBP | no finishing |
| | Armhole | ACBB | finishing |

MODIFIED LENGTH FOR SUPPORT BELT OR SUSPENSOR

Maximum modification to standard size - 40 cm / +1 5 cm.
The shortest suitable product length is chosen as the basic product for the customer.

AABLR



SENSE FUNCTIONAL SUPPORTS

WHEN TO CHOOSE FUNCTIONAL SUPPORTS?

Supports can be used when the support provided by regular product features is not sufficient or when there is a desire to have a more significant impact on the user's body posture experience.

Functional supports work best when you want to enhance body awareness in a specific area of the body or remind the user to maintain a certain posture. Please note that the experience is always individual.

FUNCTIONAL SUPPORT OR DOUBLE FABRIC?

If the aim of the Sense product is to provide a more supportive effect, you can also order parts of the product with double fabric or lining.

It should be noted that extensive reinforcements will increase the overall pressure level of the product, increase the product's stiffness, and may pose additional challenges when donning.

MORE IS NOT ALWAYS BETTER

When aiming for a localised feeling of additional support, the functional support must be targeted and proportionately small compared to the rest of the product.

If linings, reinforcements and double fabrics are added to the product excessively, it increases the overall pressure and stiffness, causing the sense of localised, targeted support to disappear. In such cases, donning may become more difficult and the sensation may even become uncomfortable.




- We recommend ordering a basic model as a first Lymed product, before trying functional supports.
- We recommend ordering functional supports primarily for long-legged and long-sleeved products.
- The intensity of the sensation provided by the supports is significantly influenced by the customer's body shape, as well as the material and pressure level of the product ordered as a base.
- Supports are made from strong fabric and sewn on the outside of the product.
- The design and size of the supports are standardised and adjusted to the custom-made product.
- Functional supports use a sturdier seam type, different from the other seams of the product.




LYMED PRODUCT INFORMATION

Sense functional supports

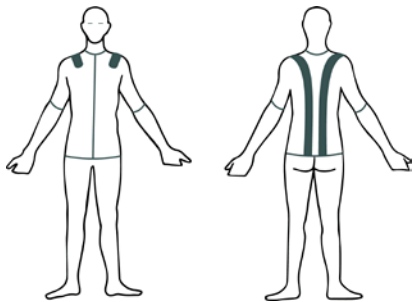


FUNCTIONAL SUPPORT, BACK

Provides a sensation of support to the upper body and shoulders, aiding in perceiving upper body posture. The most significant experiential impact is on maintaining a straight back and relaxing the shoulders.

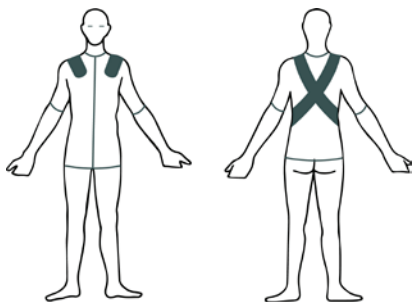


FUNCTIONAL SUPPORT, SHOULDERS

Provides a sense of support to the upper body and shoulders, while aiding in perceiving upper body posture. The most significant experiential impact is on relaxing the shoulders and guiding them backward.

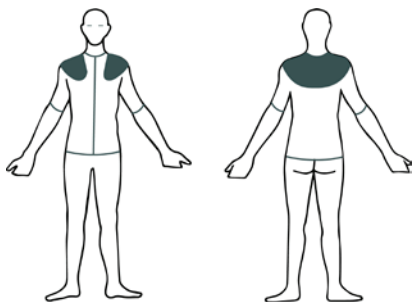


STRONG FUNCTIONAL SUPPORT, SHOULDERS

Provides an extremely strong sensation of support to the shoulders, upper arms, and upper back.

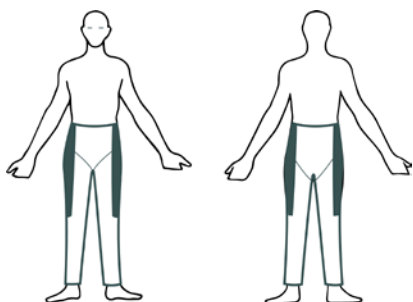

LIGHT PELVIC FUNCTIONAL SUPPORT

Provides a sense of support and posture to the midriff. Helps to perceive the position of the pelvis if the support provided by regular pressure pants is insufficient.

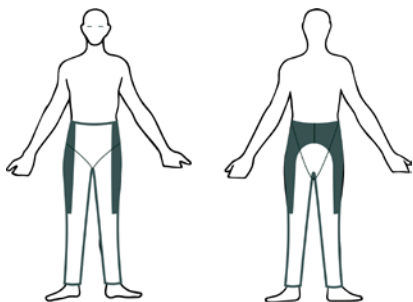

STRONG PELVIC FUNCTIONAL SUPPORT

Provides a strong sense of support and posture to the midriff. Helps to perceive the position of the pelvis if the support provided by regular pressure pants is insufficient.

Addition codes for Sense functional supports are listed on page 32

OTHER FUNCTIONAL SUPPORT

Choose the shape and location freely.



SKIN[™]

**PRESSURE GARMENTS FOR
SCAR MANAGEMENT**

CUSTOM-MADE

STANDARD-SIZE



Product shown in medium fabric without axilla construction



Product picture, code: PR22423F5

| VEST | | CUSTOM-MADE | | | |
|----------|------------------|---------------|-----------------------|--------------|--|
| Model | Product / Art.No | | | | |
| | Sleeveless | Short sleeves | Long and short sleeve | Long sleeves | |
| Women | PR21411 | PR21422 | PR21423 | PR21433 | |
| Men | PR22411 | PR22422 | PR22423 | PR22433 | |
| Children | PR10411 | PR10422 | PR10423 | PR10433 | |

Children's models are made for children under 10.

Product information:

The long-sleeved vest features anatomically shaped elbow bends. In products made from strong fabric, the axilla area uses medium fabric construction. Products made from medium fabric do not include axilla construction. The women's vest features shaping seams at the breasts. The women's model can be manufactured without bust shaping seams free of charge upon request.

| | |
|-----------------|--|
| Material: | Strong or medium |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | 2-row hooks / zipper with guard / no fastening |
| Details: | Hemline SIL 1 |
| | Neckline ELA 3 |
| | Edge of sleeve ELA 2 |
| | Sleeveless model, edging ELA 3 |



| ABDOMINAL SUPPORT | | CUSTOM-MADE |
|-------------------|---------|-------------|
| Model | Art.No | |
| Women | PR21400 | |
| Men | PR22400 | |
| Children | PR10400 | |

Children's models are made for children under 10.

Product information:

Used whenever extra pressure is needed on the midsection.

| | |
|-----------------|--|
| Material: | Strong or medium |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | 2-row hooks / zipper with guard / no fastening |
| Details: | Upper edge SIL 1 |
| | Lower edge SIL 1 |

Product picture, code: PR22400F1



Product picture, code: STPR714F2

| STERNAL STRAP NARROW FRONT | | | STANDARD-SIZE |
|------------------------------|--------------|------------------|---------------|
| Size | Width* cm | Circumf.** cm | Art.No |
| XS | 4,0 | 50 - 62 | STPR712F2 |
| S | 4,4 | 63 - 75 | STPR713F2 |
| M | 4,8 | 76 - 88 | STPR714F2 |
| L | 5,3 | 89 - 101 | STPR715F2 |
| XL | 5,7 | 102 - 114 | STPR716F2 |

Product information:

A two-piece product with hook-and-loop fastening at the sides and shoulders.

The product is designed to apply pressure to the sternal area.

A splint pocket is integrated in the sternal area.

The design is particularly suited for women.

The width of the front section at its narrowest point is 4–5.7 cm, depending on size.

*Width of the front piece at the narrowest part

**Circumference of the chest below breasts.

Material: Polyamide elastane blend

Colour: Black

Pressure level: Adjustable

Fastening: Velcro fastening



Product picture, code: STPR724F2

| STERNAL STRAP WIDE FRONT | | | STANDARD-SIZE |
|----------------------------|--------------|------------------|---------------|
| Size | Width* cm | Circumf.** cm | Art.No |
| S | 10 | 63 - 75 | STPR723F2 |
| M | 10 | 76 - 88 | STPR724F2 |
| L | 10 | 89 - 101 | STPR725F2 |
| XL | 10 | 102 - 114 | STPR726F2 |

Product information:


A two-piece product with hook-and-loop fastening at the sides and shoulders.
The product is designed to apply pressure to the sternal area.
A splint pocket with a removable pad for additional pressure is integrated in the sternal area.

*Width of the front piece is 10 cm at the narrowest part.
**Circumference of the chest below breasts.

| | |
|-----------------|---|
| Material: | Polyamide elastane blend, polyester polyurethane pad |
| Colour: | Black |
| Pressure level: | Adjustable |
| Fastening: | Velcro fastening |



Product picture, code:
PR104033F7

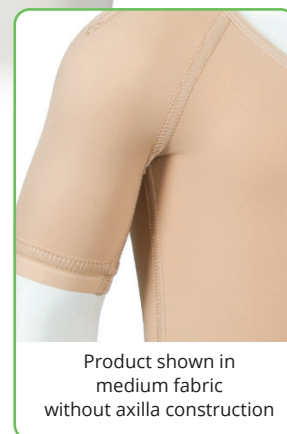

| BODY | | | | CUSTOM-MADE |
|----------|------------------|---------------|-----------------------|--------------|
| Model | Product / Art.No | | | |
| | Sleeveless | Short sleeves | Long and short sleeve | Long sleeves |
| Women | PR214011 | PR214022 | PR214023 | PR214033 |
| Children | PR104011 | PR104022 | PR104023 | PR104033 |

Children's models are made for children under 10.

Product information:

The long-sleeved body features anatomically shaped elbow bends. In products made from strong fabric, the axilla area uses medium fabric construction. Products made from medium fabric do not include axilla construction.

The women's body features shaping seams at the breasts. The women's model can be manufactured without bust shaping seams free of charge upon request. Short legs are available as an addition.

| | |
|-----------------|--|
| Material: | Strong or medium |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Women: 2-row hooks/ zipper with guard 2-row snap fasteners at the crotch Short legs: no crotch closure |
| | Children: 2-row hooks / zipper with guard 2-row snap fasteners at the crotch Short legs: 1-row snap fasteners |
| Details: | Neckline and leg opening ELA 3 Edge of sleeve ELA 2 Sleeveless model, edging ELA 3 Short legs: edge of leg ELA 2 |



Product information:

Long-leg pants with anatomical construction at the knee.
Stocking legs with anatomical construction at the ankle and knee.

When a stocking leg is ordered in strong material, the constructions are made from medium material.
Men's models with contouring at the front.

| | |
|-----------------|----------------------|
| Material: | Strong or medium |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Available separately |
| Details: | Waist ELA 1 |
| | Leg opening ELA 2 |

PANTS CUSTOM-MADE

| Model | Product / Art. No | | | |
|-------|-------------------|---------------------|--------------------------------------|--|
| | Short legs | Short and long legs | Short leg and leg with stocking OPEN | Short leg and leg with stocking CLOSED |
| Women | PR21522 | PR21523 | PR215248 | PR215249 |
| Men | PR22522 | PR22523 | PR225248 | PR225249 |
| Girls | PR11522 | PR11523 | PR115248 | PR115249 |
| Boys | PR12522 | PR12523 | PR125248 | PR125249 |

Product picture, code: PR225249F4

Children's models are made for children under 10.

| Model | Product / Art.No | | | | | | |
|-------|------------------|-------------------------------------|---------------------------------------|--|--|---|---|
| | Long legs | Long leg and leg with stocking OPEN | Long leg and leg with stocking CLOSED | Short leg and leg with toe sock, toe tips OPEN | Short leg and leg with toe sock, toe tips CLOSED | Long leg and leg with toe sock, toe tips OPEN | Long leg and leg with toe sock, toe tips CLOSED |
| Women | PR21533 | PR215348 | PR215349 | PR21524FG8 | PR21524FG9 | PR21534FG8 | PR21534FG9 |
| Men | PR22533 | PR225348 | PR225349 | PR22524FG8 | PR22524FG9 | PR22534FG8 | PR22534FG9 |
| Girls | PR11533 | PR115348 | PR115349 | PR11524FG8 | PR11524FG9 | PR11534FG8 | PR11534FG9 |
| Boys | PR12533 | PR125348 | PR125349 | PR12524FG8 | PR12524FG9 | PR12534FG8 | PR12534FG9 |

Children's models are made for children under 10.

STOCKINGS CUSTOM-MADE

| Model | Product / Art.No | | | | | |
|-------|------------------|-------------|--------------------------------------|--|-------------------------------|---------------------------------|
| | Toes OPEN | Toes CLOSED | One leg with toe sock, toe tips OPEN | One leg with toe sock, toe tips CLOSED | With toe socks, toe tips OPEN | With toe socks, toe tips CLOSED |
| Women | PR215448 | PR215449 | PR21544FG8 | PR21544FG9 | PR21544FG28 | PR21544FG29 |
| Men | PR225448 | PR225449 | PR22544FG8 | PR22544FG9 | PR22544FG28 | PR22544FG29 |
| Girls | PR115448 | PR115449 | PR11544FG8 | PR11544FG9 | PR11544FG28 | PR11544FG29 |
| Boys | PR125448 | PR125449 | PR12544FG8 | PR12544FG9 | PR12544FG28 | PR12544FG29 |

Children's models are made for children under 10.



Product picture, code: PR20603F6



Product picture, code: PR20603F1

| LEG | | CUSTOM-MADE | |
|----------|------------------|-------------|-------------|
| Model | Product / Art.No | | |
| | Ankle-knee | Knee-thigh | Ankle-thigh |
| Adults | PR20601 | PR20602 | PR20603 |
| Children | PR10601 | PR10602 | PR10603 |

Children's models are made for children under 10.

Product information:

The leg sleeve features an anatomically shaped construction in the knee bend area.

When the product is ordered in strong fabric, the knee bend construction is made from medium fabric.

Material: Strong, medium

Colours: All

Pressure level: Selectable

Fastening: Available separately

Details: Upper edge SIL 4, upper edge in knee-high SIL 1, lower edge ELA 2.



Product picture, code:
PR20639F4 A20B



Product picture, code:
PR20629F4 AZ



Product picture, code:
PR20619F4



Product picture, code:
PRA20T9F2

STOCKING

CUSTOM-MADE

| Model | Toes | Product / Art.No | | |
|----------|------------------|------------------|-----------|------------|
| | | Ankle-high | Knee-high | Thigh-high |
| Adults | Closed | PR20619 | PR20629 | PR20639 |
| Adults | Open | PR20618 | PR20628 | PR20638 |
| Adults | Toe sock, closed | PR20619FG | PR20629FG | PR20639FG |
| Adults | Toe sock, open | PR20618FG | PR20628FG | PR20638FG |
| Children | Closed | PR10619 | PR10629 | PR10639 |
| Children | Open | PR10618 | PR10628 | PR10638 |
| Children | Toe sock, closed | PR10619FG | PR10629FG | PR10639FG |
| Children | Toe sock, open | PR10618FG | PR10628FG | PR10638FG |

Children's models are made for children under 10.

Product information:

In ankle-length, knee-length, and thigh-length stockings, the ankle features an anatomically shaped construction.

In thigh-length stockings, there is additionally an anatomically shaped knee bend.

When the stocking is ordered in strong material, the constructions are made from medium material. In open-toe stockings, the toe is lightly beveled.

The ankle sock's shaft length is measured upwards from the L/R61 circumference. Please note that the shaft length can be at most the same as the L/R67 measurement.

| | |
|-----------------|---|
| Material: | Strong or medium |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Available separately |
| Details: | Upper edge SIL 1, upper edge in thigh-high SIL 4, lower edge, tip open ELA 2. |

FOOT GLOVE

CUSTOM-MADE

| Model | Toe tips | Art.No |
|----------|----------|---------|
| Adults | Closed | PRA20T9 |
| Adults | Open | PRA20T8 |
| Children | Closed | PRA10T9 |
| Children | Open | PRA10T8 |

Children's models are made for children under 10.

Product information:

The product can be used together with an open-toe stocking or separately.

| | |
|-----------------|------------------|
| Material: | Strong or medium |
| Colours: | All |
| Pressure level: | Selectable |
| Details: | No finishing |



Product picture, code: PR20303F1



Product picture, code: PR2031038F1



Product picture, code: PR2031138F1

| SLEEVE | | CUSTOM-MADE | |
|----------|------------------|--------------|--------------|
| Model | Product / Art.No | | |
| | Wrist-elbow | Elbow-axilla | Wrist-axilla |
| Adults | PR20301 | PR20302 | PR20303 |
| Children | PR10301 | PR10302 | PR10303 |

Children's models are made for children under 10.

| SLEEVE GAUNTLET | | CUSTOM-MADE | |
|-----------------|--------|------------------|-----------|
| Model | Thumb | Product / Art.No | |
| | | To elbow | To axilla |
| Adults | Closed | PR2031019 | PR2031039 |
| Adults | Open | PR2031018 | PR2031038 |
| Children | Closed | PR1031019 | PR1031039 |
| Children | Open | PR1031018 | PR1031038 |

Children's models are made for children under 10.

| SLEEVE GLOVE | | CUSTOM-MADE | |
|--------------|------------|------------------|-----------|
| Model | Fingertips | Product / Art.No | |
| | | To elbow | To axilla |
| Adults | Closed | PR2031119 | PR2031139 |
| Adults | Open | PR2031118 | PR2031138 |
| Children | Closed | PR1031119 | PR1031139 |
| Children | Open | PR1031118 | PR1031138 |

Children's models are made for children under 10.

Product information:

The wrist-to-axilla model features an anatomical elbow bend.

| | |
|-----------------|--------------------------------------|
| Material: | Strong or medium |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Available separately |
| Details: | Upper edge SIL 1 Lower edge ELA 2 |

Product information:

Axilla-length model with anatomical elbow construction. Anatomical construction in the thumb crotch.

| | |
|-----------------|--------------------------------------|
| Material: | Strong or medium |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Available separately |
| Details: | Upper edge SIL 1 Lower edge ELA 2 |

Product information:

Axilla-length model with anatomical elbow construction. Anatomical construction between the fingers and in the thumb crotch.

| | |
|-----------------|----------------------|
| Material: | Strong or medium |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Available separately |
| Details: | Upper edge SIL 1 |

SLEEVE FASTENING - MODEL 1 CUSTOM-MADE


| Model | Product / Art.No |
|--------------------------------|------------------|
| Model 1 - Shoulder flap | |
| Adults | A20S1 |
| Children | A10S1 |

Children's models are made for children under 10.

Product information:

Elastic strap running under the breast to hold the sleeve up. Fastening with velcro. Suitable especially for users with large breasts. Available for all sleeves reaching up to axilla. Women's sleeve with a bra strap holder.

| | |
|-----------|----------------|
| Material: | ELA 1 |
| Colour: | Same as sleeve |

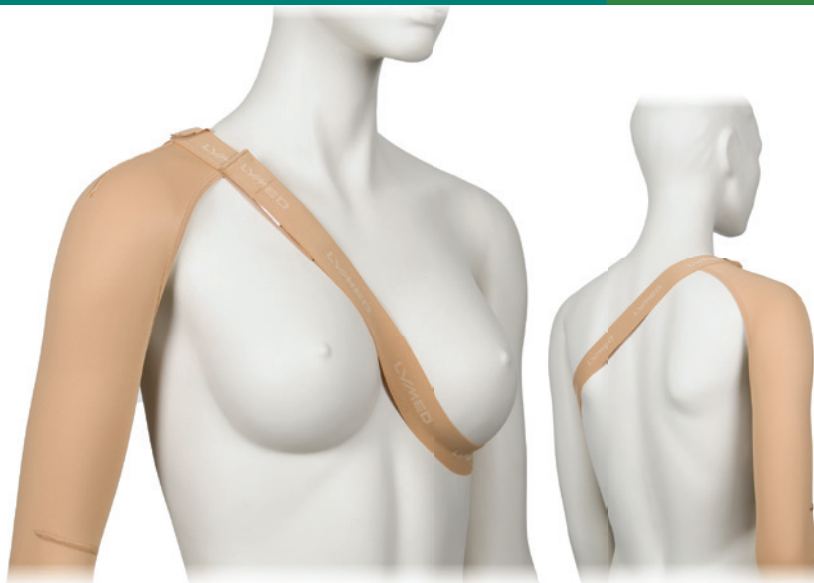

SLEEVE FASTENING - MODEL 2 CUSTOM-MADE


| Model | Product / Art.No |
|--------------------------------|------------------|
| Model 2 - Shoulder flap | |
| Adults | A20S2 |
| Children | A10S2 |

Children's models are made for children under 10.

Product information:

Elastic strap running under the axilla to hold the sleeve up. Fastening with velcro. Available for all sleeves reaching up to axilla.

| | |
|-----------|----------------|
| Material: | ELA 1 |
| Colour: | Same as sleeve |

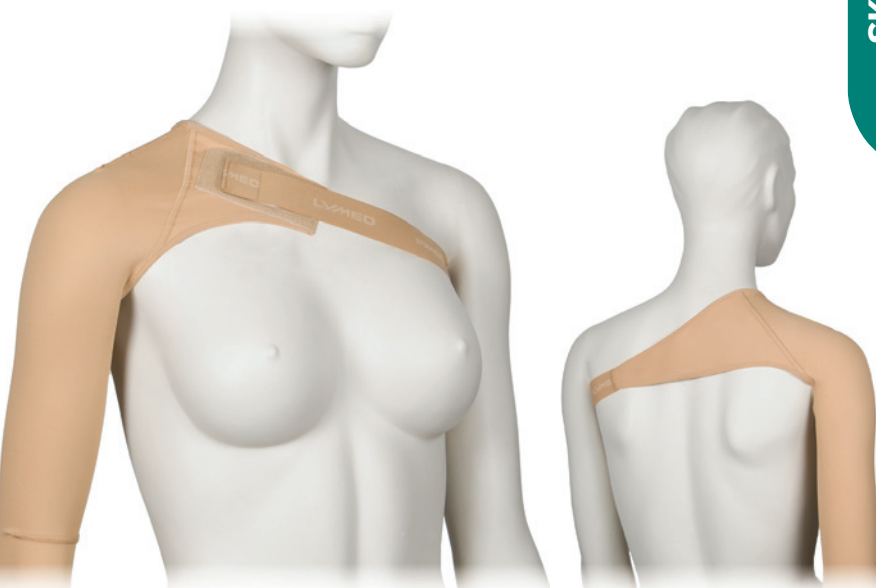

SLEEVE FASTENING - MODEL 3 CUSTOM-MADE

| Model | Product / Art.No |
|----------------------------------|------------------|
| Model 3 - Opposite sleeve | |
| Adults | A20S3 |
| Children | A10S3 |

Children's models are made for children under 10.


Product information:

In the sleeve attachment model, a counter-sleeve connects two axilla-length sleeves. The model is suitable for situations where maintaining the position of one or both sleeves is challenging. The counter-sleeve configuration can be used in different setups: to connect two sleeves or to support the positioning of a single sleeve.

Two sleeves must be ordered in conjunction with the attachment. If the ordered sleeves are made from different materials, the attachment component is manufactured by default using the lighter material.

| | |
|------------|---------------------------------|
| Material: | Strong or medium |
| Colour: | Same as sleeve |
| Fastening: | 2-row hooks / zipper with guard |
| Details: | Neckline ELA 3 |
| | Hemline SIL 2 |
| | Edge of sleeves ELA 2 |

Product pictured, made from medium material

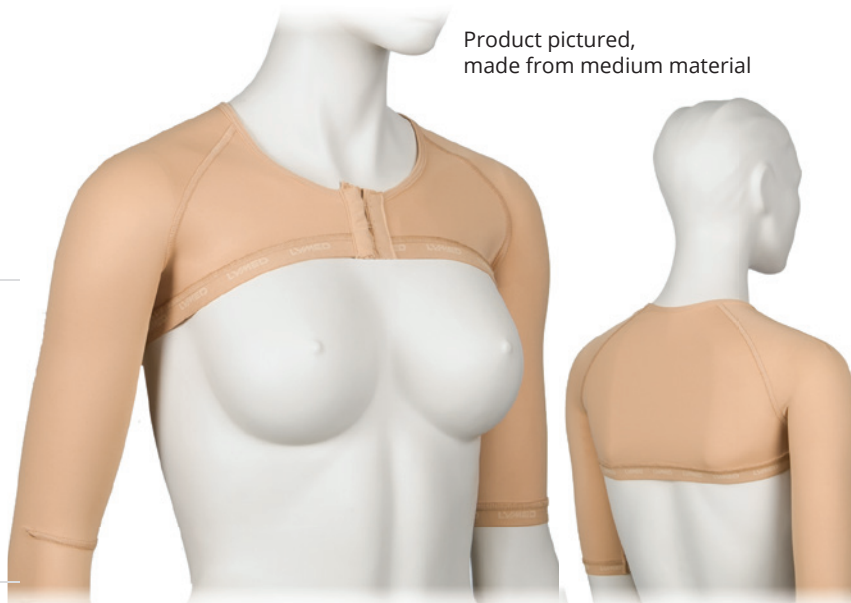

See also vest model on p. 58



GLOVE CUSTOM-MADE

| Model | Product / Art. No | |
|----------|-------------------|-----------------|
| | Fingertips closed | Fingertips open |
| Adults | PR203119 | PR203118 |
| Children | PR103119 | PR103118 |

Children's models are made for children under 10.

Product information:

Anatomical construction between the fingers and in the thumb fourchette. Number of backhand seams/darts in a glove may vary due to individual measurements and pressure/compression levels. Order a sleeve-glove when the length of the shaft is greater than the length of the palm.

| | |
|-----------------|----------------------|
| Material: | Strong or medium |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Available separately |
| Details: | Wrist edging ELA 3 |

Product picture, code:
PR203119F1 (Custom-made)
STPR31194RF1 (Standard-size)

GLOVE STANDARD-SIZE

| Size | Circumf.* | Colour | Product / Art.No | | | |
|------|-----------|--------|-------------------|--------------|-----------------|--------------|
| | | | Fingertips closed | | Fingertips open | |
| cm | | | Right | Left | Right | Left |
| XXS | 14 - 16 | Beige | STPR31191RF1 | STPR31191LF1 | STPR31181RF1 | STPR31181LF1 |
| XXS | 14 - 16 | Black | STPR31191RF2 | STPR31191LF2 | STPR31181RF2 | STPR31181LF2 |
| XS | 17 - 18 | Beige | STPR31192RF1 | STPR31192LF1 | STPR31182RF1 | STPR31182LF1 |
| XS | 17 - 18 | Black | STPR31192RF2 | STPR31192LF2 | STPR31182RF2 | STPR31182LF2 |
| S | 19 - 20 | Beige | STPR31193RF1 | STPR31193LF1 | STPR31183RF1 | STPR31183LF1 |
| S | 19 - 20 | Black | STPR31193RF2 | STPR31193LF2 | STPR31183RF2 | STPR31183LF2 |
| M | 21 - 22 | Beige | STPR31194RF1 | STPR31194LF1 | STPR31184RF1 | STPR31184LF1 |
| M | 21 - 22 | Black | STPR31194RF2 | STPR31194LF2 | STPR31184RF2 | STPR31184LF2 |
| L | 23 - 24 | Beige | STPR31195RF1 | STPR31195LF1 | STPR31185RF1 | STPR31185LF1 |
| L | 23 - 24 | Black | STPR31195RF2 | STPR31195LF2 | STPR31185RF2 | STPR31185LF2 |
| XL | 25 - 26 | Beige | STPR31196RF1 | STPR31196LF1 | STPR31186RF1 | STPR31186LF1 |
| XL | 25 - 26 | Black | STPR31196RF2 | STPR31196LF2 | STPR31186RF2 | STPR31186LF2 |
| XXL | 27 - 28 | Beige | STPR31197RF1 | STPR31197LF1 | STPR31187RF1 | STPR31187LF1 |
| XXL | 27 - 28 | Black | STPR31197RF2 | STPR31197LF2 | STPR31187RF2 | STPR31187LF2 |

Product picture, code:
PR203118F1 (Custom-made)
STPR31184RF1 (Standard-size)

Product information:

| | | |
|--|-----------------|--------------------|
| Anatomical construction between the fingers. | Material: | Strong |
| *Circumference at MCP | Colours: | Beige, black |
| | Pressure level: | 23 - 30 mmHg |
| | Details: | Wrist edging ELA 3 |



Product picture, code:
PR203108F1 (Custom-made)
STPR3104RF1 (Standard-size)

| GAUNTLET | | CUSTOM-MADE |
|----------|------------------|-------------|
| Model | Product / Art.No | |
| | Thumb closed | Thumb open |
| Adults | PR203109 | PR203108 |
| Children | PR103109 | PR103108 |

Children's models are made for children under 10.

Product information:

Anatomical construction in the thumb fourchette. Order a sleeve-gauntlet when the length of the shaft is greater than the length of the palm.

| | |
|-----------------|---|
| Material: | Strong or medium |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Available separately |
| Details: | Wrist edging ELA 3 Palm edging ELA 2 |

| GAUNTLET | | | STANDARD-SIZE | |
|----------|-----------|--------|------------------|-------------|
| Size | Circumf.* | Colour | Product / Art.No | |
| | cm | | Thumb open | |
| | | | Right | Left |
| XXS | 14 - 16 | Beige | STPR3101RF1 | STPR3101LF1 |
| XXS | 14 - 16 | Black | STPR3101RF2 | STPR3101LF2 |
| XS | 17 - 18 | Beige | STPR3102RF1 | STPR3102LF1 |
| XS | 17 - 18 | Black | STPR3102RF2 | STPR3102LF2 |
| S | 19 - 20 | Beige | STPR3103RF1 | STPR3103LF1 |
| S | 19 - 20 | Black | STPR3103RF2 | STPR3103LF2 |
| M | 21 - 22 | Beige | STPR3104RF1 | STPR3104LF1 |
| M | 21 - 22 | Black | STPR3104RF2 | STPR3104LF2 |
| L | 23 - 24 | Beige | STPR3105RF1 | STPR3105LF1 |
| L | 23 - 24 | Black | STPR3105RF2 | STPR3105LF2 |
| XL | 25 - 26 | Beige | STPR3106RF1 | STPR3106LF1 |
| XL | 25 - 26 | Black | STPR3106RF2 | STPR3106LF2 |
| XXL | 27 - 28 | Beige | STPR3107RF1 | STPR3107LF1 |
| XXL | 27 - 28 | Black | STPR3107RF2 | STPR3107LF2 |

Product information:

*Circumference at MCP

| | |
|-----------------|---|
| Material: | Strong |
| Colours: | Beige, black |
| Pressure level: | 23 - 30 mmHg |
| Details: | Wrist edging ELA 3 Palm edging ELA 2 |



Product picture, code:
STPR2037F2

| WEB SPACER | | STANDARD-SIZE |
|-----------------|---------------|-------------------------|
| <i>Model</i> | <i>Colour</i> | <i>Product / Art.No</i> |
| One size | | |
| Adults | Beige | STPR2037F1 |
| Adults | Black | STPR2037F2 |
| Children | Beige | STPR1037F1 |
| Children | Black | STPR1037F2 |

Product information:

Web spacer gives extra pressure between fingers. Product is adjustable and suitable for both hands. Intended to be worn over a pressure glove.

| | |
|-----------------|--------------------------------|
| Material: | Elastic velcro, elastane strap |
| Colours: | Beige, black |
| Pressure level: | Adjustable |
| Fastening: | Velcro |



Product picture, code:
PR203128F1 (Custom-made)
STPR334F1 (Standard-size)

| DIGITAL CAP | | CUSTOM-MADE |
|-------------|------------------|-------------|
| Model | Product / Art.No | |
| | Tip closed | Tip open |
| Adults | PR203129 | PR203128 |
| Children | PR103129 | PR103128 |

Children's models are made for children under 10.

Product information:

The digital cap is available in tip open and tip closed versions.

| | |
|-----------------|------------------|
| Material: | Strong or medium |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | No fastening |
| Details: | No finishing |


| DIGITAL CAP | | | | STANDARD-SIZE |
|-------------|----------------|-----------|--------|---------------|
| Model | Product length | Circumf.* | Colour | Art.No |
| | cm | cm | | Tip open |
| XS | 5,0 | 4 - 5 | Beige | STPR332F1 |
| S | 5,5 | 5 - 6 | Beige | STPR333F1 |
| M | 6,5 | 6 - 7 | Beige | STPR334F1 |
| L | 7,5 | 7 - 8 | Beige | STPR335F1 |
| XL | 8,5 | 8 - 9 | Beige | STPR336F1 |

Product information:

Can be used on the fingers of both hands.

*Circumference around the PIP joint.

| | |
|-----------------|--------------|
| Material: | Medium |
| Colour: | Beige |
| Pressure level: | 23 - 30 mmHg |
| Fastening: | No fastening |
| Details: | No finishing |

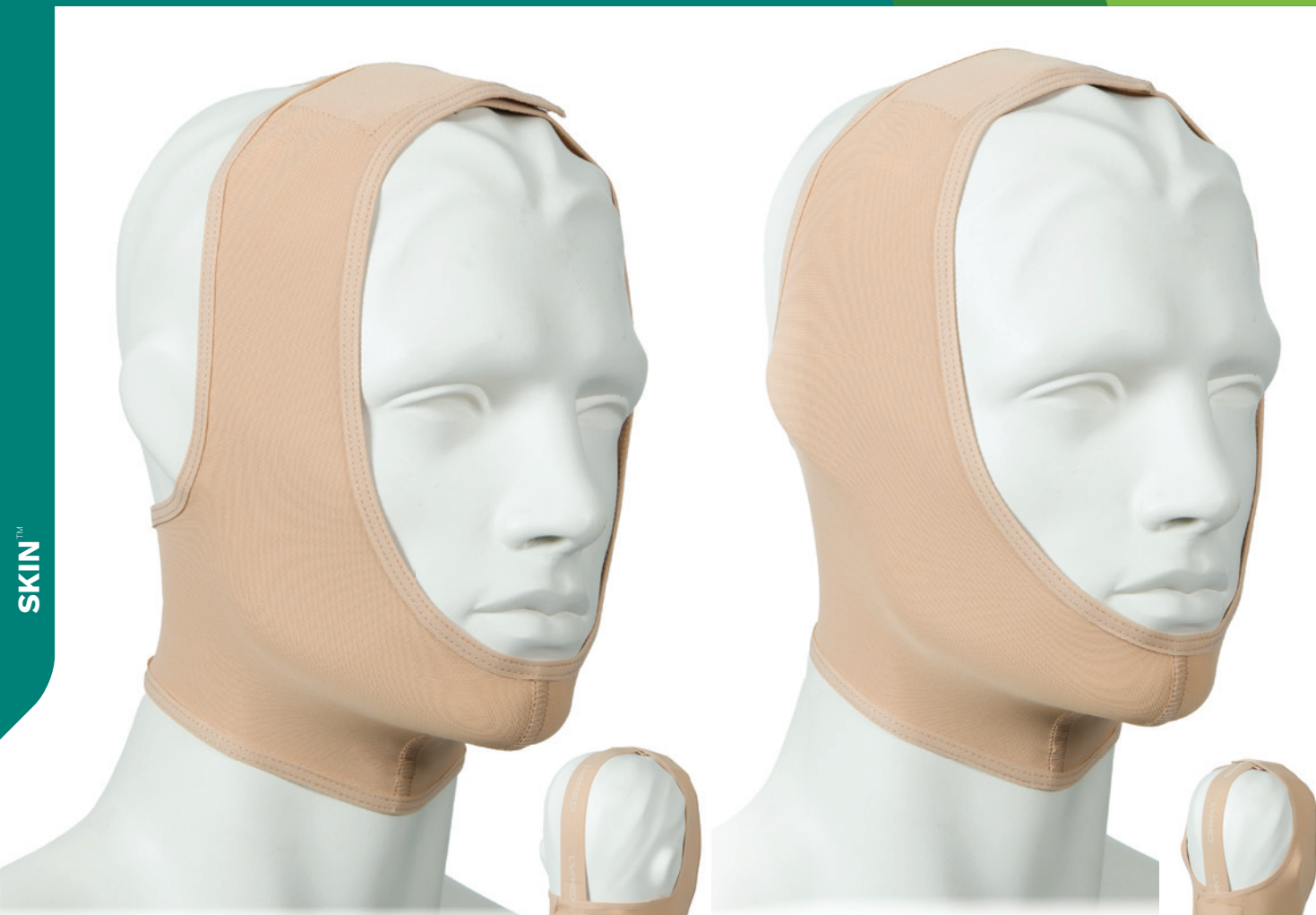

Product picture, code:
PR2011F1 (Custom-made)
STPR114F1 (Standard-size)

Product picture, code:
PR2012F1 (Custom-made)
STPR124F1 (Standard-size)

| CHIN STRAP | | CUSTOM-MADE |
|------------|------------------|-----------------|
| Model | Product / Art.No | |
| | Ear area open | Ear area closed |
| Adults | PR2011 | PR2012 |
| Children | PR1011 | PR1012 |

Children's models are made for children under 10.

Product information:

Velcro fastenings on the crown of the head and back of the neck.

| | |
|-----------------|-------------------|
| Material: | Strong or medium |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | ELA 1 with velcro |
| Details: | Edging ELA 3 |

| CHIN STRAP | | STANDARD-SIZE | |
|------------|-----------|------------------|-----------------|
| Size | Circumf.* | Product / Art.No | |
| | cm | Ear area open | Ear area closed |
| XXS | 50 - 54 | STPR111F1 | STPR121F1 |
| XS | 55 - 58 | STPR112F1 | STPR122F1 |
| S | 59 - 62 | STPR113F1 | STPR123F1 |
| M | 63 - 66 | STPR114F1 | STPR124F1 |
| L | 67 - 70 | STPR115F1 | STPR125F1 |
| XL | 71 - 74 | STPR116F1 | STPR126F1 |

Product information:

Velcro fastenings on the crown of the head and back of the neck.

*Circumference from crown to chin.

| | |
|-----------------|-------------------|
| Material: | Strong |
| Colour: | Beige |
| Pressure level: | Adjustable |
| Fastening: | ELA 1 with velcro |
| Details: | Edging ELA 3 |



Product picture, code:
PR2010F1

| HEADBAND | | CUSTOM-MADE |
|----------|------------------|-------------|
| Model | Product / Art.No | |
| Adults | PR2010 | |
| Children | PR1010 | |

Children's models are made for children under 10.

Product information:

Velcro fastening at the back of the head.

| | |
|-----------------|------------------|
| Material: | Strong or medium |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Velcro |
| Details: | Edging ELA 3 |



Product picture, code:
PR20138F1



Product picture, code:
PR20139F1


| HELMET | | CUSTOM-MADE |
|----------|------------------|-------------|
| Model | Product / Art.No | |
| | Face open | Face closed |
| Adults | PR20138 | PR20139 |
| Children | PR10138 | PR10139 |

Children's models are made for children under 10.

Product information:

Two different models are available, one of which features a closed face area. The closed-face model is designed to keep scar and wound care products, dressings, or a facial mask securely in place. If necessary, openings can be cut into the face area for the mouth, nose, and eyes. The material does not fray after cutting.

| | |
|-----------------|--|
| Material: | Strong or medium |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | 1-row hooks / zipper with guard From back of head to neck |
| Details: | Edging ELA 3 |

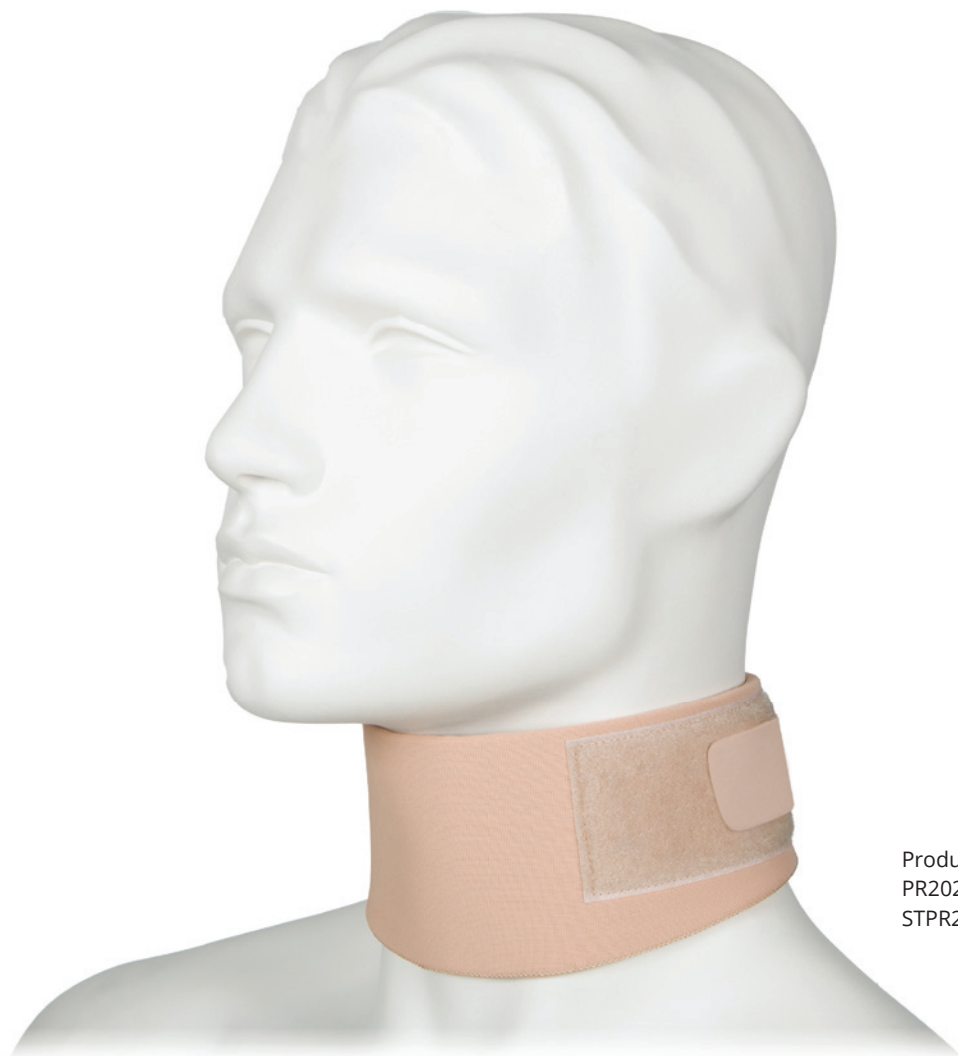

Product picture, code:
PR202F1 (Custom-made)
STPR24F1 (Standard-size)

| PRESSURE COLLAR | | CUSTOM-MADE |
|-----------------|--------|-------------|
| Model | Art.No | |
| Adults | PR202 | |
| Children | PR102 | |

Children's models are made for children under 10.

Product information:

Collar with 3 mm neoprene filling.
Velcro fastening in the back of the neck.

| | |
|-----------------|--------------------|
| Material: | Strong Neoprene |
| Colours: | All |
| Pressure level: | Selectable |
| Pressure level: | Adjustable |
| Fastening: | Velcro |

| PRESSURE COLLAR | | | | | STANDARD-SIZE |
|-----------------|----------|-----------------|--------|------|---------------|
| Size | Model | Circumf.* cm | Height | | Art.No |
| | | | Front | Back | |
| XXS | Children | 20 - 24 | 4,5 | 3,0 | STPR21F1 |
| XS | Children | 25 - 30 | 4,5 | 3,0 | STPR22F1 |
| S | Adults | 31 - 35 | 7,0 | 5,5 | STPR23F1 |
| M | Adults | 36 - 40 | 7,5 | 5,5 | STPR24F1 |
| L | Adults | 41 - 45 | 8,0 | 5,5 | STPR25F1 |
| XL | Adults | 46 - 50 | 9,0 | 6,0 | STPR26F1 |

Product information:

Collar with 3 mm neoprene filling. Velcro fastening in the back of the neck.
*Circumference around the neck.

| | |
|-----------------|----------------------|
| Material: | Strong with neoprene |
| Colour: | Beige |
| Pressure level: | Adjustable |
| Fastening: | Velcro |




COMPRESSIONTM

MEDICAL COMPRESSION GARMENTS

CUSTOM-MADE

STANDARD-SIZE

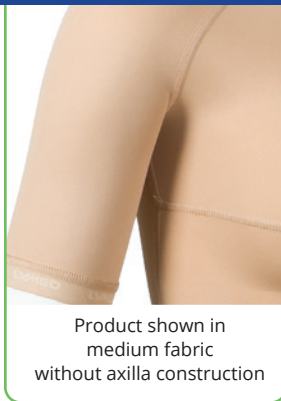

Product shown in medium fabric without axilla construction



Product picture, code: CO21433F7

Product picture, code: CO21433F1, measured to stop under the breasts
Smaller picture: Vest with mammary cups addition (MC)

VEST **CUSTOM-MADE**

| Model | Product / Art.No | | | |
|----------|------------------|---------------|-----------------------|--------------|
| | Sleeveless | Short sleeves | Long and short sleeve | Long sleeves |
| Women | CO21411 | CO21422 | CO21423 | CO21433 |
| Men | CO22411 | CO22422 | CO22423 | CO22433 |
| Children | CO10411 | CO10422 | CO10423 | CO10433 |

Children's models are made for children under 10.

Separate abdominal support available for midsection: please see product details from Lymed Skin product group page 41


Product information:

The long-sleeved shirt features anatomically shaped elbow construction. In products made from strong fabric, the axilla area uses medium fabric construction. Products made from medium fabric do not include axilla construction. The women's vest features shaping seams at the breasts. The women's model can be manufactured without bust shaping seams free of charge upon request.

| | |
|--------------------|--|
| Material: | Strong (Ccl 1-4) or medium (Ccl 1-3) |
| Colours: | All |
| Compression level: | All available |
| Fastening: | 2-row hooks / zipper with guard / none |
| Details: | Hemline: SIL 1 |
| | Neckline: ELA 3 |
| | Edge of sleeve ELA 2 |
| | Sleeveless model: edging ELA 3 |



Product picture, code: CO214022F4

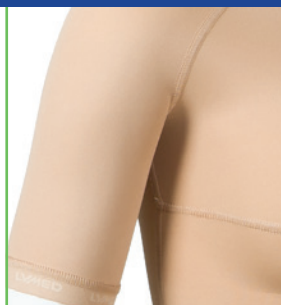

Product shown in medium fabric without axilla construction



Product information:

The long-sleeved body features anatomically shaped elbow bends. In products made from strong fabric, the underarm area uses medium fabric construction. Products made from medium fabric do not include axilla construction. The women's body features shaping seams at the breasts. The women's model can be manufactured without bust shaping seams free of charge upon request. Short legs are available as an addition.

Material: Strong (Ccl 1-4) or medium (Ccl 1-3)
 Colours: All
 Compression level: All available

Women: 2-row hooks / zipper with guard
 2-row snap fasteners at the crotch
 Short legs: no crotch closure

Children: 2-row hooks / zipper with guard
 2-row snap fasteners at the crotch
 Short legs: 1-row snap fasteners

Fastening:
 Details:
 Neckline and leg opening ELA 3
 Edge of sleeve ELA 2
 Sleeveless model: edging ELA 3
 Short legs: edge of leg ELA 2

BODY CUSTOM-MADE

| Model | Product / Art.No | | | |
|----------|------------------|---------------|-----------------------|--------------|
| | Sleeveless | Short sleeves | Long and short sleeve | Long sleeves |
| Women | CO214011 | CO214022 | CO214023 | CO214033 |
| Children | CO104011 | CO104022 | CO104023 | CO104033 |

Children's models are made for children under 10.



Product picture, code: CO7464F3



Product picture, code: CO7466F3

| BRA | | CUSTOM-MADE |
|--------------|------------------|-------------|
| Model | Product / Art.No | |
| Two cups | CO7464 | |
| One cup | CO7465 | |
| Without cups | CO7466 | |

Product information:

Sturdy compression bra designed for oedema management, particularly in the side and underarm areas. Shirt-style design with front fastening positioned between the breasts.

Available with two cups, one cup, or without cups. Prosthesis pockets are available separately as an addition.

| | |
|--------------------|--|
| Material: | Strong (Ccl 1-4) or medium (Ccl 1-3) |
| Colours: | All* |
| Compression level: | All available |
| Fastenings: | 1-row hooks / 2-row hooks / zipper with guard and two dressing aid hooks |
| Details: | Neckline ELA 3 |
| | Edging ELA 3 |
| | Lower edge selectable |

* 3 cm width adjustable straps available only in beige, black or white

BRA AND BRA TOP STANDARD SELECTION

STRAPS



Fabric, non-adjustable

Open hook, adjustable

Closed hook, adjustable

BACK NECKLINE



High

Low

FRONT NECKLINE



Medium height

Low

ARMHOLE



High armhole

Low armhole



Product picture, code: CO7474F3



Product picture, code: CO7476F3

| BRA TOP | | CUSTOM-MADE |
|--------------|------------------|-------------|
| Model | Product / Art.no | |
| Two cups | CO7474 | |
| One cup | CO7475 | |
| Without cups | CO7476 | |

* 3 cm width adjustable straps available only in beige, black or white

Product information:

A sturdy compression bra designed for oedema management, particularly in the side and underarm areas. Vest style design with front fastening positioned between the breasts. Available with two cups, one cup, or without cups. Prosthesis pockets are available separately as an addition.

| | |
|--------------------|--|
| Material: | Strong (Ccl 1-4) or medium (Ccl 1-3) |
| Colours: | All* |
| Compression level: | All available |
| Fastenings: | 1-row hooks / 2-row hooks / zipper with guard and three dressing aid hooks |
| Details: | Neckline ELA 3 |
| | Edging ELA 3 |
| | Lower edge selectable |



Product picture, code: STCO754F3H

COMPRESSION BRA

STANDARD-SIZE

Product information:

A sturdy compression bra designed for oedema management, particularly in the side and underarm areas. Vest style design with front fastening positioned between the breasts. Available with two cups, one cup, or without cups, with or without prosthesis pockets.

**Circumference at the chest below the breasts.*

| | |
|--------------------|---|
| Material: | Strong Pocket for prosthesis: Medium |
| Colours: | Black, white |
| Compression level: | Ccl 2 |
| Fastening: | 1-row hooks / zipper with guard |
| Details: | Hemline ELA 1 Edging ELA 3 |



COMPRESSION BRA | HOOKS

STANDARD-SIZE

| Size | Circumf.* cm | Colour | Product / Art.No | | | | | | |
|------|-----------------|--------|------------------|--------------|------------------|-----------------|-------------------------|--------------------------|-------------------------|
| | | | Two cups | Without cups | Cup on the right | Cup on the left | Prosthesis pocket, both | Prosthesis pocket, right | Prosthesis pocket, left |
| XS | 68 - 76 | Black | STCO752F2H | STCO762F2H | STCO762RF2H | STCO762LF2H | STCO752PPF2H | STCO752PRF2H | STCO752PLF2H |
| XS | 68 - 76 | White | STCO752F3H | STCO762F3H | STCO762RF3H | STCO762LF3H | STCO752PPF3H | STCO752PRF3H | STCO752PLF3H |
| S | 76 - 84 | Black | STCO753F2H | STCO763F2H | STCO763RF2H | STCO763LF2H | STCO753PPF2H | STCO753PRF2H | STCO753PLF2H |
| S | 76 - 84 | White | STCO753F3H | STCO763F3H | STCO763RF3H | STCO763LF3H | STCO753PPF3H | STCO753PRF3H | STCO753PLF3H |
| M | 84 - 89 | Black | STCO754F2H | STCO764F2H | STCO764RF2H | STCO764LF2H | STCO754PPF2H | STCO754PRF2H | STCO754PLF2H |
| M | 84 - 89 | White | STCO754F3H | STCO764F3H | STCO764RF3H | STCO764LF3H | STCO754PPF3H | STCO754PRF3H | STCO754PLF3H |
| L | 89 - 94 | Black | STCO755F2H | STCO765F2H | STCO765RF2H | STCO765LF2H | STCO755PPF2H | STCO755PRF2H | STCO755PLF2H |
| L | 89 - 94 | White | STCO755F3H | STCO765F3H | STCO765RF3H | STCO765LF3H | STCO755PPF3H | STCO755PRF3H | STCO755PLF3H |
| XL | 94 - 104 | Black | STCO756F2H | STCO766F2H | STCO766RF2H | STCO766LF2H | STCO756PPF2H | STCO756PRF2H | STCO756PLF2H |
| XL | 94 - 104 | White | STCO756F3H | STCO766F3H | STCO766RF3H | STCO766LF3H | STCO756PPF3H | STCO756PRF3H | STCO756PLF3H |
| XXL | 104 - 114 | Black | STCO757F2H | STCO767F2H | STCO767RF2H | STCO767LF2H | STCO757PPF2H | STCO757PRF2H | STCO757PLF2H |
| XXL | 104 - 114 | White | STCO757F3H | STCO767F3H | STCO767RF3H | STCO767LF3H | STCO757PPF3H | STCO757PRF3H | STCO757PLF3H |
| XXXL | 114 - 127 | Black | STCO758F2H | STCO768F2H | STCO768RF2H | STCO768LF2H | STCO758PPF2H | STCO758PRF2H | STCO758PLF2H |
| XXXL | 114 - 127 | White | STCO758F3H | STCO768F3H | STCO768RF3H | STCO768LF3H | STCO758PPF3H | STCO758PRF3H | STCO758PLF3H |

COMPRESSION BRA | ZIPPER

STANDARD-SIZE

| Size | Circumf.* cm | Colour | Product / Art.No | | | | | | |
|------|-----------------|--------|------------------|--------------|------------------|-----------------|-------------------------|--------------------------|-------------------------|
| | | | Two cups | Without cups | Cup on the right | Cup on the left | Prosthesis pocket, both | Prosthesis pocket, right | Prosthesis pocket, left |
| XS | 68 - 76 | Black | STCO752F2Z | STCO762F2Z | STCO762RF2Z | STCO762LF2Z | STCO752PPF2Z | STCO752PRF2Z | STCO752PLF2Z |
| XS | 68 - 76 | White | STCO752F3Z | STCO762F3Z | STCO762RF3Z | STCO762LF3Z | STCO752PPF3Z | STCO752PRF3Z | STCO752PLF3Z |
| S | 76 - 84 | Black | STCO753F2Z | STCO763F2Z | STCO763RF2Z | STCO763LF2Z | STCO753PPF2Z | STCO753PRF2Z | STCO753PLF2Z |
| S | 76 - 84 | White | STCO753F3Z | STCO763F3Z | STCO763RF3Z | STCO763LF3Z | STCO753PPF3Z | STCO753PRF3Z | STCO753PLF3Z |
| M | 84 - 89 | Black | STCO754F2Z | STCO764F2Z | STCO764RF2Z | STCO764LF2Z | STCO754PPF2Z | STCO754PRF2Z | STCO754PLF2Z |
| M | 84 - 89 | White | STCO754F3Z | STCO764F3Z | STCO764RF3Z | STCO764LF3Z | STCO754PPF3Z | STCO754PRF3Z | STCO754PLF3Z |
| L | 89 - 94 | Black | STCO755F2Z | STCO765F2Z | STCO765RF2Z | STCO765LF2Z | STCO755PPF2Z | STCO755PRF2Z | STCO755PLF2Z |
| L | 89 - 94 | White | STCO755F3Z | STCO765F3Z | STCO765RF3Z | STCO765LF3Z | STCO755PPF3Z | STCO755PRF3Z | STCO755PLF3Z |
| XL | 94 - 104 | Black | STCO756F2Z | STCO766F2Z | STCO766RF2Z | STCO766LF2Z | STCO756PPF2Z | STCO756PRF2Z | STCO756PLF2Z |
| XL | 94 - 104 | White | STCO756F3Z | STCO766F3Z | STCO766RF3Z | STCO766LF3Z | STCO756PPF3Z | STCO756PRF3Z | STCO756PLF3Z |
| XXL | 104 - 114 | Black | STCO757F2Z | STCO767F2Z | STCO767RF2Z | STCO767LF2Z | STCO757PPF2Z | STCO757PRF2Z | STCO757PLF2Z |
| XXL | 104 - 114 | White | STCO757F3Z | STCO767F3Z | STCO767RF3Z | STCO767LF3Z | STCO757PPF3Z | STCO757PRF3Z | STCO757PLF3Z |
| XXXL | 114 - 127 | Black | STCO758F2Z | STCO768F2Z | STCO768RF2Z | STCO768LF2Z | STCO758PPF2Z | STCO758PRF2Z | STCO758PLF2Z |
| XXXL | 114 - 127 | White | STCO758F3Z | STCO768F3Z | STCO768RF3Z | STCO768LF3Z | STCO758PPF3Z | STCO758PRF3Z | STCO758PLF3Z |



Product picture, code:
CO22533F1

PANTS & STOCKINGS

CUSTOM-MADE

Product information:

Long-leg pants with anatomical construction at the knee. Stocking legs with anatomical construction at the ankle and knee.

When a stocking leg is ordered in strong material with compression classes 1-2, the constructions are made from medium material.

Men's models with contouring at the front.

| | |
|--------------------|--------------------------------------|
| Material: | Strong (Ccl 1-4) or medium (Ccl 1-3) |
| Colours: | All |
| Compression level: | All available |
| Fastening: | Available separately |
| Details: | Waist ELA 1 |
| | Leg opening ELA 2 |

| PANTS | | | | | | CUSTOM-MADE |
|-------|------------------|--------------------|--------------------------------------|--|-----------|-------------|
| Model | Product / Art.No | | | | | |
| | Short legs | Short and long leg | Short leg and leg with stocking OPEN | Short leg and leg with stocking CLOSED | Long legs | |
| Women | CO21522 | CO21523 | CO215248 | CO215249 | CO21533 | |
| Men | CO22522 | CO22523 | CO225248 | CO225249 | CO22533 | |
| Girls | CO11522 | CO11523 | CO115248 | CO115249 | CO11533 | |
| Boys | CO12522 | CO12523 | CO125248 | CO125249 | CO12533 | |

Children's models are made for children under 10.

| Model | Product / Art.No | | | | | |
|-------|-------------------------------------|---------------------------------------|--|--|---|---|
| | Long leg and leg with stocking OPEN | Long leg and leg with stocking CLOSED | Short leg and leg with toe sock, toe tips OPEN | Short leg and leg with toe sock, toe tips CLOSED | Long leg and leg with toe sock, toe tips OPEN | Long leg and leg with toe sock, toe tips CLOSED |
| Women | CO215348 | CO215349 | CO21524FG8 | CO21524FG9 | CO21534FG8 | CO21534FG9 |
| Men | CO225348 | CO225349 | CO22524FG8 | CO22524FG9 | CO22534FG8 | CO22534FG9 |
| Girls | CO115348 | CO115349 | CO11524FG8 | CO11524FG9 | CO11534FG8 | CO11534FG9 |
| Boys | CO125348 | CO125349 | CO12524FG8 | CO12524FG9 | CO12534FG8 | CO12534FG9 |

Children's models are made for children under 10.

| STOCKINGS | | | | | | | CUSTOM-MADE |
|-----------|------------------|-------------|--------------------------------------|--|-------------------------------|---------------------------------|-------------|
| Model | Product / Art.No | | | | | | |
| | Toes OPEN | Toes CLOSED | One leg with toe sock, toe tips OPEN | One leg with toe sock, toe tips CLOSED | With toe socks, toe tips OPEN | With toe socks, toe tips CLOSED | |
| Women | CO215448 | CO215449 | CO21544FG8 | CO21544FG9 | CO21544FG28 | CO21544FG29 | |
| Men | CO225448 | CO225449 | CO22544FG8 | CO22544FG9 | CO22544FG28 | CO22544FG29 | |
| Girls | CO115448 | CO115449 | CO11544FG8 | CO11544FG9 | CO11544FG28 | CO11544FG29 | |
| Boys | CO125448 | CO125449 | CO12544FG8 | CO12544FG9 | CO12544FG28 | CO12544FG29 | |

Children's models are made for children under 10.



Product picture, code:
STCO21514F1



Product picture, code:
STCO21514HWF1

COMPRESSION PANTS FOR WOMEN

STANDARD-SIZE

Product information:

Normal-waist and high-waist models are available in different leg lengths.
Optional additional pressure configurations (FS, FNS, NI) are also available for the genital area.

Recommended addition: shoulder straps.

*Circumference at widest point of pelvis and upper thigh.

| | |
|--------------------|--------------------------------|
| Material: | Strong |
| Colours: | Beige, black |
| Compression level: | Ccl 2 |
| Details: | Waist SIL 1, Leg opening ELA 2 |

Product picture, code:
STCO21514FNSF1 ABS

Normal waist
with additional flap (FNS),
shoulder straps



| COMPRESSION PANTS FOR WOMEN NORMAL WAIST | | | | | | STANDARD-SIZE |
|--|-------------|-------------|--------|------------------|-------------|---------------|
| Model | Circumf.* | | Colour | Product / Art.No | | |
| | Pelvis (cm) | Thigh (cm) | | Knee length | Calf length | Ankle length |
| XS | 86 - 90 | 50,0 - 52,5 | Beige | STCO21512F1 | STCO21522F1 | STCO21532F1 |
| XS | 86 - 90 | 50,0 - 52,5 | Black | STCO21512F2 | STCO21522F2 | STCO21532F2 |
| S | 91 - 95 | 53,0 - 56,0 | Beige | STCO21513F1 | STCO21523F1 | STCO21533F1 |
| S | 91 - 95 | 53,0 - 56,0 | Black | STCO21513F2 | STCO21523F2 | STCO21533F2 |
| M | 96 - 101 | 56,5 - 59,0 | Beige | STCO21514F1 | STCO21524F1 | STCO21534F1 |
| M | 96 - 101 | 56,5 - 59,0 | Black | STCO21514F2 | STCO21524F2 | STCO21534F2 |
| L | 102 - 110 | 59,5 - 62,0 | Beige | STCO21515F1 | STCO21525F1 | STCO21535F1 |
| L | 102 - 110 | 59,5 - 62,0 | Black | STCO21515F2 | STCO21525F2 | STCO21535F2 |
| XL | 111 - 121 | 62,5 - 65,0 | Beige | STCO21516F1 | STCO21526F1 | STCO21536F1 |
| XL | 111 - 121 | 62,5 - 65,0 | Black | STCO21516F2 | STCO21526F2 | STCO21536F2 |
| XXL | 122 - 132 | 65,5 - 68,0 | Beige | STCO21517F1 | STCO21527F1 | STCO21537F1 |
| XXL | 122 - 132 | 65,5 - 68,0 | Black | STCO21517F2 | STCO21527F2 | STCO21537F2 |
| XXXL | 133 - 138 | 68,5 - 71,5 | Beige | STCO21518F1 | STCO21528F1 | STCO21538F1 |
| XXXL | 133 - 138 | 68,5 - 71,5 | Black | STCO21518F2 | STCO21528F2 | STCO21538F2 |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Beige | STCO21519F1 | STCO21529F1 | STCO21539F1 |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Black | STCO21519F2 | STCO21529F2 | STCO21539F2 |

| COMPRESSION PANTS FOR WOMEN NORMAL WAIST | | | | | | STANDARD-SIZE |
|--|-------------|-------------|--------|------------------|---------------|---------------|
| Fabric flap, velcro adjustment | | | | | | |
| Model | Circumf.* | | Colour | Product / Art.No | | |
| | Pelvis (cm) | Thigh (cm) | | Knee length | Calf length | Ankle length |
| XS | 86 - 90 | 50,0 - 52,5 | Beige | STCO21512FSF1 | STCO21522FSF1 | STCO21532FSF1 |
| XS | 86 - 90 | 50,0 - 52,5 | Black | STCO21512FSF2 | STCO21522FSF2 | STCO21532FSF2 |
| S | 91 - 95 | 53,0 - 56,0 | Beige | STCO21513FSF1 | STCO21523FSF1 | STCO21533FSF1 |
| S | 91 - 95 | 53,0 - 56,0 | Black | STCO21513FSF2 | STCO21523FSF2 | STCO21533FSF2 |
| M | 96 - 101 | 56,5 - 59,0 | Beige | STCO21514FSF1 | STCO21524FSF1 | STCO21534FSF1 |
| M | 96 - 101 | 56,5 - 59,0 | Black | STCO21514FSF2 | STCO21524FSF2 | STCO21534FSF2 |
| L | 102 - 110 | 59,5 - 62,0 | Beige | STCO21515FSF1 | STCO21525FSF1 | STCO21535FSF1 |
| L | 102 - 110 | 59,5 - 62,0 | Black | STCO21515FSF2 | STCO21525FSF2 | STCO21535FSF2 |
| XL | 111 - 121 | 62,5 - 65,0 | Beige | STCO21516FSF1 | STCO21526FSF1 | STCO21536FSF1 |
| XL | 111 - 121 | 62,5 - 65,0 | Black | STCO21516FSF2 | STCO21526FSF2 | STCO21536FSF2 |
| XXL | 122 - 132 | 65,5 - 68,0 | Beige | STCO21517FSF1 | STCO21527FSF1 | STCO21537FSF1 |
| XXL | 122 - 132 | 65,5 - 68,0 | Black | STCO21517FSF2 | STCO21527FSF2 | STCO21537FSF2 |
| XXXL | 133 - 138 | 68,5 - 71,5 | Beige | STCO21518FSF1 | STCO21528FSF1 | STCO21538FSF1 |
| XXXL | 133 - 138 | 68,5 - 71,5 | Black | STCO21518FSF2 | STCO21528FSF2 | STCO21538FSF2 |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Beige | STCO21519FSF1 | STCO21529FSF1 | STCO21539FSF1 |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Black | STCO21519FSF2 | STCO21529FSF2 | STCO21539FSF2 |

COMPRESSION PANTS FOR WOMEN | NORMAL WAIST

Flap with neoprene reinforcement, velcro adjustment

STANDARD-SIZE

| Model | Circumf.* | | Colour | Product / Art.No | | |
|-------|-------------|-------------|--------|------------------|----------------|----------------|
| | Pelvis (cm) | Thigh (cm) | | Knee length | Calf length | Ankle length |
| XS | 86 - 90 | 50,0 - 52,5 | Beige | STCO21512FNSF1 | STCO21522FNSF1 | STCO21532FNSF1 |
| XS | 86 - 90 | 50,0 - 52,5 | Black | STCO21512FNSF2 | STCO21522FNSF2 | STCO21532FNSF2 |
| S | 91 - 95 | 53,0 - 56,0 | Beige | STCO21513FNSF1 | STCO21523FNSF1 | STCO21533FNSF1 |
| S | 91 - 95 | 53,0 - 56,0 | Black | STCO21513FNSF2 | STCO21523FNSF2 | STCO21533FNSF2 |
| M | 96 - 101 | 56,5 - 59,0 | Beige | STCO21514FNSF1 | STCO21524FNSF1 | STCO21534FNSF1 |
| M | 96 - 101 | 56,5 - 59,0 | Black | STCO21514FNSF2 | STCO21524FNSF2 | STCO21534FNSF2 |
| L | 102 - 110 | 59,5 - 62,0 | Beige | STCO21515FNSF1 | STCO21525FNSF1 | STCO21535FNSF1 |
| L | 102 - 110 | 59,5 - 62,0 | Black | STCO21515FNSF2 | STCO21525FNSF2 | STCO21535FNSF2 |
| XL | 111 - 121 | 62,5 - 65,0 | Beige | STCO21516FNSF1 | STCO21526FNSF1 | STCO21536FNSF1 |
| XL | 111 - 121 | 62,5 - 65,0 | Black | STCO21516FNSF2 | STCO21526FNSF2 | STCO21536FNSF2 |
| XXL | 122 - 132 | 65,5 - 68,0 | Beige | STCO21517FNSF1 | STCO21527FNSF1 | STCO21537FNSF1 |
| XXL | 122 - 132 | 65,5 - 68,0 | Black | STCO21517FNSF2 | STCO21527FNSF2 | STCO21537FNSF2 |
| XXXL | 133 - 138 | 68,5 - 71,5 | Beige | STCO21518FNSF1 | STCO21528FNSF1 | STCO21538FNSF1 |
| XXXL | 133 - 138 | 68,5 - 71,5 | Black | STCO21518FNSF2 | STCO21528FNSF2 | STCO21538FNSF2 |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Beige | STCO21519FNSF1 | STCO21529FNSF1 | STCO21539FNSF1 |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Black | STCO21519FNSF2 | STCO21529FNSF2 | STCO21539FNSF2 |

COMPRESSION PANTS FOR WOMEN | NORMAL WAIST

Integrated centerpiece with neoprene reinforcement

STANDARD-SIZE

| Model | Circumf.* | | Colour | Product / Art.No | | |
|-------|-------------|-------------|--------|------------------|---------------|---------------|
| | Pelvis (cm) | Thigh (cm) | | Knee length | Calf length | Ankle length |
| XS | 86 - 90 | 50,0 - 52,5 | Beige | STCO21512NIF1 | STCO21522NIF1 | STCO21532NIF1 |
| XS | 86 - 90 | 50,0 - 52,5 | Black | STCO21512NIF2 | STCO21522NIF2 | STCO21532NIF2 |
| S | 91 - 95 | 53,0 - 56,0 | Beige | STCO21513NIF1 | STCO21523NIF1 | STCO21533NIF1 |
| S | 91 - 95 | 53,0 - 56,0 | Black | STCO21513NIF2 | STCO21523NIF2 | STCO21533NIF2 |
| M | 96 - 101 | 56,5 - 59,0 | Beige | STCO21514NIF1 | STCO21524NIF1 | STCO21534NIF1 |
| M | 96 - 101 | 56,5 - 59,0 | Black | STCO21514NIF2 | STCO21524NIF2 | STCO21534NIF2 |
| L | 102 - 110 | 59,5 - 62,0 | Beige | STCO21515NIF1 | STCO21525NIF1 | STCO21535NIF1 |
| L | 102 - 110 | 59,5 - 62,0 | Black | STCO21515NIF2 | STCO21525NIF2 | STCO21535NIF2 |
| XL | 111 - 121 | 62,5 - 65,0 | Beige | STCO21516NIF1 | STCO21526NIF1 | STCO21536NIF1 |
| XL | 111 - 121 | 62,5 - 65,0 | Black | STCO21516NIF2 | STCO21526NIF2 | STCO21536NIF2 |
| XXL | 122 - 132 | 65,5 - 68,0 | Beige | STCO21517NIF1 | STCO21527NIF1 | STCO21537NIF1 |
| XXL | 122 - 132 | 65,5 - 68,0 | Black | STCO21517NIF2 | STCO21527NIF2 | STCO21537NIF2 |
| XXXL | 133 - 138 | 68,5 - 71,5 | Beige | STCO21518NIF1 | STCO21528NIF1 | STCO21538NIF1 |
| XXXL | 133 - 138 | 68,5 - 71,5 | Black | STCO21518NIF2 | STCO21528NIF2 | STCO21538NIF2 |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Beige | STCO21519NIF1 | STCO21529NIF1 | STCO21539NIF1 |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Black | STCO21519NIF2 | STCO21529NIF2 | STCO21539NIF2 |

COMPRESSION PANTS FOR WOMEN | HIGH WAIST

STANDARD-SIZE

| Model | Circumf.* | | Colour | Product / Art.No | | |
|-------|-------------|-------------|--------|------------------|---------------|---------------|
| | Pelvis (cm) | Thigh (cm) | | Knee length | Calf length | Ankle length |
| XS | 86 - 90 | 50,0 - 52,5 | Beige | STCO21512HWF1 | STCO21522HWF1 | STCO21532HWF1 |
| XS | 86 - 90 | 50,0 - 52,5 | Black | STCO21512HWF2 | STCO21522HWF2 | STCO21532HWF2 |
| S | 91 - 95 | 53,0 - 56,0 | Beige | STCO21513HWF1 | STCO21523HWF1 | STCO21533HWF1 |
| S | 91 - 95 | 53,0 - 56,0 | Black | STCO21513HWF2 | STCO21523HWF2 | STCO21533HWF2 |
| M | 96 - 101 | 56,5 - 59,0 | Beige | STCO21514HWF1 | STCO21524HWF1 | STCO21534HWF1 |
| M | 96 - 101 | 56,5 - 59,0 | Black | STCO21514HWF2 | STCO21524HWF2 | STCO21534HWF2 |
| L | 102 - 110 | 59,5 - 62,0 | Beige | STCO21515HWF1 | STCO21525HWF1 | STCO21535HWF1 |
| L | 102 - 110 | 59,5 - 62,0 | Black | STCO21515HWF2 | STCO21525HWF2 | STCO21535HWF2 |
| XL | 111 - 121 | 62,5 - 65,0 | Beige | STCO21516HWF1 | STCO21526HWF1 | STCO21536HWF1 |
| XL | 111 - 121 | 62,5 - 65,0 | Black | STCO21516HWF2 | STCO21526HWF2 | STCO21536HWF2 |
| XXL | 122 - 132 | 65,5 - 68,0 | Beige | STCO21517HWF1 | STCO21527HWF1 | STCO21537HWF1 |
| XXL | 122 - 132 | 65,5 - 68,0 | Black | STCO21517HWF2 | STCO21527HWF2 | STCO21537HWF2 |
| XXXL | 133 - 138 | 68,5 - 71,5 | Beige | STCO21518HWF1 | STCO21528HWF1 | STCO21538HWF1 |
| XXXL | 133 - 138 | 68,5 - 71,5 | Black | STCO21518HWF2 | STCO21528HWF2 | STCO21538HWF2 |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Beige | STCO21519HWF1 | STCO21529HWF1 | STCO21539HWF1 |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Black | STCO21519HWF2 | STCO21529HWF2 | STCO21539HWF2 |

COMPRESSION PANTS FOR WOMEN | HIGH WAIST

Integrated centerpiece with neoprene reinforcement

STANDARD-SIZE

| Model | Circumf.* | | Colour | Product / Art.No | | |
|-------|-------------|-------------|--------|------------------|-----------------|-----------------|
| | Pelvis (cm) | Thigh (cm) | | Knee length | Calf length | Ankle length |
| XS | 86 - 90 | 50,0 - 52,5 | Beige | STCO21512HWNIF1 | STCO21522HWNIF1 | STCO21532HWNIF1 |
| XS | 86 - 90 | 50,0 - 52,5 | Black | STCO21512HWNIF2 | STCO21522HWNIF2 | STCO21532HWNIF2 |
| S | 91 - 95 | 53,0 - 56,0 | Beige | STCO21513HWNIF1 | STCO21523HWNIF1 | STCO21533HWNIF1 |
| S | 91 - 95 | 53,0 - 56,0 | Black | STCO21513HWNIF2 | STCO21523HWNIF2 | STCO21533HWNIF2 |
| M | 96 - 101 | 56,5 - 59,0 | Beige | STCO21514HWNIF1 | STCO21524HWNIF1 | STCO21534HWNIF1 |
| M | 96 - 101 | 56,5 - 59,0 | Black | STCO21514HWNIF2 | STCO21524HWNIF2 | STCO21534HWNIF2 |
| L | 102 - 110 | 59,5 - 62,0 | Beige | STCO21515HWNIF1 | STCO21525HWNIF1 | STCO21535HWNIF1 |
| L | 102 - 110 | 59,5 - 62,0 | Black | STCO21515HWNIF2 | STCO21525HWNIF2 | STCO21535HWNIF2 |
| XL | 111 - 121 | 62,5 - 65,0 | Beige | STCO21516HWNIF1 | STCO21526HWNIF1 | STCO21536HWNIF1 |
| XL | 111 - 121 | 62,5 - 65,0 | Black | STCO21516HWNIF2 | STCO21526HWNIF2 | STCO21536HWNIF2 |
| XXL | 122 - 132 | 65,5 - 68,0 | Beige | STCO21517HWNIF1 | STCO21527HWNIF1 | STCO21537HWNIF1 |
| XXL | 122 - 132 | 65,5 - 68,0 | Black | STCO21517HWNIF2 | STCO21527HWNIF2 | STCO21537HWNIF2 |
| XXXL | 133 - 138 | 68,5 - 71,5 | Beige | STCO21518HWNIF1 | STCO21528HWNIF1 | STCO21538HWNIF1 |
| XXXL | 133 - 138 | 68,5 - 71,5 | Black | STCO21518HWNIF2 | STCO21528HWNIF2 | STCO21538HWNIF2 |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Beige | STCO21519HWNIF1 | STCO21529HWNIF1 | STCO21539HWNIF1 |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Black | STCO21519HWNIF2 | STCO21529HWNIF2 | STCO21539HWNIF2 |



Product picture, code: STCO22534F1

Product information:

Men's compression pants in various leg lengths.
Men's models with contouring at the front.

Recommended addition: shoulder straps.
**Circumference at widest point of pelvis and upper thigh.*

| | |
|--------------------|-----------------------------------|
| Material: | Strong |
| Colours: | Beige, black |
| Compression level: | Ccl 2 |
| Details: | Waist SIL 1, Leg opening ELA 2 |

COMPRESSION PANTS FOR MEN | NORMAL WAIST

STANDARD-SIZE

| Model | Circumf. | | Colour | Product / Art.No | | |
|-------|-------------|-------------|--------|------------------|-------------|--------------|
| | Pelvis (cm) | Thigh (cm) | | Knee length | Calf length | Ankle length |
| XS | 84 - 89 | 55,0 - 57,5 | Beige | STCO22512F1 | STCO22522F1 | STCO22532F1 |
| XS | 84 - 89 | 55,0 - 57,5 | Black | STCO22512F2 | STCO22522F2 | STCO22532F2 |
| S | 90 - 95 | 58,0 - 60,5 | Beige | STCO22513F1 | STCO22523F1 | STCO22533F1 |
| S | 90 - 95 | 58,0 - 60,5 | Black | STCO22513F2 | STCO22523F2 | STCO22533F2 |
| M | 96 - 101 | 61,0 - 63,5 | Beige | STCO22514F1 | STCO22524F1 | STCO22534F1 |
| M | 96 - 101 | 61,0 - 63,5 | Black | STCO22514F2 | STCO22524F2 | STCO22534F2 |
| L | 102 - 107 | 64,0 - 66,5 | Beige | STCO22515F1 | STCO22525F1 | STCO22535F1 |
| L | 102 - 107 | 64,0 - 66,5 | Black | STCO22515F2 | STCO22525F2 | STCO22535F2 |
| XL | 108 - 114 | 67,0 - 69,5 | Beige | STCO22516F1 | STCO22526F1 | STCO22536F1 |
| XL | 108 - 114 | 67,0 - 69,5 | Black | STCO22516F2 | STCO22526F2 | STCO22536F2 |
| XXL | 115 - 121 | 70,0 - 72,5 | Beige | STCO22517F1 | STCO22527F1 | STCO22537F1 |
| XXL | 115 - 121 | 70,0 - 72,5 | Black | STCO22517F2 | STCO22527F2 | STCO22537F2 |

Product picture, code: STCO22534FSF1 ABS



Product information:

The front panel of the trousers features an opening instead of contouring, along with an additional adjustable support flap secured with velcro to provide genital support.

Recommended addition: shoulder straps.
*Circumference at widest point of pelvis and upper thigh.

| | |
|--------------------|-----------------------------------|
| Material: | Strong |
| Colours: | Beige, black |
| Compression level: | Ccl 2 |
| Details: | Waist SIL 1, Leg opening ELA 2 |

COMPRESSION PANTS FOR MEN | NORMAL WAIST

Fabric flap, velcro adjustment

STANDARD-SIZE

| Model | Circumf. | | Colour | Product / Art.No | | |
|-------|-------------|-------------|--------|------------------|---------------|---------------|
| | Pelvis (cm) | Thigh (cm) | | Knee length | Calf length | Ankle length |
| XS | 84 - 89 | 55,0 - 57,5 | Beige | STCO22512FSF1 | STCO22522FSF1 | STCO22532FSF1 |
| XS | 84 - 89 | 55,0 - 57,5 | Black | STCO22512FSF2 | STCO22522FSF2 | STCO22532FSF2 |
| S | 90 - 95 | 58,0 - 60,5 | Beige | STCO22513FSF1 | STCO22523FSF1 | STCO22533FSF1 |
| S | 90 - 95 | 58,0 - 60,5 | Black | STCO22513FSF2 | STCO22523FSF2 | STCO22533FSF2 |
| M | 96 - 101 | 61,0 - 63,5 | Beige | STCO22514FSF1 | STCO22524FSF1 | STCO22534FSF1 |
| M | 96 - 101 | 61,0 - 63,5 | Black | STCO22514FSF2 | STCO22524FSF2 | STCO22534FSF2 |
| L | 102 - 107 | 64,0 - 66,5 | Beige | STCO22515FSF1 | STCO22525FSF1 | STCO22535FSF1 |
| L | 102 - 107 | 64,0 - 66,5 | Black | STCO22515FSF2 | STCO22525FSF2 | STCO22535FSF2 |
| XL | 108 - 114 | 67,0 - 69,5 | Beige | STCO22516FSF1 | STCO22526FSF1 | STCO22536FSF1 |
| XL | 108 - 114 | 67,0 - 69,5 | Black | STCO22516FSF2 | STCO22526FSF2 | STCO22536FSF2 |
| XXL | 115 - 121 | 70,0 - 72,5 | Beige | STCO22517FSF1 | STCO22527FSF1 | STCO22537FSF1 |
| XXL | 115 - 121 | 70,0 - 72,5 | Black | STCO22517FSF2 | STCO22527FSF2 | STCO22537FSF2 |



Product picture, code: STCO22504F2

SUSPENSOR

STANDARD-SIZE

| Model | Estim. volume dl | Height cm | Length cm | Art.No |
|-------|------------------|-----------|-----------|-------------|
| S | 6 | 18 | 120 | STCO22503F2 |
| M | 8 | 18 | 120 | STCO22504F2 |
| L | 12 | 18 | 120 | STCO22505F2 |
| XL | 14 | 18 | 120 | STCO22506F2 |

Product information:

Suspensor for men's genital area oedema.
Adjustable velcro fastening and straps.

| | |
|-----------------|---|
| Material: | Polyamide elastane knit, support band with velcro |
| Colour: | Black |
| Pressure level: | Adjustable |
| Fastening: | Velcro fastening |
| Details: | Front edging: ELA 3 Straps: velcro straps |

PENIS TUBE

CUSTOM-MADE

| Model | Tip | Art.No |
|----------|--------|---------|
| Adults | Closed | CO22809 |
| Adults | Open | CO22808 |
| Children | Closed | CO12809 |
| Children | Open | CO12808 |

Children's models are made for children under 10.

Product information:

Anatomical construction in the base of the tube. We recommend velcro fastening for easier donning.

If the circumference of penis is over 20 cm, the product will be priced on a case-by-case basis.

| | |
|--------------------|--|
| Material: | Strong (Ccl 1-4) or medium (Ccl 1-3) |
| Colours: | All |
| Compression level: | All available |
| Fastening: | Velcro fastening available separately* |
| Details: | No finishing |

* Fastening colour: white, black or beige, which is chosen by our production.



Product picture, code: CO22808F1

Product picture, code: STCOM21534F2



COMPRESSION™

| MATERNITY PANTS | | | | STANDARD-SIZE | | |
|-----------------|-------------|-------------|--------|-------------------|--------------|--------------|
| Model | Circumf.* | | Colour | Product / Art. No | | |
| | Pelvis (cm) | Thigh (cm) | | Knee length | Calf length | Ankle length |
| XS | 86 - 90 | 50,0 - 52,5 | Beige | STCOM21512F1 | STCOM21522F1 | STCOM21532F1 |
| XS | 86 - 90 | 50,0 - 52,5 | Black | STCOM21512F2 | STCOM21522F2 | STCOM21532F2 |
| S | 91 - 95 | 53,0 - 56,0 | Beige | STCOM21513F1 | STCOM21523F1 | STCOM21533F1 |
| S | 91 - 95 | 53,0 - 56,0 | Black | STCOM21513F2 | STCOM21523F2 | STCOM21533F2 |
| M | 96 - 101 | 56,5 - 59,0 | Beige | STCOM21514F1 | STCOM21524F1 | STCOM21534F1 |
| M | 96 - 101 | 56,5 - 59,0 | Black | STCOM21514F2 | STCOM21524F2 | STCOM21534F2 |
| L | 102 - 110 | 59,5 - 62,0 | Beige | STCOM21515F1 | STCOM21525F1 | STCOM21535F1 |
| L | 102 - 110 | 59,5 - 62,0 | Black | STCOM21515F2 | STCOM21525F2 | STCOM21535F2 |
| XL | 111 - 121 | 62,5 - 65,0 | Beige | STCOM21516F1 | STCOM21526F1 | STCOM21536F1 |
| XL | 111 - 121 | 62,5 - 65,0 | Black | STCOM21516F2 | STCOM21526F2 | STCOM21536F2 |
| XXL | 122 - 132 | 65,5 - 68,0 | Beige | STCOM21517F1 | STCOM21527F1 | STCOM21537F1 |
| XXL | 122 - 132 | 65,5 - 68,0 | Black | STCOM21517F2 | STCOM21527F2 | STCOM21537F2 |
| XXXL | 133 - 138 | 68,5 - 71,5 | Beige | STCOM21518F1 | STCOM21528F1 | STCOM21538F1 |
| XXXL | 133 - 138 | 68,5 - 71,5 | Black | STCOM21518F2 | STCOM21528F2 | STCOM21538F2 |

Product information:

The maternity model features a more flexible abdominal section for a growing abdomen. The abdominal section adapts to the measurements of most expectant mothers from mid-pregnancy through to the later stages of pregnancy.

| | |
|--------------------|-----------------------|
| Material: | Strong |
| | Abdominal area: Light |
| Colours: | Beige, black |
| Compression level: | Ccl 2 |
| | Waist SIL 1 |
| Details: | Leg opening ELA 2 |

Tip! Can be used together with maternity support belt and bras.





Product picture, code: CO20603F6



Product picture, code: CO20603F1

| LEG | | CUSTOM-MADE | |
|----------|------------------|-------------|-------------|
| Model | Product / Art.No | | |
| | Ankle-knee | Knee-thigh | Ankle-thigh |
| Adults | CO20601 | CO20602 | CO20603 |
| Children | CO10601 | CO10602 | CO10603 |

Children's models are made for children under 10.

Product information:

The leg sleeve features an anatomically shaped construction in the knee bend area. When the product is ordered in strong fabric with compression class CCL 1-2, the knee bend construction is made from medium fabric.

| | |
|--------------------|---|
| Material: | Strong (Ccl 1-4) or medium (Ccl 1-3) |
| Colours: | All |
| Compression level: | All available |
| Fastening: | Available separately |
| Details: | Upper edge SIL 4, Upper edge in knee-high SIL 1, Lower edge ELA 2 |



Product picture, code:
CO20639F4 A20B



Product picture, code:
CO20629F4 AZ



Product picture, code:
CO20619F4



Product picture, code:
COA20T9F2

STOCKING CUSTOM-MADE

| Model | Toes | Product / Art.No | | |
|----------|------------------|------------------|-----------|------------|
| | | Ankle-high | Knee-high | Thigh-high |
| Adults | Closed | CO20619 | CO20629 | CO20639 |
| Adults | Open | CO20618 | CO20628 | CO20638 |
| Adults | Toe sock, closed | CO20619FG | CO20629FG | CO20639FG |
| Adults | Toe sock, open | CO20618FG | CO20628FG | CO20638FG |
| Children | Closed | CO10619 | CO10629 | CO10639 |
| Children | Open | CO10618 | CO10628 | CO10638 |
| Children | Toe sock, closed | CO10619FG | CO10629FG | CO10639FG |
| Children | Toe sock, open | CO10618FG | CO10628FG | CO10638FG |

Children's models are made for children under 10.

Product information:

The stockings feature anatomically shaped construction at the ankle and knee bend. When the product is ordered in strong fabric with compression class CCL 1-2, the ankle and knee bend constructions are made from medium fabric. In open-toe stockings, the toe is lightly beveled. The ankle sock's shaft length is measured upwards from the L/R61 circumference. Please note that the shaft length can be at most the same as the L/R67 measurement.

| | |
|--------------------|---|
| Material: | Strong (Ccl 1-4) or medium (Ccl 1-3) |
| Colours: | All |
| Compression level: | All available |
| Fastening: | Available separately |
| Details: | Upper edge SIL 1, upper edge in thigh-high SIL 4, lower edge, tip open ELA 2. |

FOOT GLOVE CUSTOM-MADE

| Model | Toe tips | Art.No |
|----------|----------|---------|
| Adults | Closed | COA20T9 |
| Adults | Open | COA20T8 |
| Children | Closed | COA10T9 |
| Children | Open | COA10T8 |

Children's models are made for children under 10.

Product information:

For the treatment of foot or toes. Can be used by itself or e.g. with an open-toe stocking.

| | |
|--------------------|--------------------------------------|
| Material: | Strong (Ccl 1-4) or medium (Ccl 1-3) |
| Colours: | All |
| Compression level: | All available |
| Details: | No finishing |



Product picture, code: CO20303F1



Product picture, code: CO2031038F1



Product picture, code: CO2031138F1

SLEEVE CUSTOM-MADE

| Model | Product / Art.No | | |
|----------|------------------|--------------|--------------|
| | Wrist-elbow | Elbow-axilla | Wrist-axilla |
| Adults | CO20301 | CO20302 | CO20303 |
| Children | CO10301 | CO10302 | CO10303 |

Children's models are made for children under 10.

Product information:

The wrist-to-axilla model features an anatomical elbow bend.

| | |
|--------------------|--------------------------------------|
| Material: | Strong (Ccl 1-4) or medium (Ccl 1-3) |
| Colours: | All |
| Compression level: | All available |
| Fastening: | Available separately |
| Details: | Upper edge SIL 1 Lower edge ELA 2 |

SLEEVE GAUNTLET CUSTOM-MADE

| Model | Thumb | Product / Art.No | |
|----------|--------|------------------|-----------|
| | | To elbow | To axilla |
| Adults | Closed | CO2031019 | CO2031039 |
| Adults | Open | CO2031018 | CO2031038 |
| Children | Closed | CO1031019 | CO1031039 |
| Children | Open | CO1031018 | CO1031038 |

Children's models are made for children under 10.

Product information:

Axilla-length model with anatomical elbow construction. Anatomical construction in the thumb fourchette.

| | |
|--------------------|--------------------------------------|
| Material: | Strong (Ccl 1-4) or medium (Ccl 1-3) |
| Colours: | All |
| Compression level: | All available |
| Fastening: | Available separately |
| Details: | Upper edge SIL 1 Lower edge ELA 2 |

SLEEVE GLOVE CUSTOM-MADE

| Model | Fingertips | Product / Art.No | |
|----------|------------|------------------|-----------|
| | | To elbow | To axilla |
| Adults | Closed | CO2031119 | CO2031139 |
| Adults | Open | CO2031118 | CO2031138 |
| Children | Closed | CO1031119 | CO1031139 |
| Children | Open | CO1031118 | CO1031138 |

Children's models are made for children under 10.

Product information:

Axilla-length model with anatomical elbow construction. Anatomical construction between the fingers and in the thumb fourchette.

| | |
|--------------------|--------------------------------------|
| Material: | Strong (Ccl 1-4) or medium (Ccl 1-3) |
| Colours: | All |
| Compression level: | All available |
| Fastening: | Available separately |
| Details: | Upper edge SIL 1 |

SLEEVE FASTENING - MODEL 1 CUSTOM-MADE


| Model | Product / Art.No |
|----------------------------------|------------------|
| Model 1 Shoulder flap | |
| Adults | A20S1 |
| Children | A10S1 |

Children's models are made for children under 10.

Product information:

Elastic strap running under the breast to hold the sleeve up. Fastening with velcro. Suitable especially for users with large breasts. Available for all sleeves reaching up to axilla. Women's sleeve with a bra strap holder.

| | |
|-----------|----------------|
| Material: | ELA 1 |
| Colour: | Same as sleeve |

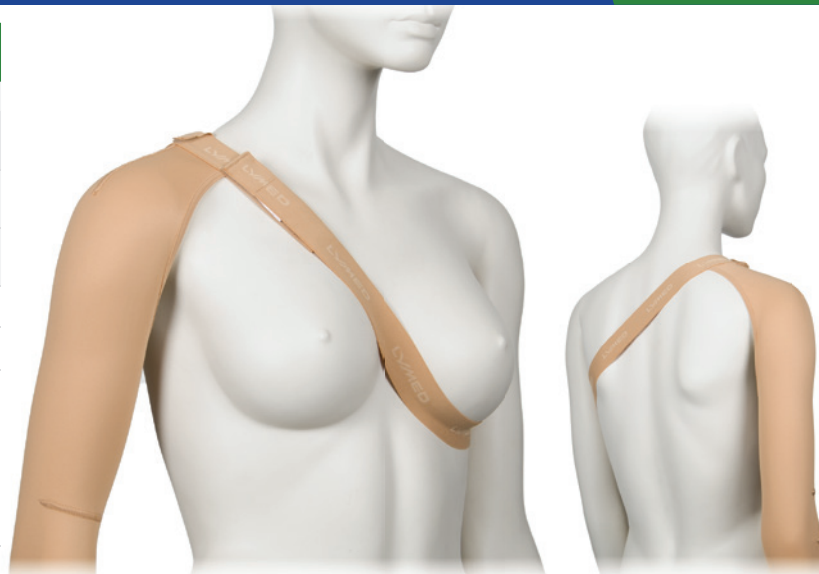

SLEEVE FASTENING - MODEL 2 CUSTOM-MADE


| Model | Product / Art.No |
|----------------------------------|------------------|
| Model 2 Shoulder flap | |
| Adults | A20S2 |
| Children | A10S2 |

Children's models are made for children under 10.

Product information:

Elastic strap running under the axilla to hold the sleeve up. Fastening with velcro. Available for all sleeves reaching up to axilla.

| | |
|-----------|----------------|
| Material: | ELA 1 |
| Colour: | Same as sleeve |

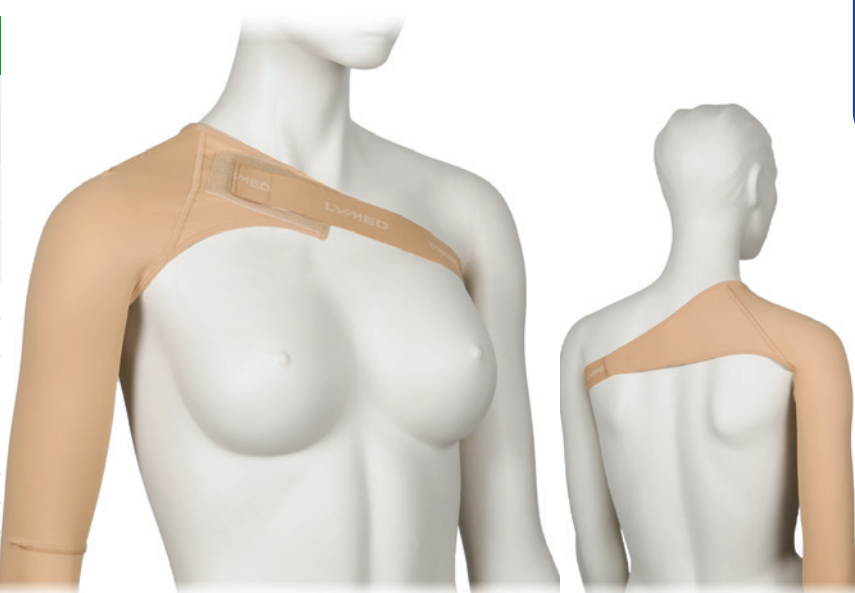

SLEEVE FASTENING - MODEL 3 CUSTOM-MADE

| Model | Product / Art.No |
|------------------------------------|------------------|
| Model 3 Opposite sleeve | |
| Adults | A20S3 |
| Children | A10S3 |

Children's models are made for children under 10.


Product information:

In the sleeve attachment model, a counter-sleeve connects two axilla-length sleeves. The model is suitable for situations where maintaining the position of one or both sleeves is challenging. The counter-sleeve configuration can be used in different setups: to connect two sleeves or to support the positioning of a single sleeve. Two sleeves must be ordered in conjunction with the attachment.

If the ordered sleeves are made from different materials, the attachment component is manufactured by default using the lighter material.

| | |
|------------|---------------------------------|
| Material: | Strong or medium |
| Colour: | Same as sleeve |
| Fastening: | 2-row hooks / zipper with guard |
| Details: | Neckline ELA 3 |
| | Hemline SIL 2 |
| | Edge of sleeves ELA 2 |

Product pictured, made from medium material

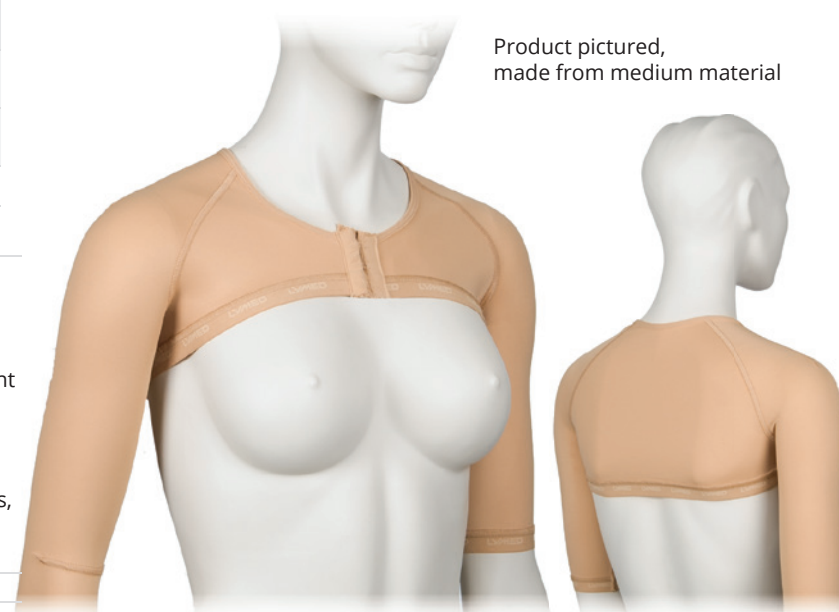

See also vest model on p. 58



Product picture, code:
CO203119F1 (Custom-made)
STCO31194RF1 (Standard-size)

Product picture, code:
CO203118F1 (Custom-made)
STCO31184RF1 (Standard-size)

| GLOVE | | CUSTOM-MADE |
|----------|-------------------|-----------------|
| Model | Product / Art.No | |
| | Fingertips closed | Fingertips open |
| Adults | CO203119 | CO203118 |
| Children | CO103119 | CO103118 |

Children's models are made for children under 10.

Product information:

Anatomical construction between the fingers and in the thumb fourchette. Number of backhand seams/darts in a glove may vary due to individual measurements and pressure/compression levels. Order a sleeve-glove when the length of the shaft is greater than the length of the palm.

| | |
|--------------------|--------------------------------------|
| Material: | Strong (Ccl 1-4) or medium (Ccl 1-3) |
| Colours: | All |
| Compression level: | All available |
| Fastening: | Available separately |
| Details: | Wrist edging ELA 3 |

| GLOVE | | STANDARD-SIZE | | | | |
|-------|-----------------|---------------|-------------------|--------------|-----------------|--------------|
| Size | Circumf.* cm | Colour | Product / Art.No | | | |
| | | | Fingertips closed | | Fingertips open | |
| | | | Right | Left | Right | Left |
| XXS | 14 - 16 | Beige | STCO31191RF1 | STCO31191LF1 | STCO31181RF1 | STCO31181LF1 |
| XXS | 14 - 16 | Black | STCO31191RF2 | STCO31191LF2 | STCO31181RF2 | STCO31181LF2 |
| XS | 17 - 18 | Beige | STCO31192RF1 | STCO31192LF1 | STCO31182RF1 | STCO31182LF1 |
| XS | 17 - 18 | Black | STCO31192RF2 | STCO31192LF2 | STCO31182RF2 | STCO31182LF2 |
| S | 19 - 20 | Beige | STCO31193RF1 | STCO31193LF1 | STCO31183RF1 | STCO31183LF1 |
| S | 19 - 20 | Black | STCO31193RF2 | STCO31193LF2 | STCO31183RF2 | STCO31183LF2 |
| M | 21 - 22 | Beige | STCO31194RF1 | STCO31194LF1 | STCO31184RF1 | STCO31184LF1 |
| M | 21 - 22 | Black | STCO31194RF2 | STCO31194LF2 | STCO31184RF2 | STCO31184LF2 |
| L | 23 - 24 | Beige | STCO31195RF1 | STCO31195LF1 | STCO31185RF1 | STCO31185LF1 |
| L | 23 - 24 | Black | STCO31195RF2 | STCO31195LF2 | STCO31185RF2 | STCO31185LF2 |
| XL | 25 - 26 | Beige | STCO31196RF1 | STCO31196LF1 | STCO31186RF1 | STCO31186LF1 |
| XL | 25 - 26 | Black | STCO31196RF2 | STCO31196LF2 | STCO31186RF2 | STCO31186LF2 |
| XXL | 27 - 28 | Beige | STCO31197RF1 | STCO31197LF1 | STCO31187RF1 | STCO31187LF1 |
| XXL | 27 - 28 | Black | STCO31197RF2 | STCO31197LF2 | STCO31187RF2 | STCO31187LF2 |

Product information:

Anatomical construction between the fingers.

*Circumference at MCP.

| | |
|--------------------|--------------------|
| Material: | Strong |
| Colours: | Beige, black |
| Compression level: | Ccl 2 |
| Details: | Wrist edging ELA 3 |



Product picture, code:
CO203108F1 (Custom-made)
STCO3104RF1 (Standard-size)

| GAUNTLET | | CUSTOM-MADE |
|----------|------------------|-------------|
| Model | Product / Art.No | |
| | Thumb closed | Thumb open |
| Adults | CO203109 | CO203108 |
| Children | CO103109 | CO103108 |

Children's models are made for children under 10.

Product information:

Anatomical construction in the thumb fourchette. Order a sleeve-gauntlet when the length of the shaft is greater than the length of the palm.

| | |
|--------------------|---|
| Material: | Strong (Ccl 1-4) or medium (Ccl 1-3) |
| Colours: | All |
| Compression level: | All available |
| Fastening: | Available separately |
| Details: | Wrist edging ELA 3 Palm edging ELA 2 |

| GAUNTLET | | | STANDARD-SIZE | |
|----------|-----------|--------|-------------------|-------------|
| Size | Circumf.* | Colour | Product / Art No. | |
| | cm | | Thumb open | |
| | | | Right | Left |
| XXS | 14 - 16 | Beige | STCO3101RF1 | STCO3101LF1 |
| XXS | 14 - 16 | Black | STCO3101RF2 | STCO3101LF2 |
| XS | 17 - 18 | Beige | STCO3102RF1 | STCO3102LF1 |
| XS | 17 - 18 | Black | STCO3102RF2 | STCO3102LF2 |
| S | 19 - 20 | Beige | STCO3103RF1 | STCO3103LF1 |
| S | 19 - 20 | Black | STCO3103RF2 | STCO3103LF2 |
| M | 21 - 22 | Beige | STCO3104RF1 | STCO3104LF1 |
| M | 21 - 22 | Black | STCO3104RF2 | STCO3104LF2 |
| L | 23 - 24 | Beige | STCO3105RF1 | STCO3105LF1 |
| L | 23 - 24 | Black | STCO3105RF2 | STCO3105LF2 |
| XL | 25 - 26 | Beige | STCO3106RF1 | STCO3106LF1 |
| XL | 25 - 26 | Black | STCO3106RF2 | STCO3106LF2 |
| XXL | 27 - 28 | Beige | STCO3107RF1 | STCO3107LF1 |
| XXL | 27 - 28 | Black | STCO3107RF2 | STCO3107LF2 |

Product information:

Thumb tip open.

*Circumference at MCP.

| | |
|--------------------|---|
| Material: | Strong |
| Colours: | Beige, black |
| Compression level: | Ccl 2 |
| Details: | Wrist edging ELA 3 Palm edging ELA 2 |



Product picture, code:
CO2011F1



Product picture, code:
CO2012F1

| CHIN STRAP | | CUSTOM-MADE |
|------------|------------------|-----------------|
| Model | Product / Art.No | |
| | Ear area open | Ear area closed |
| Adults | CO2011 | CO2012 |
| Children | CO1011 | CO1012 |

Children's models are made for children under 10.

Product information:

Velcro fastenings on the crown of the head and back of the neck.

Material: Strong (Ccl 1-4) or medium (Ccl 1-3)

Colours: All

Compression level: All available

Fastening: ELA 1 with velcro

Details: Edging: ELA 3



Product picture, code:
CO20138F1



Product picture, code:
CO20139F1

| HELMET | | CUSTOM-MADE |
|----------|------------------|-------------|
| Model | Product / Art.No | |
| | Face open | Face closed |
| Adults | CO20138 | CO20139 |
| Children | CO10138 | CO10139 |

Children's models are made for children under 10.

Product information:

Two different models are available, one of which features a closed face area. The closed-face model is designed to keep scar and wound care products, dressings, or a facial mask securely in place. If necessary, openings can be cut into the face area for the mouth, nose, and eyes. The material does not fray after cutting.

| | |
|--------------------|--|
| Material: | Strong (Ccl 1-4) or medium (Ccl 1-3) |
| Colours: | All |
| Compression level: | All available |
| Fastening: | 1-row hooks / zipper with guard From back of head to neck |
| Details: | Edging ELA 3 |

SENSE™

NEUROLOGICAL PRESSURE GARMENTS

CUSTOM-MADE

STANDARD-SIZE





Product picture, code:
SEE10422F5

| SCAPULA SUPPORT VEST | | | CUSTOM-MADE |
|----------------------|------------------|-----------------------|---------------|
| Model | Product / Art.No | | |
| | Long sleeves | Long and short sleeve | Short sleeves |
| Women | SEE21433 | SEE21423 | SEE21422 |
| Men | SEE22433 | SEE22423 | SEE22422 |
| Children | SEE10433 | SEE10423 | SEE10422 |

Children's models are made for children under 10.

Product information:

The product is intended for use in connection with Erb's palsy and serratus palsy. Adjustable, elastic hook-and-loop straps and a neoprene reinforcement in the scapular and shoulder area. The straps are made of elastane tape. Available as an additional option with bilateral scapular support. The long-sleeved shirt features anatomically shaped elbow bends. The product features axilla constructions made from light fabric. The women's shirt features shaping seams at the breasts. The women's model can be manufactured without bust shaping seams free of charge upon request.

| | |
|-----------------|---|
| Material: | Strong or medium |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | 2-row hooks / zipper with guard |
| Details: | Hemline SIL 1 Neckline ELA 3 Edge of sleeve ELA 2 |


| SCAPULA SUPPORT BODY | | | CUSTOM-MADE |
|----------------------|------------------|-----------------------|---------------|
| Model | Product / Art.No | | |
| | Long sleeves | Long and short sleeve | Short sleeves |
| Children | SEE104033 | SEE104023 | SEE104022 |

Children's models are made for children under 10.

Product information:

The product is intended for use in connection with Erb's palsy and serratus palsy. Adjustable, elastic hook-and-loop straps and a neoprene reinforcement in the scapular and shoulder area. The straps are made of elastane tape. Available with bilateral scapular support and short legs. Adjustable snap fastening in the crotch. The long-sleeved shirt features anatomically shaped elbow bends. The product features axilla constructions made from light fabric.

| | |
|-----------------|---|
| Material: | Strong or medium |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | 2-row hooks / zipper with guard 2-row snap fasteners at the crotch |
| Details: | Short legs: 1-row snap fasteners Neckline and leg opening ELA 3 Edge of sleeve ELA 2 Short legs: ELA 2 |

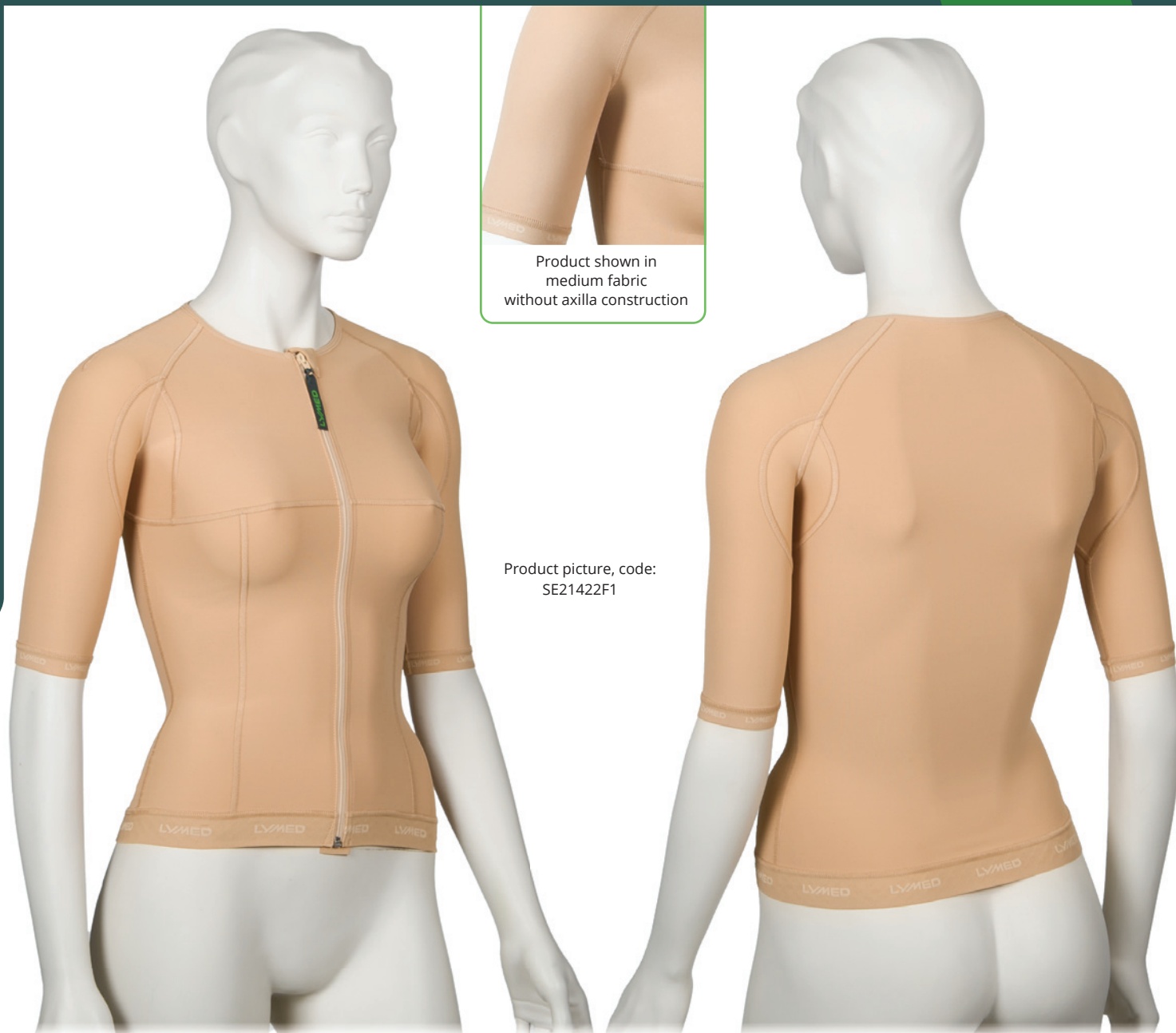

Product shown in medium fabric without axilla construction

Product picture, code: SE21422F1

| VEST | | CUSTOM-MADE | | |
|----------|------------------|---------------|-----------------------|--------------|
| Model | Product / Art.No | | | |
| | Sleeveless | Short sleeves | Long and short sleeve | Long sleeves |
| Women | SE21411 | SE21422 | SE21423 | SE21433 |
| Men | SE22411 | SE22422 | SE22423 | SE22433 |
| Children | SE10411 | SE10422 | SE10423 | SE10433 |

Children's models are made for children under 10.

Separate abdominal support available for midsection: please see product details from Lymed Skin -product group page 41

Product information:

The long-sleeved vest features anatomically shaped elbow bends. In products made from strong fabric, the axilla area uses light fabric construction. Products made from medium or light fabric do not include axilla construction. The women's vest features shaping seams at the breasts. The women's model can be manufactured without bust shaping seams free of charge upon request.

| | |
|-----------------|--|
| Material: | Strong, medium, light (only light pressure) |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | 2-row hooks / zipper with guard / no fastening |
| Details: | Helpline SIL 1 |
| | Neckline ELA 3 |
| | Edge of sleeve ELA 2 |
| | Sleeveless model: edging ELA 3 |



Product shown in medium fabric without axilla construction

Product picture, code:
SE1045333F6

| OVERALL | | CUSTOM-MADE |
|------------------------------|-------------------------|-------------|
| <i>Model</i> | <i>Product / Art.No</i> | |
| Long sleeves and legs | | |
| Children | SE10453333 | |

Children's models are made for children under 10.

Product information:

Overall with anatomical elbow and knee constructions. In products made from strong fabric, the underarm are uses medium fabric construction. Products made from medium or light fabric do not include underarm construction.

| | |
|--------------------|---|
| Material: | Strong, medium, light (only light pressure) |
| Colours available: | All |
| Pressure level: | Selectable |
| Fastening: | 2-row hooks / zipper with guard |
| Details: | Neckline ELA 3 |
| | Leg opening ELA 2 |
| | Edge of sleeve ELA 2 |



Product picture, code:
SE104011F5 SEA10L

| BODY | | CUSTOM-MADE | | |
|----------|------------------|---------------|-----------------------|--------------|
| Model | Product / Art.No | | | |
| | Sleeveless | Short sleeves | Long and short sleeve | Long sleeves |
| Women | SE214011 | SE214022 | SE214023 | SE214033 |
| Children | SE104011 | SE104022 | SE104023 | SE104033 |

Children's models are made for children under 10.

Product information:

The long-sleeved body features anatomically shaped elbow bends and in products made from strong fabric, the axilla area uses medium fabric construction. Products made from medium or light fabric do not include underarm construction.

The women's body features shaping seams at the breasts. The women's model can be manufactured without bust shaping seams free of charge upon request. Short legs are available as an addition.

| | |
|-----------------|--|
| Material: | Strong, medium, light (only light pressure) |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Women: 2-row hooks / zipper with guard 2-row snap fasteners at the crotch Short legs: no crotch closure |
| Details: | Children: 2-row hooks / zipper with guard 2-row snap fasteners at the crotch Short legs: 1-row snap fasteners |
| | Neckline and leg opening ELA 3 |
| | Edge of sleeve ELA 2 |
| | Sleeveless model: edging ELA 3 |
| | Short legs: edge of leg ELA 2 |



Product picture, code:
STSE404F7

SENSE™

ADJUSTABLE BODY

STANDARD-SIZE


| Size | Art. No | Colourcode |
|----------------------|---------|----------------------------------|
| Short sleeves | | |
| 1 | STSE401 | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 2 | STSE402 | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 3 | STSE403 | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 4 | STSE404 | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 5 | STSE405 | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 6 | STSE406 | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 7 | STSE407 | F1 / F2 / F3 / F4 / F5 / F6 / F7 |

| Sizes | Measurements (cm) | | | | | |
|-------|-------------------|------|-----|----|----|-----|
| | B30 | B11 | B17 | B2 | B9 | B12 |
| 1 | 80 - 86 | 24 | 30 | 43 | 40 | 42 |
| 2 | 84 - 90 | 25 | 31 | 47 | 43 | 45 |
| 3 | 88 - 96 | 26 | 33 | 51 | 46 | 48 |
| 4 | 94 - 102 | 27 | 35 | 55 | 49 | 52 |
| 5 | 100 - 108 | 28,5 | 37 | 59 | 52 | 56 |
| 6 | 107 - 115 | 30 | 39 | 63 | 55 | 60 |
| 7 | 114 - 122 | 31,5 | 41 | 67 | 58 | 64 |

Product information:

The product can be used in particular as an assistive garment in physiotherapy and occupational therapy. Adjustable hook-and-loop fastening at the back. The lighter fabric in the abdominal area adds adaptability to the fit. The product features axilla constructions made from medium fabric. Adjustable snap fastening in the crotch.

| | |
|-----------------|---|
| Material: | Strong Abdominal area: medium |
| Colours: | All |
| Pressure level: | Adjustable |
| Fastening: | Velcro at the back, 3- row snap fasteners at the crotch |
| Details: | Neckline and leg opening ELA 3 Edge of sleeve ELA 2 |

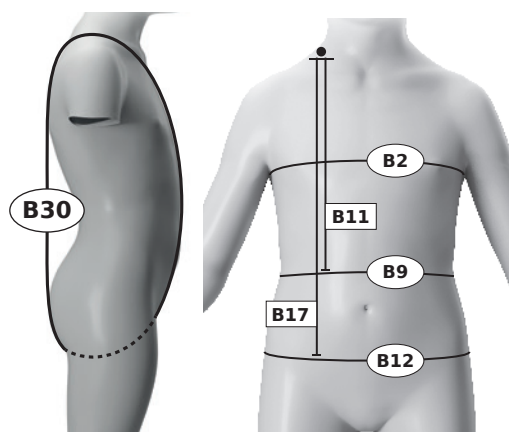



Product picture, code:
SE22522F4

Product information:

Long-leg pants with anatomical construction at the knee.
Stocking legs with anatomical construction at the ankle and knee.
When a stocking leg is ordered in strong material, the constructions are made from medium material.
Men's models with contouring at the front.

| | |
|-----------------|---|
| Material: | Strong, medium, light (only light pressure) |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Available separately |
| Details: | Waist ELA 1 Leg opening ELA 2 |

PANTS

CUSTOM-MADE

| Model | Product / Art.No | | | |
|-------|------------------|--------------------|--------------------------------------|--|
| | Short legs | Short and long leg | Short leg and leg with stocking OPEN | Short leg and leg with stocking CLOSED |
| Women | SE21522 | SE21523 | SE215248 | SE215249 |
| Men | SE22522 | SE22523 | SE225248 | SE225249 |
| Girls | SE11522 | SE11523 | SE115248 | SE115249 |
| Boys | SE12522 | SE12523 | SE125248 | SE125249 |

Children's models are made for children under 10.

| Model | Product / Art.No | | | | | | |
|-------|------------------|-------------------------------------|---------------------------------------|--|--|---|---|
| | Long legs | Long leg and leg with stocking OPEN | Long leg and leg with stocking CLOSED | Short leg and leg with toe sock, toe tips OPEN | Short leg and leg with toe sock, toe tips CLOSED | Long leg and leg with toe sock, toe tips OPEN | Long leg and leg with toe sock, toe tips CLOSED |
| Women | SE21533 | SE215348 | SE215349 | SE21524FG8 | SE21524FG9 | SE21534FG8 | SE21534FG9 |
| Men | SE22533 | SE225348 | SE225349 | SE22524FG8 | SE22524FG9 | SE22534FG8 | SE22534FG9 |
| Girls | SE11533 | SE115348 | SE115349 | SE11524FG8 | SE11524FG9 | SE11534FG8 | SE11534FG9 |
| Boys | SE12533 | SE125348 | SE125349 | SE12524FG8 | SE12524FG9 | SE12534FG8 | SE12534FG9 |

Children's models are made for children under 10.

STOCKINGS

CUSTOM-MADE

| Model | Product / Art.No | | | | | |
|-------|------------------|-------------|--------------------------------------|--|-------------------------------|---------------------------------|
| | Toes OPEN | Toes CLOSED | One leg with toe sock, toe tips OPEN | One leg with toe sock, toe tips CLOSED | With toe socks, toe tips OPEN | With toe socks, toe tips CLOSED |
| Women | SE215448 | SE215449 | SE21544FG8 | SE21544FG9 | SE21544FG28 | SE21544FG29 |
| Men | SE225448 | SE225449 | SE22544FG8 | SE22544FG9 | SE22544FG28 | SE22544FG29 |
| Girls | SE115448 | SE115449 | SE11544FG8 | SE11544FG9 | SE11544FG28 | SE11544FG29 |
| Boys | SE125448 | SE125449 | SE12544FG8 | SE12544FG9 | SE12544FG28 | SE12544FG29 |

Children's models are made for children under 10.



Product picture, code:
SE20603F1



Product picture, code:
SE10639F5



Product picture, code:
SE10629F6

| LEG | | CUSTOM-MADE | | |
|----------|------------------|-------------|-------------|--|
| Model | Product / Art.No | | | |
| | Ankle-knee | Knee-thigh | Ankle-thigh | |
| Adults | SE20601 | SE20602 | SE20603 | |
| Children | SE10601 | SE10602 | SE10603 | |

Children's models are made for children under 10.

Product information:

The leg sleeve features an anatomically shaped construction in the knee bend area. When the product is ordered in strong fabric, the knee bend construction is made from medium fabric.

| | |
|-----------------|--|
| Material: | Strong, medium, light (only light pressure) |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Available separately |
| Details: | Knee-high: upper edge SIL 1 |
| | Upper edge SIL 4, upper edge in knee-high SIL 1, lower edge ELA 2. |

| STOCKING | | CUSTOM-MADE | | |
|----------|------------------|------------------|-----------|------------|
| Model | Toes | Product / Art.No | | |
| | | Ankle-high | Knee-high | Thigh-high |
| Adults | Closed | SE20619 | SE20629 | SE20639 |
| Adults | Open | SE20618 | SE20628 | SE20638 |
| Adults | Toe sock, closed | SE20619FG | SE20629FG | SE20639FG |
| Adults | Toe sock, open | SE20618FG | SE20628FG | SE20638FG |
| Children | Closed | SE10619 | SE10629 | SE10639 |
| Children | Open | SE10618 | SE10628 | SE10638 |
| Children | Toe sock, closed | SE10619FG | SE10629FG | SE10639FG |
| Children | Toe sock, open | SE10618FG | SE10628FG | SE10638FG |

Children's models are made for children under 10.

Product information:

Stockings feature anatomical ankle and knee bend shaping. When the stocking is ordered in strong material, the anatomical shapings are made from medium material. In open-toe stockings, the toe is lightly beveled. The ankle sock's shaft length is measured upwards from the L/R61 circumference. Please note that the shaft length can be at most the same as the L/R67 measurement.

| | |
|-----------------|---|
| Material: | Strong, medium, light (only light pressure) |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Available separately |
| Details: | Upper edge SIL 1, upper-edge in thigh-high SIL 4, lower edge, tip open ELA 2. |



Product picture, code: SE20303F1



Product picture, code: SE2031038F1



Product picture, code: SE2031138F1

SLEEVE **CUSTOM-MADE**

| Model | Product / Art.No | | |
|----------|------------------|--------------|--------------|
| | Wrist-elbow | Elbow-axilla | Wrist-axilla |
| Adults | SE20301 | SE20302 | SE20303 |
| Children | SE10301 | SE10302 | SE10303 |

Children's models are made for children under 10.

Product information:

The wrist-to-axilla model features an anatomical elbow bend.

| | |
|-----------------|---|
| Material: | Strong, medium, light (only light pressure) |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Available separately |
| Details: | Upper edge SIL 1 Lower edge ELA 2 |

SLEEVE GAUNTLET **CUSTOM-MADE**

| Model | Thumb | Product / Art.No | |
|----------|--------|------------------|-----------|
| | | To elbow | To axilla |
| Adults | Closed | SE2031019 | SE2031039 |
| Adults | Open | SE2031018 | SE2031038 |
| Children | Closed | SE1031019 | SE1031039 |
| Children | Open | SE1031018 | SE1031038 |

Children's models are made for children under 10.

Product information:

Axilla-length model with anatomical elbow construction. Anatomical construction in the thumb fourchette.

| | |
|-----------------|---|
| Material: | Strong, medium, light (only light pressure) |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Available separately |
| Details: | Upper edge SIL 1 Lower edge ELA 2 |

SLEEVE GLOVE **CUSTOM-MADE**

| Model | Fingertips | Product / Art.No | |
|----------|------------|------------------|-----------|
| | | To elbow | To axilla |
| Adults | Closed | SE2031119 | SE2031139 |
| Adults | Open | SE2031118 | SE2031138 |
| Children | Closed | SE1031119 | SE1031139 |
| Children | Open | SE1031118 | SE1031138 |

Children's models are made for children under 10.

Product information:

Axilla-length model with anatomical elbow construction. Anatomical construction between the fingers and in the thumb fourchette.

| | |
|-----------------|---|
| Material: | Strong, medium, light (only light pressure) |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Available separately |
| Details: | Upper edge SIL 1 |

SLEEVE FASTENING - MODEL 1 CUSTOM-MADE


| Model | Product / Art.No |
|--------------------------------|------------------|
| Model 1 - Shoulder flap | |
| Adults | A20S1 |
| Children | A10S1 |

Children's models are made for children under 10.

Product information:

Elastic strap running under the breast to hold the sleeve up. Fastening with velcro. Suitable especially for users with large breasts. Available for all sleeves reaching up to axilla. Women's sleeve with a bra strap holder.

| | |
|-----------|----------------|
| Material: | ELA 1 |
| Colour: | Same as sleeve |

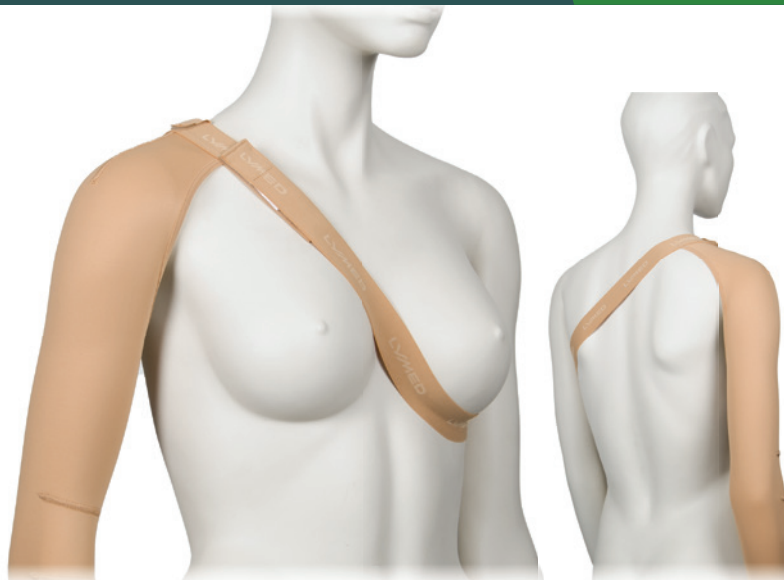

SLEEVE FASTENING - MODEL 2 CUSTOM-MADE


| Model | Product / Art.No |
|--------------------------------|------------------|
| Model 2 - Shoulder flap | |
| Adults | A20S2 |
| Children | A10S2 |

Children's models are made for children under 10.

Product information:

Elastic strap running under the axilla to hold the sleeve up. Fastening with velcro. Available for all sleeves reaching up to axilla.

| | |
|-----------|----------------|
| Material: | ELA 1 |
| Colour: | Same as sleeve |

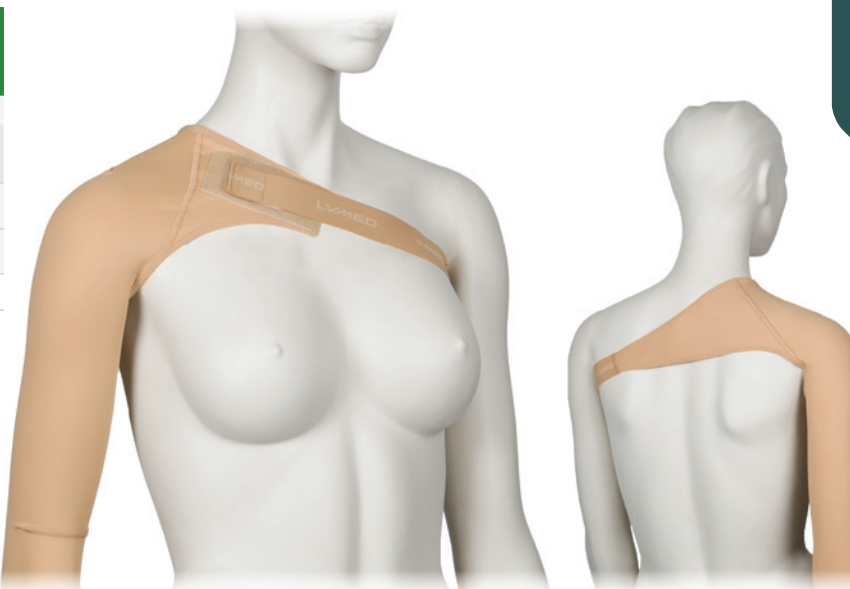

SLEEVE FASTENING - MODEL 3 CUSTOM-MADE


| Model | Product / Art.No |
|----------------------------------|------------------|
| Model 3 - Opposite sleeve | |
| Adults | A20S3 |
| Children | A10S3 |

Children's models are made for children under 10.

Product information:

In the sleeve attachment model, a counter-sleeve connects two axilla-length sleeves. The model is suitable for situations where maintaining the position of one or both sleeves is challenging. The counter-sleeve configuration can be used in different setups: to connect two sleeves or to support the positioning of a single sleeve. Two sleeves must be ordered in conjunction with the attachment. If the ordered sleeves are made from different materials, the attachment component is manufactured by default using the lighter material.

| | |
|------------|---|
| Material: | Strong, medium, light (only light pressure) |
| Colour: | Same as sleeve |
| Fastening: | 2-row hooks / zipper with guard |
| Details: | Neckline ELA 3 |
| | Hemline SIL 2 |
| | Edge of sleeves ELA 2 |

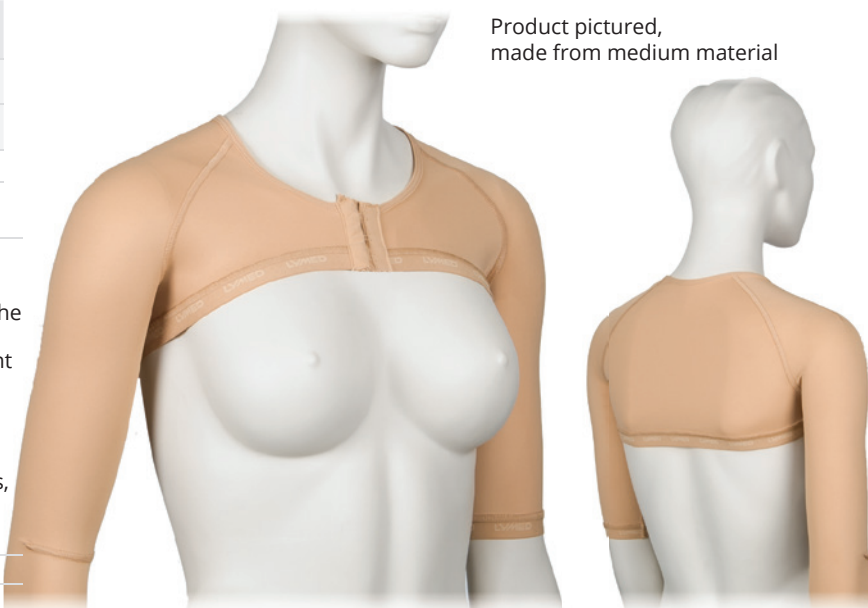

Product pictured, made from medium material

See also vest model on p. 58



Product picture, code:
SE203119F1



Product picture, code:
SE203118F1

| GLOVE | | CUSTOM-MADE |
|----------|-------------------|-----------------|
| Model | Product / Art.No | |
| | Fingertips closed | Fingertips open |
| Adults | SE203119 | SE203118 |
| Children | SE103119 | SE103118 |

Children's models are made for children under 10.

Product information:

Anatomical construction between the fingers and in the thumb fourchette. Number of backhand seams/darts in glove may vary due to individual measurements and pressure/compression levels.

Order a sleeve-glove when the length of the shaft is greater than the length of the palm.

| | |
|-----------------|---|
| Material: | Strong, medium, light (only light pressure) |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Available separately |
| Details: | Wrist edging ELA 3 |



Product picture, code:
SE203108F1

| GAUNTLET | | CUSTOM-MADE |
|----------|------------------|-------------|
| Model | Product / Art.No | |
| | Thumb closed | Thumb open |
| Adults | SE203109 | SE203108 |
| Children | SE103109 | SE103108 |

Children's models are made for children under 10.

Product information:

Anatomical construction in the thumb fourchette. Order a sleeve-glove when the length of the shaft is greater than the length of the palm.

| | |
|-----------------|---|
| Material: | Strong, medium, light (only light pressure) |
| Colours: | All |
| Pressure level: | Selectable |
| Fastening: | Available separately |
| Details: | Wrist edging ELA 3 Palm edging ELA 2 |



Product picture, code: STSE3115LF7

GLOVE WITH SPLINT POCKET STANDARD-SIZE

| Size | Product / Art.No | | Colour code |
|------|------------------|-----------|----------------------------------|
| | Right | Left | |
| 1 | STSE3111R | STSE3111L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 2 | STSE3112R | STSE3112L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 3 | STSE3113R | STSE3113L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 4 | STSE3114R | STSE3114L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 5 | STSE3115R | STSE3115L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 6 | STSE3116R | STSE3116L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 7 | STSE3117R | STSE3117L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 8 | STSE3118R | STSE3118L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 9 | STSE3119R | STSE3119L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 10 | STSE3110R | STSE3110L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |

GAUNTLET WITH SPLINT POCKET STANDARD-SIZE

| Size | Product / Art.No | | Colour code |
|------|------------------|------------|----------------------------------|
| | Right | Left | |
| 1 | STSE3101R | STSE3101L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 2 | STSE3102R | STSE3102L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 3 | STSE3103R | STSE3103L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 4 | STSE3104R | STSE3104L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 5 | STSE3105R | STSE3105L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 6 | STSE3106R | STSE3106L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 7 | STSE3107R | STSE3107L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 8 | STSE3108R | STSE3108L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 9 | STSE3109R | STSE3109L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |
| 10 | STSE31010R | STSE31010L | F1 / F2 / F3 / F4 / F5 / F6 / F7 |


Product information:

Sense glove and gauntlet include a zipper and a pocket for optional splint or other reinforcements. The product is used in connection with different tonus deficiencies or hemiplegia.

Glove with open fingertips. Gauntlet with open thumb tip.

| | |
|-----------------|--------------------|
| Material: | Strong |
| Colours: | All |
| Pressure level: | 14 - 20 mmHg |
| Fastening: | Zipper |
| Details: | Wrist edging ELA 3 |

| Sizes | Measurements (cm) | | | |
|-------|-------------------|------|------|-----|
| | MCP | L11 | L16 | L15 |
| 1 | 11 | 10,5 | 14 | 6 |
| 2 | 12 | 11 | 14,5 | 6 |
| 3 | 13 | 11,5 | 15,5 | 8,5 |
| 4 | 14 | 12 | 16 | 8,5 |
| 5 | 15 | 12,5 | 16,5 | 9 |
| 6 | 16 | 13,5 | 17 | 9 |
| 7 | 17 | 14,5 | 17,5 | 10 |
| 8 | 18 | 16 | 18,5 | 11 |
| 9 | 19 | 16,5 | 20 | 11 |
| 10 | 20 | 17,5 | 23 | 12 |

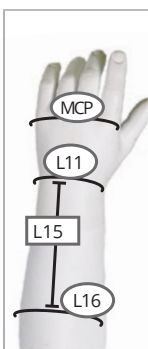

Helmets and chin straps available, please see details from Lymed Skin product group pages 54-55.



Product picture, code: STSE3133RF1

| CI GLOVE | | STANDARD-SIZE | |
|----------|-------------|------------------|-------------|
| Size | Circumf.* | Product / Art.No | |
| | cm | Right | Left |
| 1 | 12,0 - 13,5 | STSE3131RF1 | STSE3131LF1 |
| 2 | 13,5 - 15,0 | STSE3132RF1 | STSE3132LF1 |
| 3 | 15,0 - 16,5 | STSE3133RF1 | STSE3133LF1 |
| 4 | 16,5 - 18,0 | STSE3134RF1 | STSE3134LF1 |
| 5 | 18,0 - 19,5 | STSE3135RF1 | STSE3135LF1 |

Product information:

Used in constraint-included movement therapy, for the healthy hand.

*Circumference at MCP

| | |
|-----------------|--------------------|
| Material: | Strong |
| Colour: | Beige |
| Pressure level: | No pressure |
| Details: | Wrist edging ELA 3 |

INTERIM & LIGHT™

INTERIM CARE AND LIGHT PRESSURE GARMENTS

STANDARD-SIZE





Product information:

Short or long sleeved model with long hemline reaching down to the pelvis.

*Circumference of chest under axillae, above breasts.

| | |
|-----------------|---|
| Material: | Light |
| Colours: | Beige, black |
| Pressure level: | 10 - 18 mmHg |
| Fastening: | 2-row hooks / zipper with guard |
| Details: | Neckline ELA 3, hemline SIL 1, edge of sleeve ELA 2 |

Product picture, code: STIN224226F1H

MEN'S VEST - WITH HOOKS

STANDARD-SIZE

| Size | Circumf.* cm | Colour | Product / Art.No | |
|------|-----------------|--------|------------------|----------------|
| | | | Short sleeves | Long sleeves |
| 1 | 81 - 84 | Beige | STIN224221F1H | STIN224331F1H |
| 1 | 81 - 84 | Black | STIN224221F2H | STIN224331F2H |
| 2 | 85 - 88 | Beige | STIN224222F1H | STIN224332F1H |
| 2 | 85 - 88 | Black | STIN224222F2H | STIN224332F2H |
| 3 | 89 - 92 | Beige | STIN224223F1H | STIN224333F1H |
| 3 | 89 - 92 | Black | STIN224223F2H | STIN224333F2H |
| 4 | 93 - 96 | Beige | STIN224224F1H | STIN224334F1H |
| 4 | 93 - 96 | Black | STIN224224F2H | STIN224334F2H |
| 5 | 97 - 100 | Beige | STIN224225F1H | STIN224335F1H |
| 5 | 97 - 100 | Black | STIN224225F2H | STIN224335F2H |
| 6 | 101 - 104 | Beige | STIN224226F1H | STIN224336F1H |
| 6 | 101 - 104 | Black | STIN224226F2H | STIN224336F2H |
| 7 | 105 - 108 | Beige | STIN224227F1H | STIN224337F1H |
| 7 | 105 - 108 | Black | STIN224227F2H | STIN224337F2H |
| 8 | 109 - 112 | Beige | STIN224228F1H | STIN224338F1H |
| 8 | 109 - 112 | Black | STIN224228F2H | STIN224338F2H |
| 9 | 113 - 116 | Beige | STIN224229F1H | STIN224339F1H |
| 9 | 113 - 116 | Black | STIN224229F2H | STIN224339F2H |
| 10 | 117 - 120 | Beige | STIN2242210F1H | STIN2243310F1H |
| 10 | 117 - 120 | Black | STIN2242210F2H | STIN2243310F2H |

MEN'S VEST - WITH ZIPPER

STANDARD-SIZE

| Size | Circumf.* cm | Colour | Product / Art.No | |
|------|-----------------|--------|------------------|----------------|
| | | | Short sleeves | Long sleeves |
| 1 | 81 - 84 | Beige | STIN224221F1Z | STIN224331F1Z |
| 1 | 81 - 84 | Black | STIN224221F2Z | STIN224331F2Z |
| 2 | 85 - 88 | Beige | STIN224222F1Z | STIN224332F1Z |
| 2 | 85 - 88 | Black | STIN224222F2Z | STIN224332F2Z |
| 3 | 89 - 92 | Beige | STIN224223F1Z | STIN224333F1Z |
| 3 | 89 - 92 | Black | STIN224223F2Z | STIN224333F2Z |
| 4 | 93 - 96 | Beige | STIN224224F1Z | STIN224334F1Z |
| 4 | 93 - 96 | Black | STIN224224F2Z | STIN224334F2Z |
| 5 | 97 - 100 | Beige | STIN224225F1Z | STIN224335F1Z |
| 5 | 97 - 100 | Black | STIN224225F2Z | STIN224335F2Z |
| 6 | 101 - 104 | Beige | STIN224226F1Z | STIN224336F1Z |
| 6 | 101 - 104 | Black | STIN224226F2Z | STIN224336F2Z |
| 7 | 105 - 108 | Beige | STIN224227F1Z | STIN224337F1Z |
| 7 | 105 - 108 | Black | STIN224227F2Z | STIN224337F2Z |
| 8 | 109 - 112 | Beige | STIN224228F1Z | STIN224338F1Z |
| 8 | 109 - 112 | Black | STIN224228F2Z | STIN224338F2Z |
| 9 | 113 - 116 | Beige | STIN224229F1Z | STIN224339F1Z |
| 9 | 113 - 116 | Black | STIN224229F2Z | STIN224339F2Z |
| 10 | 117 - 120 | Beige | STIN2242210F1Z | STIN2243310F1Z |
| 10 | 117 - 120 | Black | STIN2242210F2Z | STIN2243310F2Z |



Product information:

Short or long sleeved model with long hemline reaching down to the pelvis. Women's vest with anatomical breast construction.

*Circumference of chest under axillae, above breasts.

| | |
|-----------------|---|
| Material: | Light |
| Colours: | Beige, black |
| Pressure level: | 10 - 18 mmHg |
| Fastening: | 2-row hooks / zipper with guard |
| Details: | Neckline ELA 3, hemline SIL 1, edge of sleeve ELA 2 |

Product picture, code: STIN214226F1H

WOMEN'S VEST - WITH HOOKS STANDARD-SIZE

| Size | Circumf.* cm | Colour | Product / Art.No | |
|------|-----------------|--------|------------------|----------------|
| | | | Short sleeves | Long sleeves |
| 1 | 76 - 80 | Beige | STIN214221F1H | STIN214331F1H |
| 1 | 76 - 80 | Black | STIN214221F2H | STIN214331F2H |
| 2 | 81 - 84 | Beige | STIN214222F1H | STIN214332F1H |
| 2 | 81 - 84 | Black | STIN214222F2H | STIN214332F2H |
| 3 | 85 - 88 | Beige | STIN214223F1H | STIN214333F1H |
| 3 | 85 - 88 | Black | STIN214223F2H | STIN214333F2H |
| 4 | 89 - 92 | Beige | STIN214224F1H | STIN214334F1H |
| 4 | 89 - 92 | Black | STIN214224F2H | STIN214334F2H |
| 5 | 93 - 96 | Beige | STIN214225F1H | STIN214335F1H |
| 5 | 93 - 96 | Black | STIN214225F2H | STIN214335F2H |
| 6 | 97 - 100 | Beige | STIN214226F1H | STIN214336F1H |
| 6 | 97 - 100 | Black | STIN214226F2H | STIN214336F2H |
| 7 | 101 - 104 | Beige | STIN214227F1H | STIN214337F1H |
| 7 | 101 - 104 | Black | STIN214227F2H | STIN214337F2H |
| 8 | 105 - 108 | Beige | STIN214228F1H | STIN214338F1H |
| 8 | 105 - 108 | Black | STIN214228F2H | STIN214338F2H |
| 9 | 109 - 112 | Beige | STIN214229F1H | STIN214339F1H |
| 9 | 109 - 112 | Black | STIN214229F2H | STIN214339F2H |
| 10 | 113 - 116 | Beige | STIN2142210F1H | STIN2143310F1H |
| 10 | 113 - 116 | Black | STIN2142210F2H | STIN2143310F2H |
| 11 | 117 - 119 | Beige | STIN2142211F1H | STIN2143311F1H |
| 11 | 117 - 119 | Black | STIN2142211F2H | STIN2143311F2H |

WOMEN'S VEST - WITH ZIPPER STANDARD-SIZE

| Size | Circumf.* cm | Colour | Product / Art.No | |
|------|-----------------|--------|------------------|----------------|
| | | | Short sleeves | Long sleeves |
| 1 | 76 - 80 | Beige | STIN214221F1Z | STIN214331F1Z |
| 1 | 76 - 80 | Black | STIN214221F2Z | STIN214331F2Z |
| 2 | 81 - 84 | Beige | STIN214222F1Z | STIN214332F1Z |
| 2 | 81 - 84 | Black | STIN214222F2Z | STIN214332F2Z |
| 3 | 85 - 88 | Beige | STIN214223F1Z | STIN214333F1Z |
| 3 | 85 - 88 | Black | STIN214223F2Z | STIN214333F2Z |
| 4 | 89 - 92 | Beige | STIN214224F1Z | STIN214334F1Z |
| 4 | 89 - 92 | Black | STIN214224F2Z | STIN214334F2Z |
| 5 | 93 - 96 | Beige | STIN214225F1Z | STIN214335F1Z |
| 5 | 93 - 96 | Black | STIN214225F2Z | STIN214335F2Z |
| 6 | 97 - 100 | Beige | STIN214226F1Z | STIN214336F1Z |
| 6 | 97 - 100 | Black | STIN214226F2Z | STIN214336F2Z |
| 7 | 101 - 104 | Beige | STIN214227F1Z | STIN214337F1Z |
| 7 | 101 - 104 | Black | STIN214227F2Z | STIN214337F2Z |
| 8 | 105 - 108 | Beige | STIN214228F1Z | STIN214338F1Z |
| 8 | 105 - 108 | Black | STIN214228F2Z | STIN214338F2Z |
| 9 | 109 - 112 | Beige | STIN214229F1Z | STIN214339F1Z |
| 9 | 109 - 112 | Black | STIN214229F2Z | STIN214339F2Z |
| 10 | 113 - 116 | Beige | STIN2142210F1Z | STIN2143310F1Z |
| 10 | 113 - 116 | Black | STIN2142210F2Z | STIN2143310F2Z |
| 11 | 117 - 119 | Beige | STIN2142211F1Z | STIN2143311F1Z |
| 11 | 117 - 119 | Black | STIN2142211F2Z | STIN2143311F2Z |



Product information:

Short or long sleeved model with long hemline reaching down to the pelvis.

*Circumference of chest under axillae, above breasts.

| | |
|-----------------|---|
| Material: | Light |
| Colours: | Beige, black |
| Pressure level: | 10 - 18 mmHg |
| Fastening: | 2-row hooks / zipper with guard |
| Details: | Neckline ELA 3, hemline SIL 1, edge of sleeve ELA 3 |

Product picture, code: STIN104226F1H

CHILDREN'S VEST - WITH HOOKS

STANDARD-SIZE

| Size | Circumf.* cm | Colour | Product / Art.No | |
|------|-----------------|--------|------------------|----------------|
| | | | Short sleeves | Long sleeves |
| 1 | 43 - 45 | Beige | STIN104221FH | STIN104331F1H |
| 1 | 43 - 45 | Black | STIN104221F2H | STIN104331F2H |
| 2 | 46 - 49 | Beige | STIN104222F1H | STIN104332F1H |
| 2 | 46 - 49 | Black | STIN104222F2H | STIN104332F2H |
| 3 | 50 - 54 | Beige | STIN104223F1H | STIN104333F1H |
| 3 | 50 - 54 | Black | STIN104223F2H | STIN104333F2H |
| 4 | 55 - 59 | Beige | STIN104224F1H | STIN104334F1H |
| 4 | 55 - 59 | Black | STIN104224F2H | STIN104334F2H |
| 5 | 60 - 63 | Beige | STIN104225F1H | STIN104335F1H |
| 5 | 60 - 63 | Black | STIN104225F2H | STIN104335F2H |
| 6 | 64 - 67 | Beige | STIN104226F1H | STIN104336F1H |
| 6 | 64 - 67 | Black | STIN104226F2H | STIN104336F2H |
| 7 | 68 - 71 | Beige | STIN104227F1H | STIN104337F1H |
| 7 | 68 - 71 | Black | STIN104227F2H | STIN104337F2H |
| 8 | 72 - 75 | Beige | STIN104228F1H | STIN104338F1H |
| 8 | 72 - 75 | Black | STIN104228F2H | STIN104338F2H |
| 9 | 76 - 79 | Beige | STIN104229F1H | STIN104339F1H |
| 9 | 76 - 79 | Black | STIN104229F2H | STIN104339F2H |
| 10 | 80 - 83 | Beige | STIN1042210F1H | STIN1043310F1H |
| 10 | 80 - 83 | Black | STIN1042210F2H | STIN1043310F2H |

CHILDREN'S VEST - WITH ZIPPER

STANDARD-SIZE

| Size | Circumf.* cm | Colour | Product / Art.No | |
|------|-----------------|--------|------------------|----------------|
| | | | Short sleeves | Long sleeves |
| 1 | 43 - 45 | Beige | STIN104221F1Z | STIN104331F1Z |
| 1 | 43 - 45 | Black | STIN104221F2Z | STIN104331F2Z |
| 2 | 46 - 49 | Beige | STIN104222F1Z | STIN104332F1Z |
| 2 | 46 - 49 | Black | STIN104222F2Z | STIN104332F2Z |
| 3 | 50 - 54 | Beige | STIN104223F1Z | STIN104333F1Z |
| 3 | 50 - 54 | Black | STIN104223F2Z | STIN104333F2Z |
| 4 | 55 - 59 | Beige | STIN104224F1Z | STIN104334F1Z |
| 4 | 55 - 59 | Black | STIN104224F2Z | STIN104334F2Z |
| 5 | 60 - 63 | Beige | STIN104225F1Z | STIN104335F1Z |
| 5 | 60 - 63 | Black | STIN104225F2Z | STIN104335F2Z |
| 6 | 64 - 67 | Beige | STIN104226F1Z | STIN104336F1Z |
| 6 | 64 - 67 | Black | STIN104226F2Z | STIN104336F2Z |
| 7 | 68 - 71 | Beige | STIN104227F1Z | STIN104337F1Z |
| 7 | 68 - 71 | Black | STIN104227F2Z | STIN104337F2Z |
| 8 | 72 - 75 | Beige | STIN104228F1Z | STIN104338F1Z |
| 8 | 72 - 75 | Black | STIN104228F2Z | STIN104338F2Z |
| 9 | 76 - 79 | Beige | STIN104229F1Z | STIN104339F1Z |
| 9 | 76 - 79 | Black | STIN104229F2Z | STIN104339F2Z |
| 10 | 80 - 83 | Beige | STIN1042210F1Z | STIN1043310F1Z |
| 10 | 80 - 83 | Black | STIN1042210F2Z | STIN1043310F2Z |



Product picture, code: STIN225336F1

Product information:

Normal waist. Men's models with contouring in the front.

*Circumference at widest point of pelvis.

| | |
|-----------------|---|
| Material: | Light |
| Colours: | Beige, black |
| Pressure level: | 10 - 18 mmHg |
| Details: | Waist: ELA 1, Leg opening ELA 2, Children's pants leg opening ELA3 |

MEN'S PANTS STANDARD-SIZE

| Size | Circumf.* cm | Colour | Product / Art.No | |
|------|-----------------|--------|------------------|---------------|
| | | | Short legs | Long legs |
| 1 | 87 - 92 | Beige | STIN225221F1 | STIN225331F1 |
| 1 | 87 - 92 | Black | STIN225221F2 | STIN225331F2 |
| 2 | 93 - 96 | Beige | STIN225222F1 | STIN225332F1 |
| 2 | 93 - 96 | Black | STIN225222F2 | STIN225332F2 |
| 3 | 97 - 100 | Beige | STIN225223F1 | STIN225333F1 |
| 3 | 97 - 100 | Black | STIN225223F2 | STIN225333F2 |
| 4 | 101 - 104 | Beige | STIN225224F1 | STIN225334F1 |
| 4 | 101 - 104 | Black | STIN225224F2 | STIN225334F2 |
| 5 | 105 - 108 | Beige | STIN225225F1 | STIN225335F1 |
| 5 | 105 - 108 | Black | STIN225225F2 | STIN225335F2 |
| 6 | 109 - 112 | Beige | STIN225226F1 | STIN225336F1 |
| 6 | 109 - 112 | Black | STIN225226F2 | STIN225336F2 |
| 7 | 113 - 116 | Beige | STIN225227F1 | STIN225337F1 |
| 7 | 113 - 116 | Black | STIN225227F2 | STIN225337F2 |
| 8 | 117 - 120 | Beige | STIN225228F1 | STIN225338F1 |
| 8 | 117 - 120 | Black | STIN225228F2 | STIN225338F2 |
| 9 | 121 - 124 | Beige | STIN225229F1 | STIN225339F1 |
| 9 | 121 - 124 | Black | STIN225229F2 | STIN225339F2 |
| 10 | 125 - 128 | Beige | STIN2252210F1 | STIN2253310F1 |
| 10 | 125 - 128 | Black | STIN2252210F2 | STIN2253310F2 |

WOMEN'S PANTS STANDARD-SIZE

| Size | Circumf.* cm | Colour | Product / Art.No | |
|------|-----------------|--------|------------------|---------------|
| | | | Short legs | Long legs |
| 1 | 87 - 91 | Beige | STIN215221F1 | STIN215331F1 |
| 1 | 87 - 91 | Black | STIN215221F2 | STIN215331F2 |
| 2 | 92 - 95 | Beige | STIN215222F1 | STIN215332F1 |
| 2 | 92 - 95 | Black | STIN215222F2 | STIN215332F2 |
| 3 | 96 - 99 | Beige | STIN215223F1 | STIN215333F1 |
| 3 | 96 - 99 | Black | STIN215223F2 | STIN215333F2 |
| 4 | 100 - 103 | Beige | STIN215224F1 | STIN215334F1 |
| 4 | 100 - 103 | Black | STIN215224F2 | STIN215334F2 |
| 5 | 104 - 107 | Beige | STIN215225F1 | STIN215335F1 |
| 5 | 104 - 107 | Black | STIN215225F2 | STIN215335F2 |
| 6 | 108 - 111 | Beige | STIN215226F1 | STIN215336F1 |
| 6 | 108 - 111 | Black | STIN215226F2 | STIN215336F2 |
| 7 | 112 - 115 | Beige | STIN215227F1 | STIN215337F1 |
| 7 | 112 - 115 | Black | STIN215227F2 | STIN215337F2 |
| 8 | 116 - 119 | Beige | STIN215228F1 | STIN215338F1 |
| 8 | 116 - 119 | Black | STIN215228F2 | STIN215338F2 |
| 9 | 120 - 123 | Beige | STIN215229F1 | STIN215339F1 |
| 9 | 120 - 123 | Black | STIN215229F2 | STIN215339F2 |
| 10 | 124 - 127 | Beige | STIN2152210F1 | STIN2153310F1 |
| 10 | 124 - 127 | Black | STIN2152210F2 | STIN2153310F2 |



Product picture, code: STIN215336F1



Product picture, code: STIN105336F1

CHILDREN'S PANTS

STANDARD-SIZE

| Size | Circumf.* cm | Colour | Product / Art.No | |
|------|-----------------|--------|------------------|---------------|
| | | | Short legs | Long legs |
| 1 | 47 - 50 | Beige | STIN105221F1 | STIN105331F1 |
| 1 | 47 - 50 | Black | STIN105221F2 | STIN105331F2 |
| 2 | 51 - 54 | Beige | STIN105222F1 | STIN105332F1 |
| 2 | 51 - 54 | Black | STIN105222F2 | STIN105332F2 |
| 3 | 55 - 58 | Beige | STIN105223F1 | STIN105333F1 |
| 3 | 55 - 58 | Black | STIN105223F2 | STIN105333F2 |
| 4 | 59 - 61 | Beige | STIN105224F1 | STIN105334F1 |
| 4 | 59 - 61 | Black | STIN105224F2 | STIN105334F2 |
| 5 | 62 - 66 | Beige | STIN105225F1 | STIN105335F1 |
| 5 | 62 - 66 | Black | STIN105225F2 | STIN105335F2 |
| 6 | 67 - 70 | Beige | STIN105226F1 | STIN105336F1 |
| 6 | 67 - 70 | Black | STIN105226F2 | STIN105336F2 |
| 7 | 71 - 74 | Beige | STIN105227F1 | STIN105337F1 |
| 7 | 71 - 74 | Black | STIN105227F2 | STIN105337F2 |
| 8 | 75 - 78 | Beige | STIN105228F1 | STIN105338F1 |
| 8 | 75 - 78 | Black | STIN105228F2 | STIN105338F2 |
| 9 | 79 - 82 | Beige | STIN105229F1 | STIN105339F1 |
| 9 | 79 - 82 | Black | STIN105229F2 | STIN105339F2 |
| 10 | 83 - 86 | Beige | STIN1052210F1 | STIN1053310F1 |
| 10 | 83 - 86 | Black | STIN1052210F2 | STIN1053310F2 |



Product picture, code:
STIN2031194RF1



Product picture, code:
STIN2031184RF1

GLOVE

STANDARD-SIZE

Product information:

Anatomical construction between the fingers.

**Circumference at MCP.*

| | |
|-----------------|--------------------|
| Material: | Light |
| Colours: | Beige, black |
| Pressure level: | 10 - 18 mmHg |
| Details: | Wrist edging ELA 3 |

| ADULT'S GLOVE | | | | | | STANDARD-SIZE |
|---------------|-----------|--------|-------------------------|------------------------|-----------------------|----------------------|
| Size | Circumf.* | Colour | Product / Art. No | | | |
| | cm | | Fingertips closed right | Fingertips closed left | Fingertips open right | Fingertips open left |
| 1 | 13 - 14 | Beige | STIN2031191RF1 | STIN2031191LF1 | STIN2031181RF1 | STIN2031181LF1 |
| 1 | 13 - 14 | Black | STIN2031191RF2 | STIN2031191LF2 | STIN2031181RF2 | STIN2031181LF2 |
| 2 | 15 - 16 | Beige | STIN2031192RF1 | STIN2031192LF1 | STIN2031182RF1 | STIN2031182LF1 |
| 2 | 15 - 16 | Black | STIN2031192RF2 | STIN2031192LF2 | STIN2031182RF2 | STIN2031182LF2 |
| 3 | 17 - 18 | Beige | STIN2031193RF1 | STIN2031193LF1 | STIN2031183RF1 | STIN2031183LF1 |
| 3 | 17 - 18 | Black | STIN2031193RF2 | STIN2031193LF2 | STIN2031183RF2 | STIN2031183LF2 |
| 4 | 19 - 20 | Beige | STIN2031194RF1 | STIN2031194LF1 | STIN2031184RF1 | STIN2031184LF1 |
| 4 | 19 - 20 | Black | STIN2031194RF2 | STIN2031194LF2 | STIN2031184RF2 | STIN2031184LF2 |
| 5 | 21 - 22 | Beige | STIN2031195RF1 | STIN2031195LF1 | STIN2031185RF1 | STIN2031185LF1 |
| 5 | 21 - 22 | Black | STIN2031195RF2 | STIN2031195LF2 | STIN2031185RF2 | STIN2031185LF2 |
| 6 | 23 - 24 | Beige | STIN2031196RF1 | STIN2031196LF1 | STIN2031186RF1 | STIN2031186LF1 |
| 6 | 23 - 24 | Black | STIN2031196RF2 | STIN2031196LF2 | STIN2031186RF2 | STIN2031186LF2 |
| 7 | 25 - 26 | Beige | STIN2031197RF1 | STIN2031197LF1 | STIN2031187RF1 | STIN2031187LF1 |
| 7 | 25 - 26 | Black | STIN2031197RF2 | STIN2031197LF2 | STIN2031187RF2 | STIN2031187LF2 |
| 8 | 27 - 28 | Beige | STIN2031198RF1 | STIN2031198LF1 | STIN2031188RF1 | STIN2031188LF1 |
| 8 | 27 - 28 | Black | STIN2031198RF2 | STIN2031198LF2 | STIN2031188RF2 | STIN2031188LF2 |

| CHILDREN'S GLOVE | | | | | | STANDARD-SIZE |
|------------------|-----------|--------|-------------------------|------------------------|-----------------------|----------------------|
| Size | Circumf.* | Colour | Product / Art. No | | | |
| | cm | | Fingertips closed right | Fingertips closed left | Fingertips open right | Fingertips open left |
| 1 | 10 - 11 | Beige | STIN1031191RF1 | STIN1031191LF1 | STIN1031181RF1 | STIN1031181LF1 |
| 1 | 10 - 11 | Black | STIN1031191RF2 | STIN1031191LF2 | STIN1031181RF2 | STIN1031181LF2 |
| 2 | 12 - 13 | Beige | STIN1031192RF1 | STIN1031192LF1 | STIN1031182RF1 | STIN1031182LF1 |
| 2 | 12 - 13 | Black | STIN1031192RF2 | STIN1031192LF2 | STIN1031182RF2 | STIN1031182LF2 |



| ADULT'S SLEEVE | | | STANDARD-SIZE |
|----------------|-----------------|--------|---------------|
| Size | Circumf.* cm | Colour | Art. No |
| 1 | 18 - 20 | Beige | STIN20301F1 |
| 1 | 18 - 20 | Black | STIN20301F2 |
| 2 | 20 - 22 | Beige | STIN20302F1 |
| 2 | 20 - 22 | Black | STIN20302F2 |
| 3 | 22 - 24 | Beige | STIN20303F1 |
| 3 | 22 - 24 | Black | STIN20303F2 |
| 4 | 24 - 26 | Beige | STIN20304F1 |
| 4 | 24 - 26 | Black | STIN20304F2 |
| 5 | 26 - 28 | Beige | STIN20305F1 |
| 5 | 26 - 28 | Black | STIN20305F2 |
| 6 | 28 - 30 | Beige | STIN20306F1 |
| 6 | 28 - 30 | Black | STIN20306F2 |
| 7 | 30 - 32 | Beige | STIN20307F1 |
| 7 | 30 - 32 | Black | STIN20307F2 |
| 8 | 32 - 34 | Beige | STIN20308F1 |
| 8 | 32 - 34 | Black | STIN20308F2 |
| 9 | 34 - 36 | Beige | STIN20309F1 |
| 9 | 34 - 36 | Black | STIN20309F2 |
| 10 | 36 - 38 | Beige | STIN203010F1 |
| 10 | 36 - 38 | Black | STIN203010F2 |
| 11 | 38 - 40 | Beige | STIN203011F1 |
| 11 | 38 - 40 | Black | STIN203011F2 |

Product information:

Sleeve length in size 6: 43,5 cm.
Wrist-axilla.
**Circumference at widest point of upper arm.*

| | |
|-----------------|--------------------------------------|
| Material: | Light |
| Colours: | Beige, black |
| Pressure level: | 10 - 18 mmHg |
| Details: | Upper edge SIL 2 Lower edge ELA 2 |

Product picture, code: STIN20305F1

| CHILDREN'S SLEEVE | | | STANDARD-SIZE |
|-------------------|-----------------|--------|---------------|
| Size | Circumf.* cm | Colour | Art. No |
| 1 | 10 - 12 | Beige | STIN10301F1 |
| 1 | 10 - 12 | Black | STIN10301F2 |
| 2 | 12 - 14 | Beige | STIN10302F1 |
| 2 | 12 - 14 | Black | STIN10302F2 |
| 3 | 14 - 16 | Beige | STIN10303F1 |
| 3 | 14 - 16 | Black | STIN10303F2 |
| 4 | 16 - 18 | Beige | STIN10304F1 |
| 4 | 16 - 18 | Black | STIN10304F2 |
| 5 | 18 - 20 | Beige | STIN10305F1 |
| 5 | 18 - 20 | Black | STIN10305F2 |

Product information:

Sleeve length in size 3: 26,5 cm.
Wrist-axilla.
**Circumference at widest point of upper arm.*

| | |
|-----------------|--------------------------------------|
| Material: | Light |
| Colours: | Beige, black |
| Pressure level: | 10 - 18 mmHg |
| Details: | Upper edge SIL 2 Lower edge ELA 3 |



Product picture, code:
STIN20692F1

Product picture, code:
STIN20682F1

ADULT'S STOCKING

STANDARD-SIZE

| Size | Circumf.* cm | Shoe size EU | Colour | Product / Art.No Toes closed |
|------|-----------------|-----------------|--------|---------------------------------|
| 1 | 28 - 32 | 35 - 38 | Beige | STIN20691F1 |
| 1 | 28 - 32 | 35 - 38 | Black | STIN20691F2 |
| 2 | 33 - 37 | 38 - 40 | Beige | STIN20692F1 |
| 2 | 33 - 37 | 38 - 40 | Black | STIN20692F2 |
| 3 | 38 - 42 | 40 - 43 | Beige | STIN20693F1 |
| 3 | 38 - 42 | 40 - 43 | Black | STIN20693F2 |
| 4 | 43 - 46 | 43 - 46 | Beige | STIN20694F1 |
| 4 | 43 - 46 | 43 - 46 | Black | STIN20694F2 |
| cm | | | | Toes open |
| 1 | 28 - 32 | | Beige | STIN20681F1 |
| 1 | 28 - 32 | | Black | STIN20681F2 |
| 2 | 33 - 37 | | Beige | STIN20682F1 |
| 2 | 33 - 37 | | Black | STIN20682F2 |
| 3 | 38 - 42 | | Beige | STIN20683F1 |
| 3 | 38 - 42 | | Black | STIN20683F2 |
| 4 | 43 - 46 | | Beige | STIN20684F1 |
| 4 | 43 - 46 | | Black | STIN20684F2 |

Product information:

Stocking that fits both left and right foot.
*Circumference of widest point of calf.

| | |
|-----------------|--|
| Material: | Light |
| Colours: | Beige, black |
| Pressure level: | 10 - 18 mmHg |
| Details: | Upper edge SIL 2 Edging, tip open ELA 2 |

CHILDREN'S STOCKING

STANDARD-SIZE

| Size | Circumf.* cm | Shoe size EU | Colour | Product / Art.No Toes closed |
|------|-----------------|-----------------|--------|---------------------------------|
| 1 | 16 - 19 | 23 - 26 | Beige | STIN10691F1 |
| 1 | 16 - 19 | 23 - 26 | Black | STIN10691F2 |
| 2 | 20 - 23 | 26 - 31 | Beige | STIN10692F1 |
| 2 | 20 - 23 | 26 - 31 | Black | STIN10692F2 |
| 3 | 24 - 27 | 31 - 36 | Beige | STIN10693F1 |
| 3 | 24 - 27 | 31 - 36 | Black | STIN10693F2 |
| cm | | | | Toes open |
| 1 | 16 - 19 | | Beige | STIN10681F1 |
| 1 | 16 - 19 | | Black | STIN10681F2 |
| 2 | 20 - 23 | | Beige | STIN10682F1 |
| 2 | 20 - 23 | | Black | STIN10682F2 |
| 3 | 24 - 27 | | Beige | STIN10683F1 |
| 3 | 24 - 27 | | Black | STIN10683F2 |

Product information:

Stocking that fits both left and right foot.
*Circumference of widest point of calf.

| | |
|-----------------|--|
| Material: | Light |
| Colours: | Beige, black |
| Pressure level: | 10 - 18 mmHg |
| Details: | Upper edge SIL 2 Edging, tip open ELA 3 |

Product information:

*Circumference of the head.

| | |
|-----------------|---------------------------|
| Material: | Light |
| Colours: | Beige, black |
| Pressure level: | 10 - 18 mmHg |
| Fastening: | Velcro in the back |
| Details: | Edges: overlocking stitch |

CHIN STRAP

STANDARD-SIZE

| Size | Circumf.* cm | Colour | Product / Art. No |
|------|-----------------|--------|-------------------|
| 1 | 47 - 50 | Beige | STIN1381F1 |
| 1 | 47 - 50 | Black | STIN1381F2 |
| 2 | 51 - 53 | Beige | STIN1382F1 |
| 2 | 51 - 53 | Black | STIN1382F2 |
| 3 | 54 - 56 | Beige | STIN1383F1 |
| 3 | 54 - 56 | Black | STIN1383F2 |
| 4 | 57 - 59 | Beige | STIN1384F1 |
| 4 | 57 - 59 | Black | STIN1384F2 |
| 5 | 60 - 62 | Beige | STIN1385F1 |
| 5 | 60 - 62 | Black | STIN1385F2 |

HELMET

STANDARD-SIZE

| Size | Circumf.* cm | Colour | Product / Art. No |
|------|-----------------|--------|-------------------|
| 1 | 47 - 50 | Beige | STIN1391F1 |
| 1 | 47 - 50 | Black | STIN1391F2 |
| 2 | 51 - 53 | Beige | STIN1392F1 |
| 2 | 51 - 53 | Black | STIN1392F2 |
| 3 | 54 - 56 | Beige | STIN1393F1 |
| 3 | 54 - 56 | Black | STIN1393F2 |
| 4 | 57 - 59 | Beige | STIN1394F1 |
| 4 | 57 - 59 | Black | STIN1394F2 |
| 5 | 60 - 62 | Beige | STIN1395F1 |
| 5 | 60 - 62 | Black | STIN1395F2 |

Product information:

Features a closed face area. The closed-face model is designed to keep scar and wound care products, dressings, or a facial mask securely in place. If necessary, openings can be cut into the face area for the mouth, nose, and eyes. The material does not fray after cutting.

*Circumference of the head.

| | |
|-----------------|---------------------------|
| Material: | Light |
| Colours: | Beige, black |
| Pressure level: | 10 - 18 mmHg |
| Fastening: | Occipital hooks |
| Details: | Edges: overlocking stitch |



Product picture, code: STIN1383F1



Product picture, code: STIN1393F1



POST-OPERATION™

POST-OPERATIONAL SUPPORT GARMENTS

STANDARD-SIZE



Product picture, code:
STPOP21535HWF2C



Product picture, code:
STPOP21525HWF2C



Product picture, code:
STPOP21515HWF2C



Product picture, code:
STPOP21505HWF2C



Ways to wear shoulder straps

PANTS FOR WOMEN | HIGH WAIST

STANDARD-SIZE

Product information:

Women's support pants intended for use in connection with mid- and lower-body surgical procedures. Also suitable for use after childbirth. The pants feature a high waist and double-layer fabric in the abdominal area. Elastic and adjustable suspenders with front opening. An additional component is included, allowing the suspenders to be configured in a Y-shape. *Circumference at widest point of pelvis and upper thigh.

| | |
|-----------------|--|
| Material: | Medium |
| Colour: | Beige, black |
| Pressure level: | 23 - 30 mmHg |
| Fastening: | Zipper with guard and hooks on the side of pelvis. |
| Details: | Waist ELA 3 |
| | Leg opening ELA 2 |

PANTS FOR WOMEN | HIGH WAIST | GENITAL AREA CLOSED
STANDARD-SIZE

| Size | Circumf.* | | Colour | Product / Art. No | | | |
|-------|-----------|-------------|--------|-------------------|-----------------|-----------------|-----------------|
| | Pelvis cm | Thigh cm | | Ankle length | Calf length | Knee length | Short length |
| XS | 86 - 90 | 50,0 - 53,0 | Beige | STPOP21532HWF1C | STPOP21522HWF1C | STPOP21512HWF1C | STPOP21502HWF1C |
| XS | 86 - 90 | 50,0 - 53,0 | Black | STPOP21532HWF2C | STPOP21522HWF2C | STPOP21512HWF2C | STPOP21502HWF2C |
| S | 91 - 95 | 53,5 - 56,0 | Beige | STPOP21533HWF1C | STPOP21523HWF1C | STPOP21513HWF1C | STPOP21503HWF1C |
| S | 91 - 95 | 53,5 - 56,0 | Black | STPOP21533HWF2C | STPOP21523HWF2C | STPOP21513HWF2C | STPOP21503HWF2C |
| M | 96 - 101 | 56,5 - 59,5 | Beige | STPOP21534HWF1C | STPOP21524HWF1C | STPOP21514HWF1C | STPOP21504HWF1C |
| M | 96 - 101 | 56,5 - 59,5 | Black | STPOP21534HWF2C | STPOP21524HWF2C | STPOP21514HWF2C | STPOP21504HWF2C |
| L | 102 - 110 | 60,0 - 62,5 | Beige | STPOP21535HWF1C | STPOP21525HWF1C | STPOP21515HWF1C | STPOP21505HWF1C |
| L | 102 - 110 | 60,0 - 62,5 | Black | STPOP21535HWF2C | STPOP21525HWF2C | STPOP21515HWF2C | STPOP21505HWF2C |
| XL | 111 - 121 | 63,0 - 66,0 | Beige | STPOP21536HWF1C | STPOP21526HWF1C | STPOP21516HWF1C | STPOP21506HWF1C |
| XL | 111 - 121 | 63,0 - 66,0 | Black | STPOP21536HWF2C | STPOP21526HWF2C | STPOP21516HWF2C | STPOP21506HWF2C |
| XXL | 122 - 132 | 66,5 - 69,0 | Beige | STPOP21537HWF1C | STPOP21527HWF1C | STPOP21517HWF1C | STPOP21507HWF1C |
| XXL | 122 - 132 | 66,5 - 69,0 | Black | STPOP21537HWF2C | STPOP21527HWF2C | STPOP21517HWF2C | STPOP21507HWF2C |
| XXXL | 133 - 138 | 69,5 - 71,5 | Beige | STPOP21538HWF1C | STPOP21528HWF1C | STPOP21518HWF1C | STPOP21508HWF1C |
| XXXL | 133 - 138 | 69,5 - 71,5 | Black | STPOP21538HWF2C | STPOP21528HWF2C | STPOP21518HWF2C | STPOP21508HWF2C |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Beige | STPOP21539HWF1C | STPOP21529HWF1C | STPOP21519HWF1C | STPOP21509HWF1C |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Black | STPOP21539HWF2C | STPOP21529HWF2C | STPOP21519HWF2C | STPOP21509HWF2C |

PANTS FOR WOMEN | HIGH WAIST | GENITAL AREA OPEN
STANDARD-SIZE

| Size | Circumf.* | | Colour | Product / Art. No | | | |
|-------|-----------|-------------|--------|-------------------|-----------------|-----------------|-----------------|
| | Pelvis cm | Thigh cm | | Ankle length | Calf length | Knee length | Short length |
| XS | 86 - 90 | 50,0 - 53,0 | Beige | STPOP21532HWF1O | STPOP21522HWF1O | STPOP21512HWF1O | STPOP21502HWF1O |
| XS | 86 - 90 | 50,0 - 53,0 | Black | STPOP21532HWF2O | STPOP21522HWF2O | STPOP21512HWF2O | STPOP21502HWF2O |
| S | 91 - 95 | 53,5 - 56,0 | Beige | STPOP21533HWF1O | STPOP21523HWF1O | STPOP21513HWF1O | STPOP21503HWF1O |
| S | 91 - 95 | 53,5 - 56,0 | Black | STPOP21533HWF2O | STPOP21523HWF2O | STPOP21513HWF2O | STPOP21503HWF2O |
| M | 96 - 101 | 56,5 - 59,5 | Beige | STPOP21534HWF1O | STPOP21524HWF1O | STPOP21514HWF1O | STPOP21504HWF1O |
| M | 96 - 101 | 56,5 - 59,5 | Black | STPOP21534HWF2O | STPOP21524HWF2O | STPOP21514HWF2O | STPOP21504HWF2O |
| L | 102 - 110 | 60,0 - 62,5 | Beige | STPOP21535HWF1O | STPOP21525HWF1O | STPOP21515HWF1O | STPOP21505HWF1O |
| L | 102 - 110 | 60,0 - 62,5 | Black | STPOP21535HWF2O | STPOP21525HWF2O | STPOP21515HWF2O | STPOP21505HWF2O |
| XL | 111 - 121 | 63,0 - 66,0 | Beige | STPOP21536HWF1O | STPOP21526HWF1O | STPOP21516HWF1O | STPOP21506HWF1O |
| XL | 111 - 121 | 63,0 - 66,0 | Black | STPOP21536HWF2O | STPOP21526HWF2O | STPOP21516HWF2O | STPOP21506HWF2O |
| XXL | 122 - 132 | 66,5 - 69,0 | Beige | STPOP21537HWF1O | STPOP21527HWF1O | STPOP21517HWF1O | STPOP21507HWF1O |
| XXL | 122 - 132 | 66,5 - 69,0 | Black | STPOP21537HWF2O | STPOP21527HWF2O | STPOP21517HWF2O | STPOP21507HWF2O |
| XXXL | 133 - 138 | 69,5 - 71,5 | Beige | STPOP21538HWF1O | STPOP21528HWF1O | STPOP21518HWF1O | STPOP21508HWF1O |
| XXXL | 133 - 138 | 69,5 - 71,5 | Black | STPOP21538HWF2O | STPOP21528HWF2O | STPOP21518HWF2O | STPOP21508HWF2O |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Beige | STPOP21539HWF1O | STPOP21529HWF1O | STPOP21519HWF1O | STPOP21509HWF1O |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Black | STPOP21539HWF2O | STPOP21529HWF2O | STPOP21519HWF2O | STPOP21509HWF2O |

PANTS FOR WOMEN | NORMAL WAIST

STANDARD-SIZE

Product information:

Women's support pants intended for use in connection with mid- and lower-body surgical procedures. The pants feature a normal waist. Available with the genital area open or closed. Zipper on the right side.

**Circumference at widest point of pelvis and upper thigh.*

| | |
|-----------------|--|
| Material: | Medium |
| Colour: | Beige, black |
| Pressure level: | 23 - 30 mmHg |
| Fastening: | Zipper with guard and hooks on the side of pelvis. |
| Details: | Waist SIL 1 |
| | Leg opening ELA 2 |

POST-OPERATION™



Product picture, code: STPOP21535F2ZC



Product picture, code: STPOP21525F2ZC



Product picture, code: STPOP21515F2ZC

PANTS FOR WOMEN | NORMAL WAIST | WITHOUT ZIPPER | GENITAL AREA CLOSED
STANDARD-SIZE

| Size | Circumf.* | | Colour | Product / Art. No | | |
|-------|-----------|-------------|--------|-------------------|---------------|---------------|
| | Pelvis cm | Thigh cm | | Ankle length | Calf length | Short length |
| XS | 86 - 90 | 50,0 - 53,0 | Beige | STPOP21532F1C | STPOP21522F1C | STPOP21512F1C |
| XS | 86 - 90 | 50,0 - 53,0 | Black | STPOP21532F2C | STPOP21522F2C | STPOP21512F2C |
| S | 91 - 95 | 53,5 - 56,0 | Beige | STPOP21533F1C | STPOP21523F1C | STPOP21513F1C |
| S | 91 - 95 | 53,5 - 56,0 | Black | STPOP21533F2C | STPOP21523F2C | STPOP21513F2C |
| M | 96 - 101 | 56,5 - 59,5 | Beige | STPOP21534F1C | STPOP21524F1C | STPOP21514F1C |
| M | 96 - 101 | 56,5 - 59,5 | Black | STPOP21534F2C | STPOP21524F2C | STPOP21514F2C |
| L | 102 - 110 | 60,0 - 62,5 | Beige | STPOP21535F1C | STPOP21525F1C | STPOP21515F1C |
| L | 102 - 110 | 60,0 - 62,5 | Black | STPOP21535F2C | STPOP21525F2C | STPOP21515F2C |
| XL | 111 - 121 | 63,0 - 66,0 | Beige | STPOP21536F1C | STPOP21526F1C | STPOP21516F1C |
| XL | 111 - 121 | 63,0 - 66,0 | Black | STPOP21536F2C | STPOP21526F2C | STPOP21516F2C |
| XXL | 122 - 132 | 66,5 - 69,0 | Beige | STPOP21537F1C | STPOP21527F1C | STPOP21517F1C |
| XXL | 122 - 132 | 66,5 - 69,0 | Black | STPOP21537F2C | STPOP21527F2C | STPOP21517F2C |
| XXXL | 133 - 138 | 69,5 - 71,5 | Beige | STPOP21538F1C | STPOP21528F1C | STPOP21518F1C |
| XXXL | 133 - 138 | 69,5 - 71,5 | Black | STPOP21538F2C | STPOP21528F2C | STPOP21518F2C |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Beige | STPOP21539F1C | STPOP21529F1C | STPOP21519F1C |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Black | STPOP21539F2C | STPOP21529F2C | STPOP21519F2C |

PANTS FOR WOMEN | NORMAL WAIST | WITHOUT ZIPPER | GENITAL AREA OPEN
STANDARD-SIZE

| Size | Circumf.* | | Colour | Product / Art. No | | |
|-------|-----------|-------------|--------|-------------------|---------------|---------------|
| | Pelvis cm | Thigh cm | | Ankle length | Calf length | Short length |
| XS | 86 - 90 | 50,0 - 53,0 | Beige | STPOP21532F1O | STPOP21522F1O | STPOP21512F1O |
| XS | 86 - 90 | 50,0 - 53,0 | Black | STPOP21532F2O | STPOP21522F2O | STPOP21512F2O |
| S | 91 - 95 | 53,5 - 56,0 | Beige | STPOP21533F1O | STPOP21523F1O | STPOP21513F1O |
| S | 91 - 95 | 53,5 - 56,0 | Black | STPOP21533F2O | STPOP21523F2O | STPOP21513F2O |
| M | 96 - 101 | 56,5 - 59,5 | Beige | STPOP21534F1O | STPOP21524F1O | STPOP21514F1O |
| M | 96 - 101 | 56,5 - 59,5 | Black | STPOP21534F2O | STPOP21524F2O | STPOP21514F2O |
| L | 102 - 110 | 60,0 - 62,5 | Beige | STPOP21535F1O | STPOP21525F1O | STPOP21515F1O |
| L | 102 - 110 | 60,0 - 62,5 | Black | STPOP21535F2O | STPOP21525F2O | STPOP21515F2O |
| XL | 111 - 121 | 63,0 - 66,0 | Beige | STPOP21536F1O | STPOP21526F1O | STPOP21516F1O |
| XL | 111 - 121 | 63,0 - 66,0 | Black | STPOP21536F2O | STPOP21526F2O | STPOP21516F2O |
| XXL | 122 - 132 | 66,5 - 69,0 | Beige | STPOP21537F1O | STPOP21527F1O | STPOP21517F1O |
| XXL | 122 - 132 | 66,5 - 69,0 | Black | STPOP21537F2O | STPOP21527F2O | STPOP21517F2O |
| XXXL | 133 - 138 | 69,5 - 71,5 | Beige | STPOP21538F1O | STPOP21528F1O | STPOP21518F1O |
| XXXL | 133 - 138 | 69,5 - 71,5 | Black | STPOP21538F2O | STPOP21528F2O | STPOP21518F2O |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Beige | STPOP21539F1O | STPOP21529F1O | STPOP21519F1O |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Black | STPOP21539F2O | STPOP21529F2O | STPOP21519F2O |

PANTS FOR WOMEN | NORMAL WAIST | WITH ZIPPER | GENITAL AREA CLOSED

STANDARD-SIZE

| Size | Circumf.* | | Colour | Product / Art. No | | |
|-------|-----------|-------------|--------|-------------------|----------------|----------------|
| | Pelvis cm | Thigh cm | | Ankle length | Calf length | Short length |
| XS | 86 - 90 | 50,0 - 53,0 | Beige | STPOP21532F1ZC | STPOP21522F1ZC | STPOP21512F1ZC |
| XS | 86 - 90 | 50,0 - 53,0 | Black | STPOP21532F2ZC | STPOP21522F2ZC | STPOP21512F2ZC |
| S | 91 - 95 | 53,5 - 56,0 | Beige | STPOP21533F1ZC | STPOP21523F1ZC | STPOP21513F1ZC |
| S | 91 - 95 | 53,5 - 56,0 | Black | STPOP21533F2ZC | STPOP21523F2ZC | STPOP21513F2ZC |
| M | 96 - 101 | 56,5 - 59,5 | Beige | STPOP21534F1ZC | STPOP21524F1ZC | STPOP21514F1ZC |
| M | 96 - 101 | 56,5 - 59,5 | Black | STPOP21534F2ZC | STPOP21524F2ZC | STPOP21514F2ZC |
| L | 102 - 110 | 60,0 - 62,5 | Beige | STPOP21535F1ZC | STPOP21525F1ZC | STPOP21515F1ZC |
| L | 102 - 110 | 60,0 - 62,5 | Black | STPOP21535F2ZC | STPOP21525F2ZC | STPOP21515F2ZC |
| XL | 111 - 121 | 63,0 - 66,0 | Beige | STPOP21536F1ZC | STPOP21526F1ZC | STPOP21516F1ZC |
| XL | 111 - 121 | 63,0 - 66,0 | Black | STPOP21536F2ZC | STPOP21526F2ZC | STPOP21516F2ZC |
| XXL | 122 - 132 | 66,5 - 69,0 | Beige | STPOP21537F1ZC | STPOP21527F1ZC | STPOP21517F1ZC |
| XXL | 122 - 132 | 66,5 - 69,0 | Black | STPOP21537F2ZC | STPOP21527F2ZC | STPOP21517F2ZC |
| XXXL | 133 - 138 | 69,5 - 71,5 | Beige | STPOP21538F1ZC | STPOP21528F1ZC | STPOP21518F1ZC |
| XXXL | 133 - 138 | 69,5 - 71,5 | Black | STPOP21538F2ZC | STPOP21528F2ZC | STPOP21518F2ZC |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Beige | STPOP21539F1ZC | STPOP21529F1ZC | STPOP21519F1ZC |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Black | STPOP21539F2ZC | STPOP21529F2ZC | STPOP21519F2ZC |

PANTS FOR WOMEN | NORMAL WAIST | WITH ZIPPER | GENITAL AREA OPEN

STANDARD-SIZE

| Size | Circumf.* | | Colour | Product / Art. No | | |
|-------|-----------|-------------|--------|-------------------|----------------|----------------|
| | Pelvis cm | Thigh cm | | Ankle length | Calf length | Short length |
| XS | 86 - 90 | 50,0 - 53,0 | Beige | STPOP21532F1ZO | STPOP21522F1ZO | STPOP21512F1ZO |
| XS | 86 - 90 | 50,0 - 53,0 | Black | STPOP21532F2ZO | STPOP21522F2ZO | STPOP21512F2ZO |
| S | 91 - 95 | 53,5 - 56,0 | Beige | STPOP21533F1ZO | STPOP21523F1ZO | STPOP21513F1ZO |
| S | 91 - 95 | 53,5 - 56,0 | Black | STPOP21533F2ZO | STPOP21523F2ZO | STPOP21513F2ZO |
| M | 96 - 101 | 56,5 - 59,5 | Beige | STPOP21534F1ZO | STPOP21524F1ZO | STPOP21514F1ZO |
| M | 96 - 101 | 56,5 - 59,5 | Black | STPOP21534F2ZO | STPOP21524F2ZO | STPOP21514F2ZO |
| L | 102 - 110 | 60,0 - 62,5 | Beige | STPOP21535F1ZO | STPOP21525F1ZO | STPOP21515F1ZO |
| L | 102 - 110 | 60,0 - 62,5 | Black | STPOP21535F2ZO | STPOP21525F2ZO | STPOP21515F2ZO |
| XL | 111 - 121 | 63,0 - 66,0 | Beige | STPOP21536F1ZO | STPOP21526F1ZO | STPOP21516F1ZO |
| XL | 111 - 121 | 63,0 - 66,0 | Black | STPOP21536F2ZO | STPOP21526F2ZO | STPOP21516F2ZO |
| XXL | 122 - 132 | 66,5 - 69,0 | Beige | STPOP21537F1ZO | STPOP21527F1ZO | STPOP21517F1ZO |
| XXL | 122 - 132 | 66,5 - 69,0 | Black | STPOP21537F2ZO | STPOP21527F2ZO | STPOP21517F2ZO |
| XXXL | 133 - 138 | 69,5 - 71,5 | Beige | STPOP21538F1ZO | STPOP21528F1ZO | STPOP21518F1ZO |
| XXXL | 133 - 138 | 69,5 - 71,5 | Black | STPOP21538F2ZO | STPOP21528F2ZO | STPOP21518F2ZO |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Beige | STPOP21539F1ZO | STPOP21529F1ZO | STPOP21519F1ZO |
| XXXXL | 139 - 144 | 72,0 - 74,5 | Black | STPOP21539F2ZO | STPOP21529F2ZO | STPOP21519F2ZO |



Product picture, code: STPOP2134F1

SLEEVES FOR WOMEN


STANDARD-SIZE

| Size | Circumf. 1* cm | Circumf. 2** cm | Colour | Art. No |
|------|-------------------|--------------------|--------|-------------|
| XS | 76 - 84 | 24,0 - 26,0 | Beige | STPOP2132F1 |
| XS | 76 - 84 | 24,0 - 26,0 | Black | STPOP2132F2 |
| S | 85 - 92 | 26,5 - 28,5 | Beige | STPOP2133F1 |
| S | 85 - 92 | 26,5 - 28,5 | Black | STPOP2133F2 |
| M | 93 - 100 | 29,0 - 31,0 | Beige | STPOP2134F1 |
| M | 93 - 100 | 29,0 - 31,0 | Black | STPOP2134F2 |
| L | 101 - 107 | 31,5 - 33,5 | Beige | STPOP2135F1 |
| L | 101 - 107 | 31,5 - 33,5 | Black | STPOP2135F2 |
| XL | 108 - 114 | 34,0 - 36,0 | Beige | STPOP2136F1 |
| XL | 108 - 114 | 34,0 - 36,0 | Black | STPOP2136F2 |
| XXL | 115 - 124 | 36,5 - 38,5 | Beige | STPOP2137F1 |
| XXL | 115 - 124 | 36,5 - 38,5 | Black | STPOP2137F2 |

Product information:

Women's support sleeves intended for use in connection with upper limb surgical procedures.
Anatomically shaped elbow bends.

*Circumference 1 under axillae, above breasts.
*Circumference 2 at widest point of upper arm.

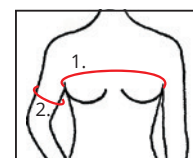

| | |
|-----------------|---|
| Material: | Sleeves: medium Bodice: light |
| Colours: | Beige, black |
| Pressure level: | 23 - 30 mmHg in sleeves No pressure in torso |
| Fastening: | 2-row hooks Hemline ELA 1 |
| Details: | Neckline ELA 3 Edge of sleeve ELA 2 |



Product picture, code:
STPOP2244F2

VEST FOR MEN

STANDARD-SIZE

| Size | Chest circumference under axillae | Waist circumference | Vest length from shoulder to hem | Colour | Product / Art. No |
|------|-----------------------------------|---------------------|----------------------------------|--------|-------------------|
| | cm | cm | cm | | |
| XXXS | 75 - 79 | 69 - 72 | 53,5 | Beige | STPOP2240F1 |
| XXXS | 75 - 79 | 69 - 72 | 53,5 | Black | STPOP2240F2 |
| XXS | 80 - 84 | 72 - 75 | 54,5 | Beige | STPOP2241F1 |
| XXS | 80 - 84 | 72 - 75 | 54,5 | Black | STPOP2241F2 |
| XS | 85 - 89 | 75 - 78 | 55,5 | Beige | STPOP2242F1 |
| XS | 85 - 89 | 75 - 78 | 55,5 | Black | STPOP2242F2 |
| S | 90 - 95 | 79 - 84 | 56,5 | Beige | STPOP2243F1 |
| S | 90 - 95 | 79 - 84 | 56,5 | Black | STPOP2243F2 |
| M | 96 - 101 | 84 - 90 | 59,5 | Beige | STPOP2244F1 |
| M | 96 - 101 | 84 - 90 | 59,5 | Black | STPOP2244F2 |
| L | 102 - 107 | 90 - 96 | 62,5 | Beige | STPOP2245F1 |
| L | 102 - 107 | 90 - 96 | 62,5 | Black | STPOP2245F2 |
| XL | 108 - 113 | 96 - 102 | 65,5 | Beige | STPOP2246F1 |
| XL | 108 - 113 | 96 - 102 | 65,5 | Black | STPOP2246F2 |
| XXL | 114 - 119 | 102 - 108 | 68,5 | Beige | STPOP2247F1 |
| XXL | 114 - 119 | 102 - 108 | 68,5 | Black | STPOP2247F2 |
| XXXL | 120 - 125 | 108 - 114 | 71,5 | Beige | STPOP2248F1 |
| XXXL | 120 - 125 | 108 - 114 | 71,5 | Black | STPOP2248F2 |

Product information:

Men's support shirt intended for use in connection with post-operative care following upper body surgical procedures.

Material: Medium

Colour: Beige, black

Pressure level: 23 - 30 mmHg

Fastening: 2-row hooks

Details: Hemline SIL 1

Neckline and armholes ELA 3



Product picture, code:
STPOP2271VF2

SHORT VEST FOR MEN

STANDARD-SIZE

| Size | Chest circumference under axillae | Circumference at the hem of the vest | Vest length from shoulder to hem* | Colour | Product / Art. No |
|------|-----------------------------------|--------------------------------------|-----------------------------------|--------|-------------------|
| | cm | cm | cm | | |
| XXXS | 75 - 79 | 69 - 72 | 35,5 | Beige | STPOP2270VF1 |
| XXXS | 75 - 79 | 69 - 72 | 35,5 | Black | STPOP2270VF2 |
| XXS | 80 - 84 | 72 - 75 | 36 | Beige | STPOP2271VF1 |
| XXS | 80 - 84 | 72 - 75 | 36 | Black | STPOP2271VF2 |
| XS | 85 - 89 | 75 - 78 | 36,5 | Beige | STPOP2272VF1 |
| XS | 85 - 89 | 75 - 78 | 36,5 | Black | STPOP2272VF2 |
| S | 90 - 95 | 79 - 84 | 37 | Beige | STPOP2273VF1 |
| S | 90 - 95 | 79 - 84 | 37 | Black | STPOP2273VF2 |
| M | 96 - 101 | 84 - 90 | 39,5 | Beige | STPOP2274VF1 |
| M | 96 - 101 | 84 - 90 | 39,5 | Black | STPOP2274VF2 |
| L | 102 - 107 | 90 - 96 | 42 | Beige | STPOP2275VF1 |
| L | 102 - 107 | 90 - 96 | 42 | Black | STPOP2275VF2 |
| XL | 108 - 113 | 96 - 102 | 44,5 | Beige | STPOP2276VF1 |
| XL | 108 - 113 | 96 - 102 | 44,5 | Black | STPOP2276VF2 |
| XXL | 114 - 119 | 102 - 108 | 47 | Beige | STPOP2277VF1 |
| XXL | 114 - 119 | 102 - 108 | 47 | Black | STPOP2277VF2 |
| XXXL | 120 - 125 | 108 - 114 | 49,5 | Beige | STPOP2278VF1 |
| XXXL | 120 - 125 | 108 - 114 | 49,5 | Black | STPOP2278VF2 |

Product information:

Men's support shirt intended for use in connection with post-operative care following upper body surgical procedures. Adjustable velcro straps.

*The shoulder adjustment affects the length of the vest.

| | |
|-----------------|---------------------------------------|
| Material: | Medium |
| Colour: | Beige, black |
| Pressure level: | 23 - 30 mmHg |
| Fastening: | 2-row hooks, adjustable velcro straps |
| Details: | Hemline SIL 1 |
| | Neckline and armholes ELA 3 |

Product picture, code: STPOP714F3



BRA | LIGHT SUPPORT

STANDARD-SIZE

Product information:

The light support bra is intended for use after surgery to keep dressings in place. Adjustable 3 cm wide front-opening straps allow inspection and changing of dressings without removing the bra.

An adjustable hook-and-loop fastening at the back facilitates donning. Available with two cups and with one or two prosthesis pockets.

**Circumference at the chest below the breasts.*

| | |
|-----------------|---------------------------------|
| Material: | Light |
| Colours: | Black, white |
| Pressure level: | No pressure |
| Fastening: | Velcro at the back, width 7 cm |
| Details: | Lower end ELA 1 Edging ELA 3 |



Product picture, code: STPOP714F3

BRA | LIGHT SUPPORT

STANDARD-SIZE

| Size | Circumf.* cm | Colour | Product / Art. No | Two cups | Prosthesis pocket, right | Prosthesis pocket, left | Prosthesis pocket, both |
|------|-----------------|--------|-------------------|--------------|-----------------------------|----------------------------|----------------------------|
| XS | 70 - 75 | Black | STPOP712F2 | STPOP712PRF2 | STPOP712PLF2 | STPOP712PPF2 | STPOP712PPF2 |
| XS | 70 - 75 | White | STPOP712F3 | STPOP712PRF3 | STPOP712PLF3 | STPOP712PPF3 | STPOP712PPF3 |
| S | 73 - 78 | Black | STPOP713F2 | STPOP713PRF2 | STPOP713PLF2 | STPOP713PPF2 | STPOP713PPF2 |
| S | 73 - 78 | White | STPOP713F3 | STPOP713PRF3 | STPOP713PLF3 | STPOP713PPF3 | STPOP713PPF3 |
| M | 76 - 83 | Black | STPOP714F2 | STPOP714PRF2 | STPOP714PLF2 | STPOP714PPF2 | STPOP714PPF2 |
| M | 76 - 83 | White | STPOP714F3 | STPOP714PRF3 | STPOP714PLF3 | STPOP714PPF3 | STPOP714PPF3 |
| L | 81 - 88 | Black | STPOP715F2 | STPOP715PRF2 | STPOP715PLF2 | STPOP715PPF2 | STPOP715PPF2 |
| L | 81 - 88 | White | STPOP715F3 | STPOP715PRF3 | STPOP715PLF3 | STPOP715PPF3 | STPOP715PPF3 |
| XL | 86 - 93 | Black | STPOP716F2 | STPOP716PRF2 | STPOP716PLF2 | STPOP716PPF2 | STPOP716PPF2 |
| XL | 86 - 93 | White | STPOP716F3 | STPOP716PRF3 | STPOP716PLF3 | STPOP716PPF3 | STPOP716PPF3 |
| XXL | 91 - 98 | Black | STPOP717F2 | STPOP717PRF2 | STPOP717PLF2 | STPOP717PPF2 | STPOP717PPF2 |
| XXL | 91 - 98 | White | STPOP717F3 | STPOP717PRF3 | STPOP717PLF3 | STPOP717PPF3 | STPOP717PPF3 |

Product picture, code: STPOP724F3



BRA | MEDIUM SUPPORT

STANDARD-SIZE

Product information:

The medium support bra is intended for use after surgery to keep dressings in place.

The material is stronger than in the light support bra.

Adjustable 3 cm wide front-opening straps allow inspection and changing of dressings without removing the bra.

An adjustable hook-and-loop fastening at the back facilitates donning.

Also available with one or two prosthesis pockets.

**Circumference at the chest below the breasts.*

| | |
|-----------------|---------------------------------|
| Material: | Medium |
| Colours: | Black, white |
| Pressure level: | No pressure |
| Fastening: | Velcro at the back, width 7 cm |
| Details: | Lower end ELA 1 Edging ELA 3 |



Product picture, code: STPOP724F3

BRA | MEDIUM SUPPORT

STANDARD-SIZE

| Size | Circumf.* cm | Colour | Product / Art. No | Two cups | Prosthesis pocket, right | Prosthesis pocket, left | Prosthesis pocket, both |
|------|-----------------|--------|-------------------|--------------|-----------------------------|----------------------------|----------------------------|
| XS | 70 - 75 | Black | STPOP722F2 | STPOP722PRF2 | STPOP722PLF2 | STPOP722PPF2 | |
| XS | 70 - 75 | White | STPOP722F3 | STPOP722PRF3 | STPOP722PLF3 | STPOP722PPF3 | |
| S | 73 - 78 | Black | STPOP723F2 | STPOP723PRF2 | STPOP723PLF2 | STPOP723PPF2 | |
| S | 73 - 78 | White | STPOP723F3 | STPOP723PRF3 | STPOP723PLF3 | STPOP723PPF3 | |
| M | 76 - 83 | Black | STPOP724F2 | STPOP724PRF2 | STPOP724PLF2 | STPOP724PPF2 | |
| M | 76 - 83 | White | STPOP724F3 | STPOP724PRF3 | STPOP724PLF3 | STPOP724PPF3 | |
| L | 81 - 88 | Black | STPOP725F2 | STPOP725PRF2 | STPOP725PLF2 | STPOP725PPF2 | |
| L | 81 - 88 | White | STPOP725F3 | STPOP725PRF3 | STPOP725PLF3 | STPOP725PPF3 | |
| XL | 86 - 93 | Black | STPOP726F2 | STPOP726PRF2 | STPOP726PLF2 | STPOP726PPF2 | |
| XL | 86 - 93 | White | STPOP726F3 | STPOP726PRF3 | STPOP726PLF3 | STPOP726PPF3 | |
| XXL | 91 - 98 | Black | STPOP727F2 | STPOP727PRF2 | STPOP727PLF2 | STPOP727PPF2 | |
| XXL | 91 - 98 | White | STPOP727F3 | STPOP727PRF3 | STPOP727PLF3 | STPOP727PPF3 | |

Product picture, code: STPOP7380DF3H



Product information:

Strong support bras for breast support after surgery.
Can also be used as a dressing bra.
Front fastening between the breasts and a supportive back panel.
3 cm wide, adjustable and openable straps.

**Circumference at the chest below the breasts.*

| | |
|-----------------|---|
| Material: | Strong |
| Colours: | Black, white |
| Pressure level: | No pressure |
| Fastening: | 1-row hooks / zipper with guard and assistive hooks |
| Details: | Lower end ELA 1 Edging ELA 3 |

BRA | STRONG SUPPORT | SHORT

STANDARD-SIZE

| <i>Circumf.*</i> | <i>Art. No (Code + Cup size + Colour + Fastening)</i> | | | |
|------------------|---|---------------------------|----------------------|--------------------------|
| cm | Code | Cup Size | Colour black / white | Fastening hooks / zipper |
| 65 - 69 | STPOP7365 | A / B / C / D / E / F / G | F2 / F3 | H / Z |
| 70 - 74 | STPOP7370 | A / B / C / D / E / F / G | F2 / F3 | H / Z |
| 75 - 79 | STPOP7375 | A / B / C / D / E / F / G | F2 / F3 | H / Z |
| 80 - 84 | STPOP7380 | A / B / C / D / E / F / G | F2 / F3 | H / Z |
| 85 - 89 | STPOP7385 | A / B / C / D / E / F / G | F2 / F3 | H / Z |
| 90 - 94 | STPOP7390 | A / B / C / D / E / F / G | F2 / F3 | H / Z |
| 95 - 99 | STPOP7395 | A / B / C / D / E / F / G | F2 / F3 | H / Z |
| 100 - 104 | STPOP73100 | A / B / C / D / E / F / G | F2 / F3 | H / Z |

Product picture, code: STPOP7480DF3Z



POST-OPERATION™

Product information:

Strong support bras for breast support after surgery.
The long model is longer than the short model and is particularly suitable for larger bust sizes. Can also be used as a dressing bra. Front fastening with auxiliary hooks between the breasts and a supportive back panel. 3 cm wide, adjustable and openable straps.

*Circumference at the chest below the breasts.

| | |
|-----------------|---|
| Material: | Strong |
| Colours: | Black, white |
| Pressure level: | No pressure |
| Fastening: | Zipper with guard and assistive hooks / 1-row hooks |
| Details: | Lower end ELA 1 Edging ELA 3 |

BRA | STRONG SUPPORT | LONG

STANDARD-SIZE

| <i>Circumf.*</i> | <i>Art. No (Code + Cup size + Colour + Fastening)</i> | | | |
|------------------|---|---------------------------|----------------------|--------------------------|
| cm | Code | Cup Size | Colour black / white | Fastening hooks / zipper |
| 65 - 69 | STPOP7465 | A / B / C / D / E / F / G | F2 / F3 | H / Z |
| 70 - 74 | STPOP7470 | A / B / C / D / E / F / G | F2 / F3 | H / Z |
| 75 - 79 | STPOP7475 | A / B / C / D / E / F / G | F2 / F3 | H / Z |
| 80 - 84 | STPOP7480 | A / B / C / D / E / F / G | F2 / F3 | H / Z |
| 85 - 89 | STPOP7485 | A / B / C / D / E / F / G | F2 / F3 | H / Z |
| 90 - 94 | STPOP7490 | A / B / C / D / E / F / G | F2 / F3 | H / Z |
| 95 - 99 | STPOP7495 | A / B / C / D / E / F / G | F2 / F3 | H / Z |
| 100 - 104 | STPOP74100 | A / B / C / D / E / F / G | F2 / F3 | H / Z |



S-SHAPED BREAST WRAP

STANDARD-SIZE

| Size | Circumf.* cm | Art.No |
|------|-----------------|------------|
| XS | 60 - 75 | STPOP752F3 |
| S | 70 - 90 | STPOP753F3 |
| M | 85 - 105 | STPOP754F3 |

Product information:

S-shape band to support the breasts in the aftercare of breast reconstructions. Intended to be worn on top of the bra to provide extra support after surgery. Can be used with strong support bra (see previous page).

Separate band enables the use of the products separately.

Fabric on the sides, adjustable velcro fastening at the front and back. Height of the product is 8 cm from the front.

**Circumference above the breasts.*

| | |
|-----------------|--|
| Material: | Strong, elastic 8 cm wide support band |
| Colour: | White |
| Pressure level: | Adjustable |
| Fastening: | Velcro |
| Details: | Upper and lower edge ELA 3 |

Product picture, code: STPOP753F3



LIGHT ABDOMINAL BAND

STANDARD-SIZE

| <i>Prod.length</i> cm | <i>Height</i> cm | <i>Art.No</i> |
|--------------------------|---------------------|----------------|
| 100 | 18 | STPOP4042100F2 |
| 120 | 18 | STPOP4042120F2 |
| 140 | 18 | STPOP4042140F2 |

Product information:

An adjustable light support belt with a two-part hook-and-loop fastening.
The fastening attaches along the entire length of the product.
Transparent and breathable material.
Select the belt length closest to the required circumference.

| | |
|-----------------|---------------------|
| Material: | Velcro support band |
| Colour: | Black |
| Pressure level: | Adjustable |
| Fastening: | Velcro |

Product picture, code: STPOP4042100F2



Product picture, code: STPOP403180F1



Product picture, code: STPOP403380F1

STRONG ABDOMINAL BAND

STANDARD-SIZE

| Prod.length cm | Height cm | Art.No |
|-------------------|--------------|----------------|
| 80 | 16 | STPOP403180F1 |
| 100 | 16 | STPOP4031100F1 |
| 120 | 16 | STPOP4031120F1 |
| 140 | 16 | STPOP4031140F1 |
| 80 | 26 | STPOP403380F1 |
| 100 | 26 | STPOP4033100F1 |
| 120 | 26 | STPOP4033120F1 |
| 140 | 26 | STPOP4033140F1 |

Product information:

A strong and flexible support belt.

Wide, adjustable hook-and-loop fastening across the full width of the belt. Can also be used, for example, as a back support and as a light hernia belt. Select the belt length closest to the required circumference.

For the lowest height option, select a belt shorter than the circumference.

| | |
|-----------------|---|
| Material: | Strong support band polyamide 86 %, elasthane 9 %, polypropylene 5 % |
| Colour: | Beige |
| Pressure level: | Adjustable |
| Fastening: | Velcro |

Product picture, code: STM4041110F1

MATERNITY SUPPORT BELT

STANDARD-SIZE

| Prod.length cm | Height cm | Art.No |
|-------------------|--------------|--------------|
| 110 | 10 | STM4041110F1 |

Product information:

Strong elastic band to support the growing belly.

| | |
|-----------------|---|
| Material: | Strong support band polyamide 86 %, elasthane 9 %, polypropylene 5 % |
| Colour: | Beige |
| Pressure level: | Adjustable |
| Fastening: | Velcro |





Product picture, code:
STPOP1114F1

Product picture, code:
STPOP1124F1

FACIAL SUPPORT

STANDARD-SIZE

| Size | Circumf.* | Product / Art.No | |
|------|-----------|------------------|-----------------|
| | cm | Ear area open | Ear area closed |
| S | 58 - 62 | STPOP1113F1 | STPOP1123F1 |
| M | 62 - 66 | STPOP1114F1 | STPOP1124F1 |
| L | 66 - 72 | STPOP1115F1 | STPOP1125F1 |
| XL | 72 - 76 | STPOP1116F1 | STPOP1126F1 |

Product information:

Velcro fastenings on the crown of the head and back of the neck.

*Circumference from crown to chin.

| | |
|-----------------|------------------------------------|
| Material: | Medium |
| Colour: | Beige |
| Pressure level: | Adjustable |
| Fastening: | Velcro |
| Details: | Neck strap: ELA 1 Edging: ELA 3 |



Product picture, code:
STPOP115F1

STRONG CHIN STRAP

STANDARD-SIZE

| Size | Colour | Art. No |
|----------|--------|------------|
| One size | Beige | STPOP115F1 |
| One size | White | STPOP115F3 |

Product information:

Consists of several bands with adjustable velcro fastenings.

| | |
|-----------------|--|
| Material: | Strong, elastic 8 cm wide support band |
| Colour: | Beige, white |
| Pressure level: | Adjustable |
| Fastening: | Velcro |

**HELMET FOR HAIR
TRANSPLANT OPERATIONS**

STANDARD-SIZE

| Size | Circumf.* cm | Art.No |
|------|-----------------|-------------|
| S | 50 - 54 | STPOP1393F2 |
| M | 54 - 58 | STPOP1394F2 |

Product information:

Helmet for keeping dressings in place.

**Circumference of the head.*

| | |
|-----------------|-----------------------|
| Material: | Light |
| Colour: | Black |
| Pressure level: | No pressure |
| Fastening: | Velcro below the chin |



Product picture, code: STPOP1394F2

| | |
|------------------|-----|
| SKIN™ | 40 |
| COMPRESSION™ | 57 |
| SENSE™ | 82 |
| INTERIM & LIGHT™ | 96 |
| POST-OPERATION™ | 107 |

LYMED®

Manufacturer:

Lymed Oy

Pyhäjärvenkatu 5 A , 33200 Tampere Finland

Tel.: +358 20 779 2233

Email: orders@lymed.fi

Distributors worldwide: www.lymed.fi