FI	Käyttöohjeet	2
EN	Instructions for Use	5
DA	Brugsanvisning	9
DE	Gebrauchsanweisung	13
ES	Instrucciones de uso	17
ET	Kasutusjuhend	21
FR	Mode d'emploi	24
IT	Istruzioni per l'uso	28
LV	Lietošanas instrukcija	32
NL	Gebruiksaanwijzing	36
NO	Bruksanvisning	40
PL	Instrukcja użytkowania	44
RU	Инструкция по применению	48
SV	Bruksanvisning	52
TR	Kullanım Talimatları	56
AR	مادخت سالات امي لعت	59
FA	هدافت سالام علاروت سد	62

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These instructions apply to the following standard-sized Lymed® products: Skin™, Compression™, Sense™, Interim & Light™, and Post-operation™, which are classified as Class I medical devices in accordance with Regulation (EU) 2017/745 (EU MDR).

Intended users and patient population: These instructions for use are intended for users of Lymed products and the healthcare professionals responsible for their care. In these instructions, the terms pressure garment and product refer to a medical device.

Manufacturer: Lymed Oy, Pyhäjärvenkatu 5, 33200 Tampere, Finland, info@lymed.fi, +358 20 779 2233

Information regarding the warranty conditions applicable to your product and the availability of repair services can be obtained from the granting authority of the Lymed product, the Lymed distributor, or the party responsible for your care.

Date of instructions for use: 27.7.2026
Further information and donning instructions:
www.lymed.fi/manuals

1. About pressure garments – intended use

1.1 A pressure garment is a soft and elastic medical device used to support bodily function by utilising externally applied pressure. Pressure garments can be used by users of different ages in a wide range of use contexts as part of a care and rehabilitation process related to diseases, injuries, and functional capacity challenges.

1.2 In practice, a pressure garment is a closely fitting product that can be designed according to its primary intended use, the intended support effect, and the body area to which pressure is applied. The general term pressure garment may be used to describe both pressure and support garments that provide even pressure, as well as compression garments in which the pressure is graduated and decreases towards the central parts of the body.

1.3 Pressure garments are used as part of a treatment plan guided by a healthcare professional, based on an individual assessment and the recommendation of the treating professional. Pressure garments do not replace other medical treatment or the instructions provided by healthcare professionals.

2. Before first use

2.1 Please note that a Lymed product is a medical device, even if it may appear to be an ordinary garment. Carefully read these instructions before using the Lymed product. Retain these instructions for use for the entire period of product use. If you have any questions regarding your product or its use, contact the healthcare professional responsible for your care, the granting authority of the product, your doctor, or the Lymed distributor who supplied the product.

2.2 The need for use, method of use, and duration of use of the Lymed product are determined by the party responsible for care. If use of the product is not based on the guidance of a healthcare professional, the user must carefully follow these instructions for use and seek assessment by a healthcare professional where necessary.

2.3 Review the donning instructions at www.lymed.fi before use. If a custom-made product has been manufactured for you, ensure that your personal identifier is indicated on the product information label attached to the garment. If user identification is based on a pseudonym, the person or organisation supplying the product must ensure that the product is unequivocally assigned to the correct user. If you are using a standard-sized product, verify from the product information label attached to the garment that the product corresponds to your order. The product must be washed before first use. The feel of the product may change during the first washes as the fabric structure settles.

2.4 The Lymed product is manufactured from materials suitable for its intended purpose. The materials do not contain natural rubber, nickel or other commonly known allergens. See the product-specific fabric composition on the product information label attached to the product. Depending on the product model and selected features, the product may also contain silicone, metal, or plastic.

3. Use

3.1 The Lymed product must be used in the manner described in these instructions for use. Use in accordance with the instructions supports product safety as well as its intended performance and service life.

3.2 The Lymed product is worn as the innermost layer of clothing. Lightweight undergarments may normally be worn underneath the Lymed product. Protect the product from mechanical stress and friction that may occur, for example, during household tasks or when using abrasive footwear or assistive devices. The product may be worn continuously during the day and night as required or in accordance with the instructions of the party responsible for care.

3.3 Always ensure before use that the product has been donned correctly. If the fit changes during use, it is recommended that the user's measurements or the product size be reviewed.

3.4 When using skin care products such as creams, ensure that they are fully absorbed or dry before donning the Lymed product. Do not apply fragrances or textile treatment agents to the product. Chemicals and cosmetic products may damage the material fibres, thereby reducing the product's service life and performance.

3.5 Simultaneous or layered use of multiple Lymed products is permitted only under the guidance of the party responsible for care. In such cases, Lymed products are designed and manufactured for concurrent use. Products must not be layered or used together with products from other manufacturers without specific guidance from the party responsible for care.

3.6 Do not use the product directly over an open wound or broken skin. Protect the affected area appropriately before use. Combined use of the Lymed product with wound care or scar management products and dressings requires assessment by the party responsible for care. Combined use may affect product fit; therefore, any bulky products or dressings must be taken into account during measurement or when selecting the size.

3.7 The product may be used in water, provided that it is washed as soon as possible after use.

4. Donning

4.1 Carefully review the donning instructions before use. Donning instructions are available at www.lymed.fi. Donning a Lymed product requires time and proper technique. Learning the correct donning technique may initially take time. Avoid forcefully pulling or stretching the product, particularly at the seams. An incorrectly donned product may feel uncomfortable or appear to be the wrong size.

4.2 Allow sufficient time for dressing. Remove sharp jewellery before dressing and ensure that your nails are smooth and short. Open any fastenings. Position the product gradually without pulling at the seams. More extensive garments may, if necessary, be turned partially or completely inside out and gradually rolled into place while avoiding pulling. Close any fastenings and ensure that the edges of the product are positioned correctly. Check that joint areas and anatomical contours are properly aligned, and that seams are not twisted. Smooth out any wrinkles in the fabric. The product is correctly positioned when the smoother and flatter side of the seam is against the skin.

4.3 Permitted separate donning aids include fabric sliding aids, friction-enhancing mats or grip-enhancing gloves, such as household gloves. Do not use donning aids that may damage or overstretch the product, such as donning frames.

5. Repair

5.1 As medical devices, Lymed products must not be repaired by the user or by a third party. Only the manufacturer is authorised to repair or modify Lymed products.

6. Service life

6.1 Lymed products are designed for long-term use. However, their service life cannot be compared to that of ordinary garments. With regular and typical use, the estimated service life of the product is 3–6 months. The estimated service life is based on reasonably foreseeable external influences and conditions of use. In products subject to wear and tear, such as stockings and gloves, service life may vary considerably between users. Service life is influenced by, among other factors, purpose and frequency of use, level of pressure or compression, conditions of use, user anatomy, skin characteristics, donning technique, washing routine, and prolonged exposure to sunlight.

6.2 The Lymed product must be replaced regularly in order to maintain performance. The condition and fit of the product must be monitored during use. The product must be replaced at the latest if elasticity decreases, the user's measurements or weight change, fit changes, the product can no longer be properly cleaned, the material feels greasy, or the product becomes damaged.

7. Washing instructions

7.1 Follow the washing instructions to ensure that the product's properties are maintained and to prevent premature wear and

tear. The product is recommended to be washed after each use in order to maximise its durability. This may mean machine washing the product even daily. It is recommended to have more than one product in use so that products can be washed and worn alternately, thereby extending their combined service life. Failure to wash the product may cause skin symptoms, particularly in users with sensitive skin.

7.2 Hand washing is not recommended, as the material will not be properly cleaned. If the product must occasionally be washed by hand, it is important to return to machine washing as soon as possible.

- Machine wash at 40°C using a normal cycle.
- Wash inside out in a laundry bag. Close any zips, velcro fastenings and hooks.
- Use a mild detergent that does not contain fabric conditioner, bleach, or strong fragrances.
- The use of fabric conditioner, detergents containing fabric conditioner, and bleach is prohibited.
- Line dry, dry flat or tumble dry at low temperature only. Do not wring when wet.
- Do not dry the product in direct sunlight, in a microwave, in an oven, or on a radiator.
- Do not iron or steam.
- Wash with similar colours. Colours may fade or transfer pigment.

8. Contexts of use

8.1 The party responsible for care may prescribe a Lymed product for indications other than those listed below, based on an individual assessment.

8.2 Scar management

8.2.1 Lymed products are used as part of scar management to support scar control and the scar maturation process, particularly following burns, extensive skin grafts and surgical procedures. They are intended for the management of hypertrophic scars and keloids and may also be used preventively when wound or graft healing is delayed, and the risk of hypertrophic scar formation is increased. Even pressure may support scar maturation, help manage scar thickening and stiffness, and alleviate scar-related discomfort such as itching and redness.

8.3 Oedema management

8.3.1 Lymed products are used as part of oedema management and prevention in various situations associated with swelling and circulatory load. The products support blood and fluid circulation through external compression. They may also be used in venous conditions where lower limb wound dressings are applied, to support oedema management, and to help maintain the position of dressings and wound coverings as part of a comprehensive treatment or recovery plan.

8.4 Sensorimotor and neurological application

8.4.1 Lymed products are used as rehabilitation aids and therapeutic tools in neurological physiotherapy and occupational

therapy to support body control, movement regulation, and body awareness as part of a professionally guided rehabilitation and treatment plan. The pressure applied by the product may enhance proprioceptive feedback, which may support muscle activation and movement control during rehabilitation. The products may also provide external support to joints, which can help manage joint hypermobility and support predictability and control of movement in everyday situations. In clinical practice, the products are also used as part of pain management alongside other treatment modalities, particularly in situations where pain is associated with impaired body control, joint hypermobility, sensorimotor deviations, or tissue load.

8.5 Post-operative aftercare

8.5.1 Lymed products may be used as part of aftercare following various invasive surgical procedures. They provide external pressure and/or external support, which supports post-operative recovery by offering external support to the operated body area. Pressure and external support may help support the management of oedema, swelling, bruising, and scars, alleviate recovery-related discomfort, and support movement and functional activity during recovery. The products may also be used to assist in keeping dressings or wound coverings in place as part of an overall treatment or recovery plan.

9. Contraindications, warnings and restrictions of use

9.1 The risks associated with the use of a Lymed product are generally low. The product may be used alongside many medical treatments, subject to assessment by the party responsible for care. Suitability, implementation, and individual instructions regarding use are always determined by the party responsible for care. Use of the product may increase urinary frequency and may cause physical urticaria.

9.2 Contraindications to the use of a Lymed product may include untreated heart failure, severe arterial disease, acute deep vein thrombosis, acute skin infection, pressure urticaria, or practical limitations to use, such as significant difficulty with dressing. Particular caution must be exercised when a Lymed product is used in individuals with cognitive impairment, reduced capacity for self-determination, neuropathy, sensory deficits, fragile skin, acute lymphostasis, or acute erysipelas. Inform the party responsible for care of all medical conditions or health-related factors that may affect the use of the Lymed product.

9.3 Discontinue use of the Lymed product and contact the party responsible for care or a doctor if severe pain, significant discomfort, numbness, tingling, coldness or bluish discolouration of a limb, abnormal swelling, circulatory disturbance, weakening or loss of pulse in a limb, dizziness, chest pain, breathing difficulties or shortness of breath, skin breakdown, other skin symptoms, or allergic reactions occur during use. This also applies to symptoms not specifically mentioned in these instructions for use.

9.4 If the expected therapeutic effect is not achieved, or if the effect of the product changes during use, contact the healthcare professional responsible for your care or your doctor. The suitability, size, pressure level, or other treatment solution may require reassessment. In sensorimotor and neurological contexts of use, the body may adapt to pressure, and the perceived effect of the

product may decrease over time. In such cases, it may be necessary to pause use or alternate between products with different pressure levels. In these situations, contact the party responsible for your care or a doctor.

9.5 Do not use a damaged product. If any part becomes detached from the product, discontinue use immediately and keep the detached part out of the reach of children. Do not allow children to chew or suck the product.

9.6 The material of the products may be more slippery than ordinary clothing. In gloves and stockings, this may reduce grip and must be taken into account during use.

9.7 The person placing the order for the Lymed product and the party responsible for care are responsible for ensuring that the planned user-specific product solutions are appropriate, usable, and safe in daily life. This applies particularly to situations where combinations of multiple additional features or otherwise complex product solutions are planned. Such configurations may increase, for example, the risk of entanglement or tripping.

9.8 Please note that:

9.8.1 The product must not be transferred, lent, or used by another person. The product has been selected or manufactured based on the measurements, intended use, and treatment plan of an individual user. When used by another person, the product may be unsuitable and may cause harm, even if the symptoms or need for use appear similar.

9.8.2 Device identification information: Do not remove the product information label attached to the product. It contains important information about the product. Removing the label eliminates product identification and traceability, as well as the possibility for modifications and repairs.

9.8.3 If a serious incident occurs in relation to the product or its use, an appropriate incident report must be submitted, in accordance with Regulation (EU) 2017/745 (MDR), to the competent authority of your country of residence as well as to the manufacturer: Lymed Oy.

9.8.4 Labelling and symbols: The meanings of the symbols used on the product packaging label are provided in text beneath the symbols. If there is any uncertainty regarding the meaning of the symbols, contact the manufacturer or the distributor that supplied the device.

10. Storage and disposal

10.1 Store the unused product protected from sunlight, in a dry place at room temperature.

10.2 Do not use the product after the expiry date indicated on the packaging.

10.3 Retain the product packaging for the entire service life of the product, as it contains product identification information.

10.4 A used product may be recycled as textile waste or disposed of with household waste in accordance with local practices in your country of residence. Remove product identification details before disposal.

11. Providing feedback

11.1 Provide feedback on your product and your experience of using it at www.lymed.fi

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